Note: Document originally drafted in the English language.  
注释：文件最初用英语起草。

**Product Description**

The B-100 housing is designed for Varex Imaging rotating Anode inserts having 71 mm (2.8 inches) and 80 mm (3.0 inches) diameter targets.

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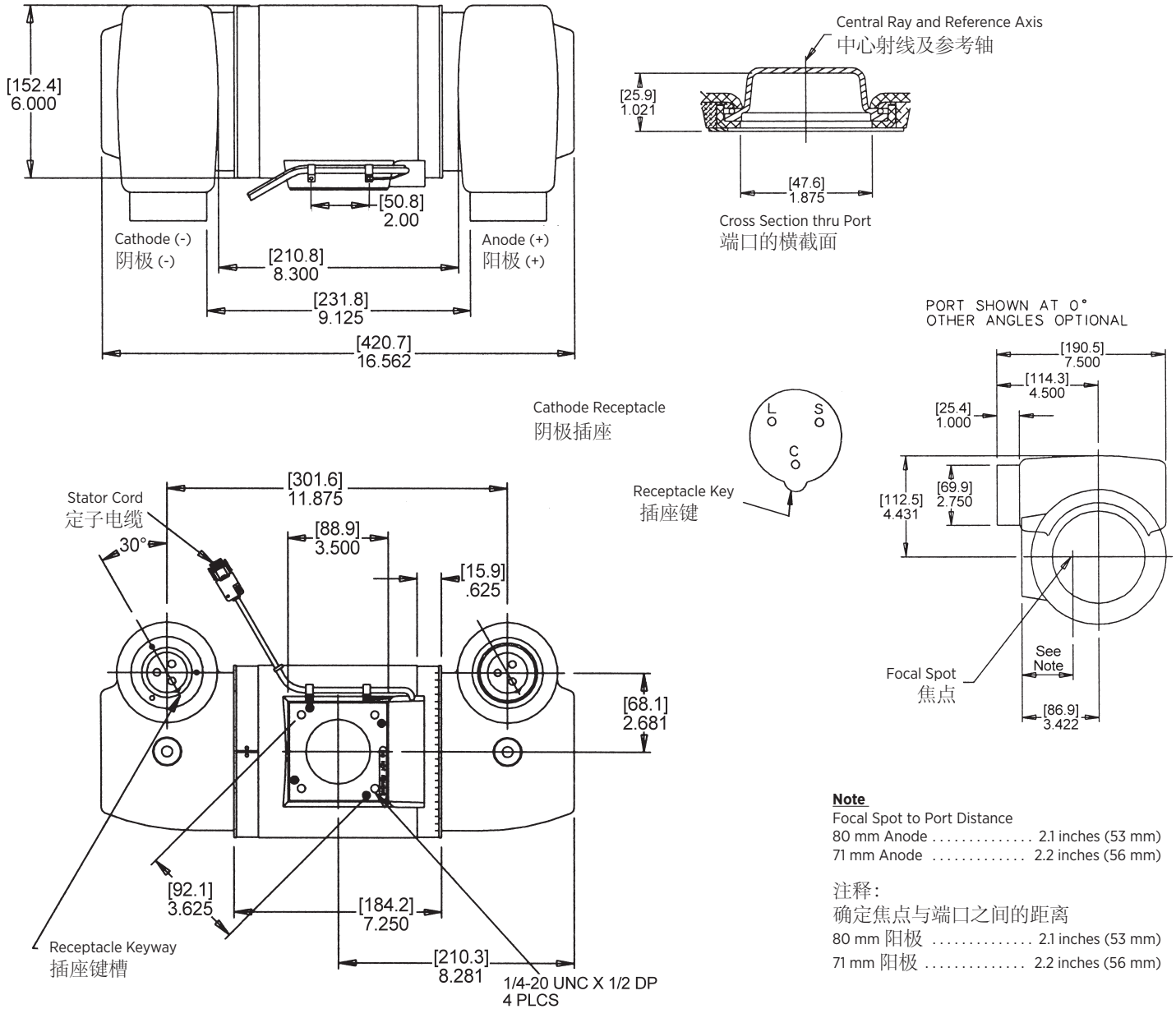
<b>Product Description</b>	
Nominal X-ray Tube Voltage .....	150 kV
Maximum Cathode to Ground .....	75 kV
Maximum Anode to Ground .....	75 kV
Grid to Cathode (if applicable) .....	-4 kV
<b>Grid Control Voltages</b>	
Typical Bias Voltage for Cutoff at 150 kV .....	-3600 Vdc
Grid Voltage for Exposure .....	0 Vdc
Housing Heat Storage Capacity .....	926 kJ (1,250 kHU)
<b>Nominal Continuous Input Power</b>	
without air circulator	
.....	185 Watts (250 HU/sec) IEC 60613:2010
with air circulator	
.....	370 Watts (500 HU/sec) IEC 60613:2010
<b>X-Ray Tube Assembly</b>	
Permanent Filtration ...	0.7 mm Al/75 kV IEC 60522/1999
Leakage Technique Factors .....	150 kV, 3.3 mA
<b>Temperature Limits for Storage and Transport</b>	
.....	-20°C to +75°C
Humidity .....	+10% to +90%
Atmospheric Pressure Range .....	70 kPa to 106 kPa
<b>Thermal Switch</b> .....	
Normally Closed ...	Opens @ 85°C ±3.9°C (185°F ±7°F)

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Dimensions are for Reference only  
维度是供仅供参考


**Note:**

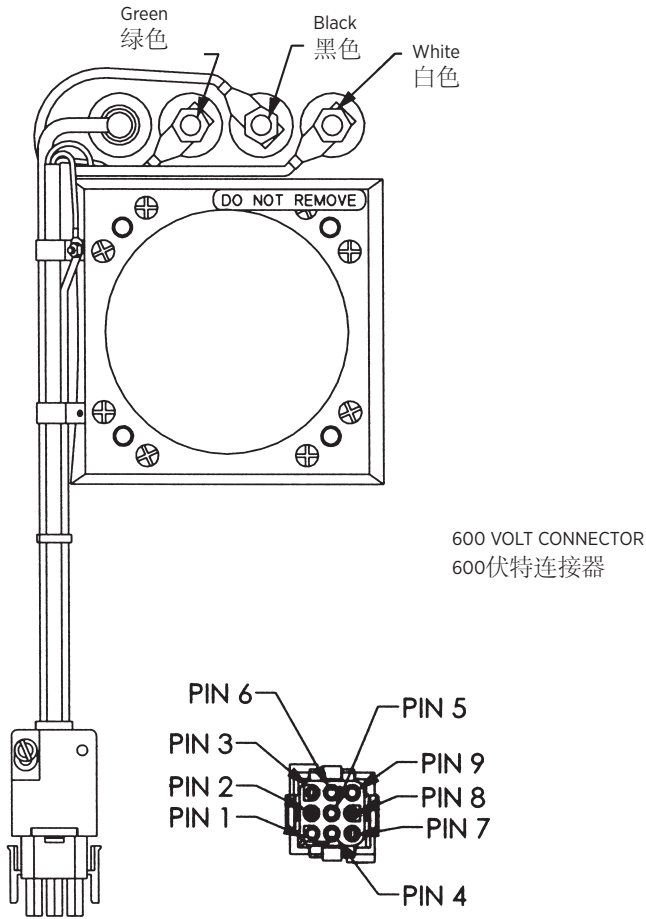- Reference dimensions are for information purposes and are not required for inspection.
- Aperture size optional  
0.937 Sq. in. 1.250 Sq. in.
- Federal Standard Cable Receptacles:  
2 1/2 -20 UNF -2B THREAD

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Stator Ratings and Characteristics  
定子额定值及特征

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To ensure proper target rotation, each stator cord is equipped with a special twist-locking connector and a mating connector.

**Stator Power:**  
Time to full speed of the anode is a function of the power rating of the “starter” and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed for 71mm (2.8 inch) and 80mm (3.0 inch) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 500 volts rms.

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
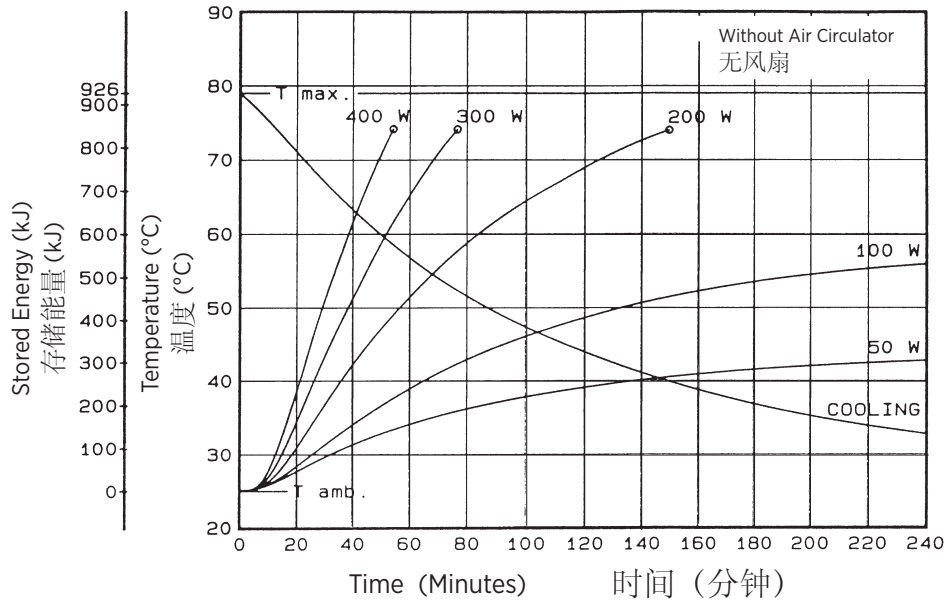
Tube Housing Assembly Heating and Cooling  
管套组件加热与冷却

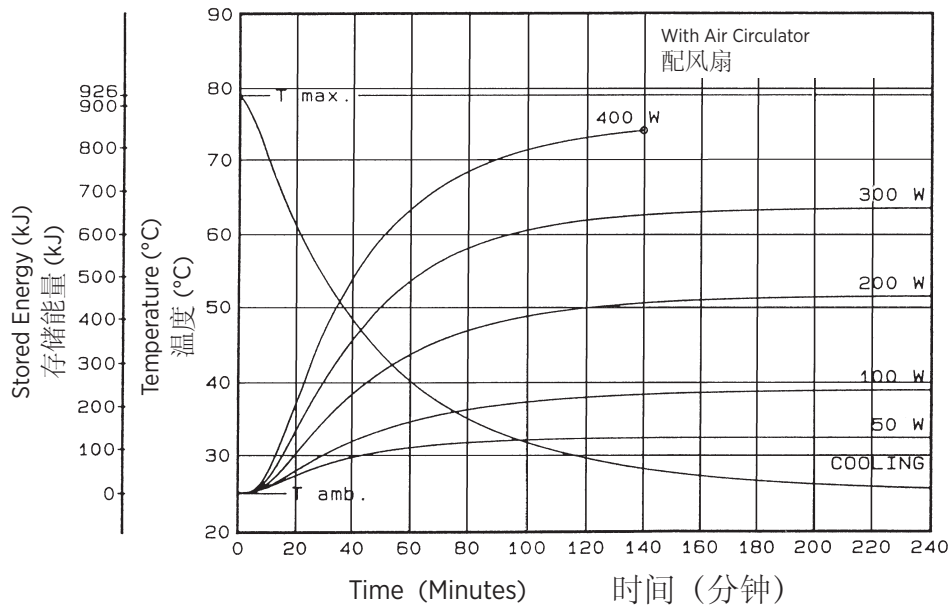Tube Housing Assembly Heating and Cooling  
管套组件加热与冷却

**Note:**  
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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