Note: Document originally drafted in the English language.  
注释：文件最初用英语起草。

### Product Description

The B-150H housing is designed for Varex Imaging rotating anode inserts having 102mm (4 inch) diameter targets.

IEC Classification ..... Class 1

Weight, Approximate:

Housing ..... 19.5 kg (43 lbs)

with Trunnion Rings ..... 22.9 kg (50.5 lbs)

with Trunnion Rings and Fan Assembly  
..... 24.1 kg (53.2 lbs)

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



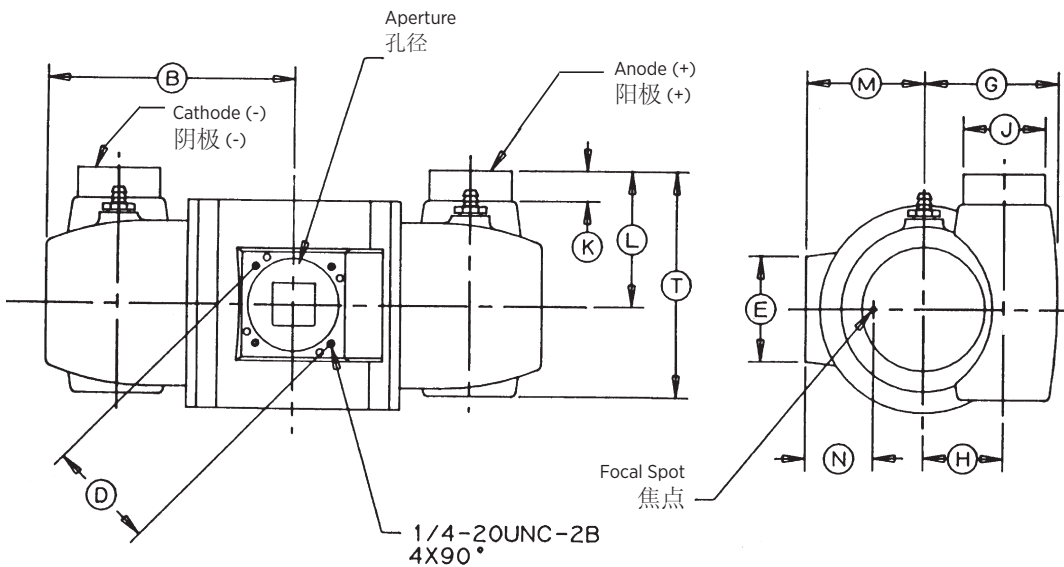
<b>Product Description</b>	
Nominal X-ray Tube Voltage .....	150 kV
Maximum Cathode to Ground .....	75 kV
Maximum Anode to Ground .....	75 kV
Grid to Cathode (If applicable) .....	-4 kV
<b>Grid Control Voltages</b>	
Typical Bias Voltage for Cutoff at 150 kV .....	-3600 Vdc
Grid Voltage for Exposure .....	0 Vdc
<b>Maximum X-ray Tube Assembly Heat Content</b>	
.....	1,111 kJ (1,500 kHU)
<b>Nominal Continuous Input Power</b> (max. housing temperature 78°C)	
without air circulator .....	200 Watts (270 HU/sec)
with air circulator .....	400 Watts (540 HU/sec)
<b>X-Ray Tube Assembly</b>	
Permanent Filtration .....	0.7mm Al/75 kV IEC 60522
<b>Loading Factors for Leakage Radiation</b> .... 150 kV, 4.0 mA	
<b>Ambient Air Temperature Limits for Operation</b>	
.....	5°C to 40°C
<b>Temperature Limits for Storage and Transport</b>	
.....	-20°C to +75°C
Humidity .....	+10% to +90%
Atmospheric Pressure Range .....	70 kPa to 106 kPa
<b>X-Ray Tube Assembly (Complies to)</b> .....	
	IEC 60601-2-28

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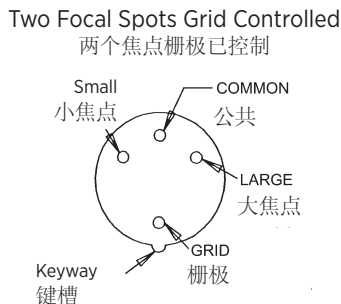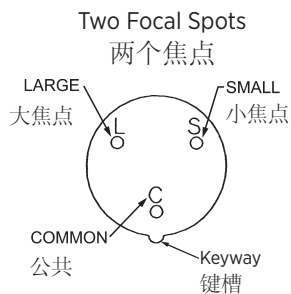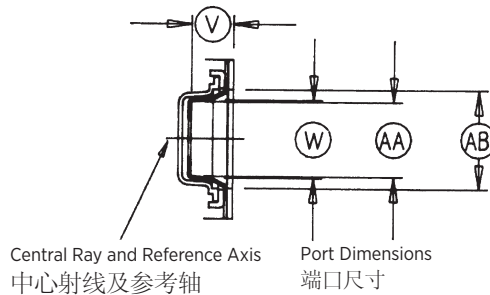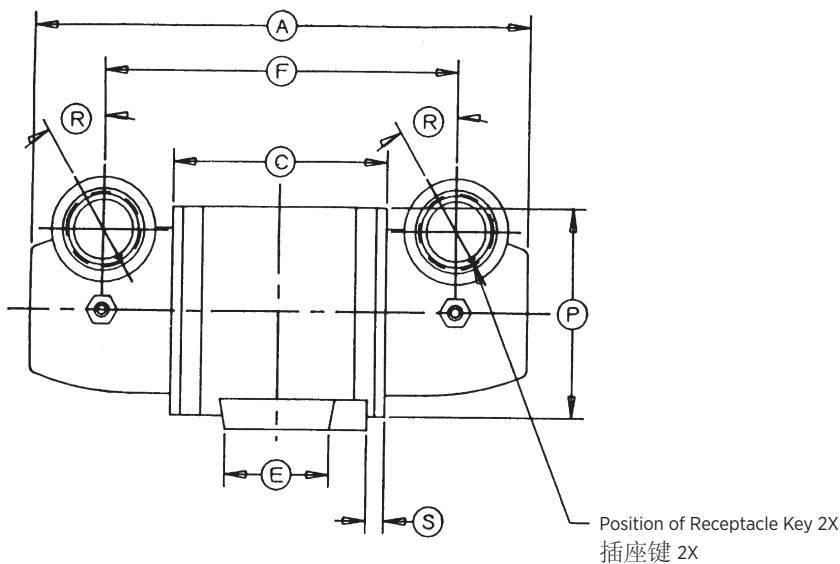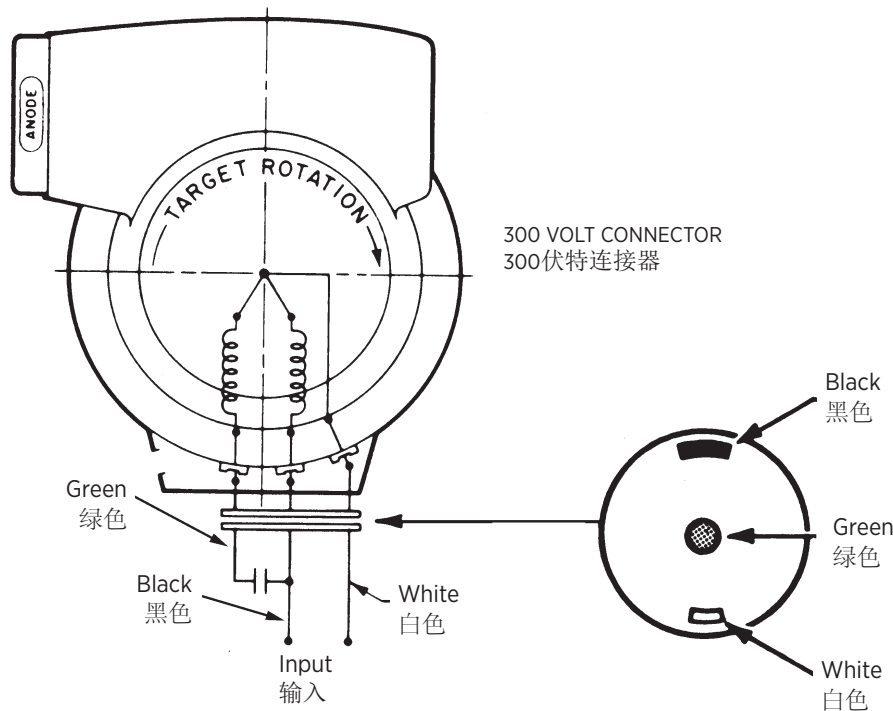
TO ENSURE PROPER TARGET ROTATION, EACH STATOR CORD IS EQUIPPED WITH A SPECIAL TWIST-LOCKING CONNECTOR AND A MATING CONNECTOR.

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**Stator Power:**  
Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 102 mm (4 inch) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

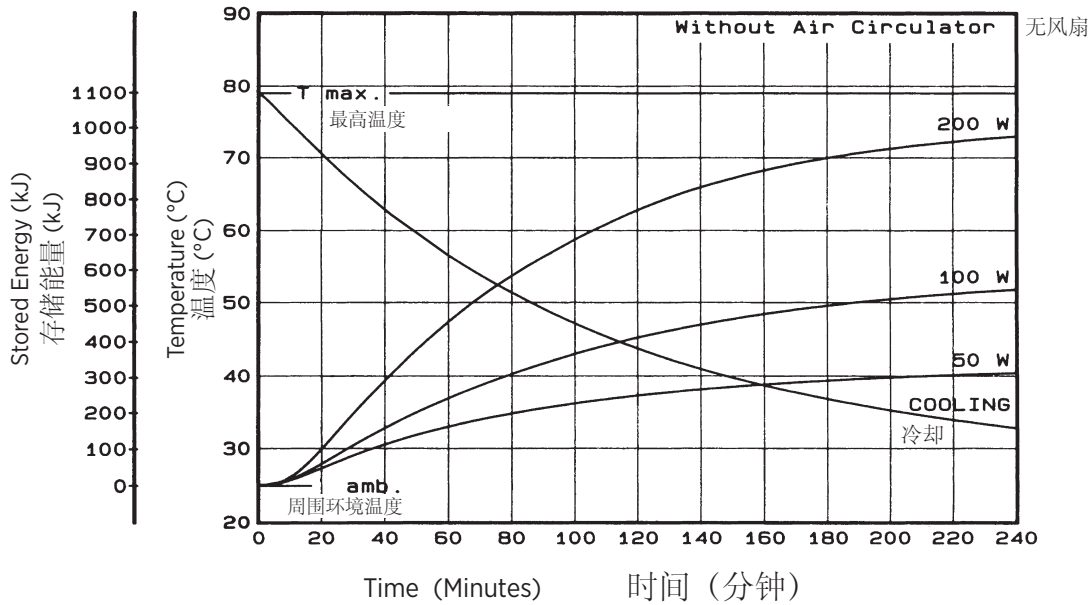
No more than two high speed starts per minute are permissible. The starting voltage must never exceed 500 volts rms.

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
Tube Housing Assembly Heating and Cooling  
管套组件加热与冷却

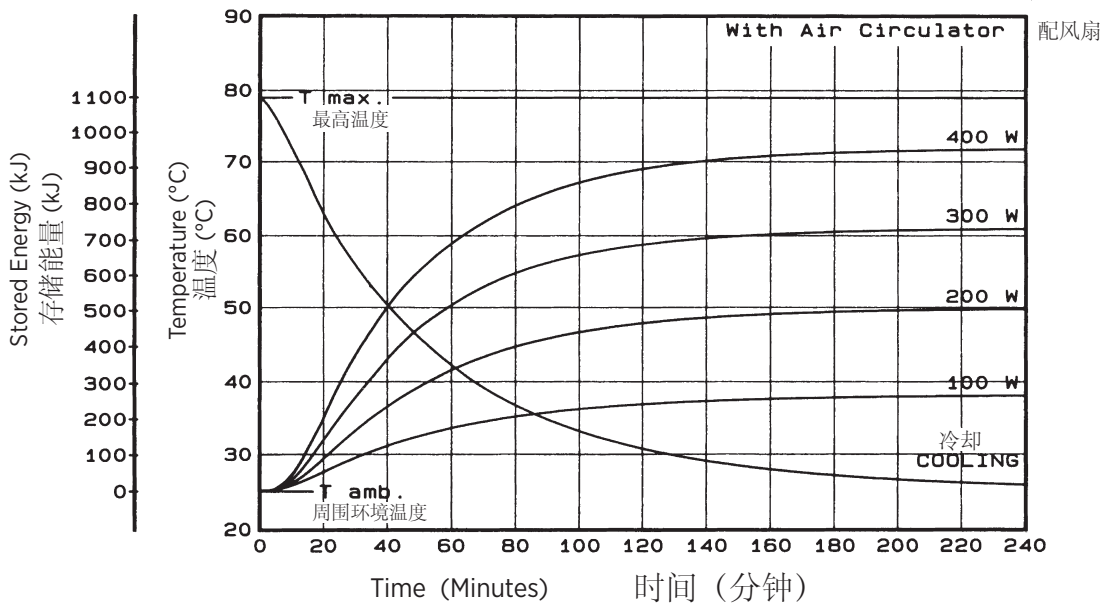Tube Housing Assembly Heating and Cooling  
管套组件加热与冷却

**Note:**  
 1. Heat inputs into housing include tube power, filament power, and stator power.  
 2. Heating curves based on no restrictions of natural convection around tube housing assembly.

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