Note: Document originally drafted in the English language.
注释：文件最初用英语起草。

Product Description

The B-160H housing is designed for Varex Imaging rotating Anode inserts having 108 mm (4.25 inch) diameter targets.

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Product Description

Nominal X-ray Tube Voltage 150 kV
 Maximum Cathode to Ground 75 kV
 Maximum Anode to Ground 75 kV
 Grid to Cathode (If applicable) -4 kV

Grid Control Voltages
 Typical Bias Voltage for Cutoff at 125 kV -3700 Vdc
 Grid Voltage for Exposure 0 Vdc

Maximum X-ray Tube Assembly Heat Content
 1,100 kJ (1,500 kHU)

Nominal Continuous Input Power
 (max. housing temperature 78°C)
 without air circulator ... 200 Watts (270 HU/sec) IEC 60613:2010
 with air circulator 400 Watts (540 HU/sec) IEC 60613:2010
 with HE-100 1000 Watts (1400 HU/sec) IEC 60613:2010

X-ray tube assembly cooling is provided by ambient air flow.

X-Ray Tube Assembly
 Permanent Filtration 1.0mm Al IEC 60522/1999

High Voltage Cable Receptacles Per IEC 60526

Loading Factors for Leakage Radiation 150 kV, 4 mA

Temperature Limits for Storage and Transport ... -20°C to +75°C
 Humidity +10% to +90%
 Atmospheric Pressure Range 70 kPa to 106 kPa

Thermal Switch Normally Closed
 3 A @ 120 Vac or 30 Vdc max.
 Open 76.7°C ±2.8°C (170°F ±5°F)

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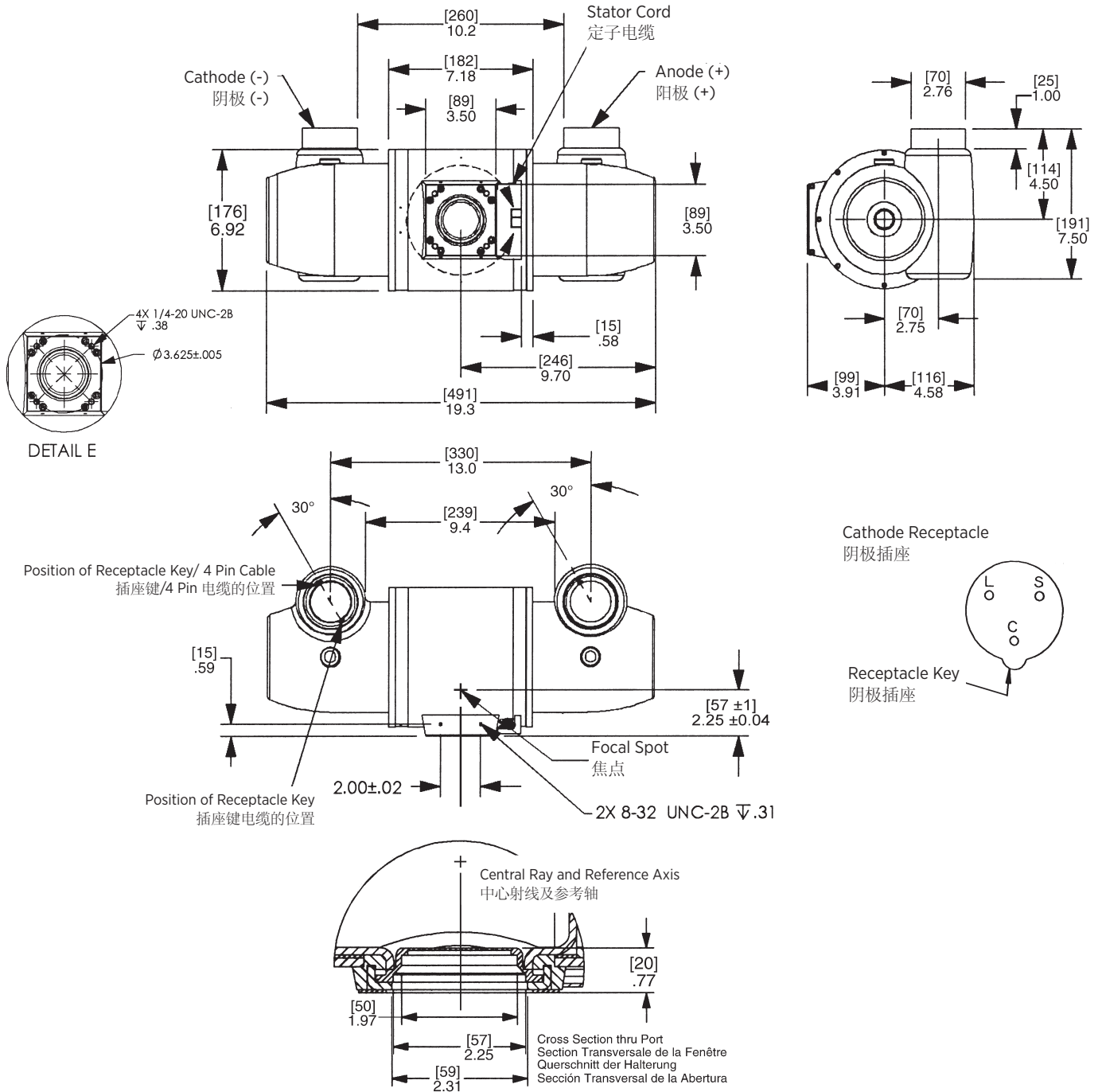
Dimensions are for Reference only
维度是供仅供参考


Note:

- Reference dimensions are for information purposes and are not required for inspection
- Aperture size optional:
0.937 Sq. In. 1.250 Sq. In.
- Cable receptacle
2 1/2 -20 Unf -2b thread

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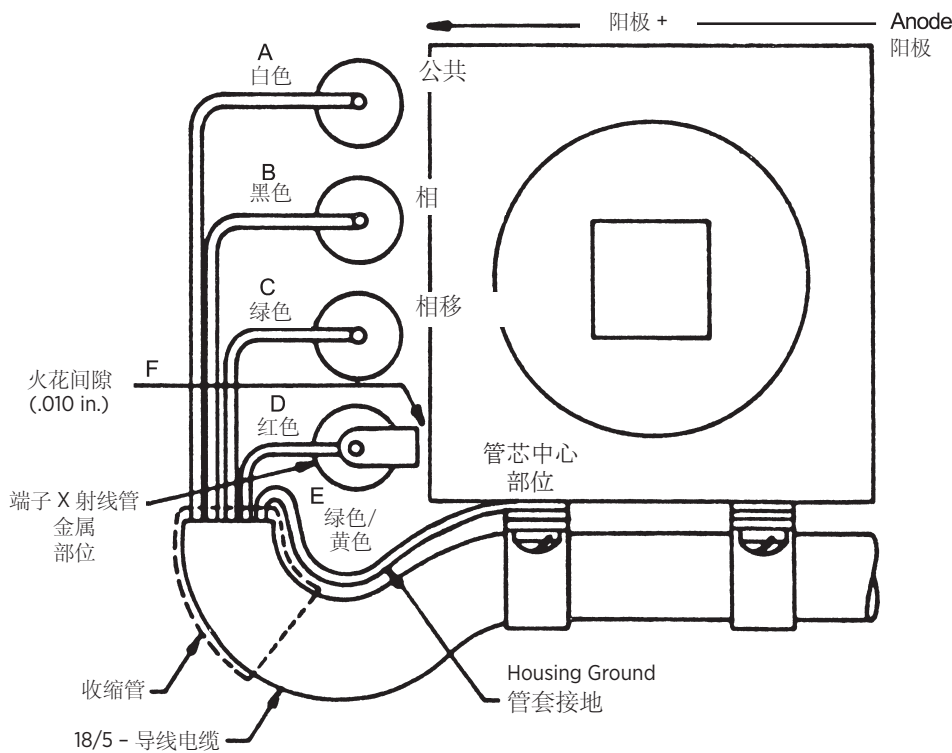Stator Power:

Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 108 mm (4.125 inch) anode series tubes is between 2 and 4.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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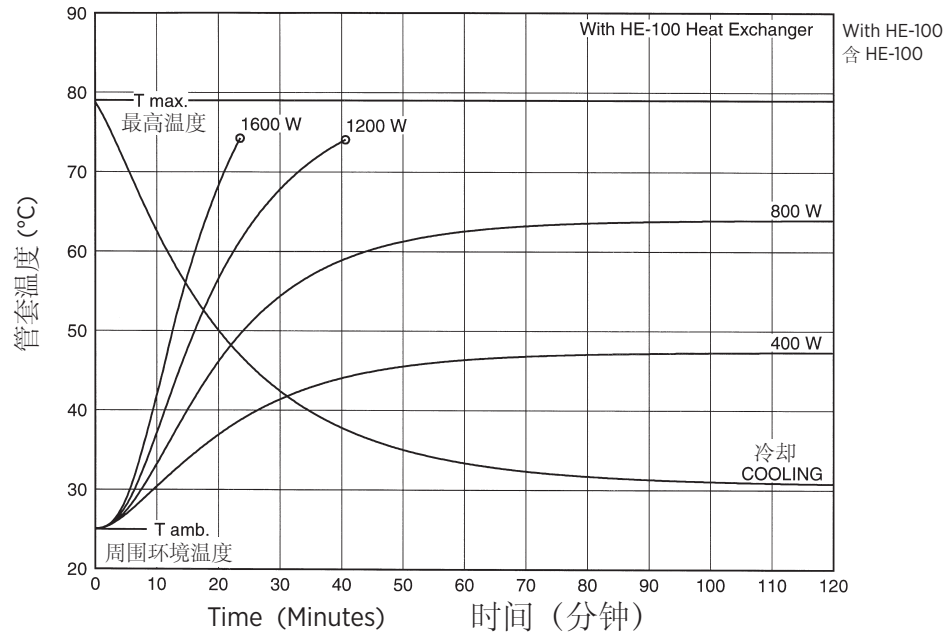
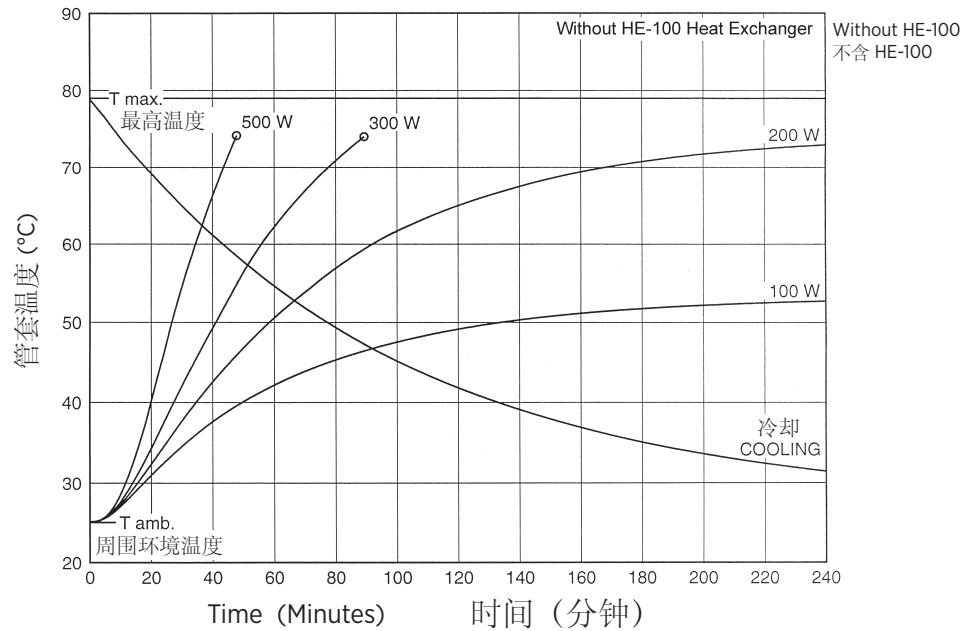
X-ray Tube Assembly Heating and Cooling Curve 管套组件加热与冷却

Caution:

Housings configured with heat exchangers having long hoses (> 5 ft or 1.5 m) are not configured for operation separate from the associated heat exchanger. In such cases, the tube unit must not be operated unless properly coupled to the heat exchanger. Reference the specific heat exchanger's product data sheet for the associated cooling curves.

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Note:

Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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