Note: Document originally drafted in the English language.  
注释：文件最初用英语起草。

**Product Description**

The B-110 housing is designed for Varex Imaging mammography inserts.

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管套 ..... 10.2 kg (22.5 lbs)

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<b>Product Description</b>	
Maximum Potential Difference	
Cathode to Ground .....	0 kV
Anode to Ground .....	52 kV
Grid Control Voltages: (If Applicable)	
Typical Bias Voltage for Focals .....	-5 to -100 Vdc
Housing Heat Storage Capacity .....	370 kJ (500 kHU)
Housing Continuous Heat Dissipation	
.....	100 Watts (135 HU/sec)
Maximum Housing Temperature .....	78°C
X-Ray Tube Assembly	
Permanent Filtration .....	0.63 mm Be IEC 60522
Leakage Technique Factors .....	55 kV, 4.0 mA
Cable Receptacles .....	Non Federal Standard 55 kV - Rating
Thermal Switch .....	
.....	normally closed Rating - 10 A @ 240Vac
Open .....	79.4°C ±3.9°C (175°F ±7°F)
Ambient Air Temperature Limits for Operation	
.....	5°C to 40°C
Temperature Limits for Storage and Transportation	
.....	-10°C to 75°C
Humidity .....	+10% to +90%
Atmospheric Pressure Range .....	70 kPa to 106 kPa
X-Ray Tube Assembly (Complies to) .....	IEC 60601-2-28

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**For Determining Focal Spot to Port Distance**  
确定焦点与端口之间的距离

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
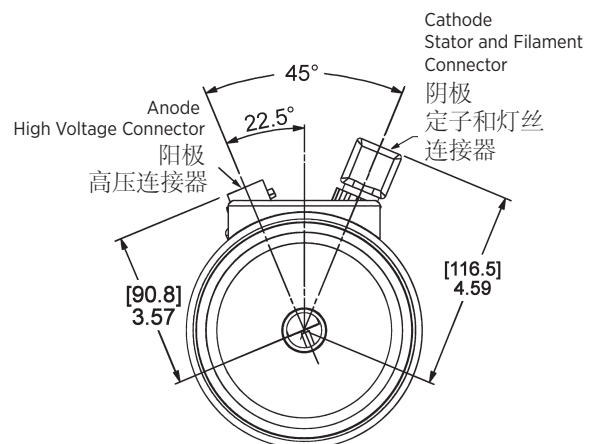
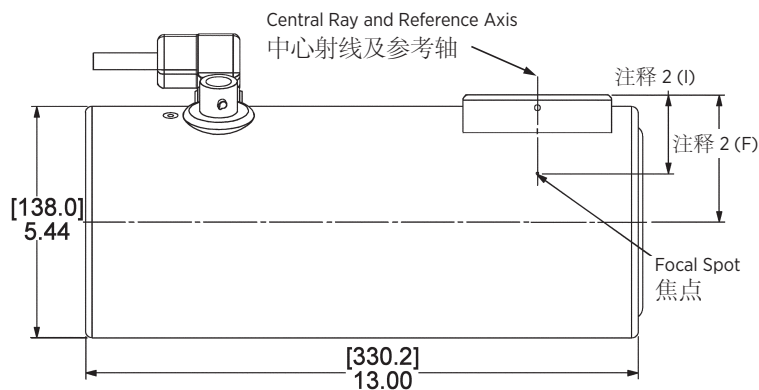
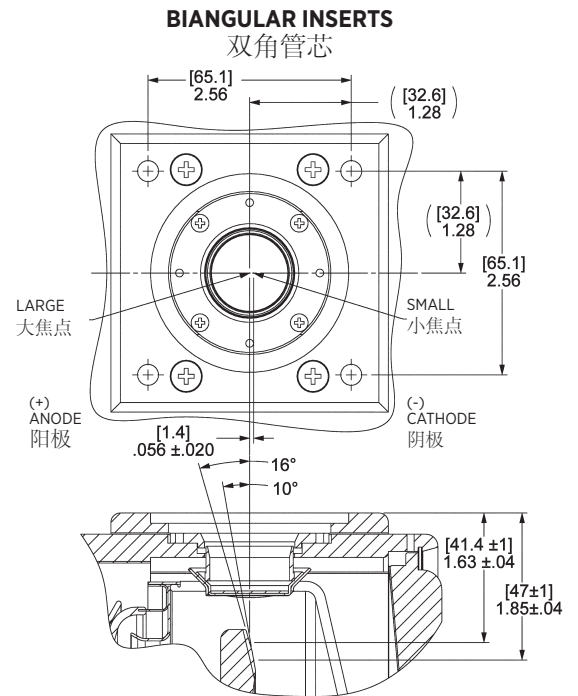
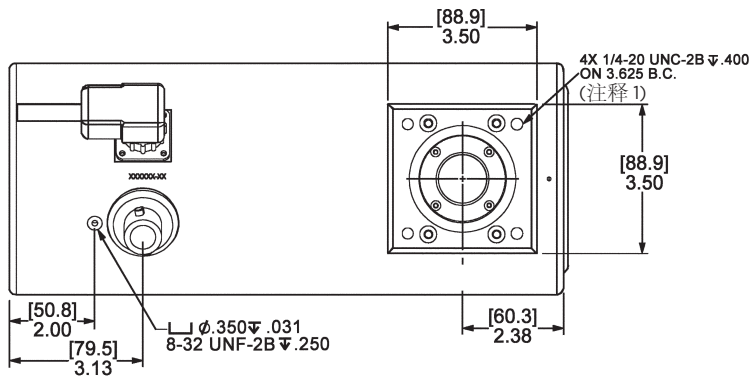
**Note:** Dimensions in [mm] Inches  
注释: 尺寸 (英寸)

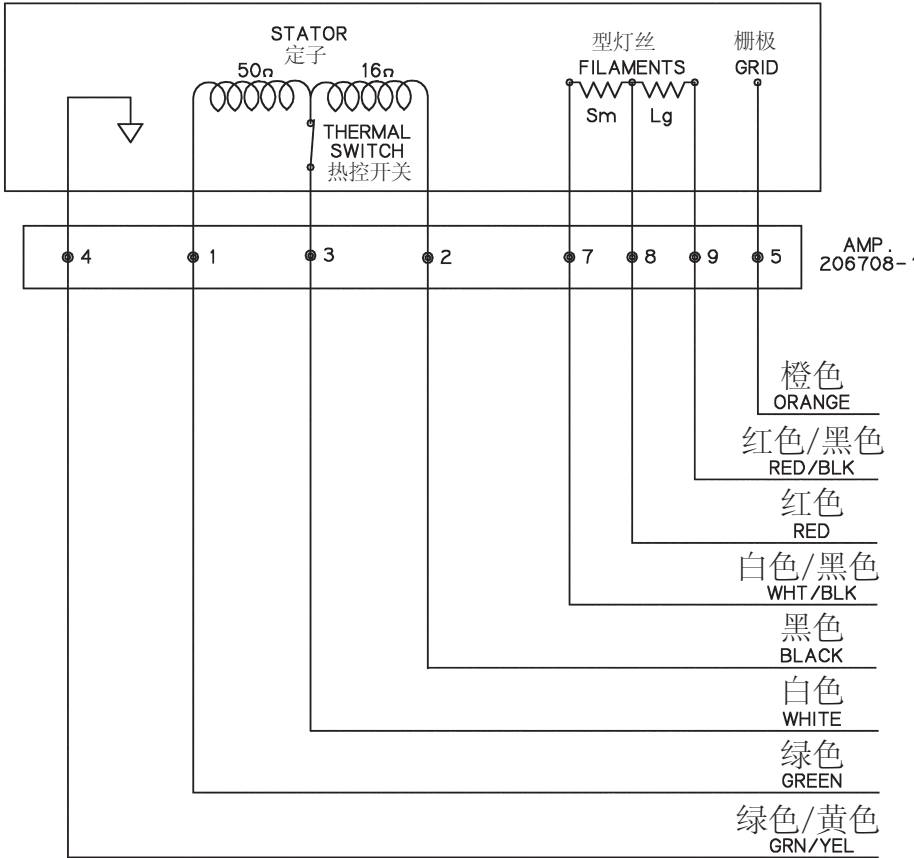Stator Ratings and Characteristics  
定子额定值及特征

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**Stator Power:**  
Time to full speed of the anode is a function of the power rating of the “starter” and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible.

Starter:	Start	Run
50/60 Hz	115 VAC	45 VAC
150/180 Hz	230 VAC	90 VAC

**NOTE:**  
For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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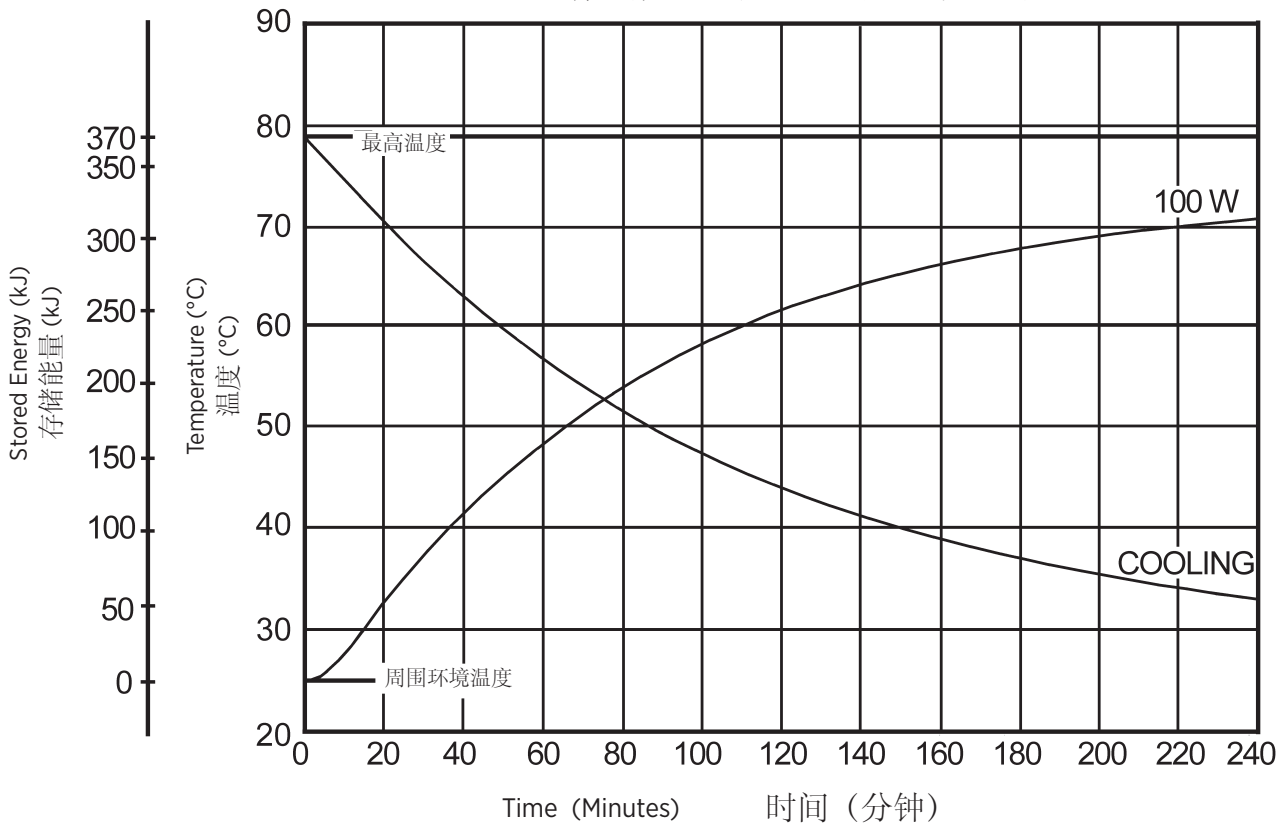
X-Ray Tube Assembly Heating and Cooling Curves  
X 射线管汇编热化/冷却的曲线

**Note:**  
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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Manufactured by Varex Imaging Corporation  
由万睿视影像有限公司生产

Specifications subject to change without notice.  
规格如有更改，恕不另行通知。