Note: Document originally drafted in the English language.
注释：文件最初用英语起草。

Product Description

The ROT350™ housing is used for rotating anode inserts having 90mm (3.5 inch) diameter targets.

IEC Classification Class 1

Weight, Approximate:
Housing & Tube 24 kg (55 lbs)

Mounting Trunnion

™All trademarks property of the respective manufacturer.

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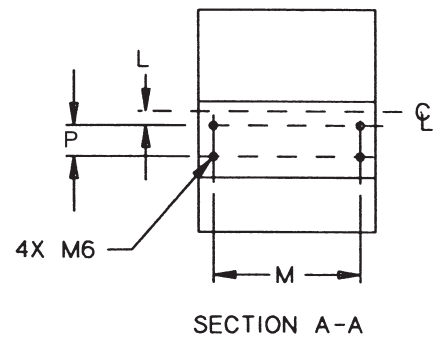
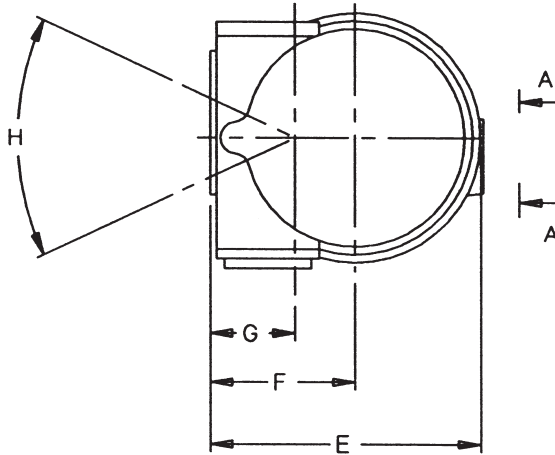
Product Description	
Maximum Potential Difference	150 kV
Cathode to Ground	85 kV
Anode to Ground	85 kV
Maximum X-Ray Tube Assembly Heat Content	1,250 kJ (1,700 kHU)
Maximum Housing Temperature	85°C
Maximum Continuous Heat Dissipation	
without Air Circulator	250 W (350 HU/sec)
with Air Circulator	350 W (490 HU/sec)
X-Ray tube assembly	
Permanent Filtration	0.7mm Al IEC 60522
Loading Factors for Leakage Radiation	150 kV, 4.0 mA
Temperature Limits for Storage and Transport	-25°C to + 70°C
Humidity	10% to 90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Thermal Switch	normally Closed
	5A @ 125 Vac
	(Activates on Bellow extension near 75 +/-5°C)
X-Ray Tube Assembly (Complies to)	IEC 60601-2-28

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


(As viewed looking down into the receptacle)
向插座内俯视进行查看

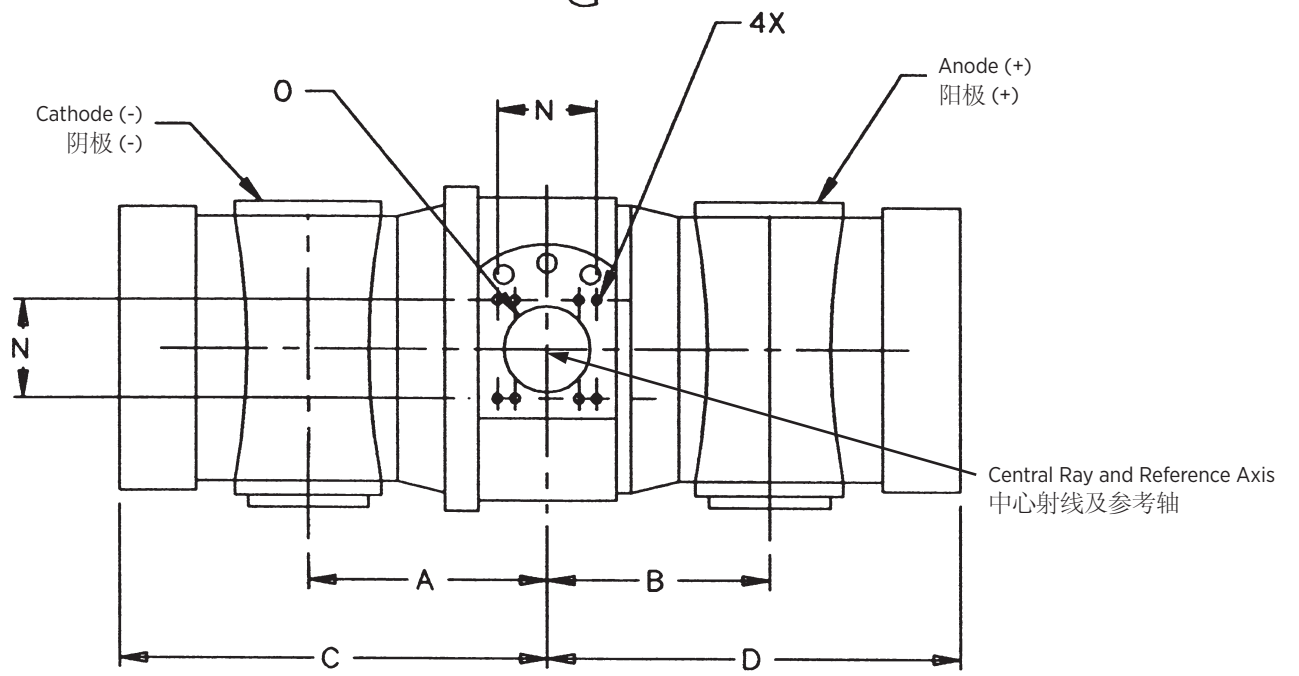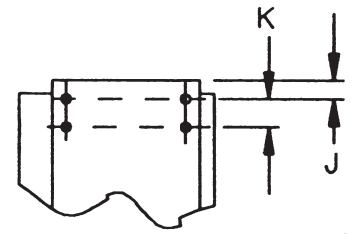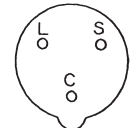Receptacle Key
插座键

Stator - Wiring Diagram - Refer to the equipment manufacturer's instructions.

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Stator Drive Frequency 定子驱动频率	RPM
50/60 Hz 150/180 Hz	2800/3400 ±10% 8200/9600 ±10%

Stator Power:

Time to full speed of the anode is a function of the power rating of the “starter” and the weight / diameter of the anode. The ROT350 stator is rated for regular speed and high speed starters.

Voltage:	Start	Run
60Hz	230 VAC	75 VAC
180Hz	400 VAC	85 VAC

Time to Full Speed:

50/60 Hz	1.0 sec.
150/180 Hz	2.0 sec.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

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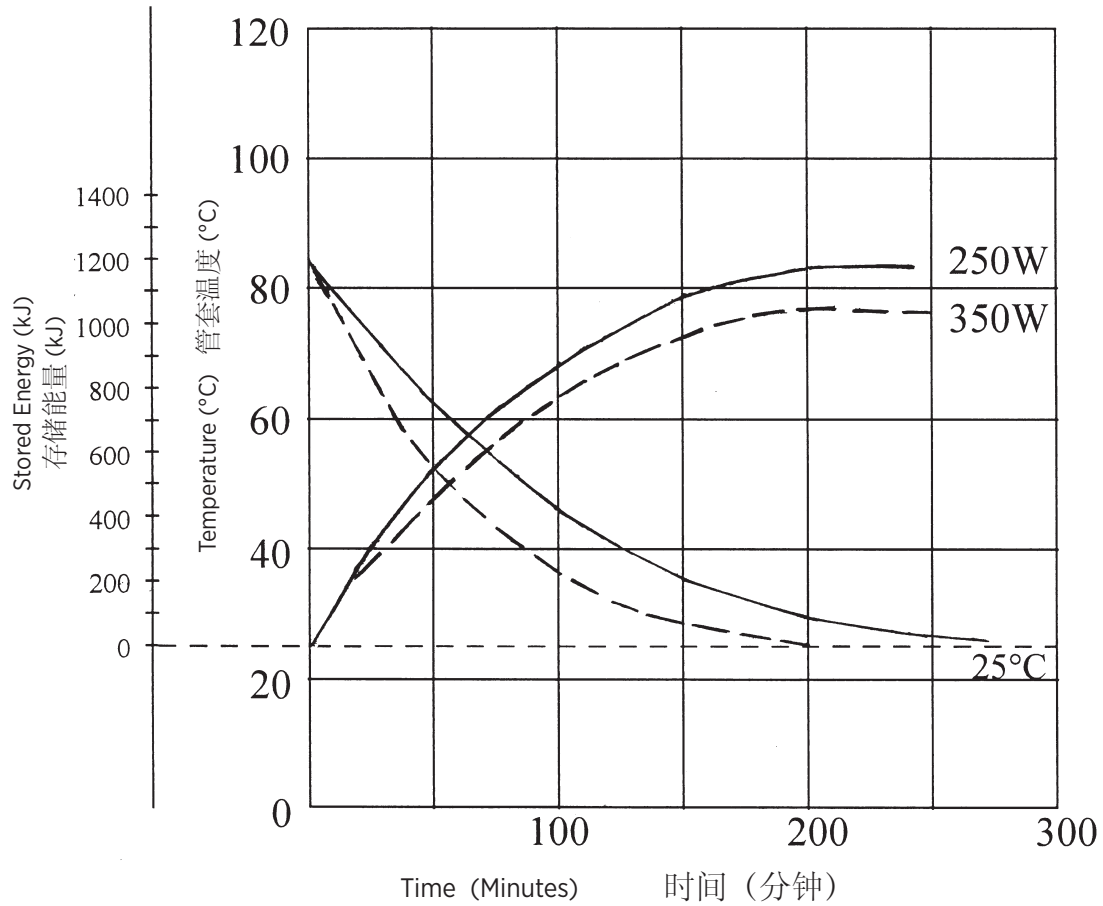
Note:
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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