Note: Document originally drafted in the English language.
注释：文件最初用英语起草。

Product Description

The B-115 housing is designed for Varex Imaging mammography inserts.

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Product Description

Nominal X-ray Tube Voltage
 Maximum Cathode to Ground 0 kV
 Maximum Anode to Ground 39 - 49 kV

Grid Control Voltages: (If Applicable)
 Typical Bias Voltage for Focals -5 to -100 Vdc

Maximum X-Ray Tube Heat Content 370 kJ (500 kHU)

Nominal Continuous Input Power
 100 Watts (135 HU/sec)
 IEC 60613:2010

Maximum Housing Temperature 78°C

X-Ray Tube Assembly
 Permanent Filtration ... 0.63 mm Be IEC 60522/1999
 (0.0mm Al/49 kV)

Leakage Technique Factors 49 kV, 2.0 mA

Cable Receptacles Non Federal Standard
 55 kV - Rating

Thermal Switch normally closed
 Open 79.4°C ±3.9°C (175°F ±7°F)
 Rating - 10 A @ 240Vac

Ambient Air Temperature Limits for Operation
 5°C to 40°C

Temperature Limits for Storage and Transportation
 -10°C to 75°C
 Humidity +10% to +90%
 Atmospheric Pressure Range 70 kPa to 106 kPa

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For Determining Focal Spot to Port Distance
确定焦点与端口之间的距离

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



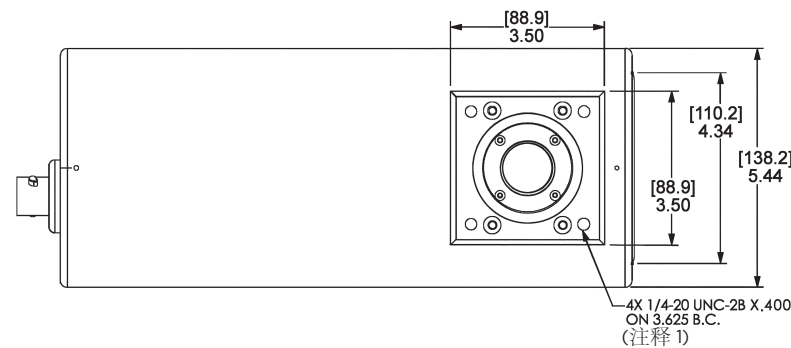
BIANGULAR INSERTS
双角管芯

FRONT VIEW
看前部

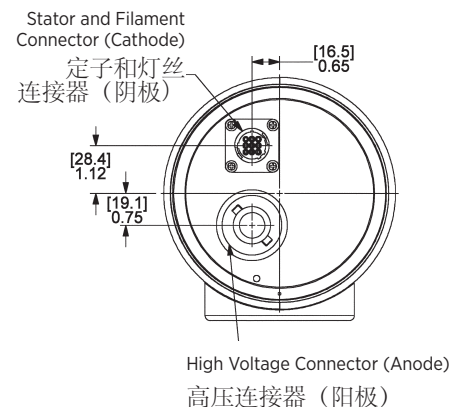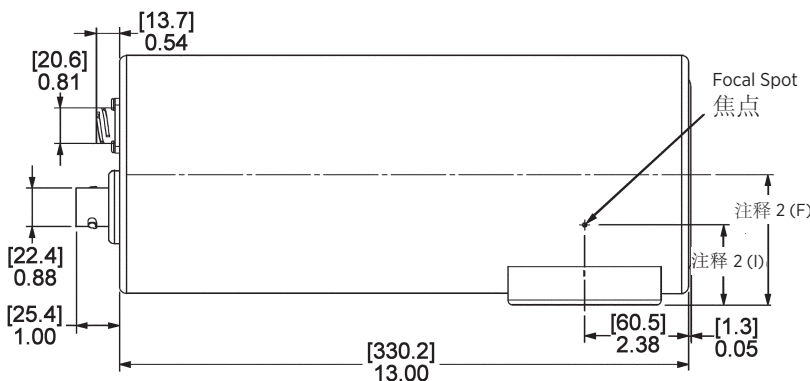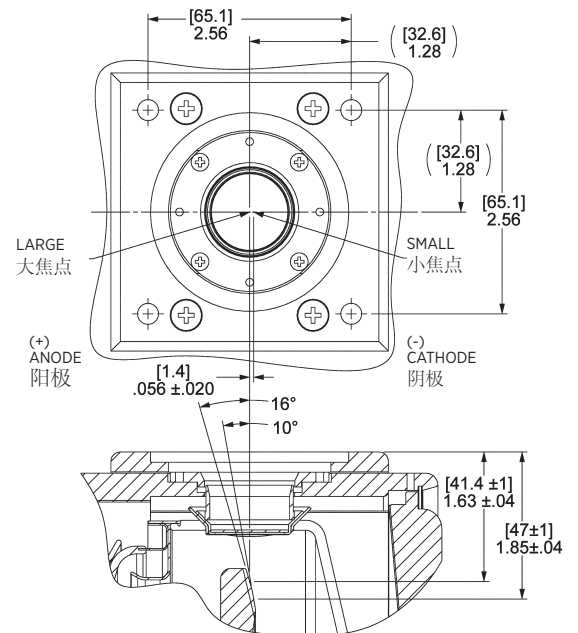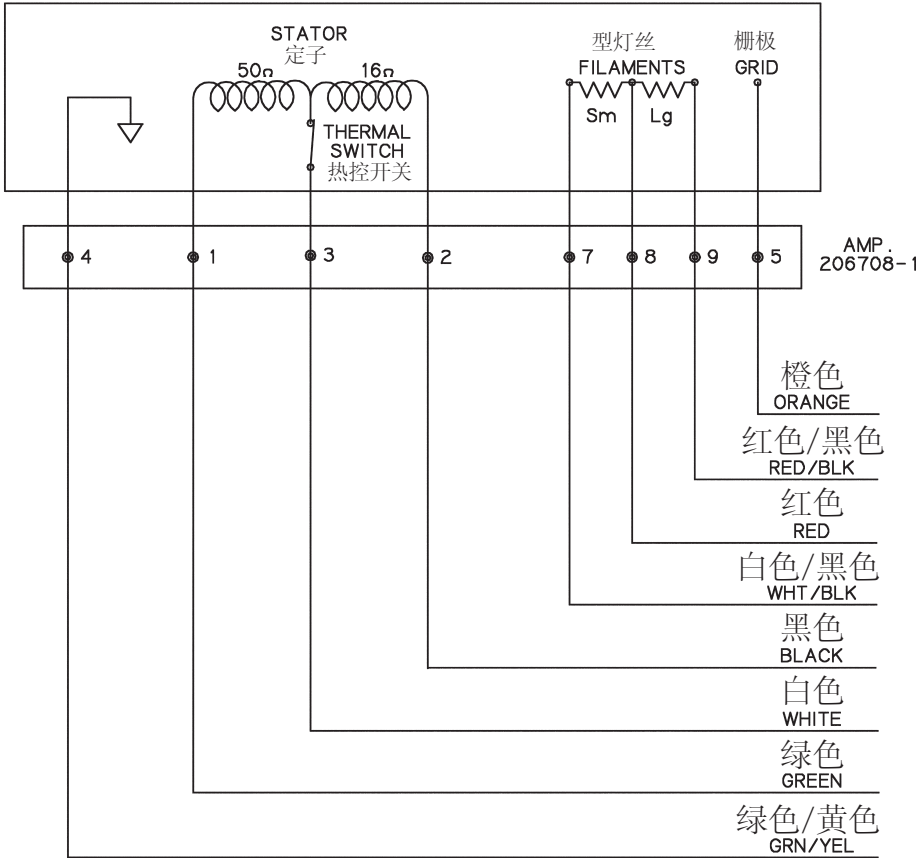Stator Ratings and Characteristics
定子额定值及特征

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Stator Power:
Time to full speed of the anode is a function of the power rating of the “starter” and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible.

Starter:	Start	Run
50/60 Hz	115 VAC	45 VAC
150/180 Hz	230 VAC	90 VAC

NOTE:
For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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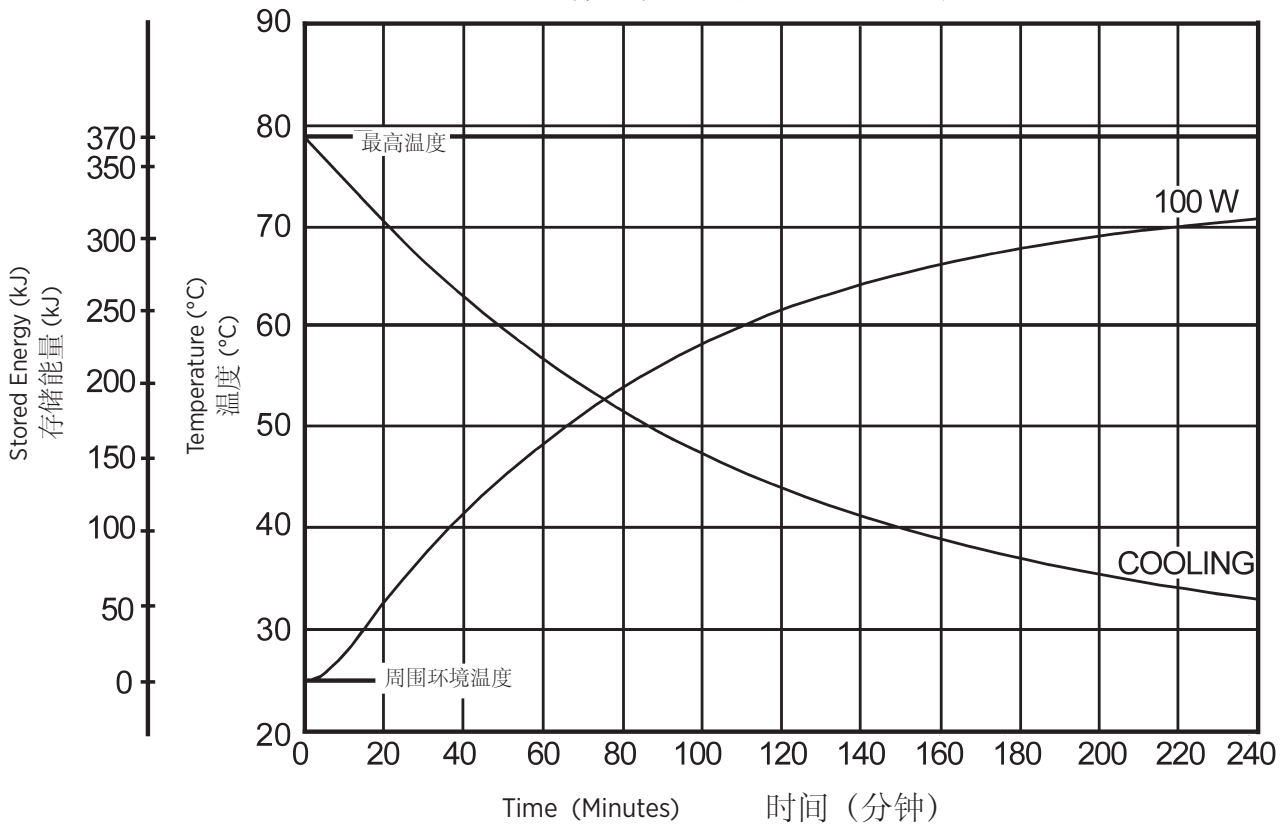
X-Ray Tube Assembly Heating and Cooling Curves
X 射线管汇编热化/冷却的曲线

Note:
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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