Note: Document originally drafted in the English language.
 注释：文件最初用英语起草。

Product Description

The B-185H housing is designed for Varex Imaging rotating Anode inserts having 133 mm (5.253 inch) diameter targets.

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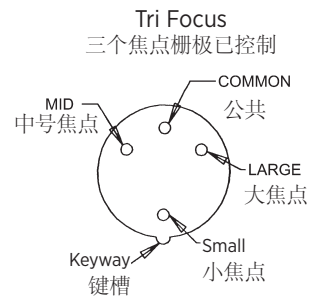
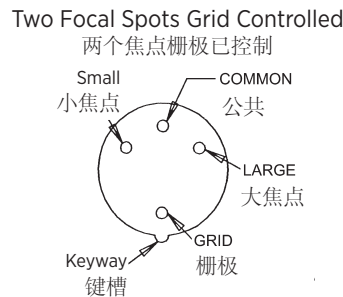
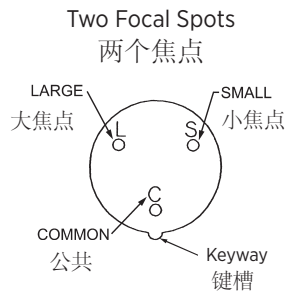
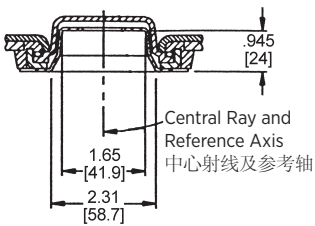
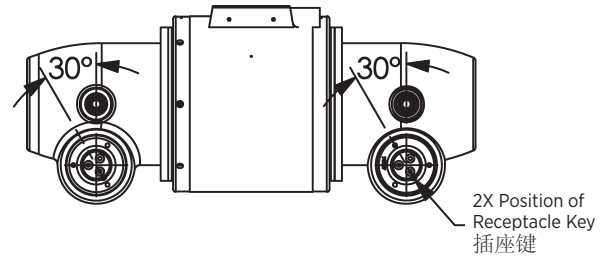
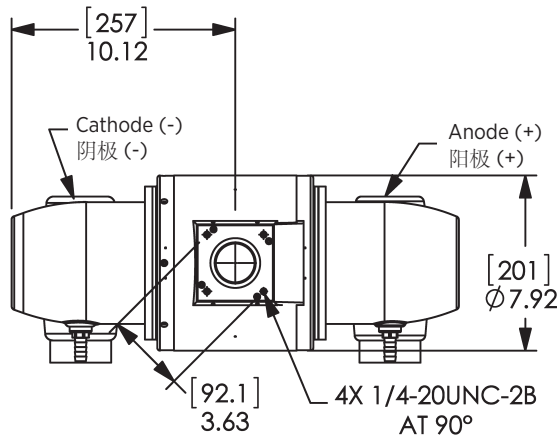
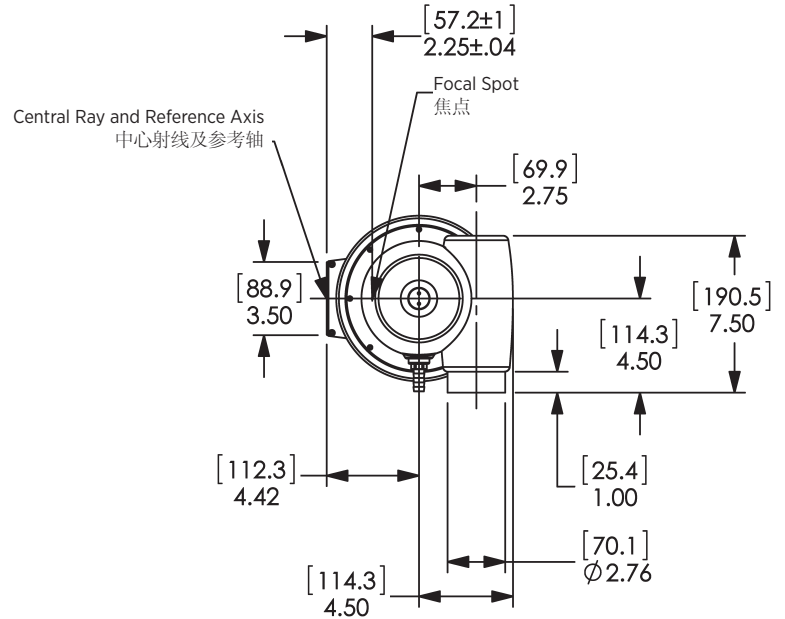
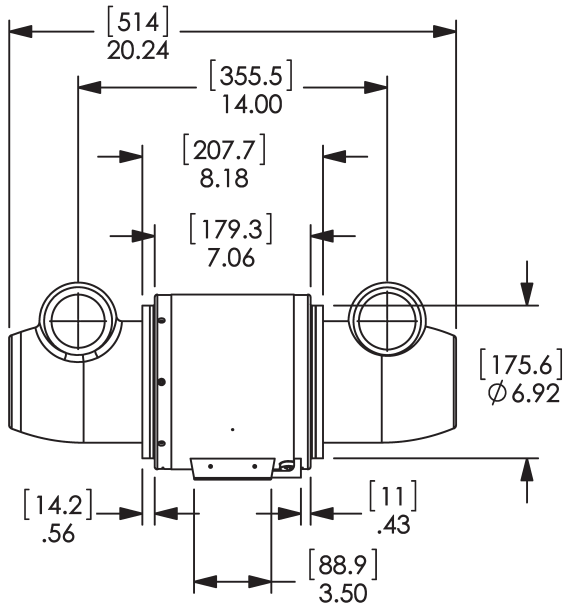
Product Description	
Nominal X-ray Tube Voltage	150kV
Maximum Cathode to Ground	75 kV
Maximum Anode to Ground	75 kV
Grid to Cathode (if applicable)	-4 kV
Grid Control Voltages	
Typical Bias Voltage for Cutoff at 125 kV	-3600 Vdc
Grid Voltage for Exposure	0 Vdc
Maximum X-ray Tube Assembly Heat Content	
.....	1,482 kJ (2,000 kHU)
Nominal Continuous Input Power (max. housing temperature 78°C)	
without HE-100	250 W IEC 60613:2010
with HE-100	1,130 W IEC 60613:2010
X-Ray Tube Assembly	
Permanent Filtration	1.0mm Al IEC 60522/1999
Loading Factors for Leakage Radiation	
	150 kV, 7.2 mA
High Voltage Cable Receptacles	
	Per IEC 60526
Temperature Limits for Storage and Transport ...	
	-20°C to +75°C
Humidity	
	+10% to +90%
Atmospheric Pressure Range	
	70 kPa to 106 kPa
Federal Standard High Voltage Cable	
	72
Thermal Switch	
	normally closed
Open	76.7°C ±2.8°C (170°F ±5°F)
	3A @ 120Vac or 30 Vdc max.
High Voltage Cable Receptacles	
	Per IEC 60526

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Dimensions are for Reference only
维度是供仅参考

Note: Dimensions in Inches [mm]
注: 维度在英寸 [mm]


Note: B-185H

- Reference dimensions are for information purposes and are not required for inspection
- Aperture size optional: 0.937 Sq. In. 1.250 Sq. In.
- Cable receptacles 2 1/2 -20 UNF -2B thread

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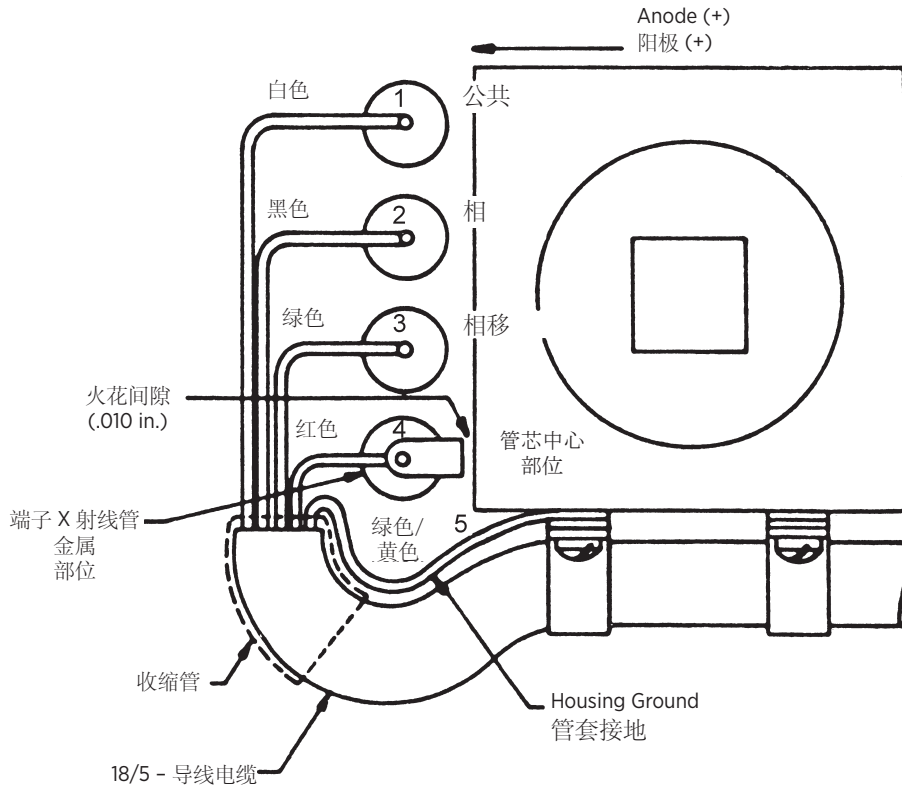Stator Ratings and Characteristics 定子额定值及特征

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Stator Power:

Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 133 mm (5.25 inch) anode series tubes is between 5 and 7 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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
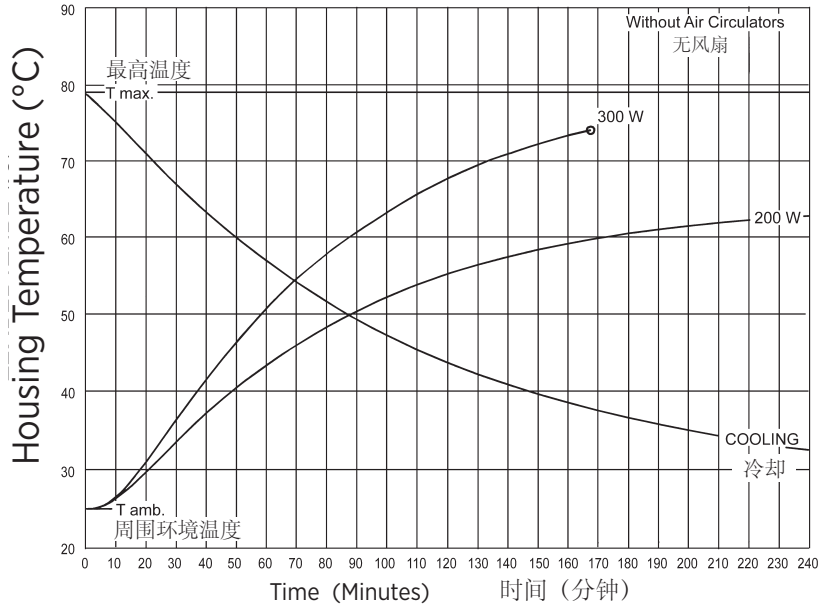
X-ray Tube Assembly Heating and Cooling Curve
X 射线管汇编热化/冷却的曲线

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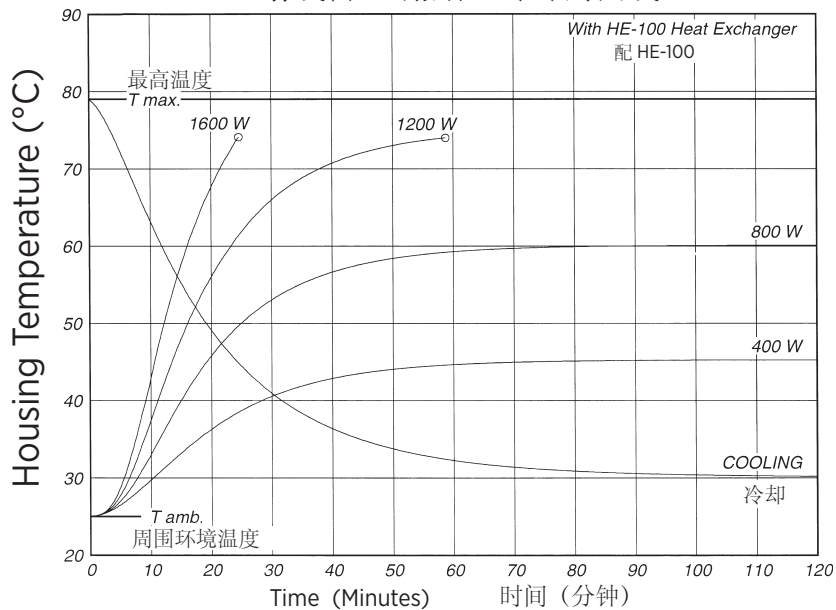Caution:

Housings configured with heat exchangers having long hoses (> 5 ft or 1.5 m) are not configured for operation separate from the associated heat exchanger. In such cases, the tube unit must not be operated unless properly coupled to the heat exchanger. Reference the specific heat exchanger's product data sheet for the associated cooling curves.

Note:

Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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