Note: Document originally drafted in the English language.
注释：文件最初用英语起草。

Product Description

The B-121 housing is designed for Varex Imaging mammography inserts.

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Product Description	
Maximum Potential Difference	
Cathode to Ground	0 kV
Anode to Ground	39 - 49 kV
Grid Control Voltages: (If Applicable)	
Typical Bias Voltage for Focals	-5 to -100 Vdc
Housing Heat Storage Capacity	370 kJ (500 kHU)
Nominal Continuous Input Power .	300 Watts (405 HU/sec) IEC 60613:2010
Maximum Housing Temperature	78°C
X-Ray Tube Assembly	
Permanent Filtration	0.63 mm Be IEC 60522/1999
Leakage Technique Factors	49 kV, 6.0 mA
Cable Receptacles	Non Federal Standard 55 kV - Rating
Thermal Switch	normally closed Rating - 10 A @ 240Vac
Open	79.4°C ±3.9°C (175°F ±7°F)
Ambient Air Temperature Limits for Operation	
.....	5°C to 40°C
Temperature Limits for Storage and Transportation	
.....	-10°C to 75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
X-Ray Tube Assembly (Complies to)	IEC 60601-2-28

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2. **PORT BOSS DIMENSION**
端口轴套尺寸

For Determining Focal Spot to Port Distance
确定焦点与端口之间的距离

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

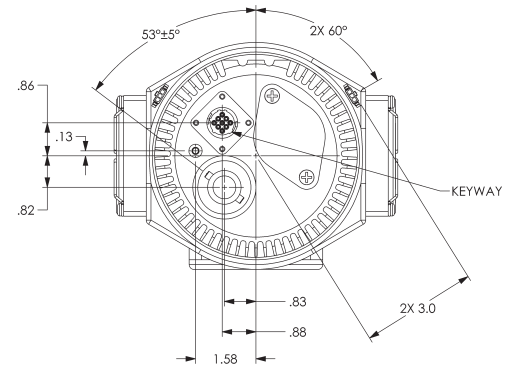
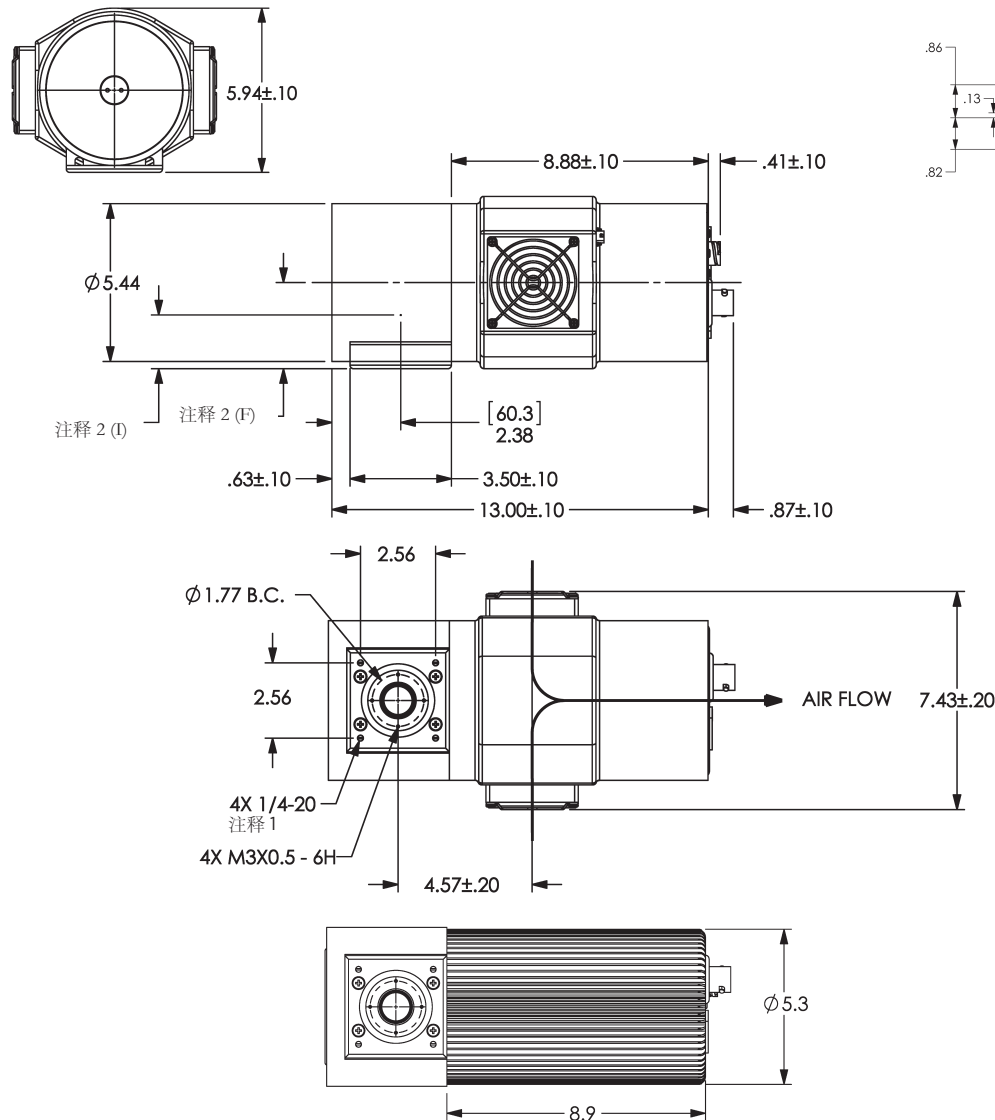
Note: Dimensions in [mm] Inches
注释: 尺寸 (英寸)


BIANGULAR INSERTS

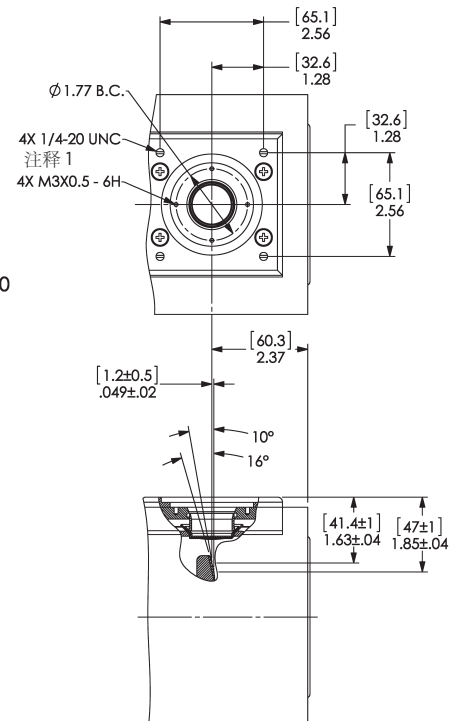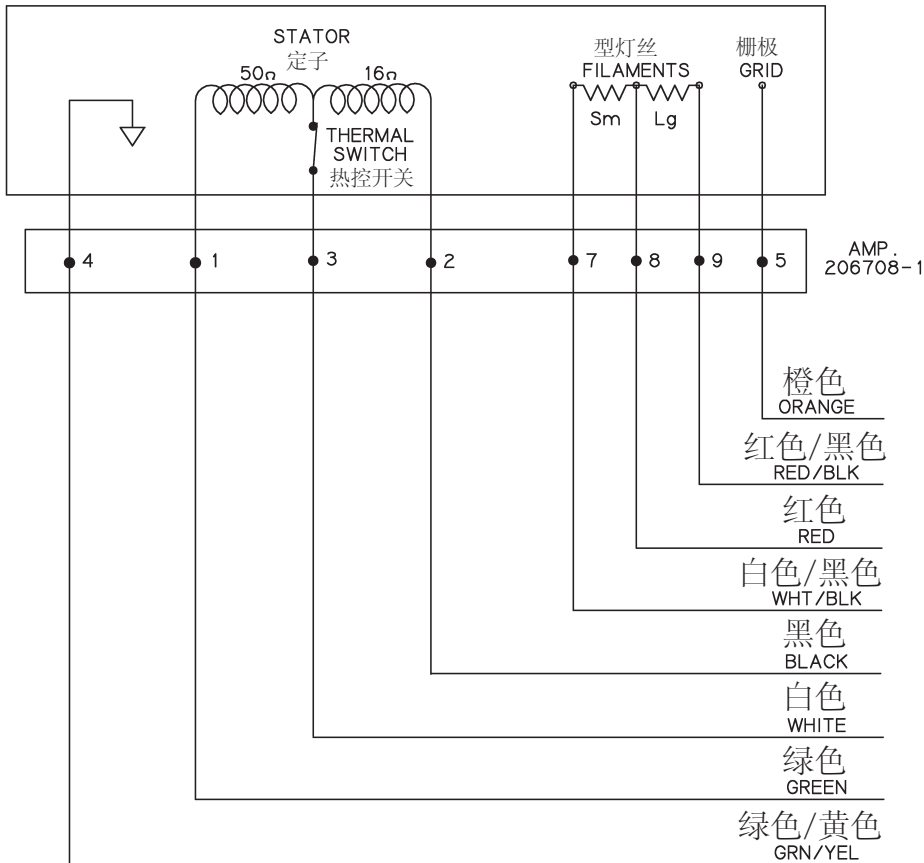Stator Ratings and Characteristics
定子额定值及特征

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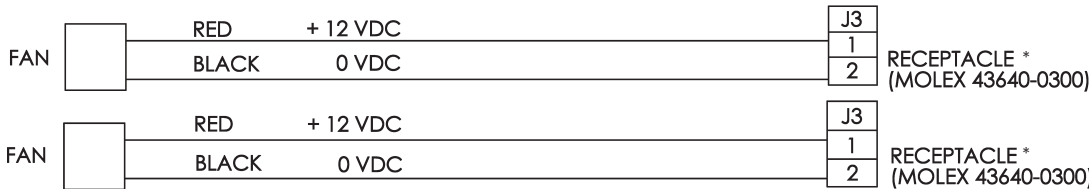Stator Power:
Time to full speed of the anode is a function of the power rating of the "starter" and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible.

Starter:	Start	Run
50/60 Hz	115 VAC	45 VAC
150/180 Hz	230 VAC	90 VAC

NOTE:
For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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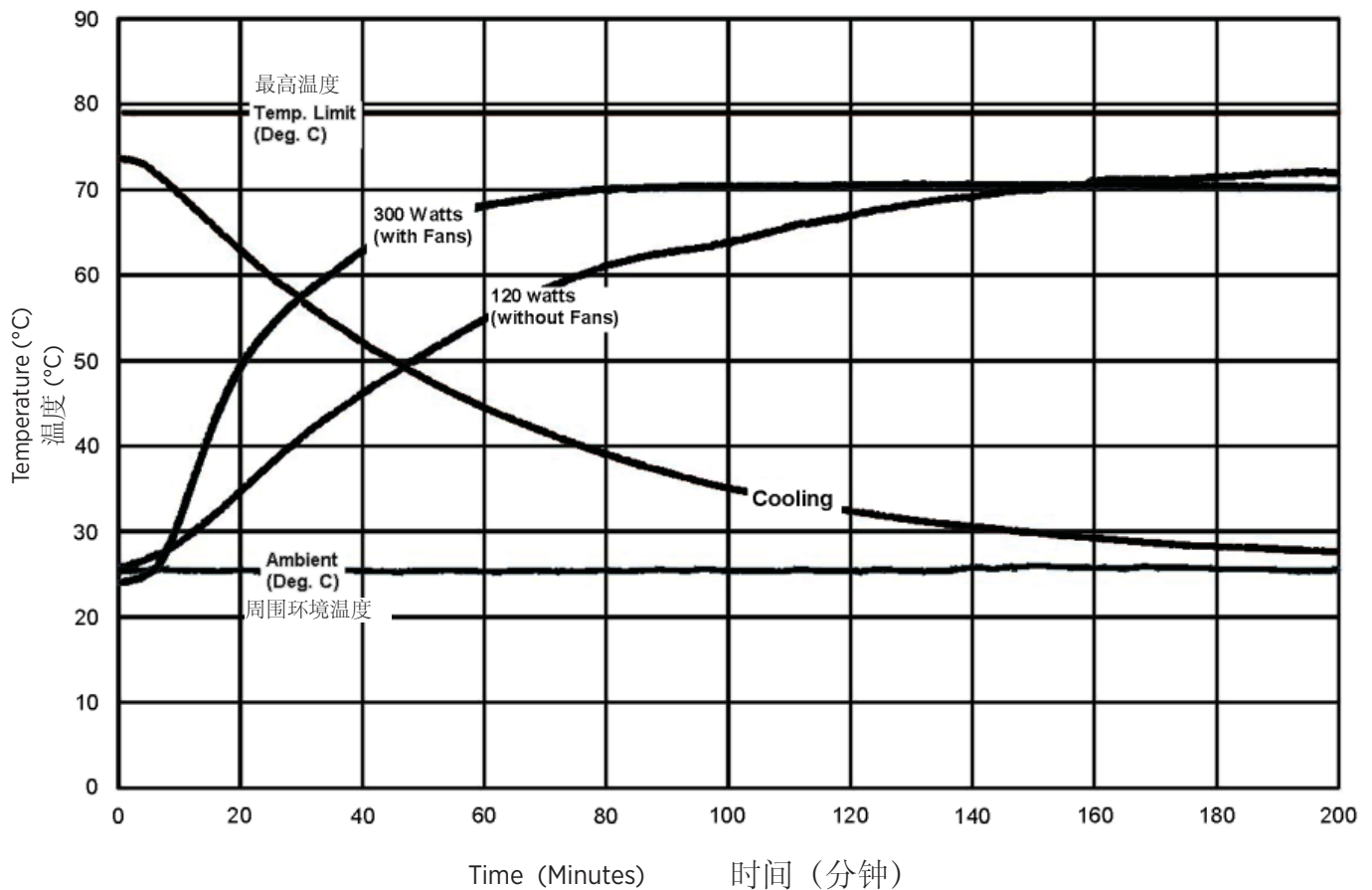
X-Ray Tube Assembly Heating and Cooling Curves
X 射线管汇编热化/冷却的曲线

Note:
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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