Note: Document originally drafted in the English language.
注释：文件最初用英语起草。

Product Description

The B-283H housing is designed for Varex Imaging rotating Anode inserts having 133 mm (5.253 inch) diameter targets.

IEC Classification Class 1

Weight, Approximate:
Housing and Tube 40 kg (88.2 lbs)

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Product Description

Nominal X-ray Tube Voltage 125 kV
 Maximum Cathode to Ground 63 kV
 Maximum Anode to Ground 63 kV
 Grid to Cathode (If applicable) -3 kV

Grid Control Voltages

Typical Bias Voltage for Cutoff at 150 kV -3000 Vdc
 Grid Voltage for Exposure 0 Vdc

Maximum X-Ray Tube Assembly Heat Content
 1,800 kJ (2,520 kHU)

Maximum Continuous Heat Dissipation
 (max. housing temperature 78°C)
 with HE100 1,200 W (1,680 HU/sec)

X-Ray Tube Assembly

Permanent Filtration 1.0 mm Al IEC 60522

Loading Factors for Leakage Radiation
 125 kV, 22 mA

Ambient Air Temperature Limits for Operation
 5°C to 40°C

Temperature Limits for Storage and Transport
 -20°C to +75°C

Humidity +10% to +90%

Atmospheric Pressure Range 70 kPa to 106 kPa

Safety Devices

Thermal Switch Normally Closed Contact
 500mA @ 50VDC or 100mA @ 120VAC

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



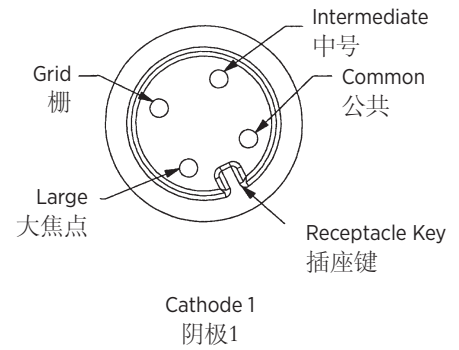
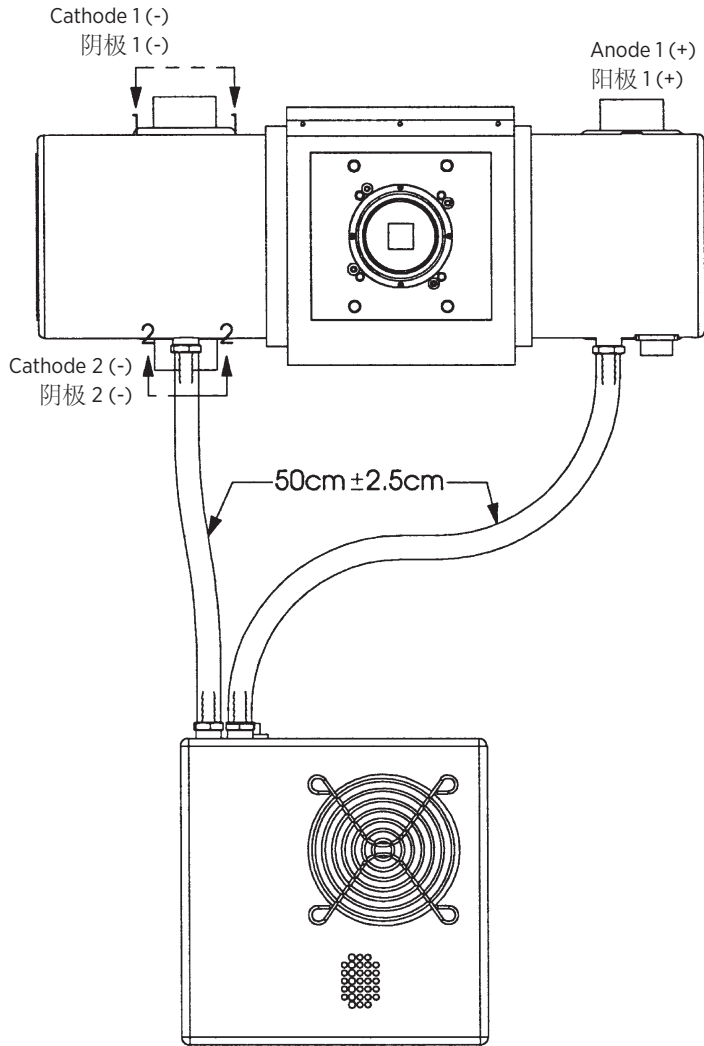
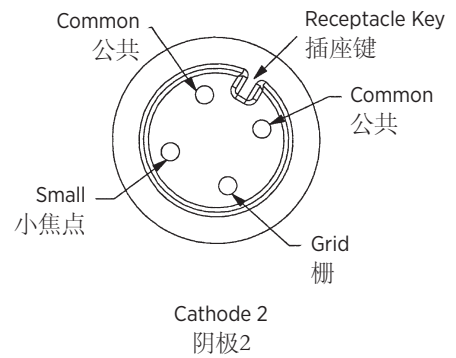
Bottom of Receptacle

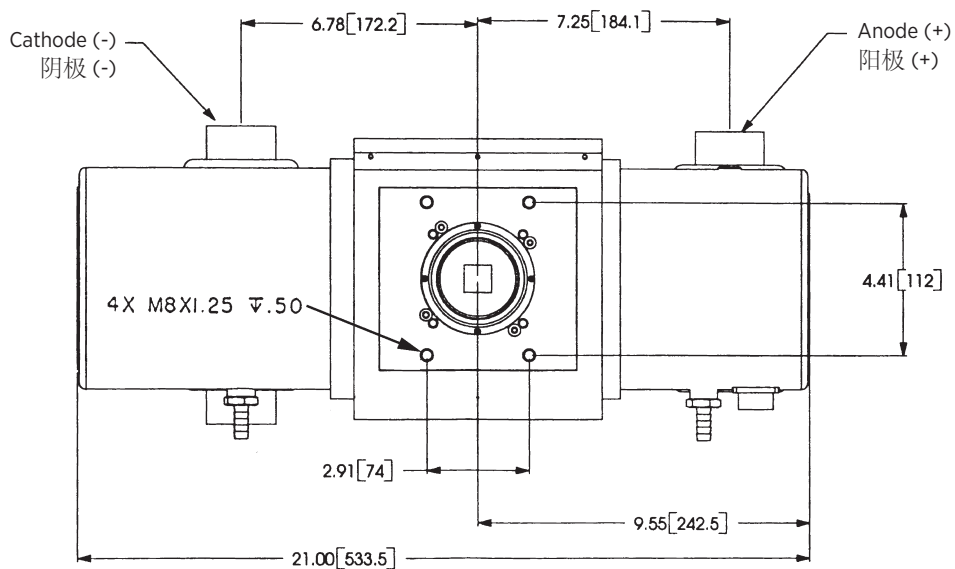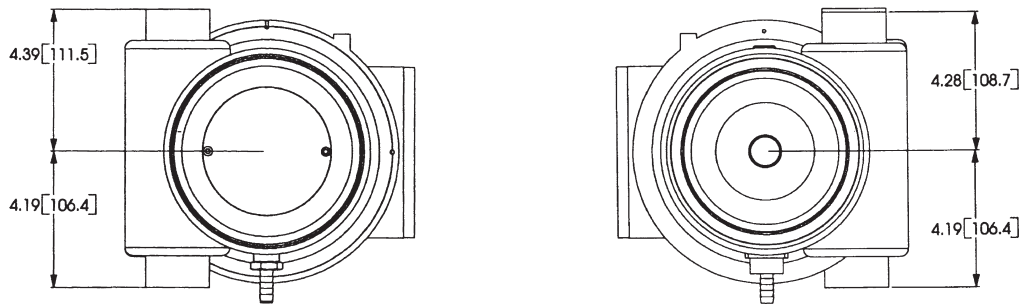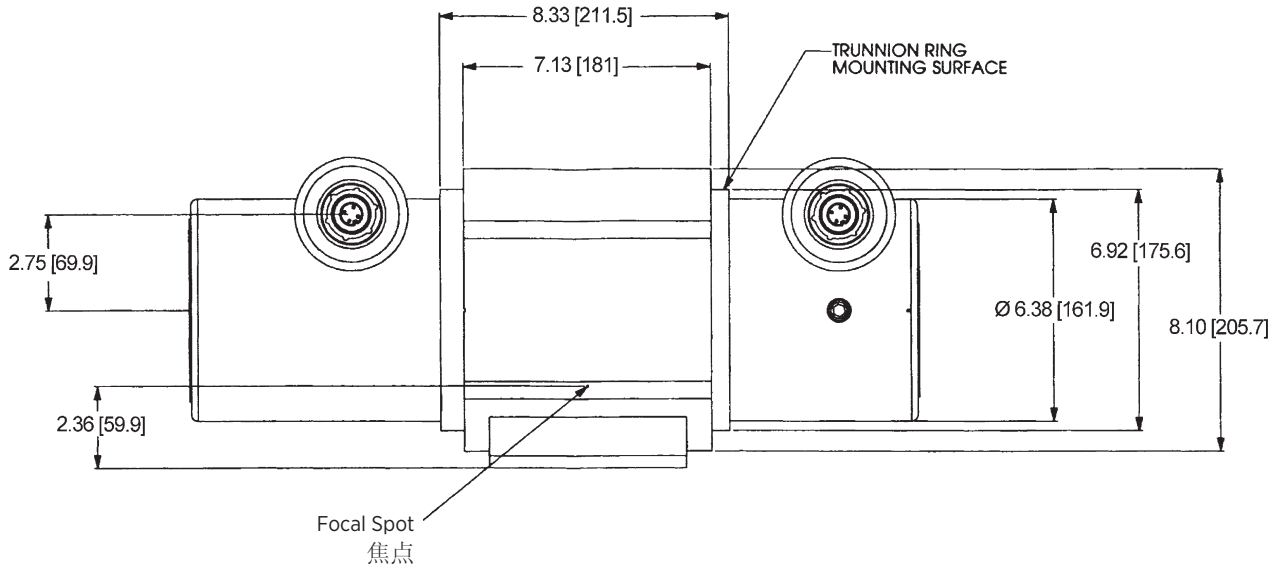Dimensions are for Reference only
维度是供仅参考


Stator Ratings and Characteristics
定子额定值及特征

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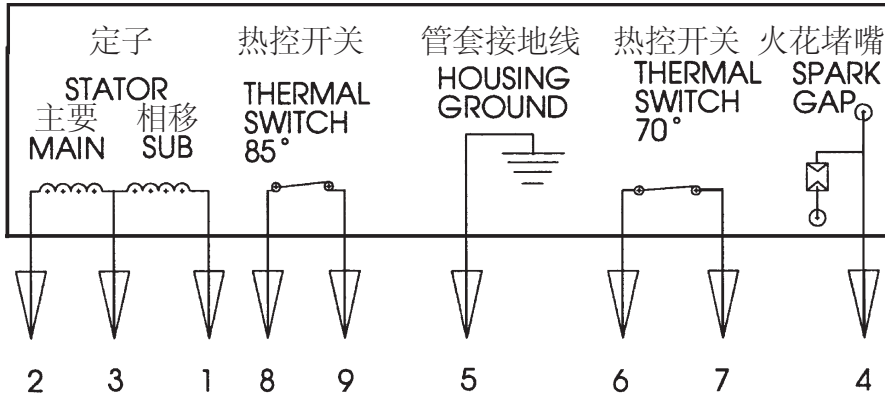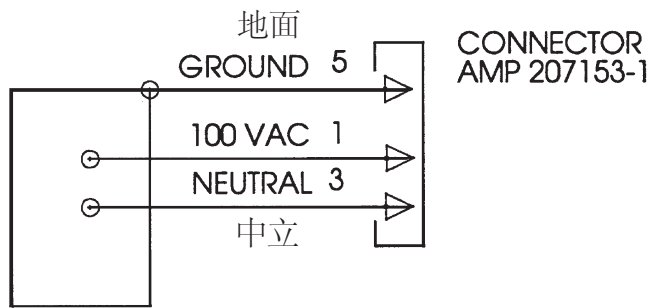
Stator Power:
Time to full speed of the anode is a function of the power rating of the “starter” and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters.
Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.
No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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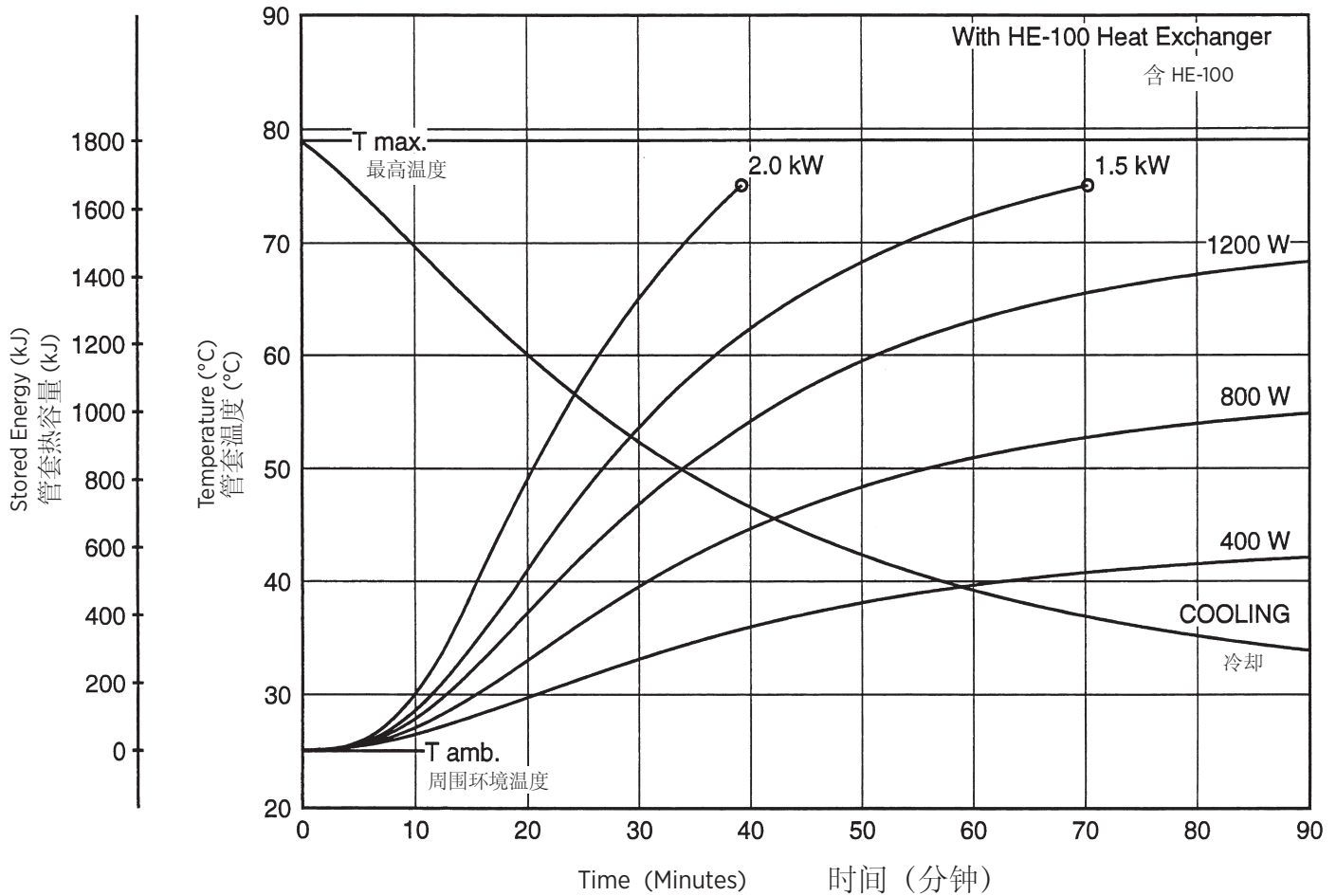Note:
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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