Note: Document originally drafted in the English language.
 注释：文件最初用英语起草。

Product Description

The B-145A housing is designed for Varex Imaging rotating anode inserts having 71mm (3 inch) diameter targets.

IEC Classification Class 1

Weight, Approximate:
 Housing & Tube 14.51 kg (32 lbs)

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



Product Description	
Maximum Potential Difference	125 kV
Cathode to Ground	63 kV
Anode to Ground	63 kV
Maximum X-Ray Tube Assembly Heat Content	500 kJ (675 kHU)
Nominal Continuous Input Power (Maximum Housing Temperature 78°C)	140 W (189 HU/sec) IEC 60613:2010
X-ray tube assembly cooling is provided by ambient air flow.	
X-Ray Tube Assembly Permanent Filtration 0.7mm Al/75 kV IEC 60522/1999	
Loading Factors for Leakage Radiation 125 kV, 4.0 mA	
Thermal Switch	Normally Closed 7A @ 120 Vac or 30 Vdc Max.
Open	85°C ±3.9°C (185°F ±7°F)
X-Ray Tube Assembly (Complies to) IEC 60601-2-28	

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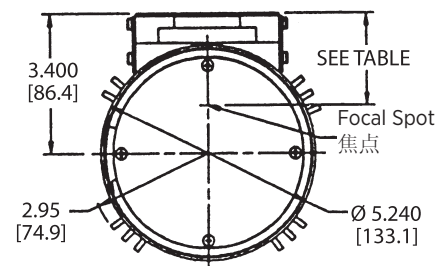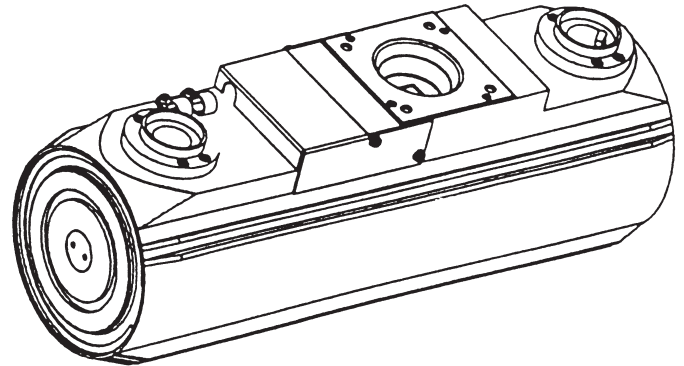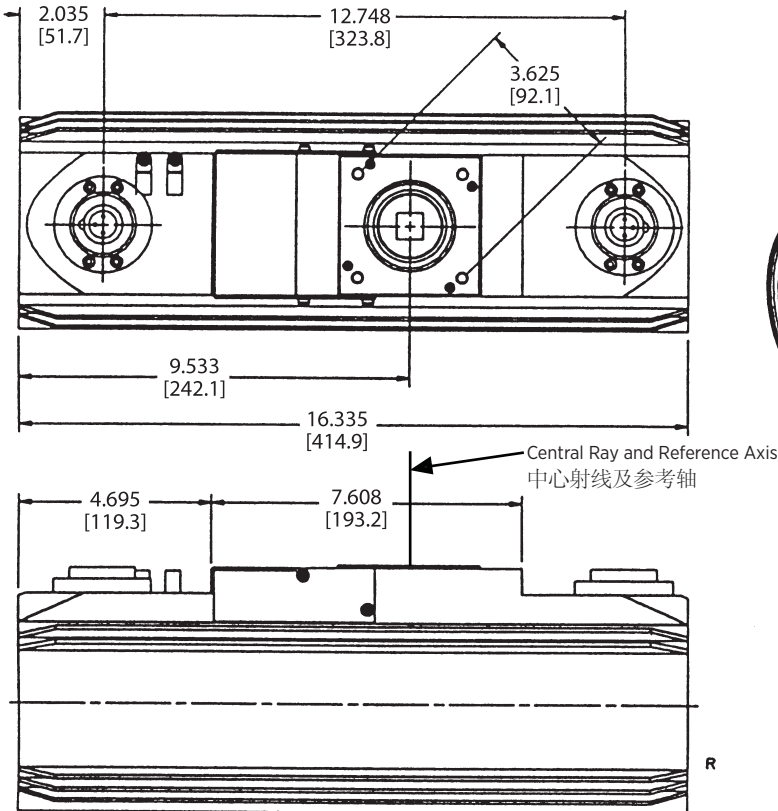Dimensions are for Reference only
维度是供仅参考

Insert 管芯	Focal Spot to Port Distance 焦点与端口的距离
3" Series	2.25" ±0.40 (57.1mm ±1.0mm)
Rad Series	2.12" ±0.40 (53.8 mm ±1.0mm)

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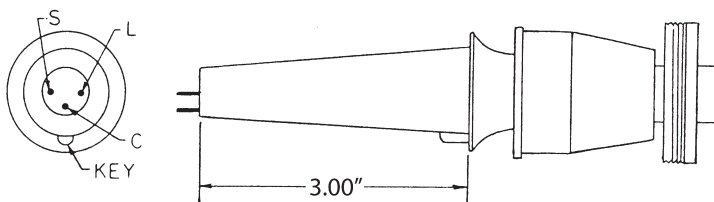
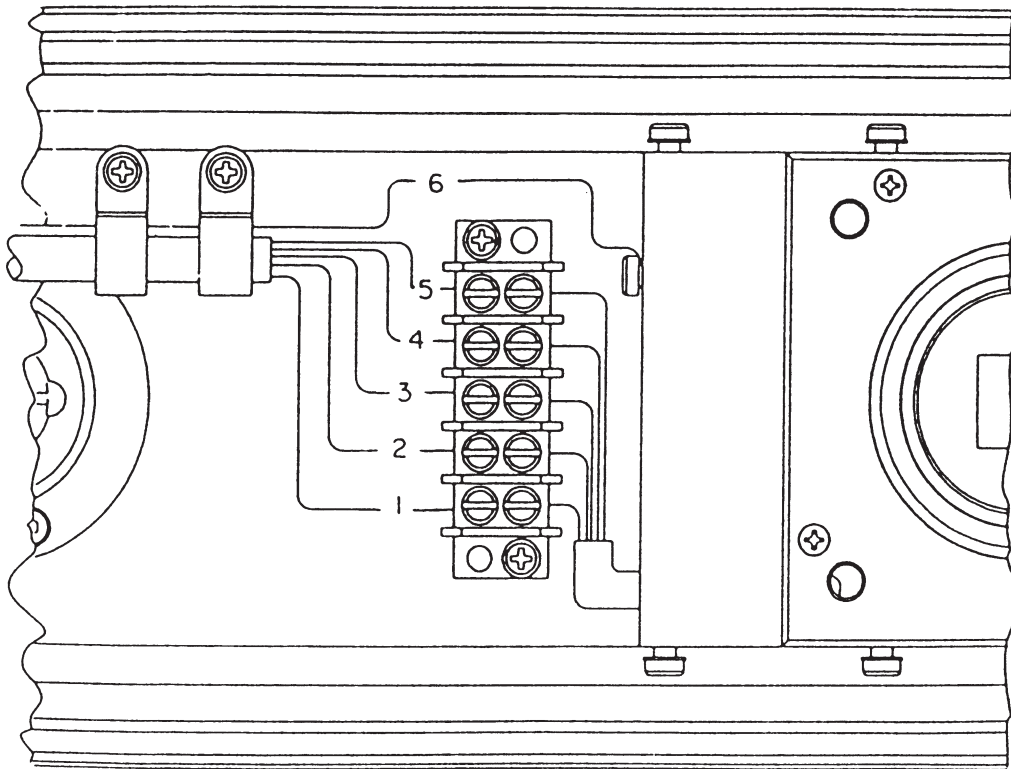
Stator Power: 50/60 Hz Only
Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. Time to full speed of 3 inch (71mm) anode series tubes is between 0.5 and 1.0 seconds.

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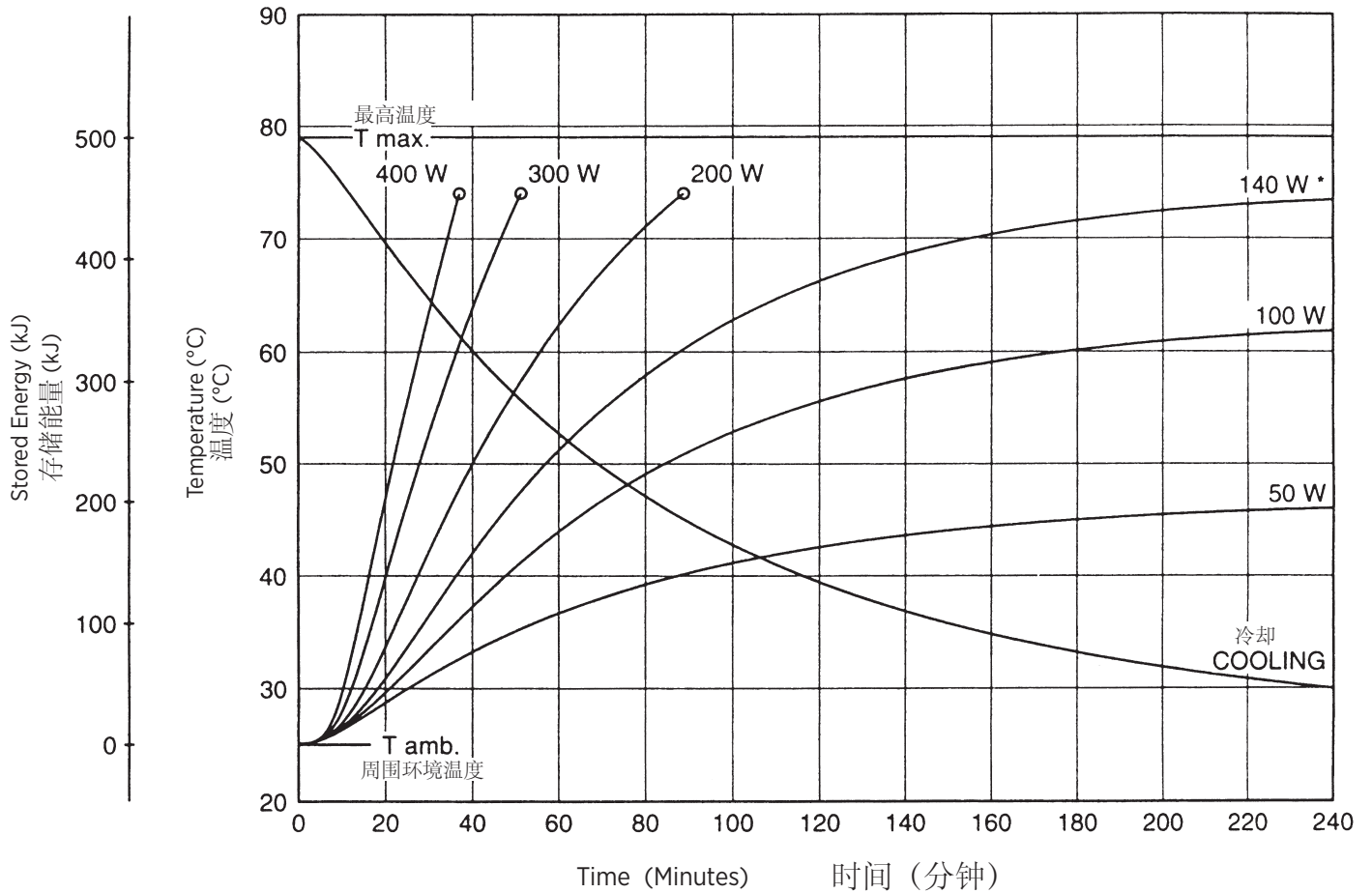Note:

Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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Parker Medical, Inc.