Note: Document originally drafted in the English language.
注释：文件最初用英语起草。

Product Description

The Diamond® housing is designed for Varex Imaging rotating anode inserts having 71 mm (2.8 inches) or 80 mm (3 inches) diameter targets.

IEC Classification Class 1

Weight, Approximate:
Housing & Tube 17 kg (37.4 lbs)

Mounting Port plate or Trunnion
Standard Housing - 1/4-20 Screws
Metric Housing - M6-1 screws

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Product Description

Maximum Peak Voltage 150 kV
 Cathode to Ground 82.5 kV
 Anode to Ground 82.5 kV

Maximum X-Ray Tube Assembly Heat Content
 930 kJ (1.3 MHU)

Nominal Continuous Input Power
 (maximum housing temperature 78°C)
 200 W (278 HU/sec)
 IEC 60613:2010 @ Sea Level

X-Ray tube assembly cooling is provided by ambient air flow.

X-Ray Tube Assembly (Insert and Housing)
 Permanent Filtration ... 0.7mm Al/75kV IEC 60522/1999

Loading Factors for Leakage Radiation 150 kV, 3.3 mA

Temperature Limits for Storage and Transport
 -20°C to +75°C
 Humidity +10% to +90%
 Atmospheric Pressure Range 70 kPa to 106 kPa

Thermal Switch Normally Closed
 Contact Rating - 7A @ 120Vac
 Open 85°C ±3.9°C (185°F ±7°F)

High Voltage Receptacle Cables ... Complies to IEC 60526
 NEMA Standard XR7-1979 (R1984, 1990)

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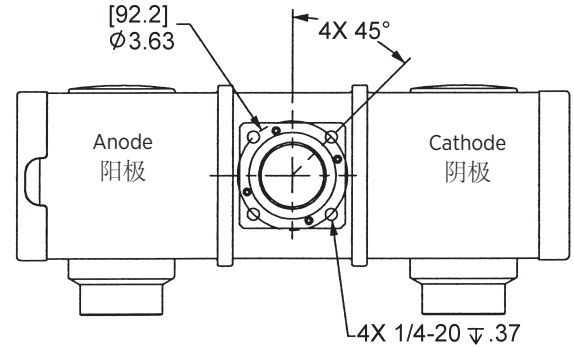
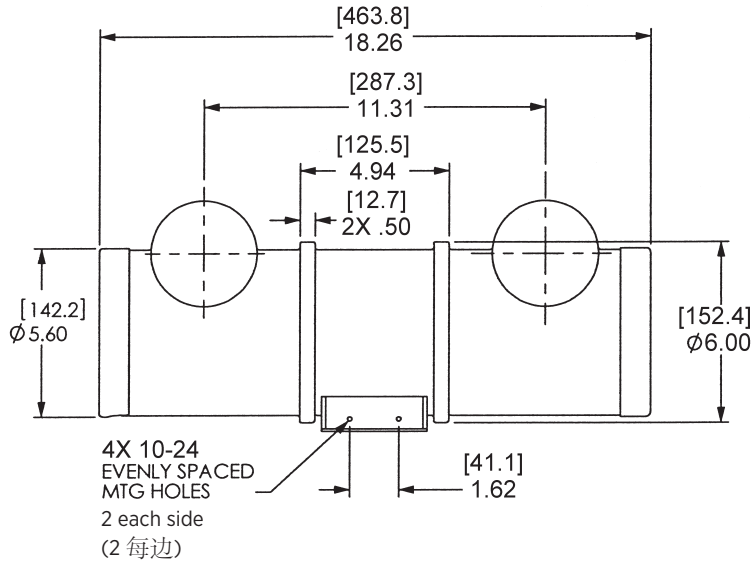
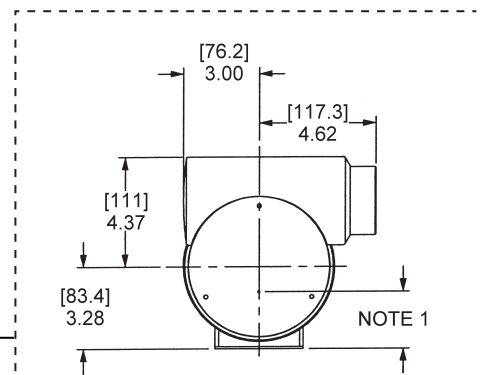
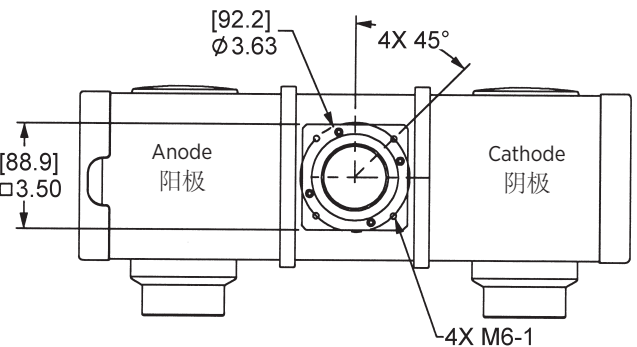
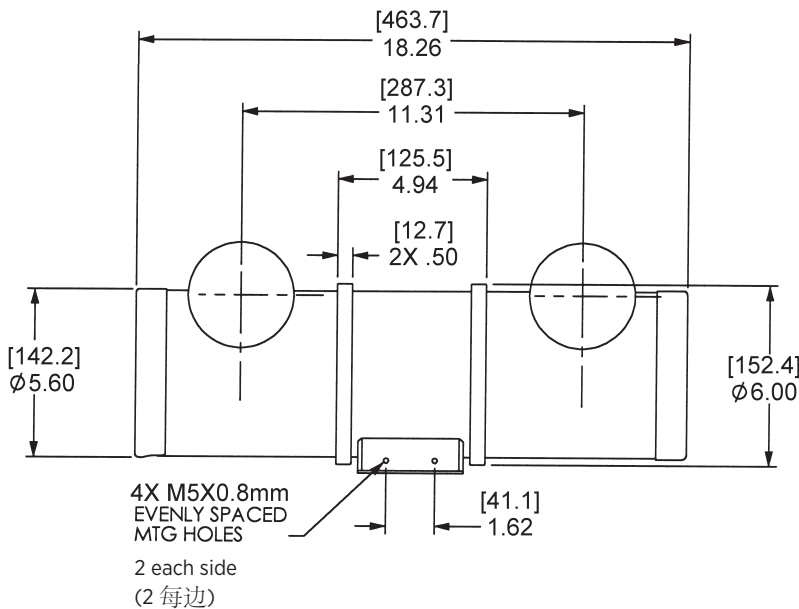
Metric Port Housing 米制端口管套


Note 1

Focal Spot to Port Distance
80 mm Anode [53.0 ±1.0] 2.10 ±0.040
71 mm Anode (RAD-8 & 12) ... [56.0 ±1.0] 2.20 ±0.040

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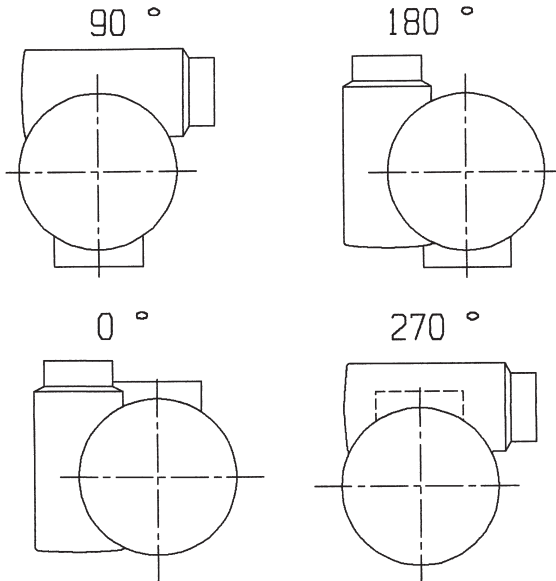
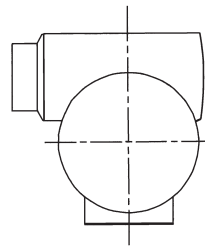
DIAMOND-M (ONLY)
REVERSED 90°

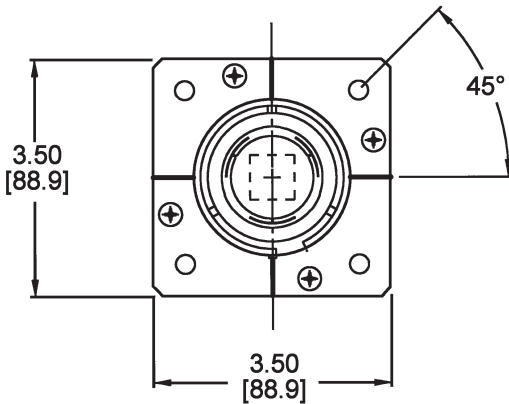
Dimensions are for Reference only
维度是供仅参考


Note: Window aperture is .875" (22mm) square on 16° anode and .656" (16.7mm) square on 10° and 12° anodes.

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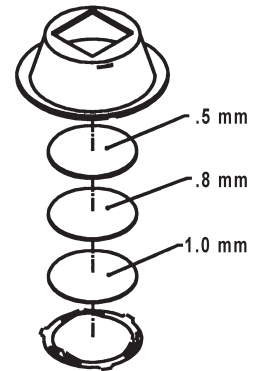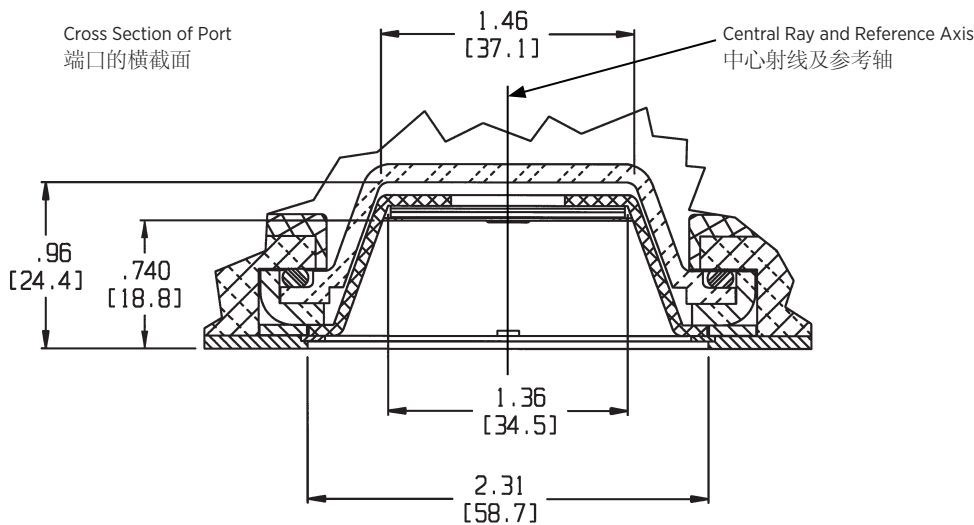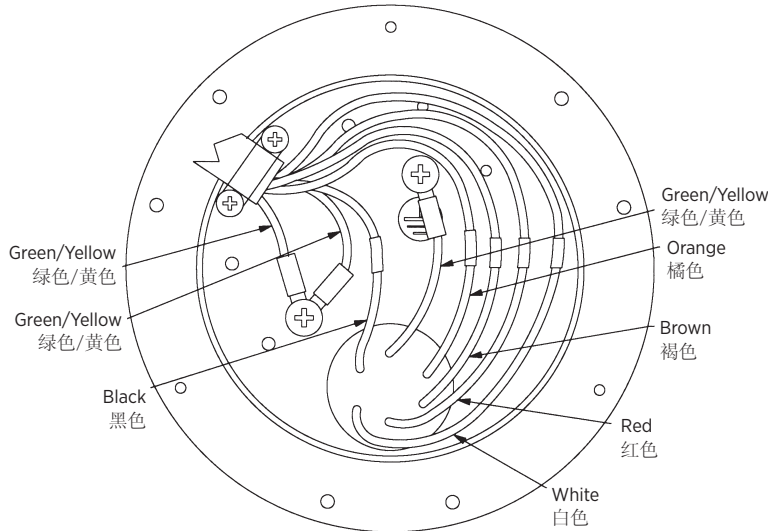
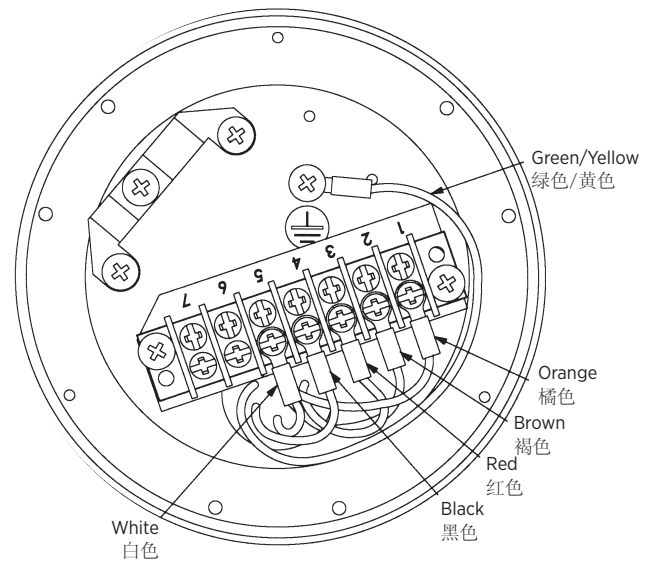
Stator Cord:

A six wire shield-grounded stator cord should be connected from the stator terminal to the motor control. All listed voltage and amperage measurements should be taken at the housing end.

If the stator cord is not provided, the cord used must be UL listed or a UL recognized component for all products used in the USA. Outside the USA, the cord used must comply with all applicable regulatory and statutory requirements for electrical and safety. The cord must have the following characteristics: six conductor, 18AWG (0.9mm²) Cu, rated 600V, 90°C minimum, shielded cable with PVC jacket 0.032 inch (0.081mm) thick minimum. Lead terminations are provided and must be used with wire that matches the above description.

Stator Power:

Time to full speed of the anode is a function of the power rating of the “starter” and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed for 71mm (2.8 inch) and 80mm (3.0 inch) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 500 volts rms.

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
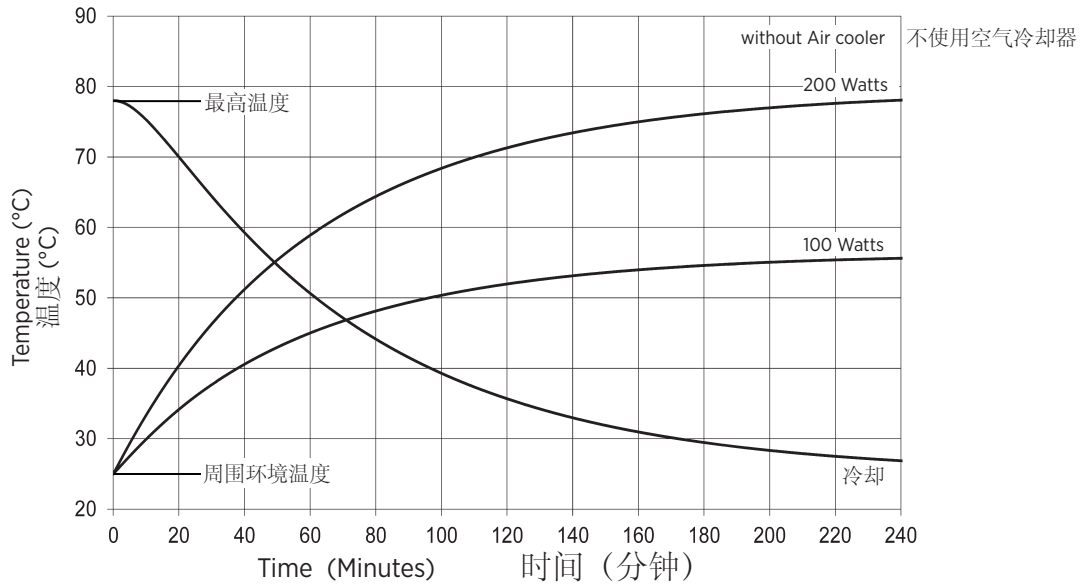
X-Ray Tube Assembly Heating and Cooling Curve
X 射线管汇编热化/冷却的曲线

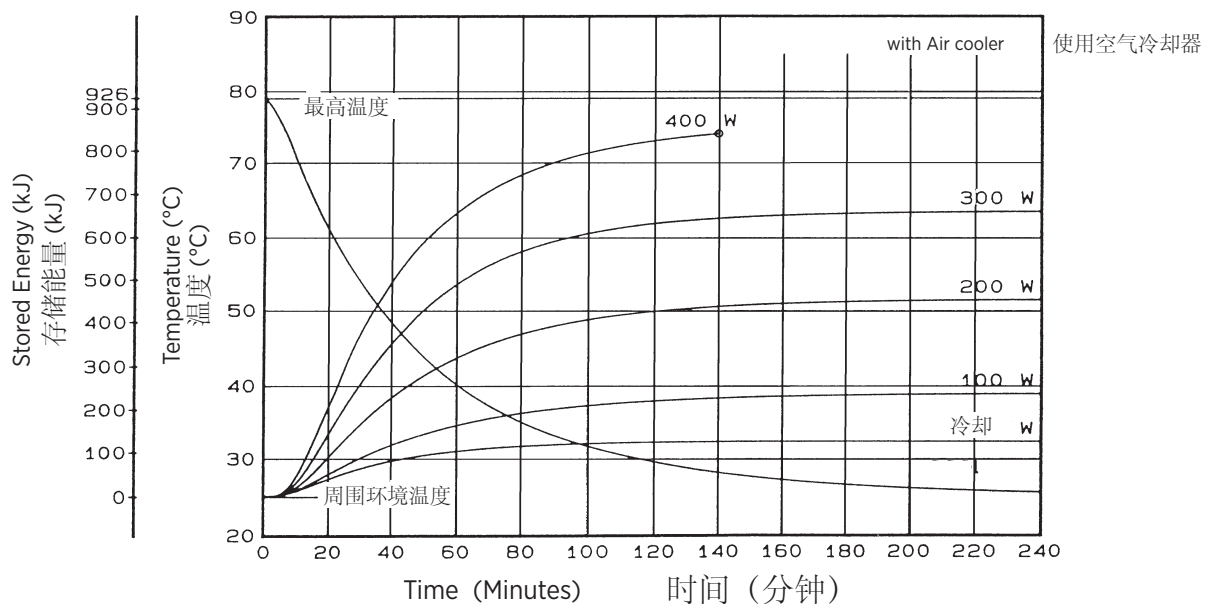X-Ray Tube Assembly Heating and Cooling Curve
X 射线管汇编热化/冷却的曲线

Note:
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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