Surgical C-Arm  
Rotating Anode X-Ray Tube Housing  
外科 C 型臂, 旋转阳极 X 射线管套

Note: Document originally drafted in the English language.  
注释: 文件最初用英语起草。

**Product Description**

The ORIII is a uniquely designed x-ray tube housing developed for use in surgical C-Arm applications. This design utilizes a high performance rotating anode tube, with inherently greater target loading capabilities than that of stationary anode tube designs. This housing provides ease of use at a nominal patient table height.

**Housing:** Shockproof, Aluminum. Lead lined; filled under vacuum with high dielectric insulating oil and hermetically sealed. The housing contains a stator for driving the rotor.

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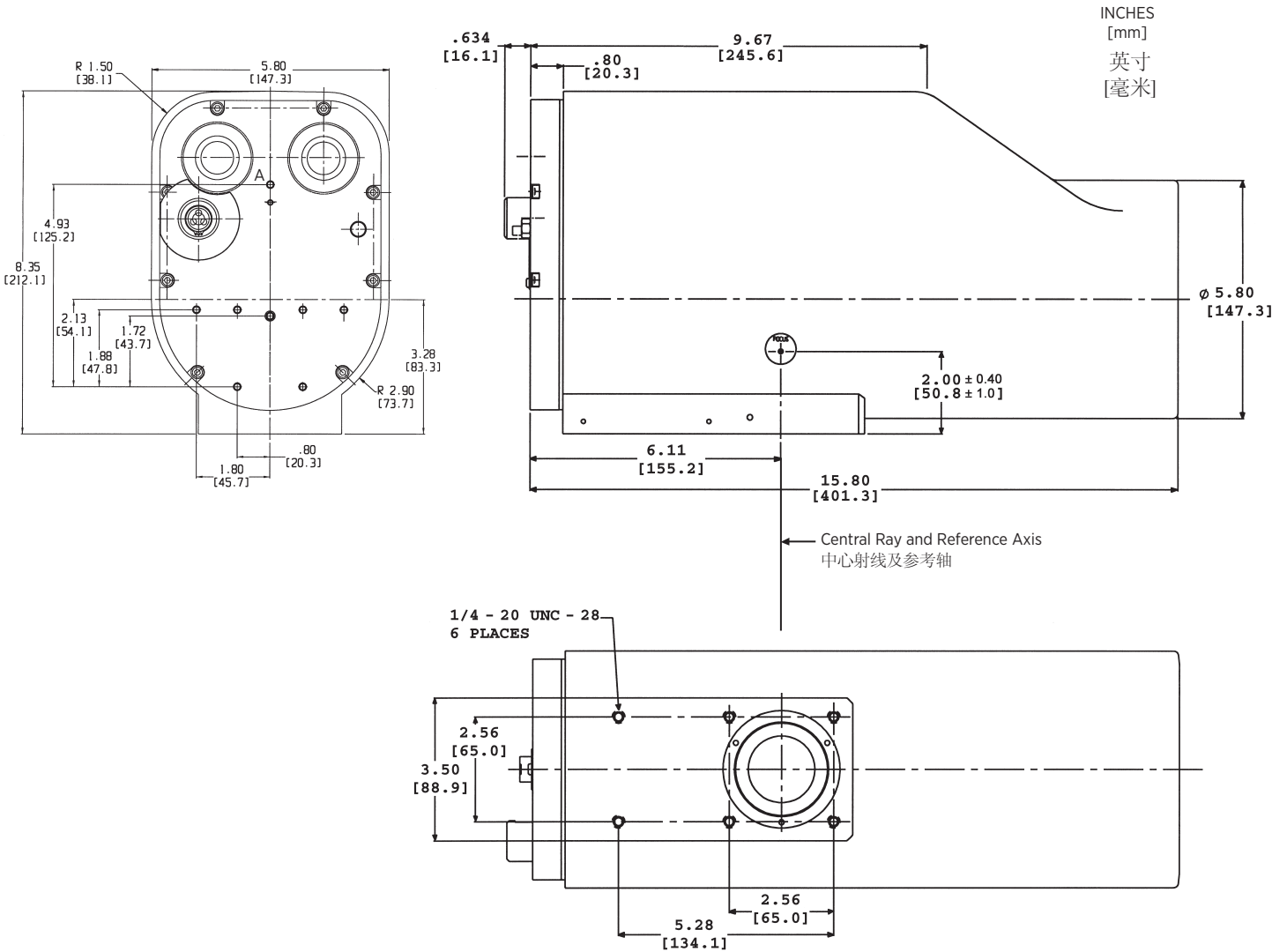
| <b>Product Description</b>  |   |
|---|---|
| <b>Maximum Peak Voltage</b>   |   |
| Anode to Cathode .....  | 125 kV  |
| Anode to Ground .....   | 62.5 kV   |
| Cathode to Ground .....   | 62.5 kV   |
| <b>Nominal Continuous Input Power</b>   |   |
| .....   | 185 W (15 kHU**/Min.)<br>IEC 60613:2010         |
| Maximum Housing Temperature .....   | 78°C  |
| X-ray tube assembly cooling provided by ambient air flow.   |   |
| <b>Thermal Characteristics</b>  |   |
| Housing heat storage capacity .....   | 1,185 kJ (1.6 MHU**)                            |
| Weight .....  | 19.5 Kg (43 lbs.) without cables or accessories |
| <b>X-Ray Tube Assembly</b>  |   |
| Permanent Filtration .....  |   |
| .....   | 0.7mm Al minimum at 75 kV - IEC 60522/1999      |
| Loading Factors for Leakage Radiation .....   |   |
| .....   | 125 kV, 1.5 mA                                  |
| <b>X-Ray Coverage:</b>  |   |
| 8.5° target - 12 in (30 cm) at 40 in (100 cm) Distance Source-Image   |   |
| 10° target - 14 in (35 cm) at 40 in (100 cm) Distance Source-Image  |   |
| <b>Thermal Switch</b>   |   |
| The switch must be used in conjunction with the control manufacturer's interlock circuitry. The thermal switch is normally closed. The switch will open when the temperature reaches 85°C ±2.8°C. |   |
| <b>CAUTION</b>  |   |
| The thermal switch does not protect the insert against overheating.   |   |
| IEC Classification .....  | Class 1   |
| **H.U. (Heat Units) = (kV) x (mA) x 1.35 x Time in seconds (3Ø operation)   |   |

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Dimensions are for Reference only  
维度是供仅供参考


**Cable Receptacles:** Alden type, P702-1 (cathode) and P702 (anode), high voltage receptacles. The receptacles mate with Alden type (P7010S) high voltage connector and cable.

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**Rayproofing:** Stray radiation complies with DHHS standards under the "Radiation Control for Health and Safety Act 1968" (U.S.A.)

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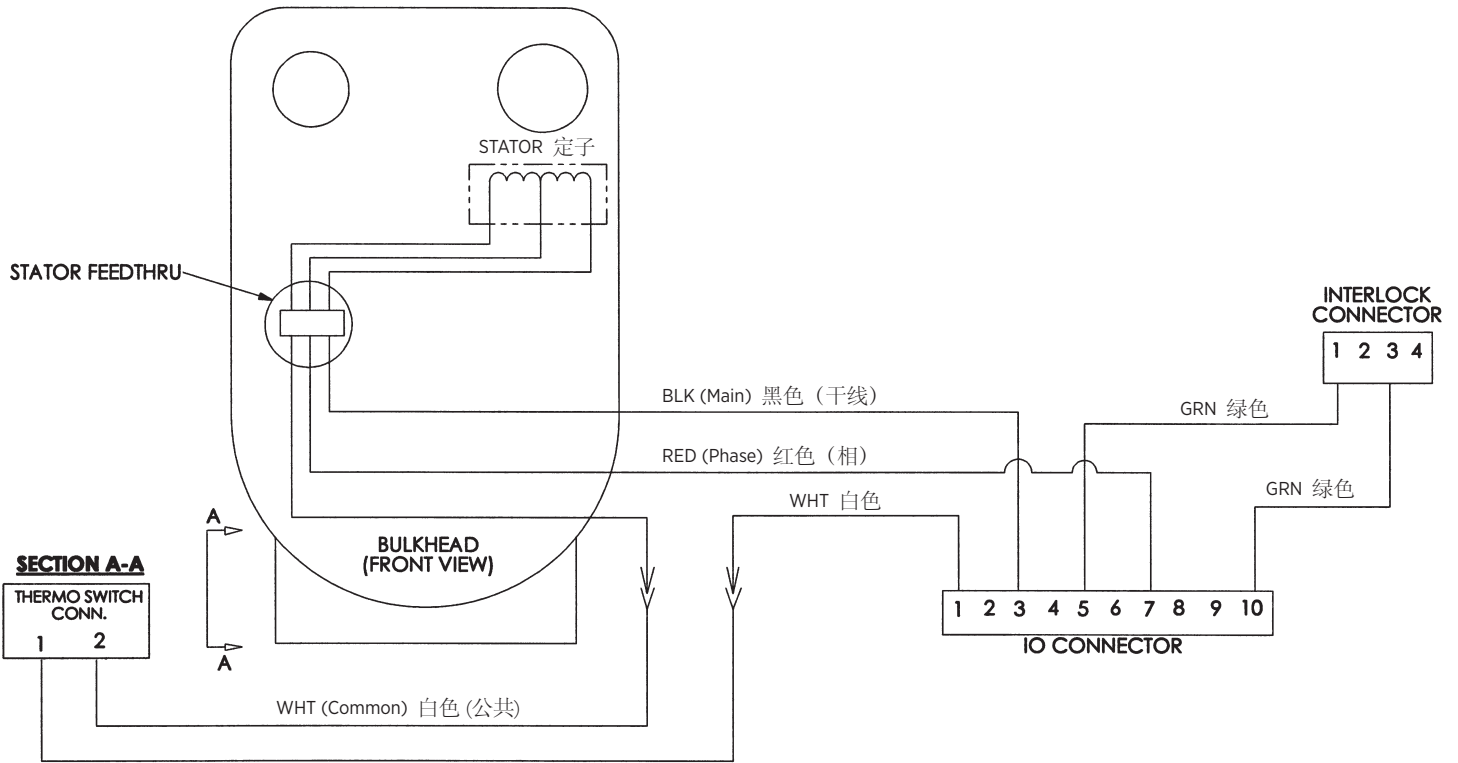**Stator:** The ORIII housing is equipped with a standard Varex Imaging stator requiring a 30 microfarad capacitor, providing 90° phase shift for 50/60 Hz., 2,850/3,450 rpm operation.

**Stator Motor Ratings:**

| Frequency (Hz.) | Volts | Common Leg Current |
|-----------------|-------|--------------------|
| 50-60           | 60*   | Approx. = 2.0 Amps |
| 50-60           | 120   | Approx. = 4.0 Amps |
| 50-60           | 240   | Approx. = 8.0 Amps |

\*Running Voltage

**Stator Cord:** A six wire shield-grounded stator cord should be connected from the stator terminal to the motor control. All listed voltage and amperage measurements should be taken at the housing end.

If the stator cord is not provided, the cord used must be UL listed or a UL recognized component for all products used in the USA. Outside the USA, the cord used must comply with all applicable regulatory and statutory requirements for electrical and safety. The cord must have the following characteristics: six conductor, 18AWG (0.9mm<sup>2</sup>) Cu, rated 600V, 90°C minimum, shielded cable with PVC jacket 0.032 inch (0.081mm) thick minimum. Lead terminations are provided and must be used with wire that matches the above description.

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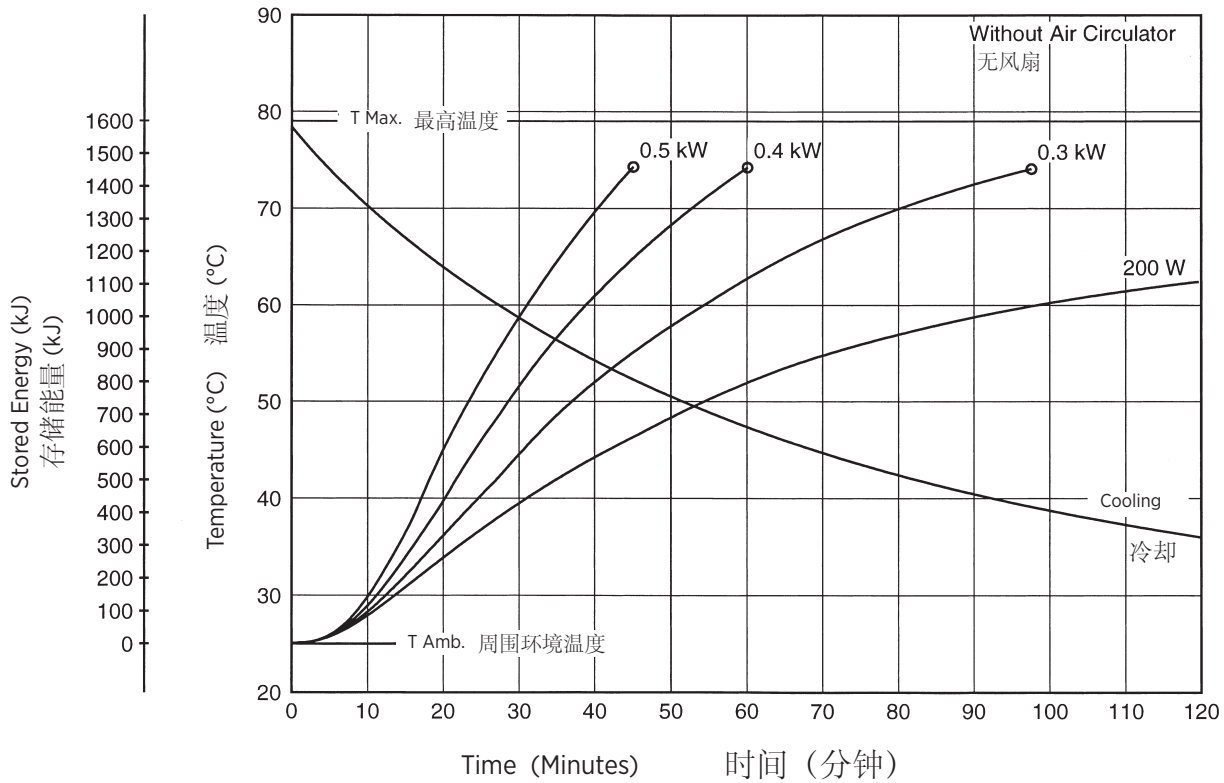
Tube Housing Assembly Heating and Cooling  
X 射线管汇编热化/冷却的曲线

**Operating Temperature**  
Minimum ambient temperature for storage and transportation is -20°C (0°F). Operating range of the ORIII housing is 16° - 72°C (60° - 167°F).

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