Note: Document originally drafted in the English language.

Product Description

The B-280H/B-282H housing is designed for Varex Imaging rotating anode inserts having 133mm (5.25 inch) diameter targets.

IEC Classification Class 1

Weight (approximate)
Housing & Tube 40 kg (88.2 lbs)

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Soporte Entrada
(use tornillos metricas M8 x 1.25)

Product Description

Nominal X-ray Tube Voltage	125 kV
Maximum Cathode to Ground	63 kV
Maximum Anode to Ground	63 kV
Grid to Cathode (If applicable)	-3 kV
Grid Control Voltages	
Typical Bias Voltage for Cutoff at 150 kV	-3000 Vdc
Grid Voltage for Exposure	0 Vdc
Maximum X-Ray Tube Assembly Heat Content	
	1,800 kJ (2,520 kWh)
Maximum Continuous Heat Dissipation	2,700 W (3,780 HU/sec)
X-Ray Tube Assembly	
Permanent Filtration	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation	125 kV, 22 mA
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Safety Devices	
Thermal Switch	Normally Closed Contact 500mA @ 50VDC or 100mA @ 120VAC
Thermal Switch #1	
Open	85°C ±3.9°C (185°F ±7°F)
Thermal Switch #2	
Open	70°C ±3.9°C (158°F ±7°F)

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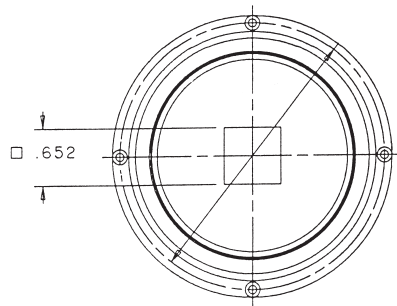
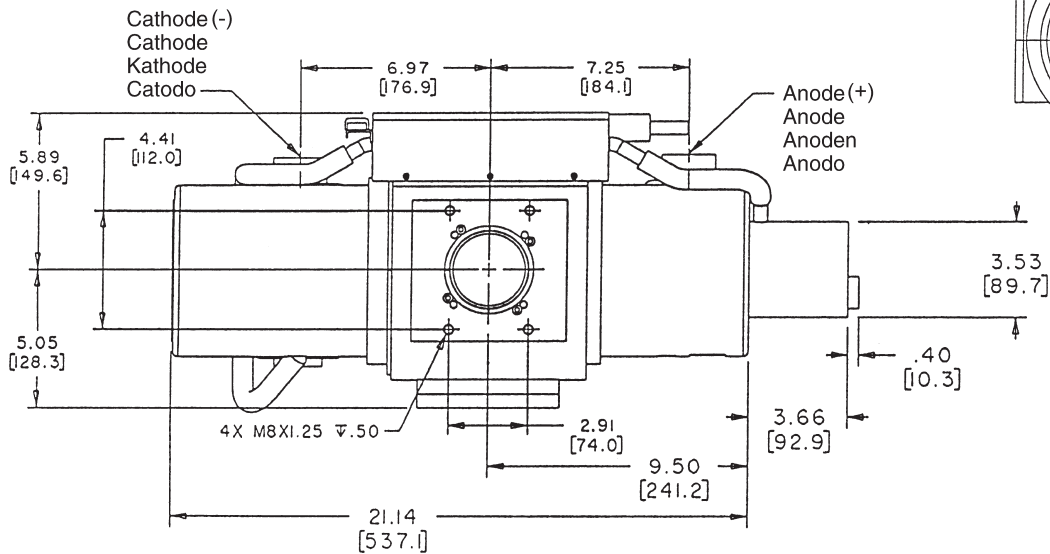
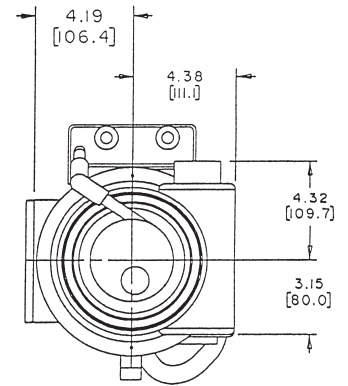
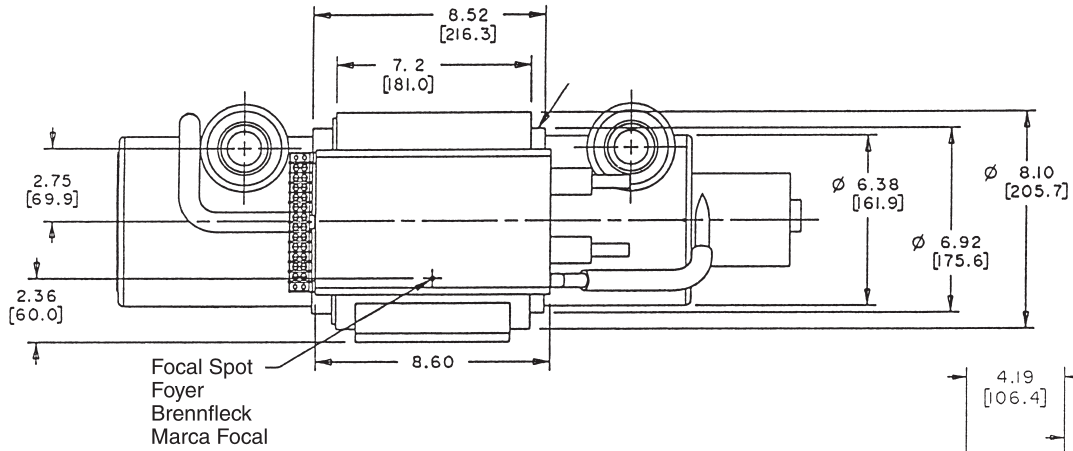
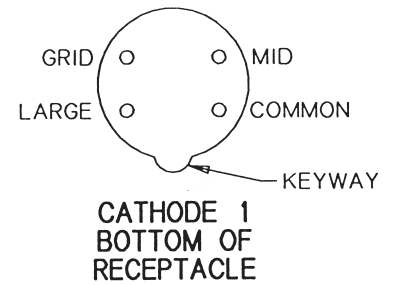
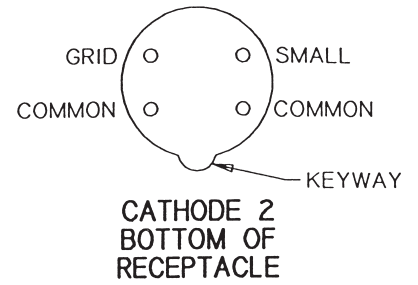
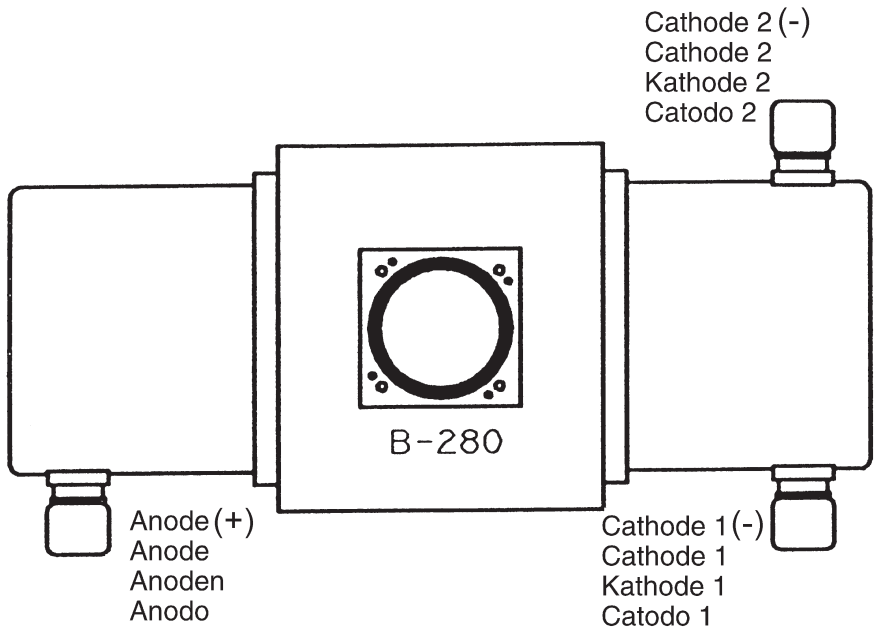
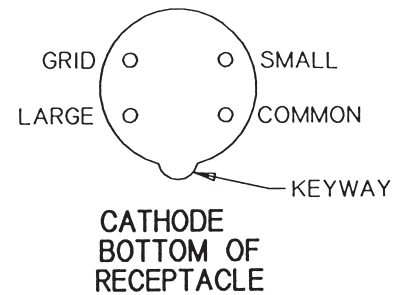
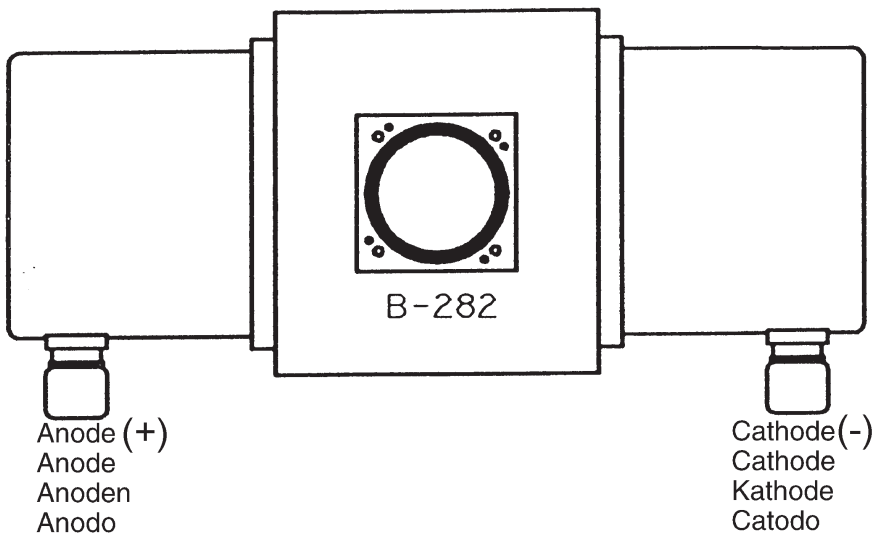
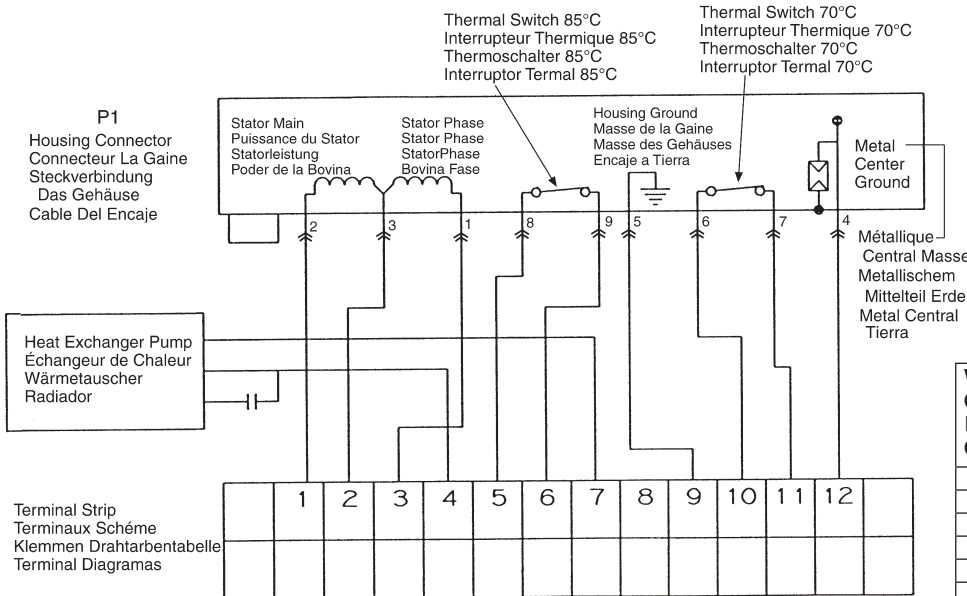
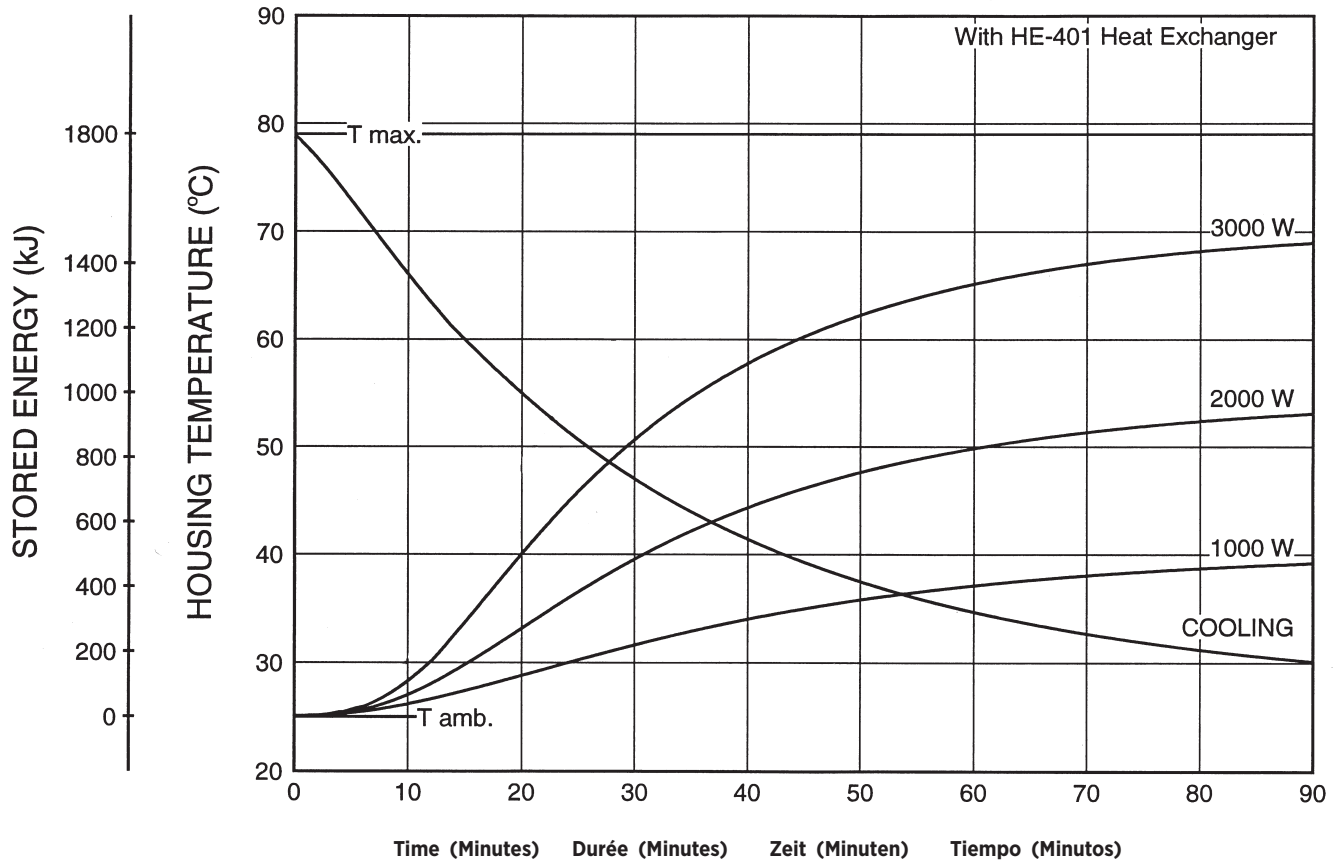
Stator Power:
 Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters.
 Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.
 No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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TUBE HOUSING ASSEMBLY HEATING AND COOLING

Note:
 1. Heat inputs into housing include tube power, filament power, and stator power.
 2. Heating curves based on no restrictions of natural convection around tube housing assembly.

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