Note: Document originally drafted in the English language.

Product Description

The MX100™ housing is used for rotating anode inserts having 100mm (4 inch) diameter targets.

IEC Classification Class 1

Weight, Approximate:
Housing & Tube 26.3 kg (58 lbs)

Mounting Housing Bosses

™All trademarks property of the respective manufacturer.

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Classification CEI Classe 1

Poids, Approximatif:
Gaine et Tube 26.3 kg (58 lbs)

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Product Description

Maximum Potential Difference	150 kV
Cathode to Ground	82 kV
Anode to Ground	82 kV
Grid to Cathode (If Applicable)	-4 kV
Grid Control Voltages:	
Typical Bias Voltage for Cutoff at 150 kV	-3600 Vdc
Grid Voltage for Exposure	0 Vdc
Maximum X-Ray Tube Assembly Heat Content	1,110 kJ (1,554 kHU)
Maximum Continuous Heat Dissipation with Air Circulator	
.....	740 W (1,036 HU/sec)
Maximum Housing Temperature	78°C
X-Ray Tube Assembly	
Permanent Filtration	0.7mm Al IEC 60522
Loading Factors for Leakage Radiation	150 kV, 4.0 mA
Temperature Limits for Storage and Transport	-9°C to +70°C
Humidity	10% to 90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Thermal Switch	
.....	Normally Open
.....	6A @ 125 Vac or 7A @ 30 Vac/dc
Close	130°F (±5.0°F)
Open	110°F (±5.0°F)
Pressure Switch	
.....	Normally Closed
.....	1A @ 125 Vac or 30 Vdc
Open	5 PSIG (±1.0 PSIG)
Close	4 PSIG (±1.0 PSIG)
X-Ray Tube Assembly (Complies to)	IEC 60601-2-28

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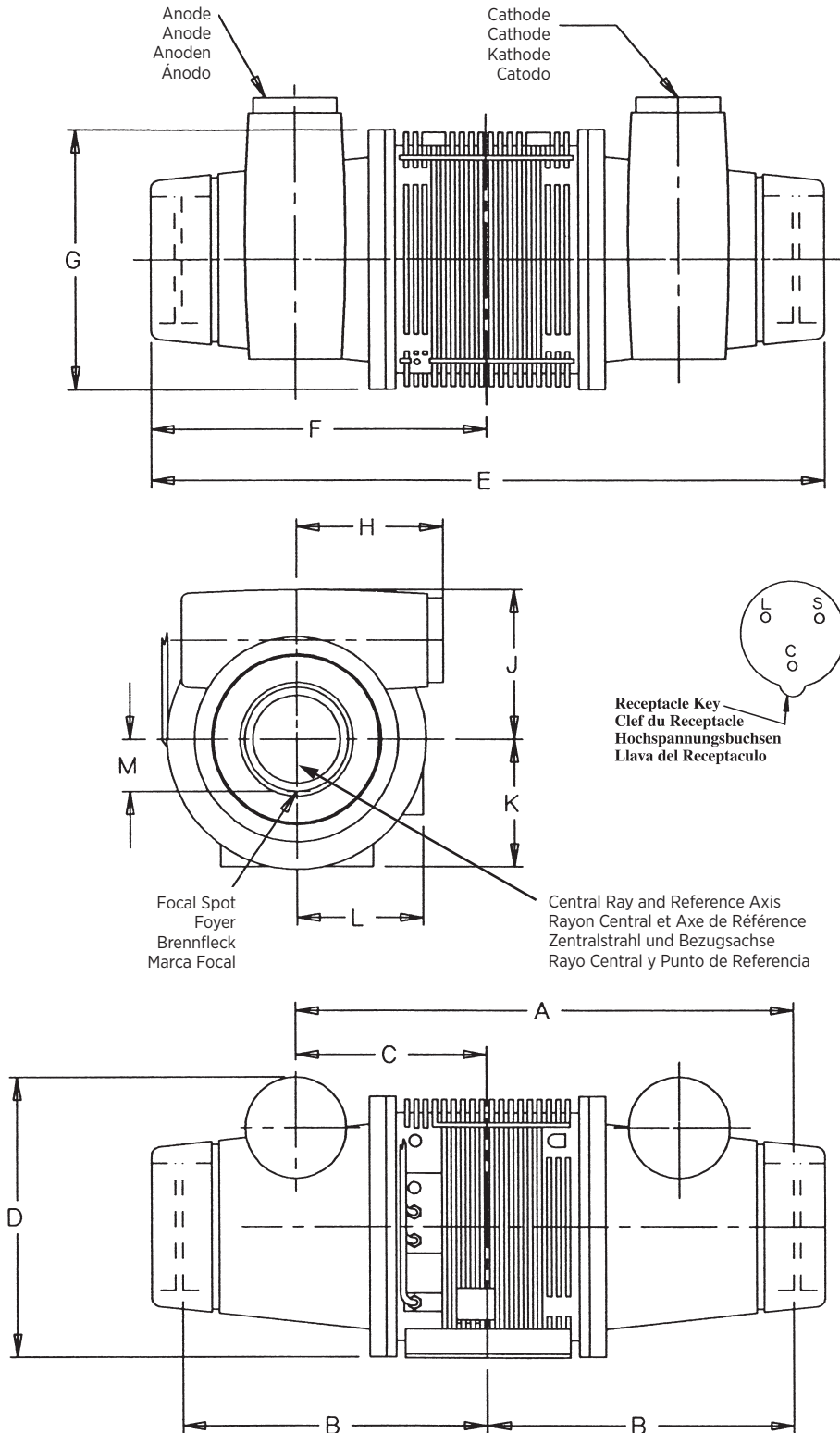
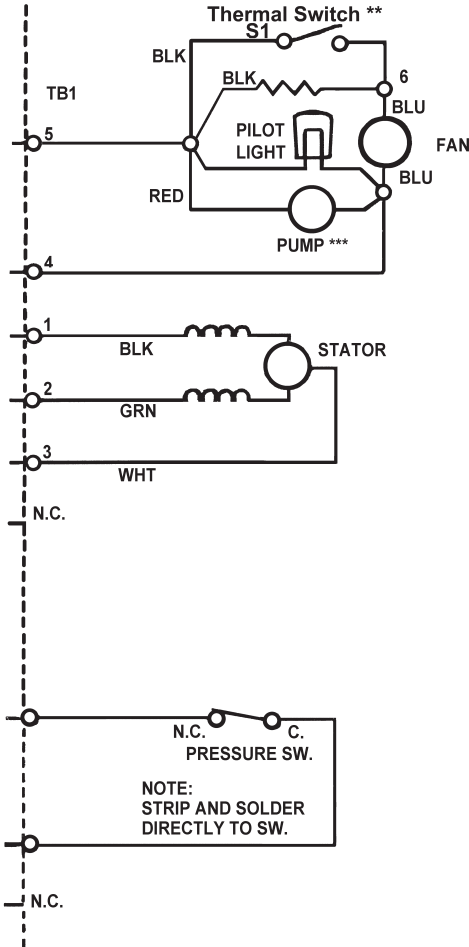
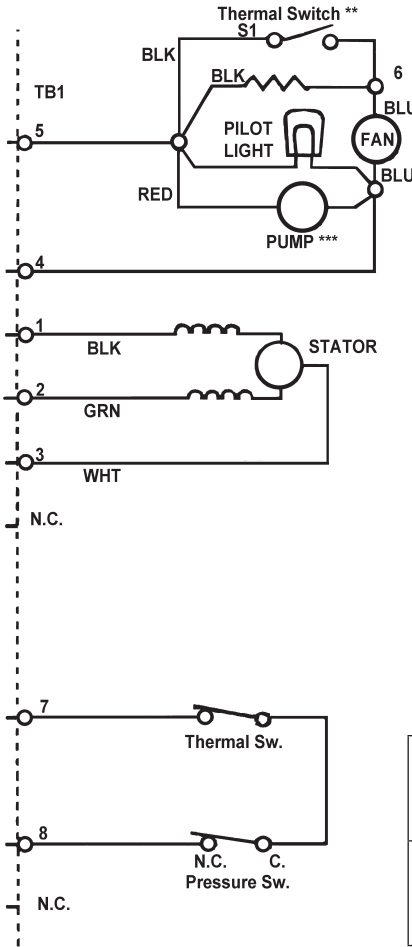
Note:
 Check wiring of tube and change as necessary to make it look like above diagram.
 **Used on tubes with 2 speed fans
 ***May not be present

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Brake Voltage 106 VDC

Low Speed Time	2 sec.
High Speed Time	3.5 sec.

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