Note: Document originally drafted in the English language.

Product Description

The S260 housing is designed for rotating anode inserts having 90mm (3.5 inch) diameter targets.

IEC Classification..... Class 1

Weight, Approximate:
Housing & Tube..... 23 kg (51 lbs)

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Classification CEI Classe 1

Poids, Approximatif:
Gaine et Tube 23 kg (51 lbs)

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Peso, Aproximado:
Encaje y Tubo 23 kg (51 lbs)

Soporte..... Rotable

Product Description	
Maximum Potential Difference	150 kV
Cathode to Ground	82 kV
Anode to Ground	82 kV
Maximum X-Ray Tube Assembly Heat Content	1,400 kJ (1,960 kHU)
Maximum Continuous Heat Dissipation (max. housing temperature 78°C) with 25°C Cooling Water	700 W (980 HU/sec)
X-Ray Tube Assembly Permanent filtration	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation	150 kV, 4.0 mA
Temperature Limits for Storage and Transport	-9°C to +70°C
Humidity	10% to 90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Thermal Switch: (Overload)	Normally Closed 15A @ 125 Vac or 10A @30 Vac/dc
Open	70°C (±2.8°C)
Closed	59°C (±2.8°C)
Thermal Switch: (cooling control)	Normally Closed 15A @ 125 Vac or 10A @30 Vac/dc
Open	40°C (±2.8°C)
Closed	33°C (±2.8°C)
X-Ray Tube Assembly (Complies to)	IEC 60601-2-28

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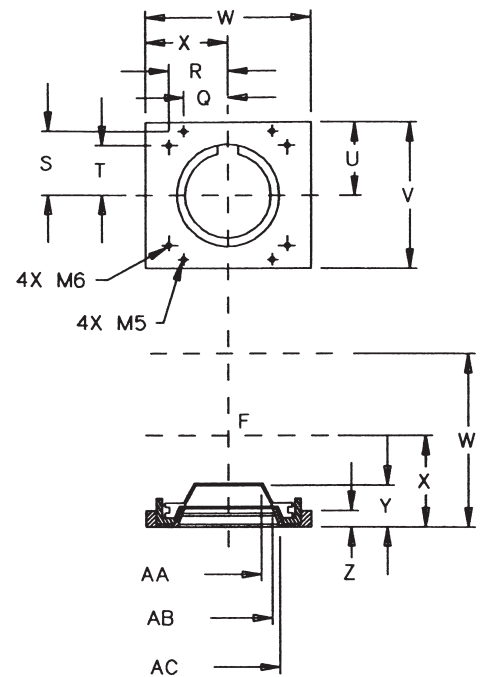
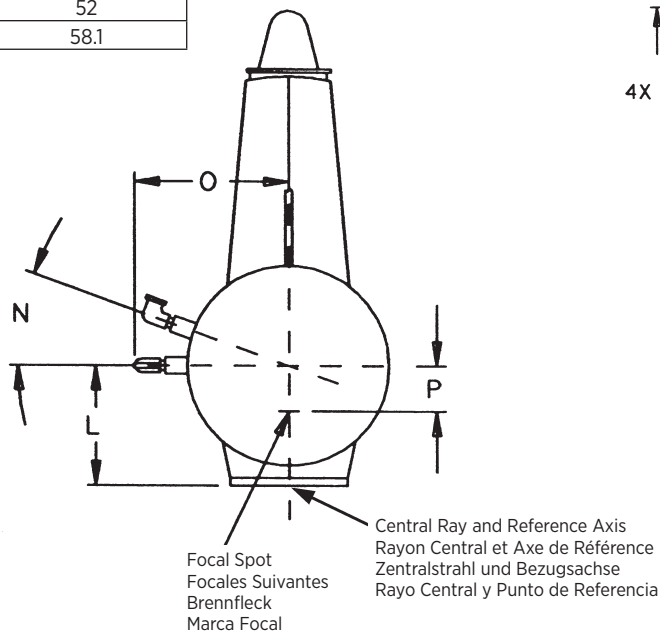
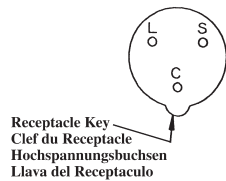
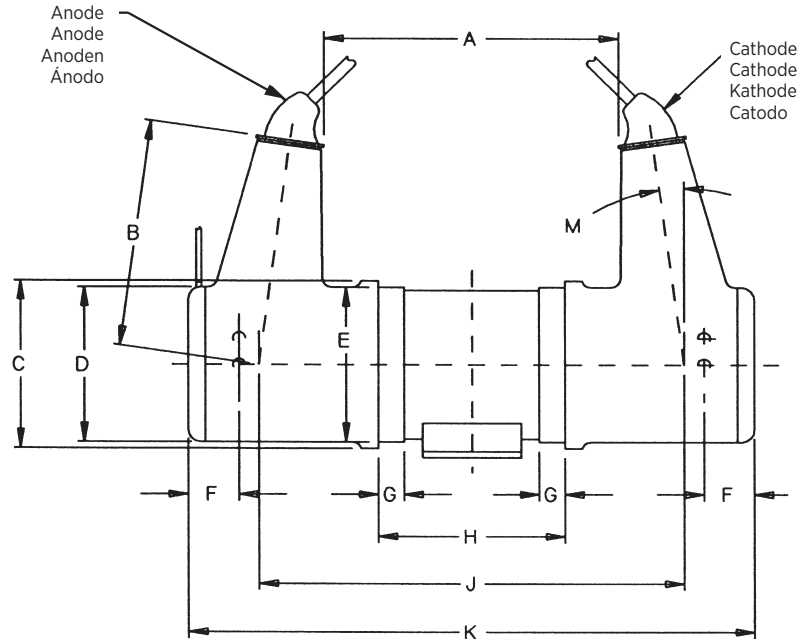
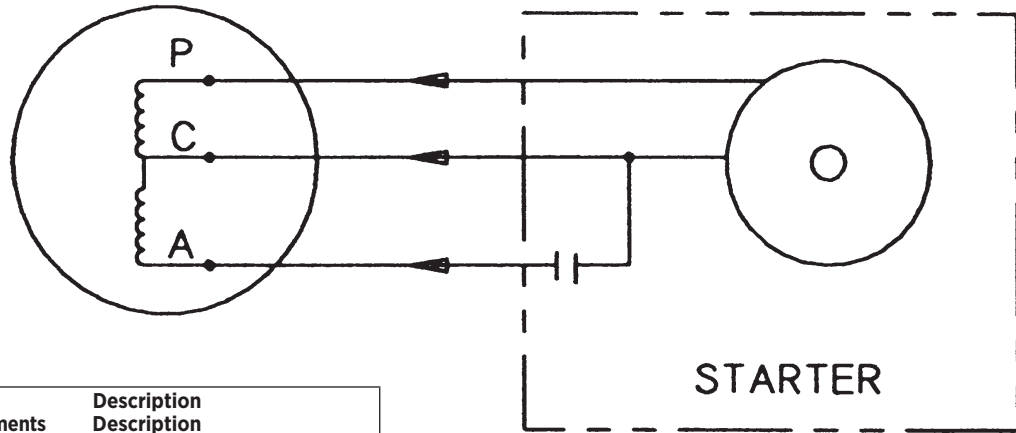
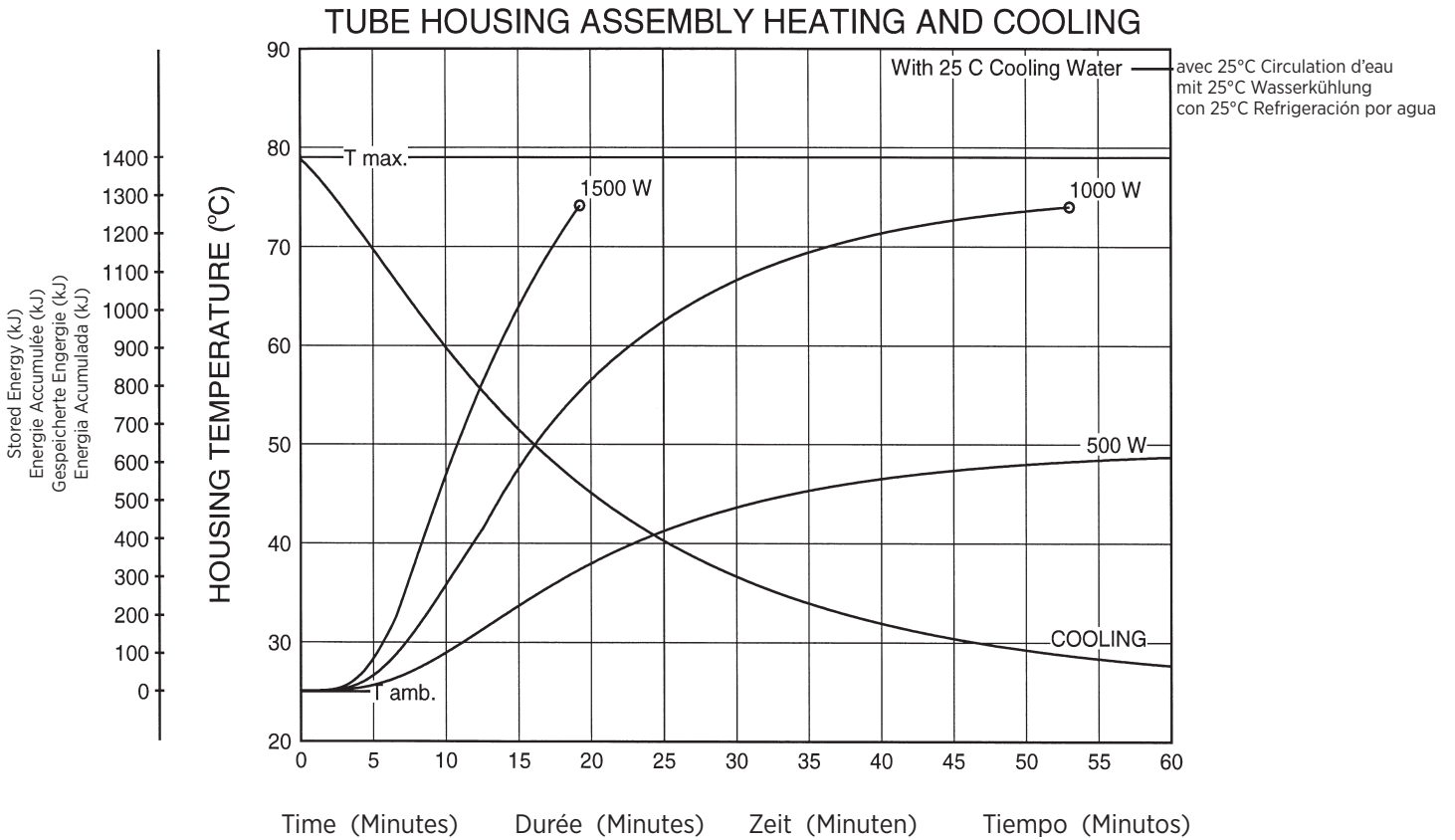
Note:

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.

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