Note: Document originally drafted in the English language.

Product Description

The B-115 housing is designed for Varex Imaging mammography inserts.

IEC Classification Class 1
 Weight, Approximate:
 Housing 10.2 kg (22.5 lbs)
 Stator Cord 4.5 m (14 Ft)
 Mounting Port
 X-Ray Tube Assembly (Complies to) . . IEC 60601-2-28

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Product Description

Nominal X-ray Tube Voltage	
Maximum Cathode to Ground	0 kV
Maximum Anode to Ground	39 - 49 kV
Grid Control Voltages: (If Applicable)	
Typical Bias Voltage for Focals	-5 to -100 Vdc
Maximum X-Ray Tube Heat Content	370 kJ (500 kHU)
Nominal Continuous Input Power	100 Watts (135 HU/sec)
	IEC 60613:2010
Maximum Housing Temperature	78°C
X-Ray Tube Assembly	
Permanent Filtration	0.63 mm Be IEC 60522/1999
	(0.0mm Al/49 kV)
Leakage Technique Factors	49 kV, 2.0 mA
Cable Receptacles	Non Federal Standard
	55 kV - Rating
Thermal Switch	normally closed
Open	79.4°C ±3.9°C (175°F ±7°F)
	Rating - 10 A @ 240Vac
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transportation	-10°C to 75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa

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Notes:

1. Dimensions listed are for the standard port boss
 Metric M6 X 1.0 available upon request

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For Determining Focal Spot to Port Distance

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DIM. F = 3.20 INCHES [81.3mm]
 DIM. I = 1.89 INCHES [48mm]

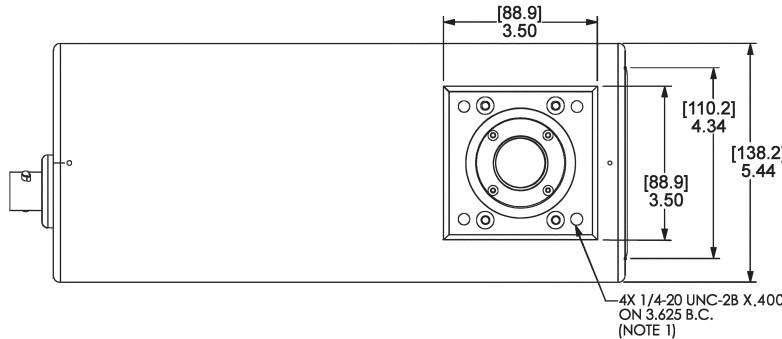
BIANGULAR INSERTS M-113 Series, M-171

STANDARD/METRIC PORT BOSS
 DIM. F = 2.97 INCHES [75.4mm]
 DIM. I = Refer to Drawing


EXTENDED PORT BOSS

DIM. F = 3.20 INCHES [81.3mm]
 DIM. I = Refer to Drawing


Note: Dimensions in [mm] Inches
Remarque: Dimensions en [mm] Pouces
Hinweis: Abmessungen in [mm] Zoll
Nota: Dimensiones en [mm] Pulgadas

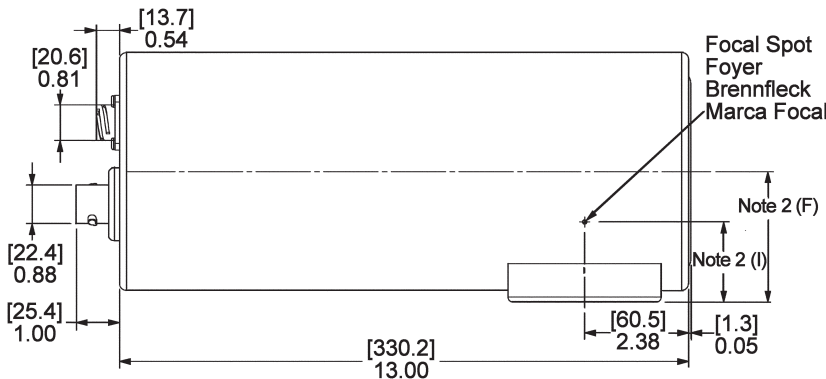FRONT VIEW
 VUE DE FACE
 VONDERANSICHT
 DELANTER EXPUESTA

BIANGULAR INSERTS

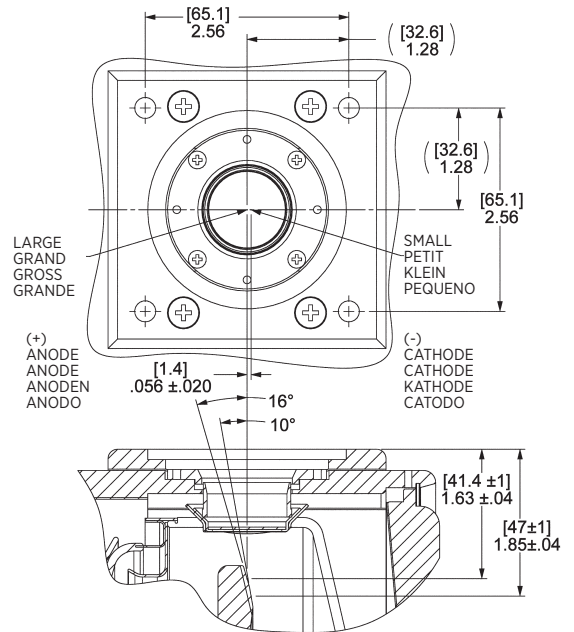
Stator and Filament Connector (Cathode)

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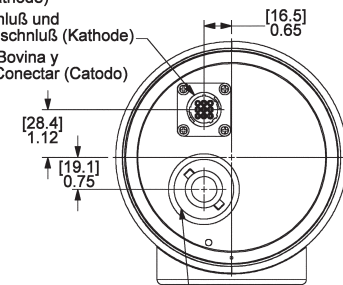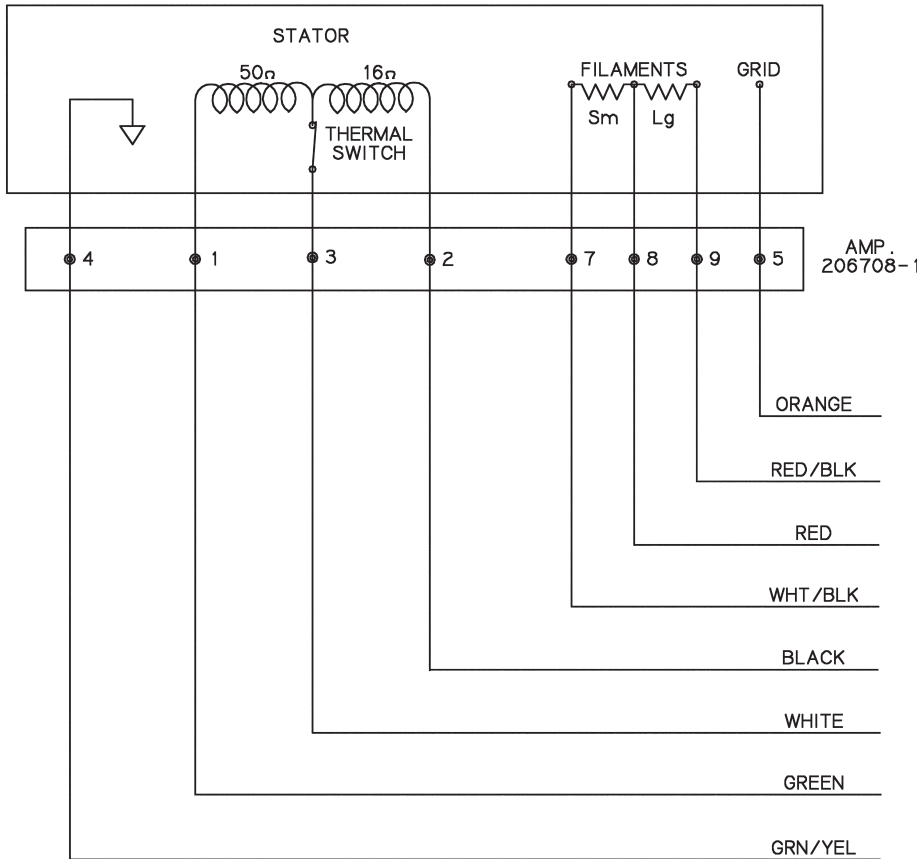
Stator Power:
 Time to full speed of the anode is a function of the power rating of the "starter" and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible.

Starter:	Start	Run
50/60 Hz	115 VAC	45 VAC
150/180 Hz	230 VAC	90 VAC

NOTE:
 For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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