Note: Document originally drafted in the English language.

Product Description

The B-185H housing is designed for Varex Imaging rotating Anode inserts having 133 mm (5.25 inch) diameter targets.

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Soporte Rotable ó de entrada

Product Description

Nominal X-ray Tube Voltage 150kV
 Maximum Cathode to Ground 75 kV
 Maximum Anode to Ground 75 kV
 Grid to Cathode
 If applicable -4 kV
 Grid Control Voltages
 Typical Bias Voltage for Cutoff at 125 kV -3600 Vdc
 Grid Voltage for Exposure 0 Vdc
 Maximum X-Ray Tube Assembly Heat Content 1,482 kJ (2,000 kHU)
 Nominal Continuous Input Power
 (maximum housing temperature 78°C)
 without HE-100 250 W IEC 60613:2010
 with HE-100 1,130 W IEC 60613:2010
 X-Ray Tube Assembly
 Permanent Filtration 1.0mm Al IEC 60522
 Leakage Technique Factors 150 kV, 7.2 mA
 High Voltage Cable Receptacles Per IEC 60526
 Temperature Limits for Storage and Transport -20°C to +75°C
 Humidity +10% to +90%
 Atmospheric Pressure Range 70 kPa to 106 kPa
 Thermal Switch Normally Closed
 Open 76.7°C ±2.8°C (170°F ±5°F)
 3A @ 120Vac or 30 Vdc max.
 X-Ray Tube Assembly (Complies to) IEC 60601-2-28

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
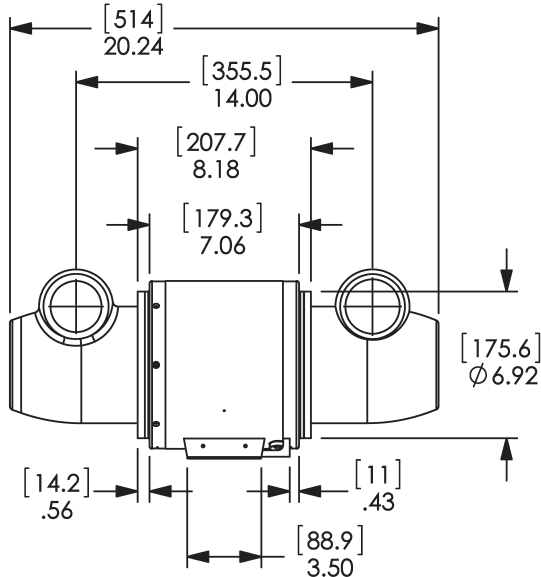
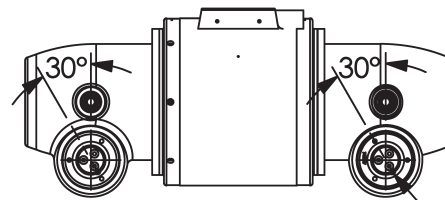
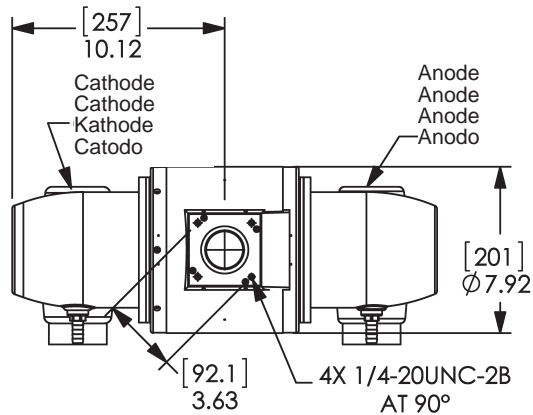
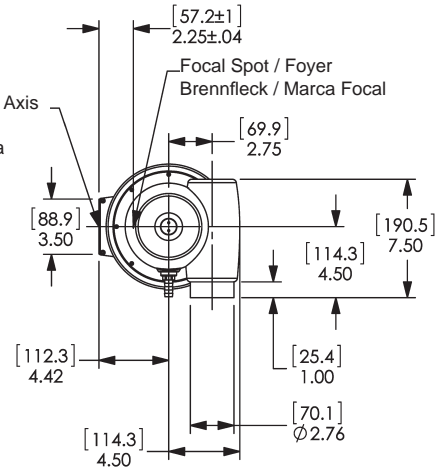
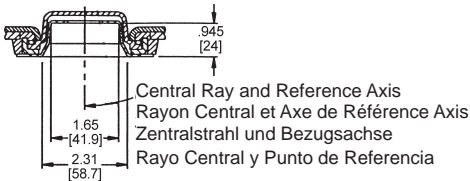
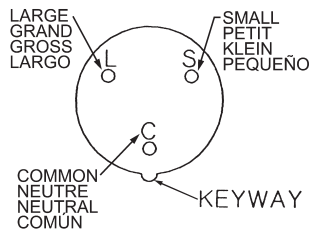
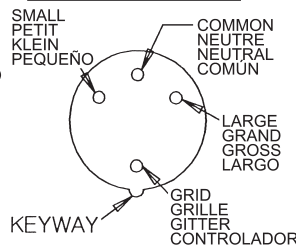
TRI FOCUS

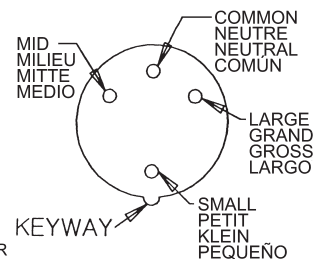NOTE:

- Reference dimensions are for information purposes and are not required for inspection.
- Aperature size optional
0.937 Sq. in.
1.250 Sq. in.
- Cable receptacles
2 1/2 -20 UNF -2B THREAD

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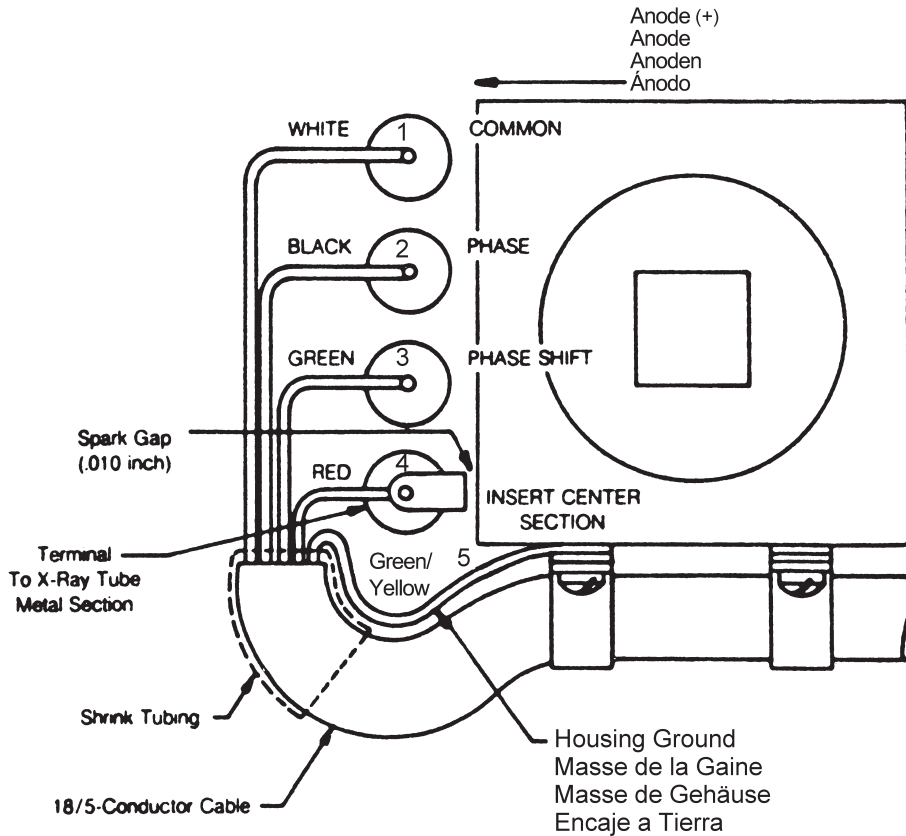
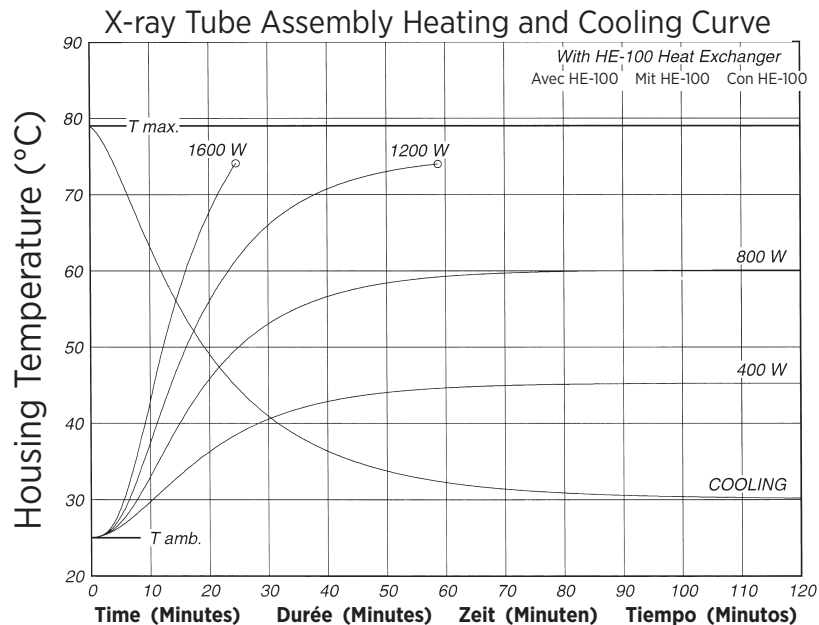
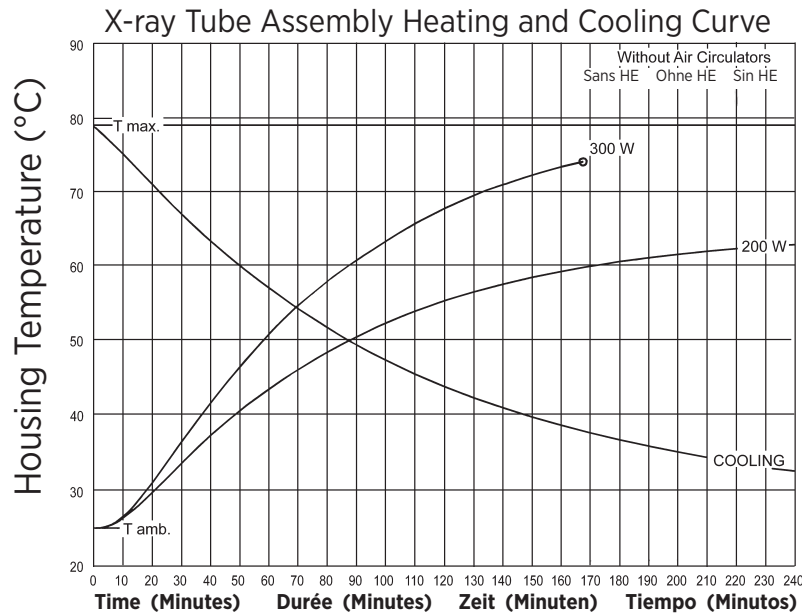
Stator Power:
 Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 102 mm (4 inch) anode series tubes is between 1.3 and 2.0 seconds.
 Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.
 No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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Caution:
 Housings configured with heat exchangers having long hoses (> 5 ft or 1.5 m) are not configured for operation separate from the associated heat exchanger. In such cases, the tube unit must not be operated unless properly coupled to the heat exchanger. Reference the specific heat exchanger's product data sheet for the associated cooling curves.

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Note:
 Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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