Note: Document originally drafted in the English language.

**Product Description**

The Leo housing is designed for Varex Imaging rotating anode inserts having 71 mm (2.8 inches) or 80 mm (3 inches) diameter targets.

IEC Classification ..... Class 1

Weight, Approximate:

Housing & Tube ..... 17.4 kg (38.5 lbs)

Mounting ..... Port plate or Trunnion  
Metric Housing - M6-1 screws

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**Product Description**

Maximum Peak Voltage ..... 150 kV  
 Cathode to Ground ..... 75 kV  
 Anode to Ground ..... 75 kV

Maximum X-Ray Tube Assembly Heat Content ..... 900 kJ (1,250 kHU)

Nominal Continuous Input Power  
 (maximum housing temperature 78°C) ..... 200 W (278 HU/sec)  
 IEC 60613:2010 @ sea Level

X-ray tube assembly cooling is provided by ambient air flow.

X-Ray Tube Assembly (Insert and Housing)  
 Permanent Filtration ..... 0.7mm Al/75kV IEC 60522/1999

Loading Factors for Leakage Radiation ..... 150 kV, 2.0 mA

Temperature Limits for Storage and Transport ..... -20°C to +75°C  
 Humidity ..... +10% to +90%  
 Atmospheric Pressure Range ..... 70 kPa to 106 kPa

Thermal Switch ..... Normally Closed  
 Contact Rating - 10A @ 240Vac

Open ..... 80°C ±3.0°C (176°F ±6°F)

Federal Standard High Voltage Receptacles .... (Complies to IEC 60526)  
 NEMA Standard XR7-1979 (R1984, 1990)

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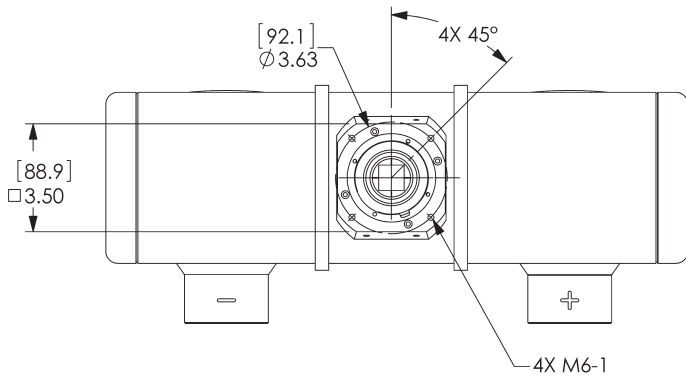
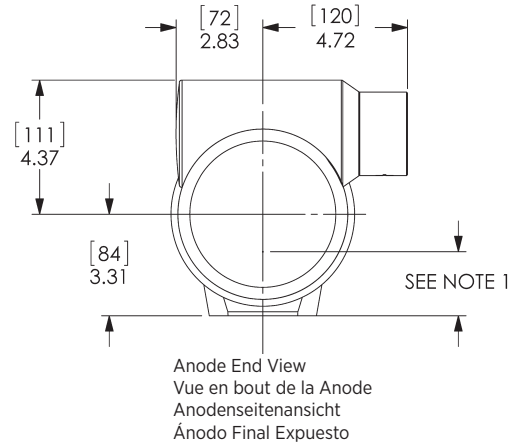
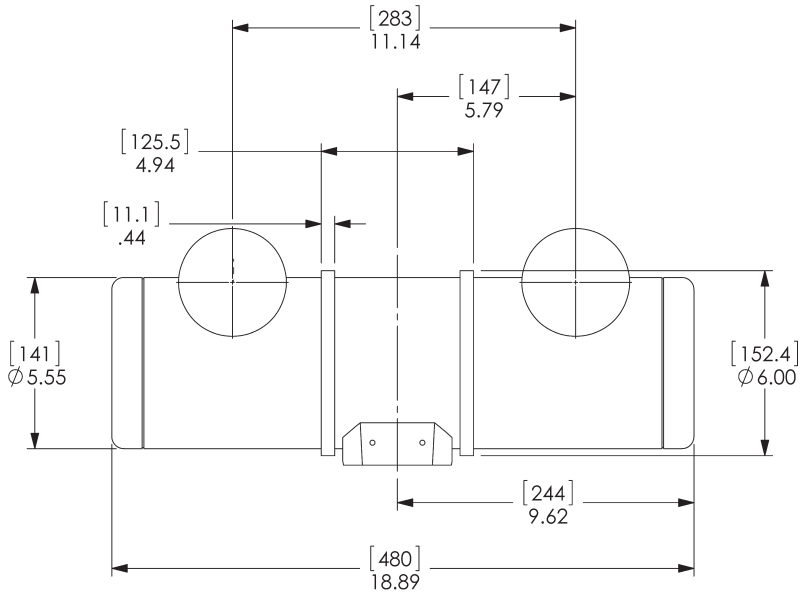
**Note 1**

Focal Spot to Port Distance  
 80 mm Anode ..... [53.0 ±1.0] 2.10 ±0.040  
 71 mm Anode (RAD-8 & 12) .... [56.0 ±1.0] 2.20 ±0.040

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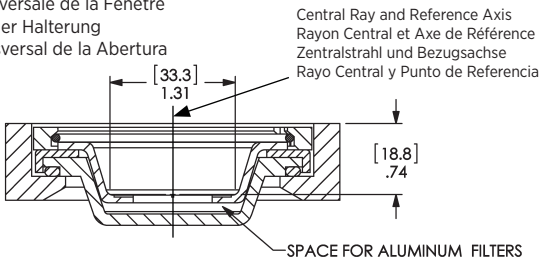
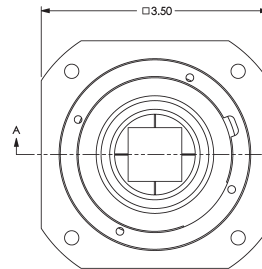
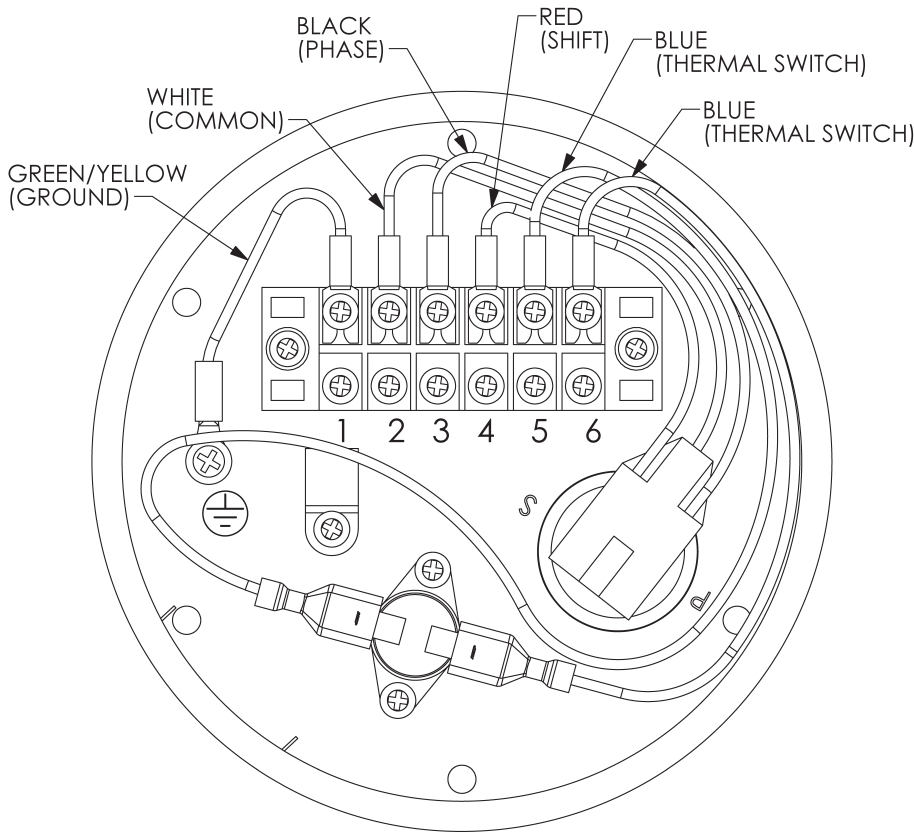
**Stator Cord:**

A six wire shield-grounded stator cord should be connected from the stator terminal to the motor control. All listed voltage and amperage measurements should be taken at the housing end.

If the stator cord is not provided, the cord used must be UL listed or a UL recognized component for all products used in the USA. Outside the USA, the cord used must comply with all applicable regulatory and statutory requirements for electrical and safety. The cord must have the following characteristics: six conductor, 18AWG (0.9mm<sup>2</sup>) Cu, rated 600V, 90°C minimum, shielded cable with PVC jacket 0.032 inch (0.081mm) thick minimum. Lead terminations are provided and must be used with wire that matches the above description.

**Stator Power:**

Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed for 71mm (2.8 inch) and 80mm (3.0 inch) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 400 volts rms.

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