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Note: Document originally drafted in the English language.

Product Description

The ORIIIB is a uniquely designed x-ray tube housing developed for use in surgical C-Arm applications. This design utilizes a high performance rotating anode tube, with inherently greater target loading capabilities than that of stationary anode tube designs. This housing provides ease of use at a nominal patient table height.

Housing: Shockproof, Aluminum. Lead lined; filled under vacuum with high dielectric insulating oil and hermetically sealed. The housing contains a stator for driving the rotor.

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Tube Housing Specifications

Maximum Peak Voltage	
Anode to Cathode	125 kV
Anode to Ground	62.5 kV
Cathode to Ground	62.5 kV
Nominal Continuous Input Power	185 W (15 kHU**/Min.) IEC 60613:2010
Maximum Housing Temperature	78°C
X-ray tube assembly cooling provided by ambient air flow.	
Thermal Characteristics	
Housing heat storage capacity	1,185 kJ (1.6 MHU**)
Weight	43 lbs. without cables or accessories
X-Ray Tube Assembly	
Permanent Filtration	0.7mm Al minimum at 75 kV - IEC 60522/1999
Loading Factors for Leakage Radiation	125 kV, 1.5 mA
X-Ray Coverage:	
8.5° target - 12 in (30 cm) at 40 in (100 cm) Distance Source-Image	
10° target - 14 in (35 cm) at 40 in (100 cm) Distance Source-Image	
Thermal Switch	
The switch must be used in conjunction with the control manufacturer's interlock circuitry. The thermal switch is normally closed. The switch will open when the temperature reaches the set point.	
CAUTION	
The thermal switch does not protect the insert against overheating.	
IEC Classification	Class 1
**H.U. (Heat Units) = (kV) x (mA) x 1.35 x Time in seconds (3Ø operation)	

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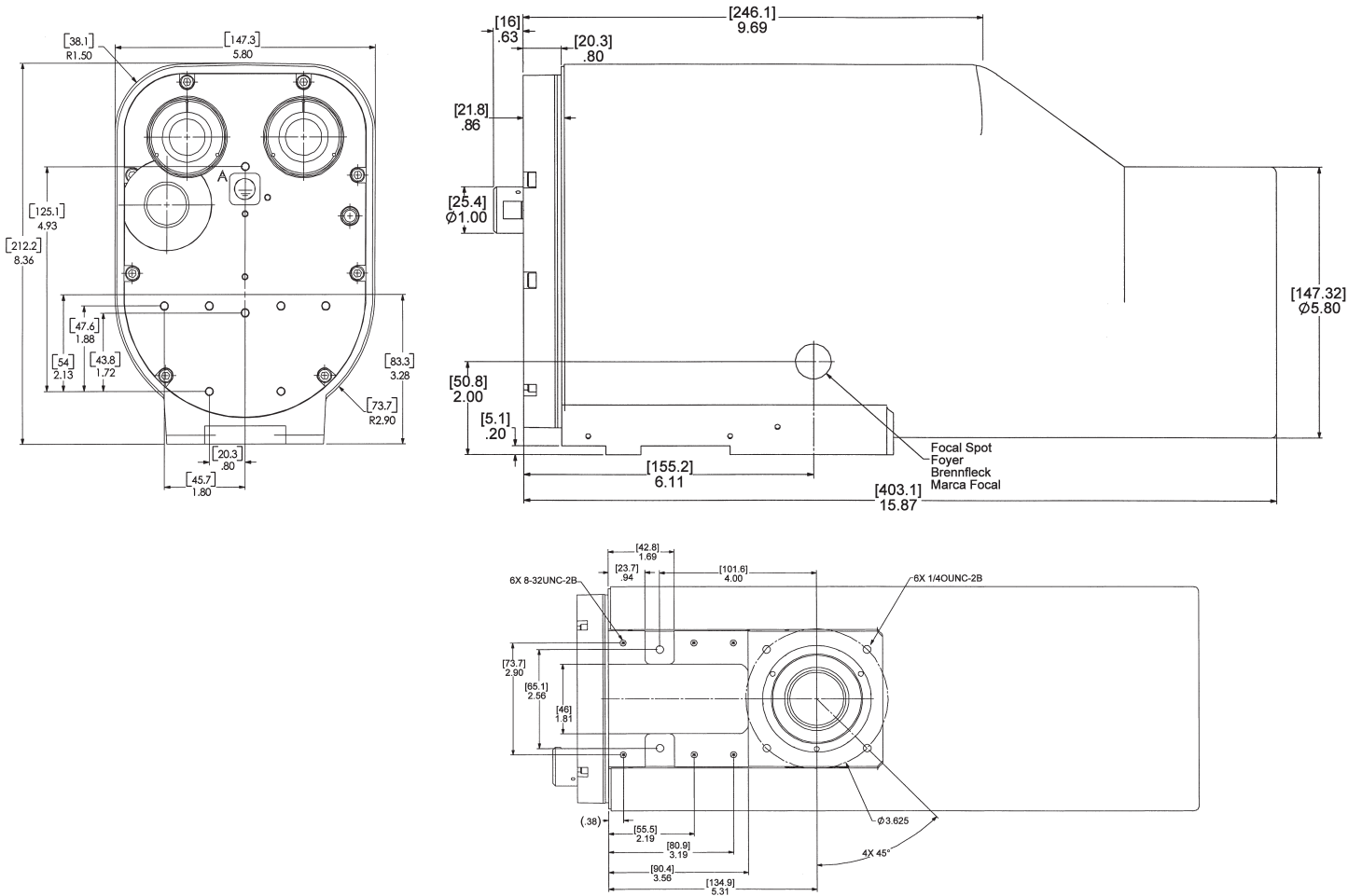
[M\mm]
 INCHES

Cable Receptacles:
 Alden type, P702-1 (cathode) and P702 (anode), high voltage receptacles.
 The receptacles mate with Alden type (P7010S) high voltage connector and cable.

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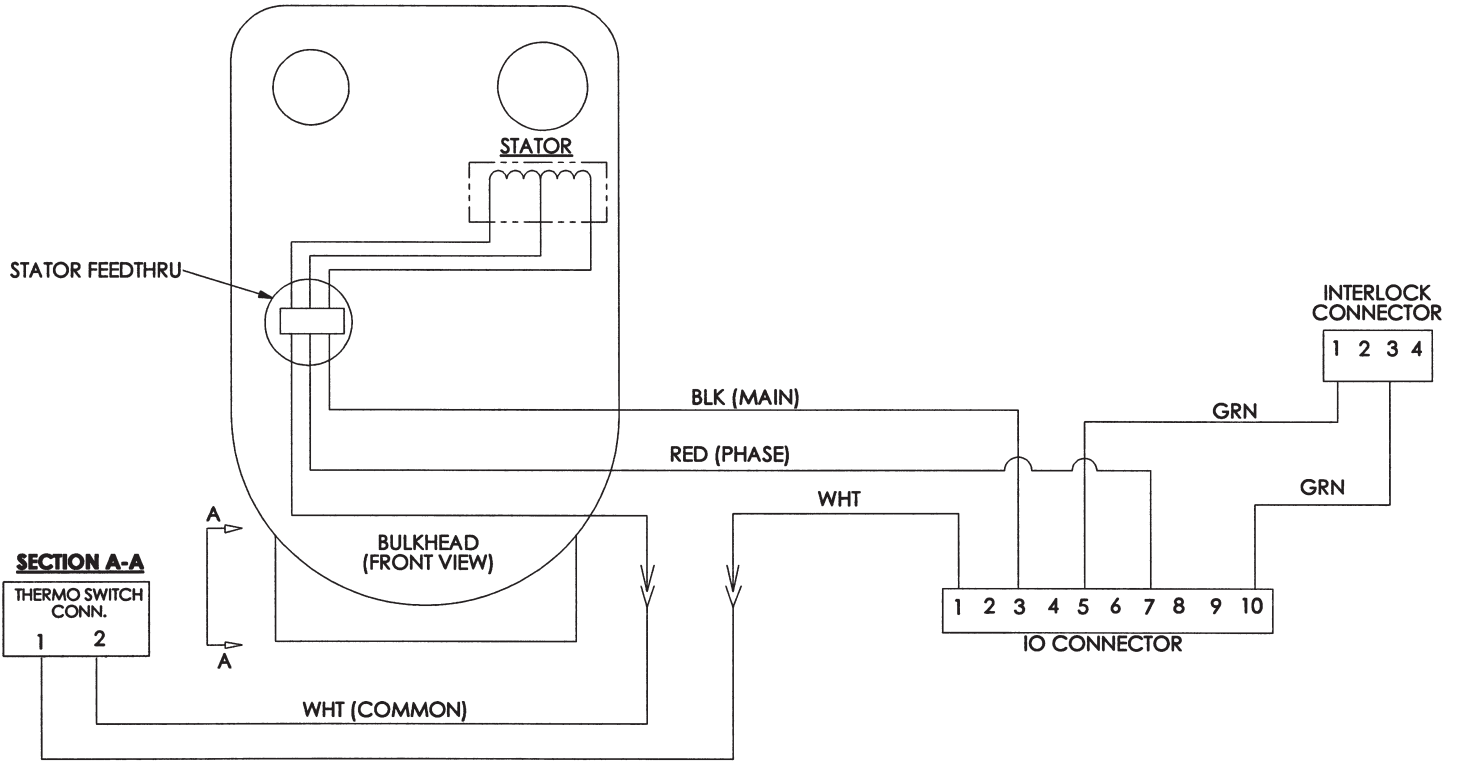
Rayproofing: Stray radiation complies with DHHS standards under the
 "Radiation Control for Health and Safety Act 1968" (U.S.A.)

Etanchéité Radioactive:
 Les pertes en radiation sont conformes aux normes DHHS du
 "Radiation Control for Health and Safety Act 1968" (U.S.A.).

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Stator: The ORIIIB housing is equipped with a standard Varex Imaging stator requiring a 30 microfarad capacitor, providing 90° phase shift for 50/60 Hz., 2,850/3,450 rpm operation.

Stator Motor Ratings:

Frequency (Hz.)	Volts	Common Leg Current
50-60	60*	Approx. = 2.0 Amps
50-60	120	Approx. = 4.0 Amps
50-60	240	Approx. = 8.0 Amps

*Running Voltage

Stator Cord: A six wire shield-grounded stator cord should be connected from the stator terminal to the motor control. All listed voltage and amperage measurements should be taken at the housing end.

If the stator cord is not provided, the cord used must be UL listed or a UL recognized component for all products used in the USA. Outside the USA, the cord used must comply with all applicable regulatory and statutory requirements for electrical and safety. The cord must have the following characteristics: six conductor, 18AWG (0.9mm²) Cu, rated 600V, 90°C minimum, shielded cable with PVC jacket 0.032 inch (0.081mm) thick minimum. Lead terminations are provided and must be used with wire that matches the above description.

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Frequenz (Hz)	Volts	Klemmen
50-60	60*	Approx. = 2.0 Amps
50-60	120	Approx. = 4.0 Amps
50-60	240	Approx. = 8.0 Amps

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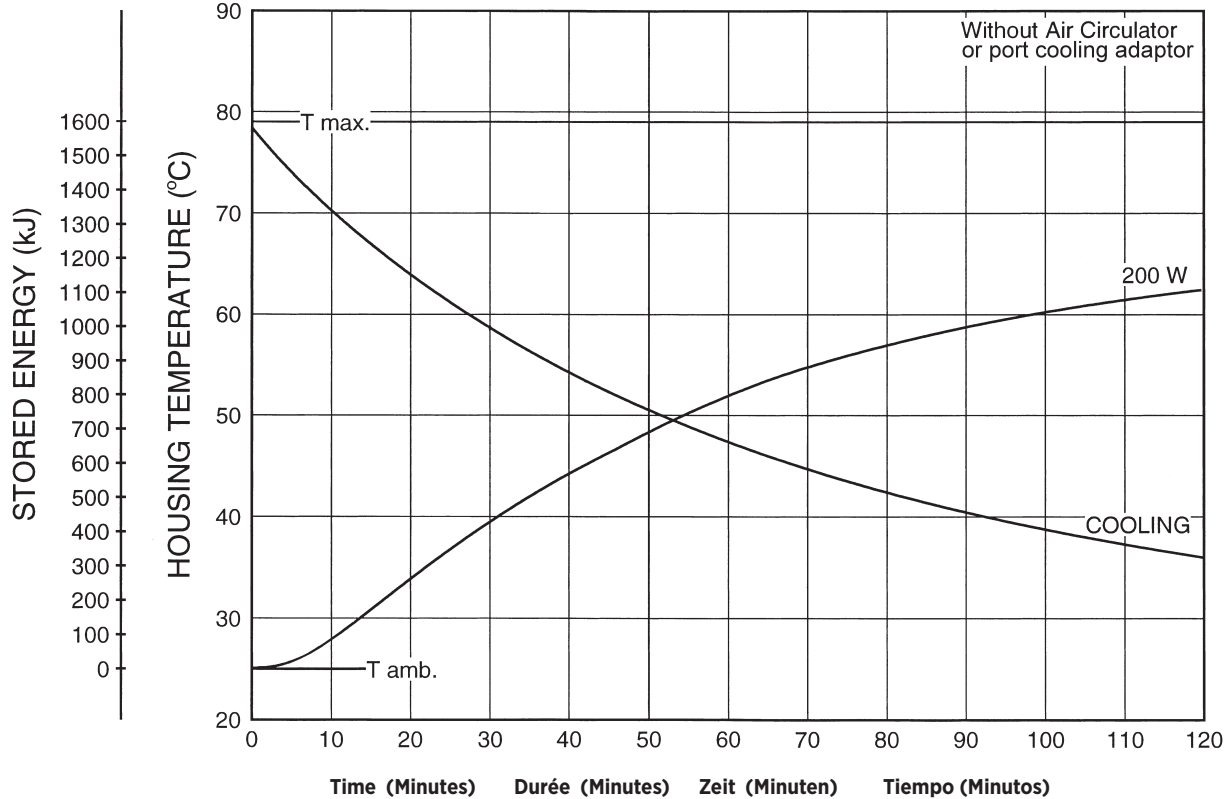
TUBE HOUSING ASSEMBLY HEATING AND COOLING

Operating Temperature: Minimum ambient temperature for storage and transportation is -20°C (0°F). Operating range of the ORIIIB housing is 16° - 72°C (60° - 167°F).

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