Note: Document originally drafted in the English language.  
注释：文件最初用英语起草。

**Product Description**

The M-152 is a 2.75" (70 mm) 49.0 kV, 97 kJ (135 kHU) maximum anode heat content, rotating anode insert. This insert is specifically designed for use in Mammography systems. The insert features two targets, one of Molybdenum and the other Rhodium and is available with the following nominal focal spots:

Target: Mo - 0.1 (6°) - 0.3 (20°)  
Target: Rh - 0.1 (6°) - 0.3 (20°)  
IEC 60336

**Nominal Anode Input Power**

Small (Mo) - 2.5 kW IEC 60613  
Large (Mo) - 5.5 kW IEC 60613  
Small (Rh) - 2.0 kW IEC 60613  
Large (Rh) - 4.1 kW IEC 60613

For the equivalent anode input power of 80 Watts

This insert is intended for use in M52.2 housings.

All trademarks property of the respective manufacturer

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

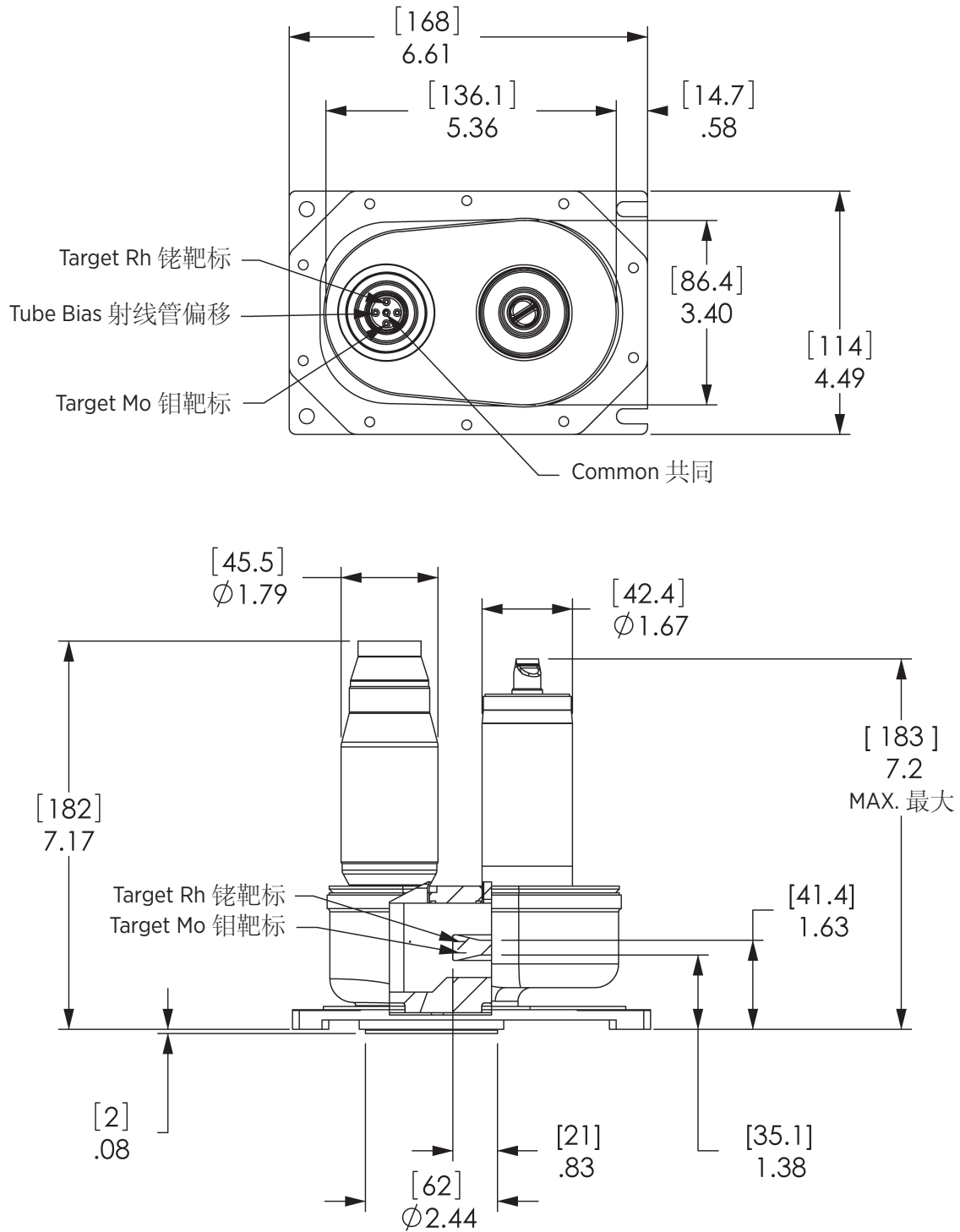
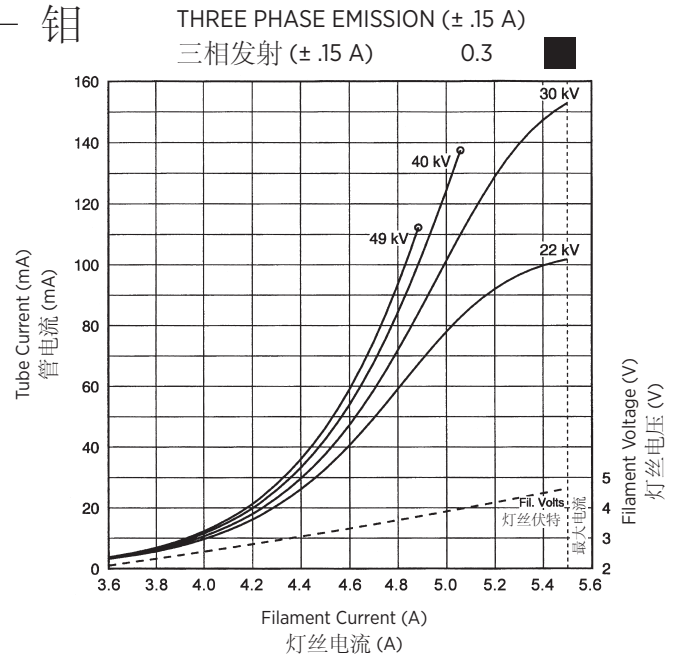
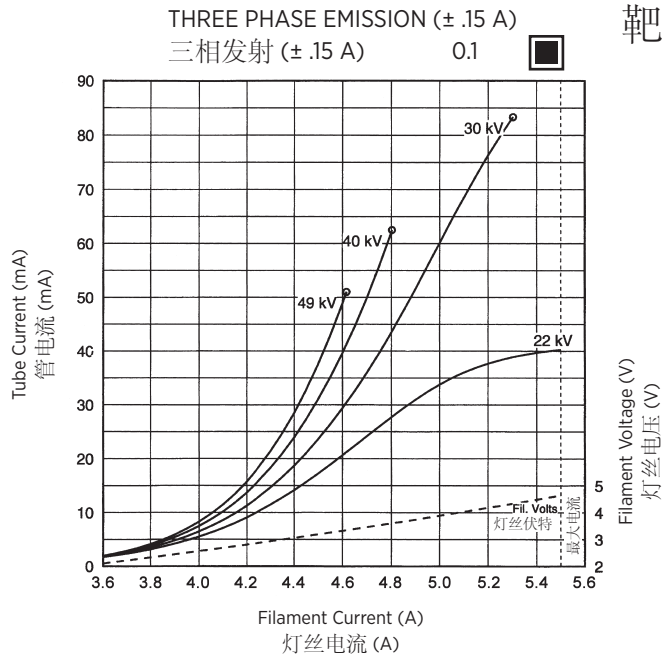
Target - Mo.  
靶标 - 钼

Target - Rh.  
靶标 - 铑

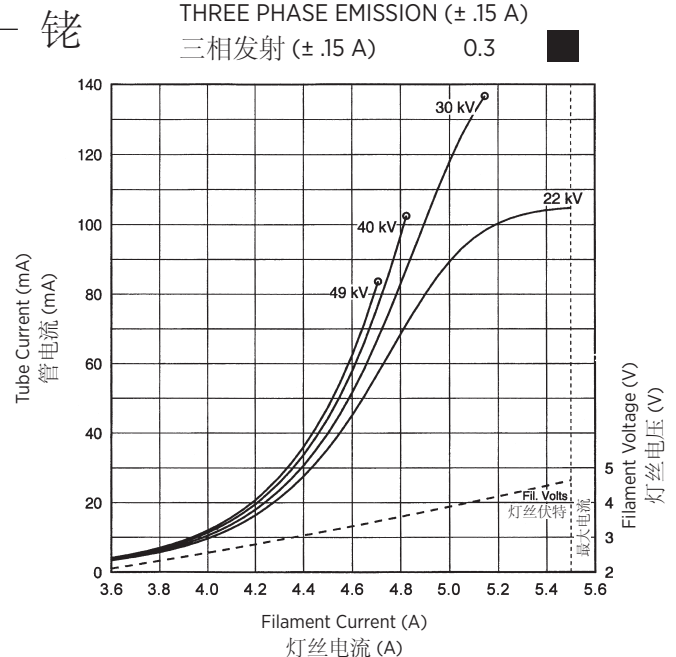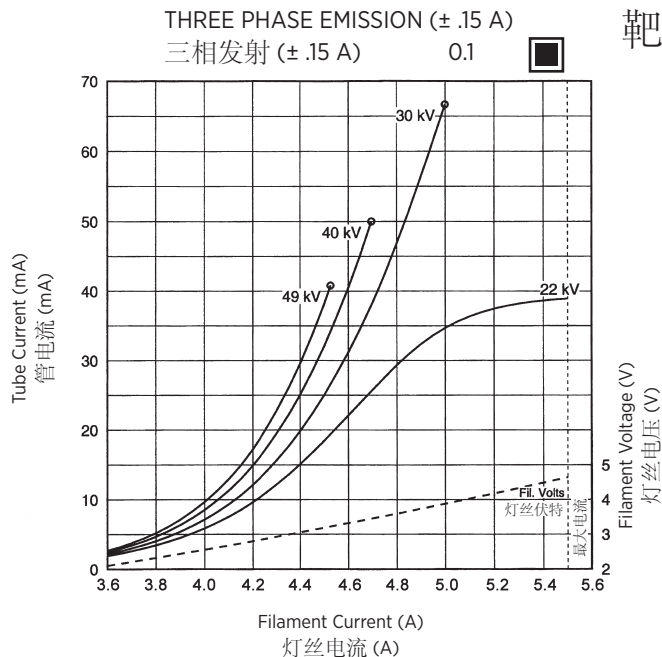
Note: When using these emission curves for trial exposures, refer to the power rating curves shown for maximum kV, tube emission, filament current, exposure time, and target speed.

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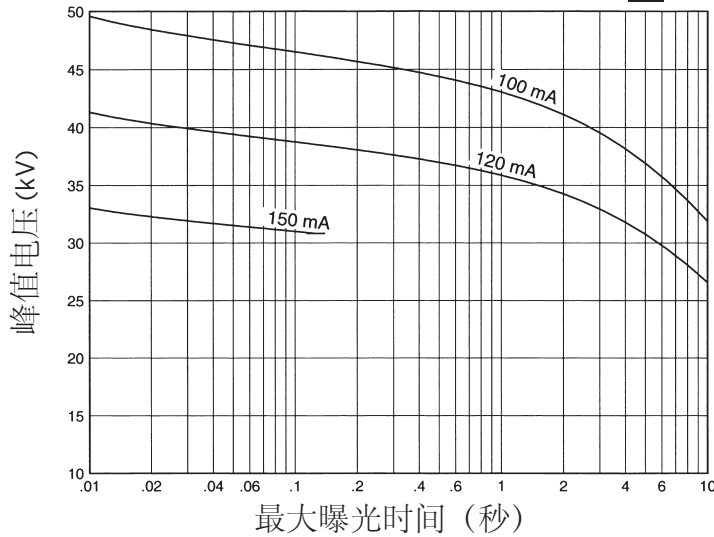
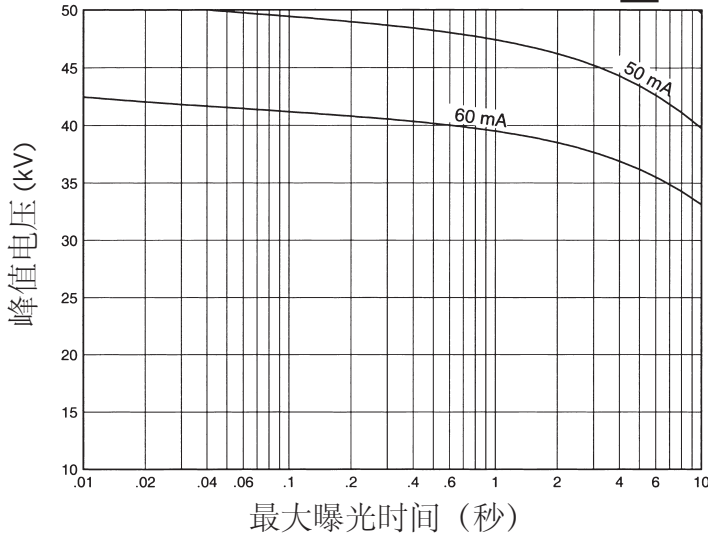
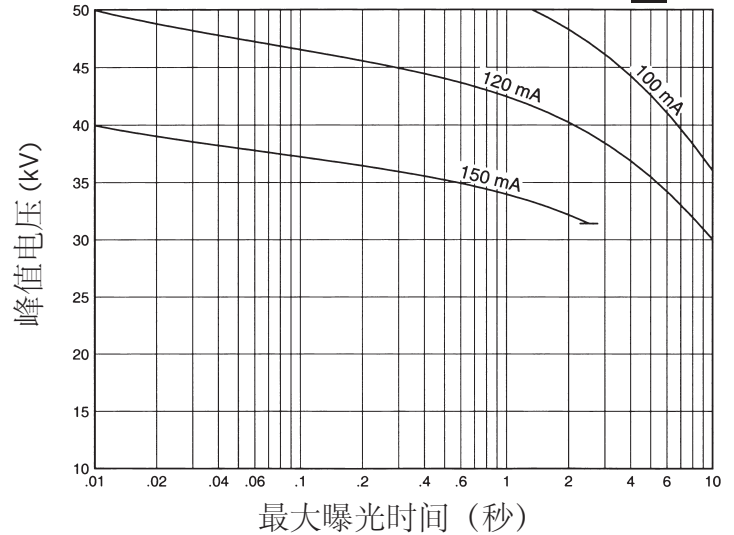
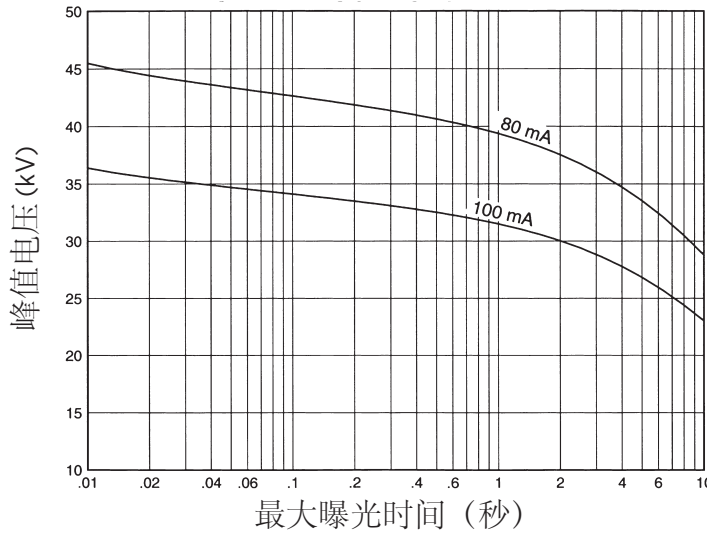
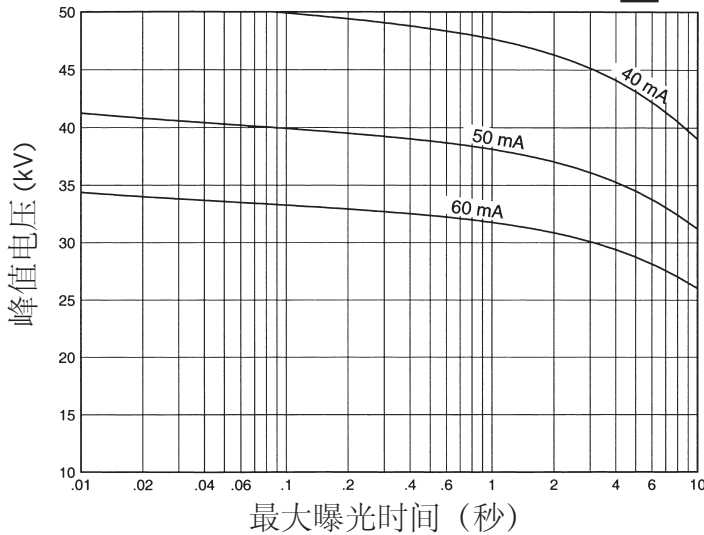
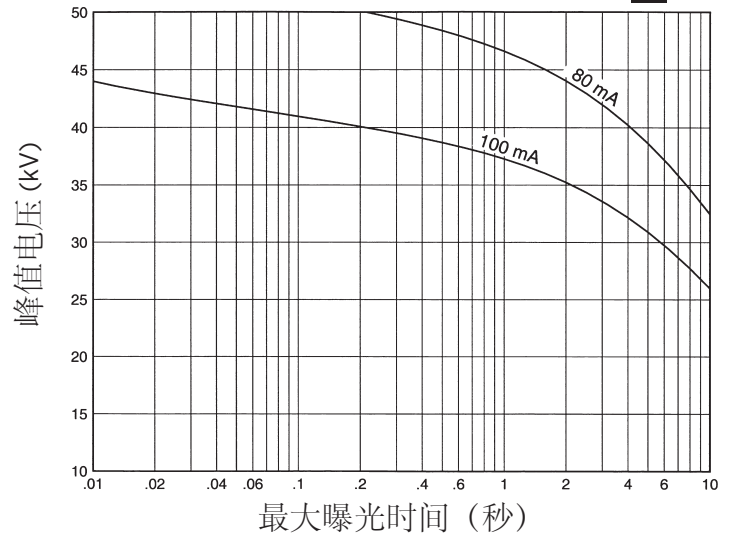
<b>Product Description</b>	
Maximum Potential Difference	
Cathode to Ground .....	49 kV
Anode to Ground .....	0 kV
Heat Storage Capacity .....	336 kJ (470 kHU)
Continuous Heat Dissipation ...	144 W (200 HU/sec)
Maximum Housing Temperature .....	78°C
X-Ray Tube Assembly	
Permanent Filtration .....	0.63 mm Be IEC 60522
Leakage Technique Factors .....	49 kV, 3.0 mA
IEC Classification .....	Class 1
Weight (Approximate) .....	10.5 kg (23.1 lbs)
Safety Devices: Thermal Switch	
Thermal Switch	
Normally Closed Contact .....	Opens at 75°C±3°C
Pressure Switch	
Normally Closed Contact .....	Opens at 345 mbar
Ambient Air Temperature Limits for Operation	
.....	5°C to 40°C
Temperature Limits for Storage and Transportation	
.....	-20°C to 80°C
Humidity .....	10% to 90%
Atmospheric Pressure Range .....	70 kPa to 106 kPa
Mounting .....	Housing

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
Dimensions are for Reference only  
维度是供仅参考

**M52.2**

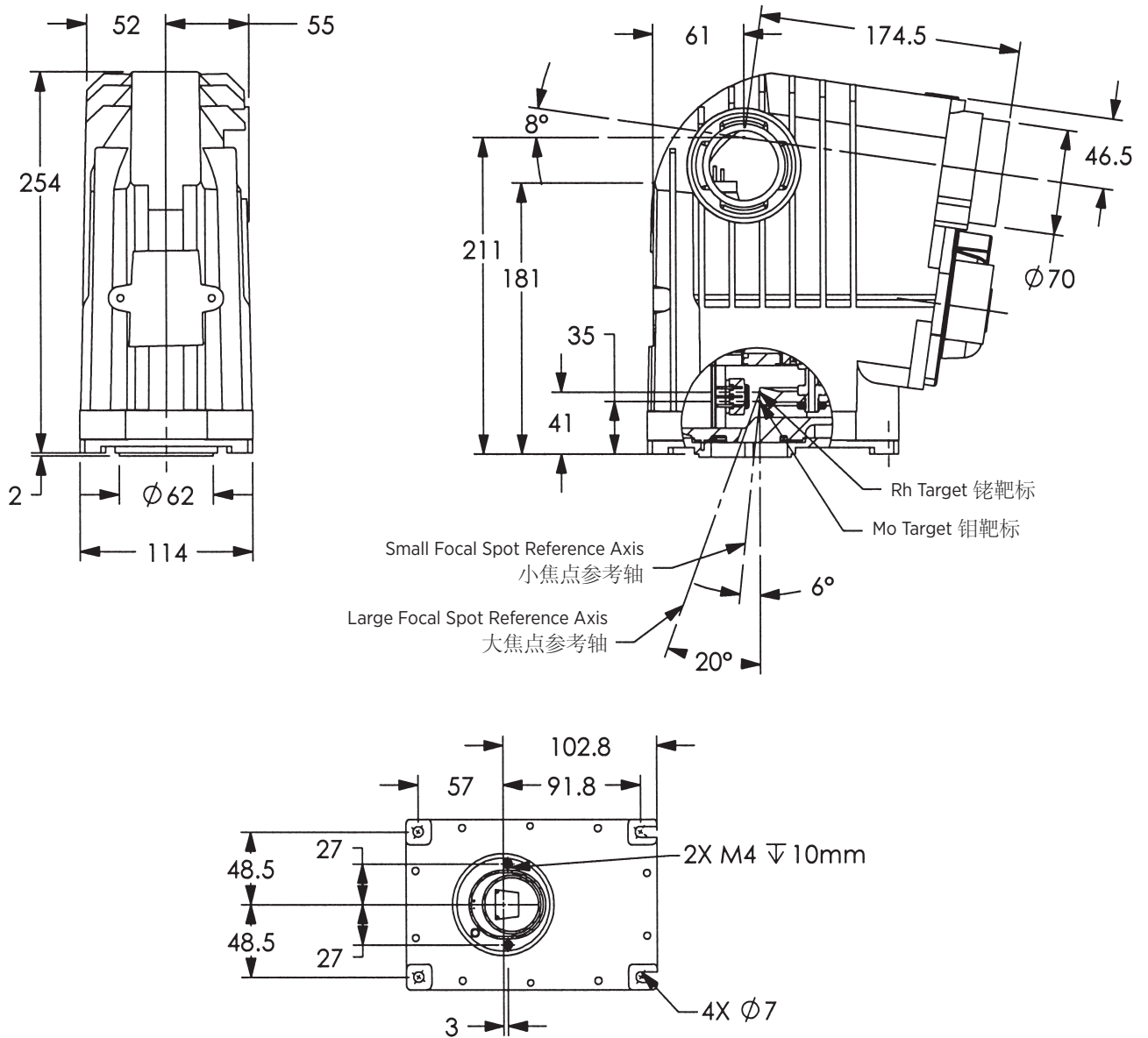
### Housing Assembly Heating and Cooling 管套组件加热与冷却

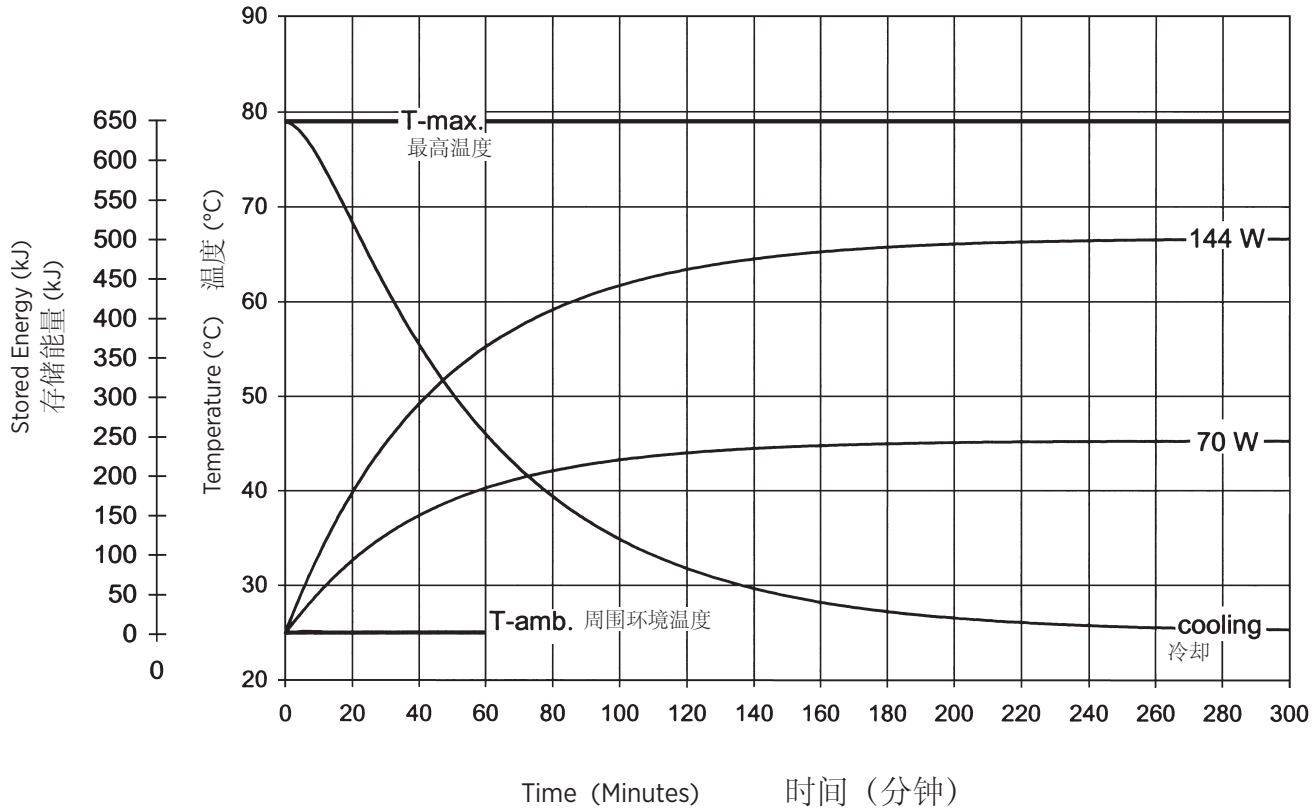**Note:**  
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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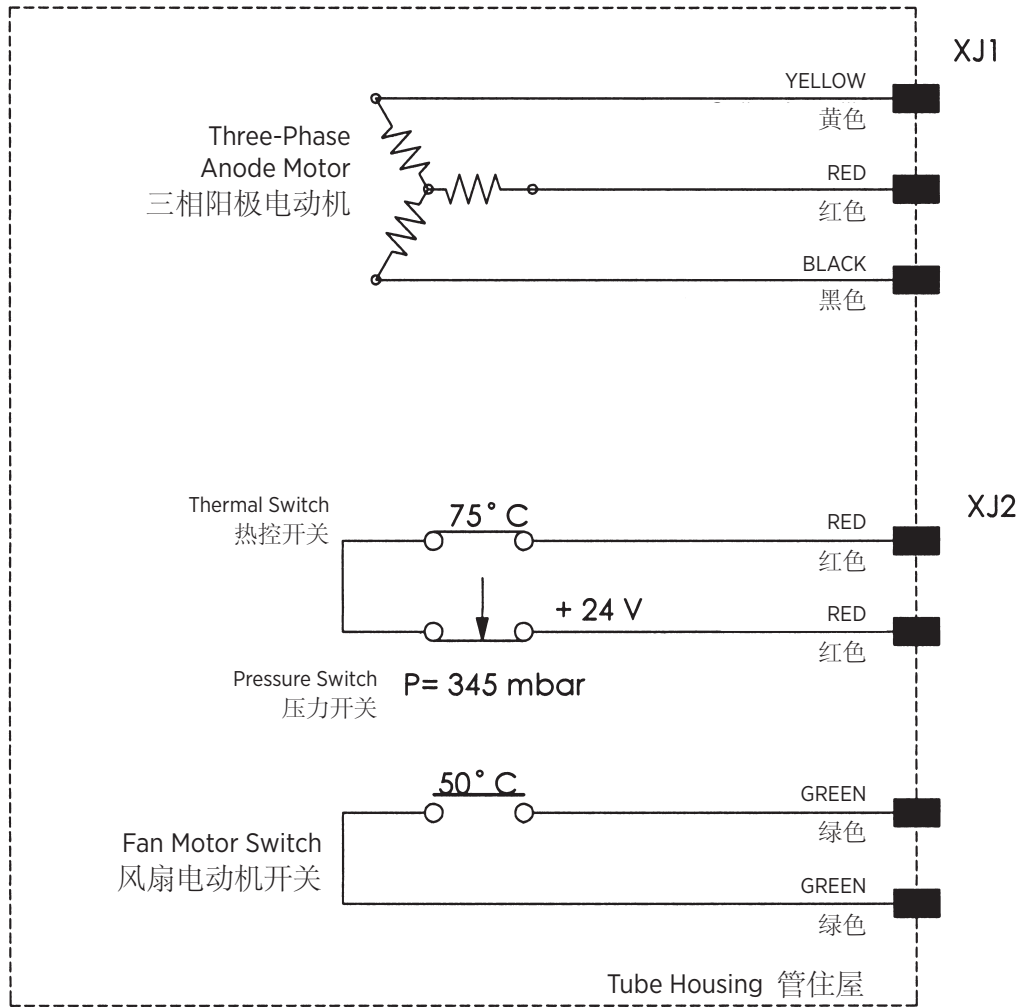	Stator Drive Frequency 100 Hz = 5600 - 6000 RPM		Stator Drive Frequency 180 Hz = 9600 - 10,800 RPM	
	Power (W)	Time (s)	Power (W)	Time (s)
<b>Start</b>	1200	0.8	900	1.5
<b>Continuous operation</b>	110	-	70	-
<b>Brake</b>	100	5	300	2.5

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### Anode Heating and Cooling Curves 阳极加热与冷却曲线