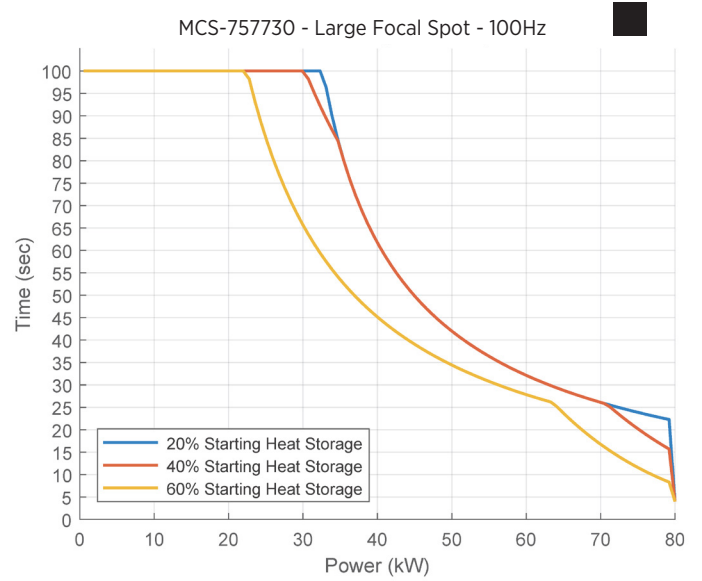
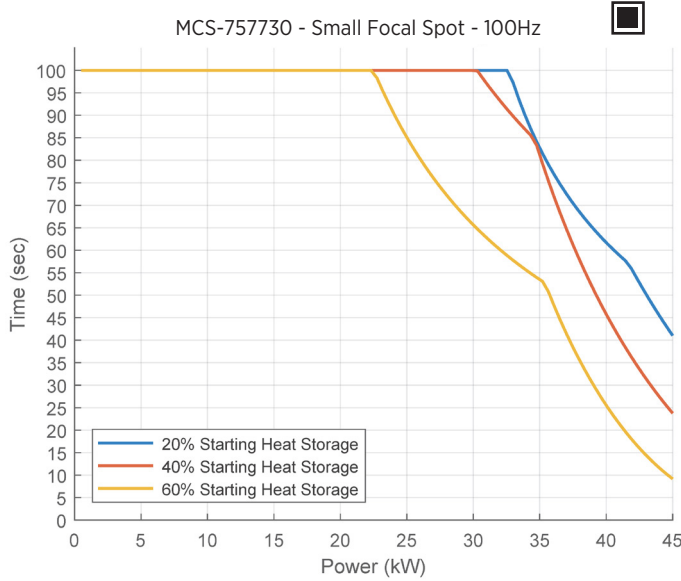
Rotating Anode X-Ray Tube
Tubes Radiogènes à Anode Tournante
Drehanoden - Röntgenröhre
Tubos de Rayos-X con Ánodo Giratorio



Note: Document originally drafted in the English language.

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3 Ø 100 Hz

3 Ø 130 Hz

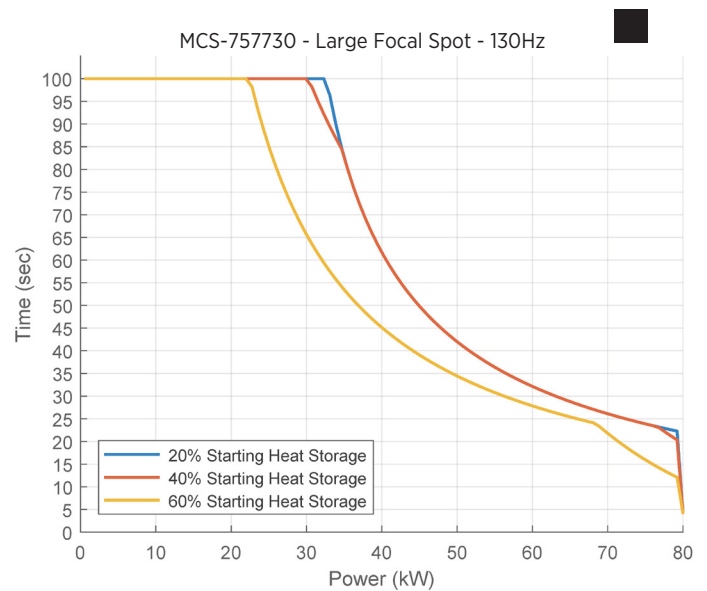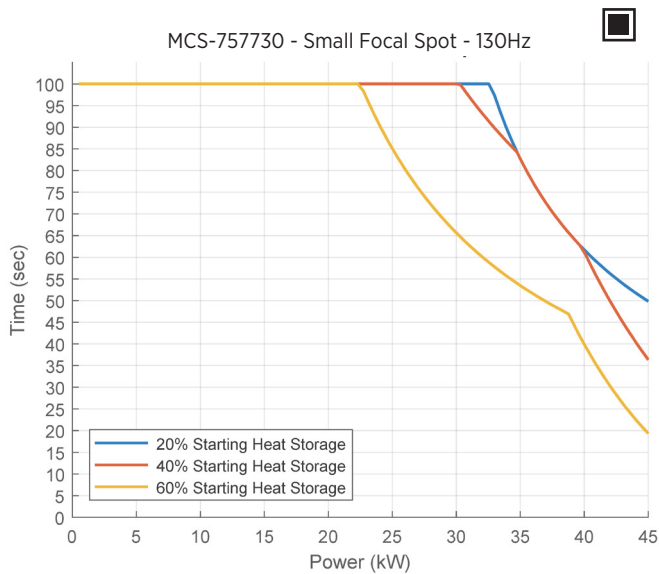Note:
 Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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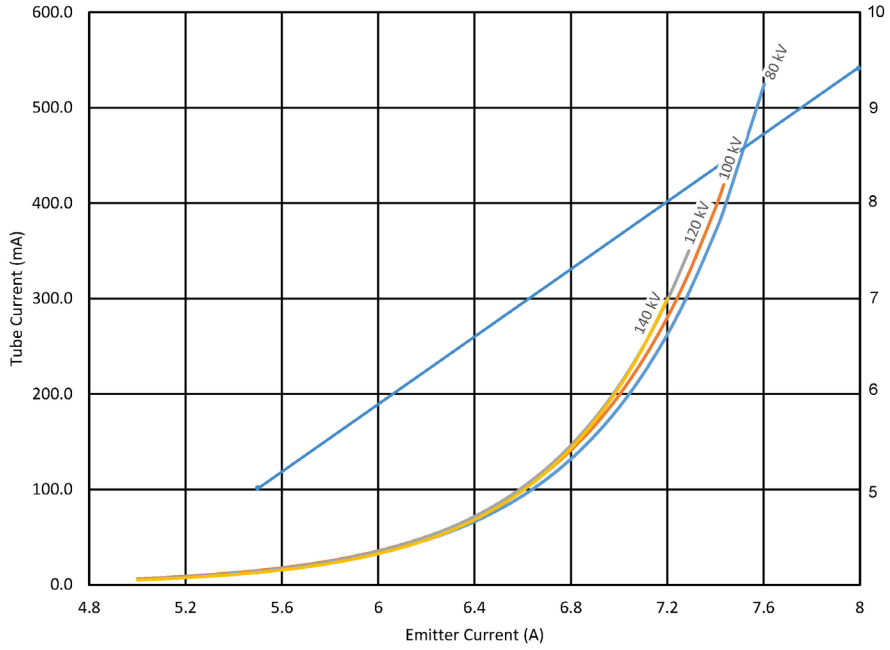
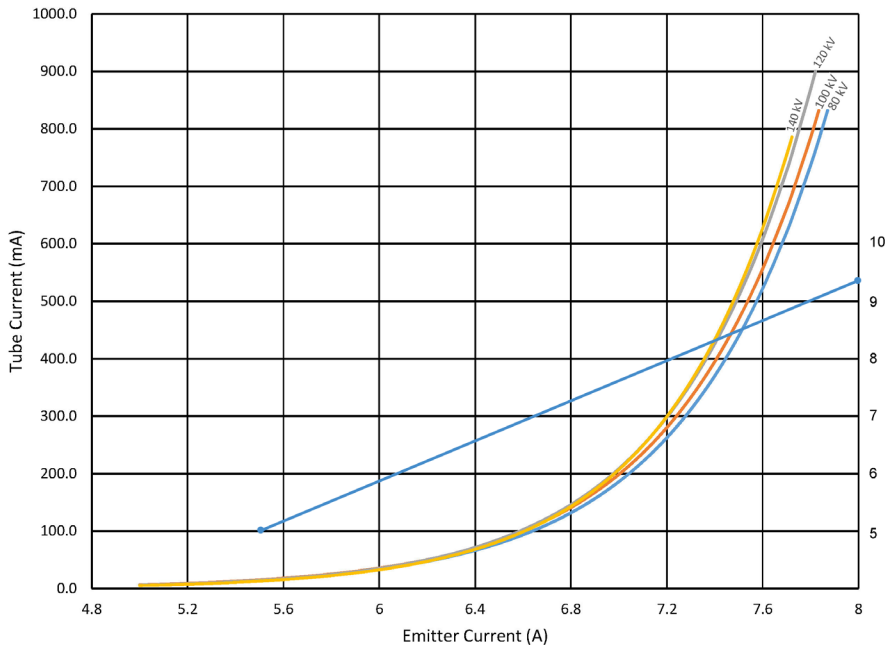
La Gaine B-903H

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Encaje de B-903H

Maximum Peak Voltage	140 kV
Anode to Ground	0 kV
Cathode to Ground	140 kV
Maximum X-ray Tube Assembly Heat Content	8.3 MJ (11.7 MHU)
Nominal Continuous Input Power (Includes stator heat)	4.0 kW (5.6 kHU/sec) IEC 60613:2010
Maximum Housing Temperature	78°C
Maximum Heat Exchanger Dissipation (at 86 kPa and inlet temperature difference of 31°C)	5.0 kW
Focal Point Position (Central Ray) Within 1mm (X, Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent filtration	1.0 mm Al IEC 60522/1999
Loading Factors for Leakage Radiation	140 kV, 29 mA
High Voltage Cable	Special
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity (non-condensing)	10% - 90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight - Housing with High Voltage Cable	64.7 kg (143 lbs)
Heat Exchanger Weight	22.8 kg (50 lbs)
TCU Weight	6.8 kg (15.0 lbs)
IEC Classification	Class 1
Safety Devices	
Housing	
Thermal Switch - Normally Closed Contacts	Opens at 85°C
Pressure Relief Valve	Opens at 42 PSI
Heat Exchanger	
Flow Switch - Normally Open Contacts, closes with adequate fluid flow	
Filament Frequency Limits	50 HZ - 45 kHz
Power Supply	DC

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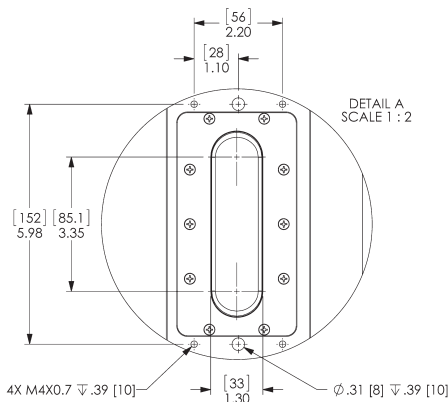
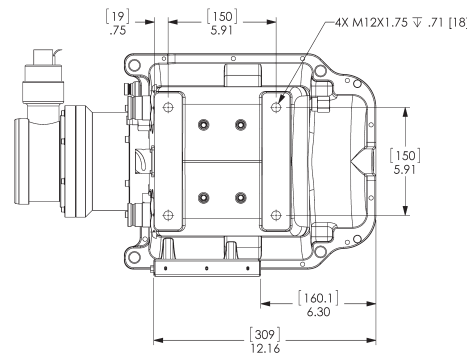
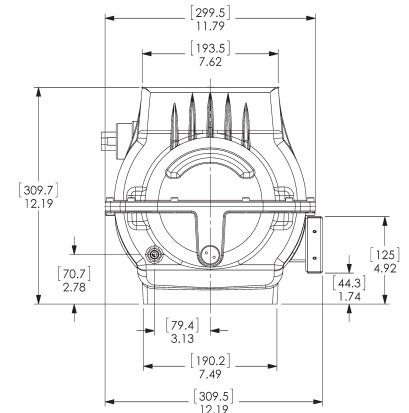
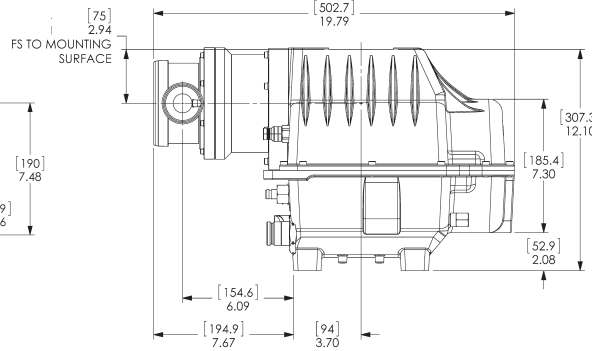
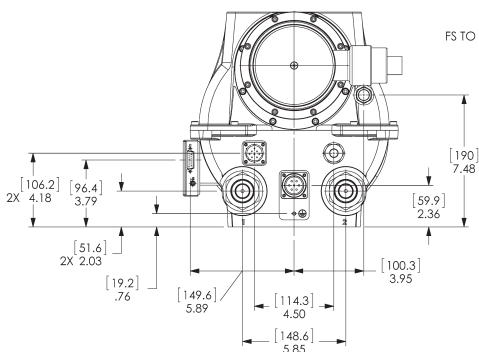
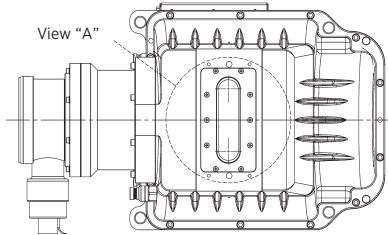
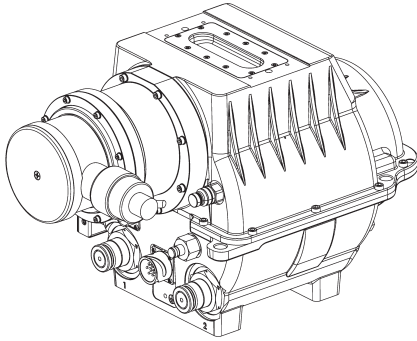
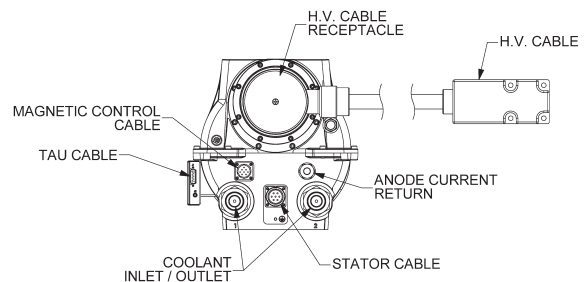
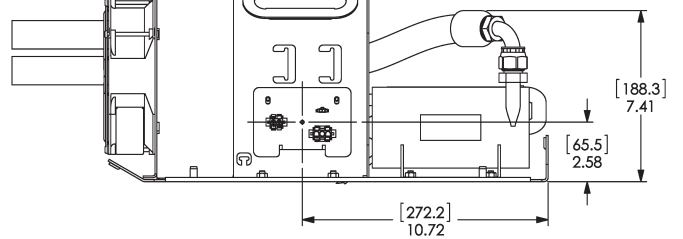
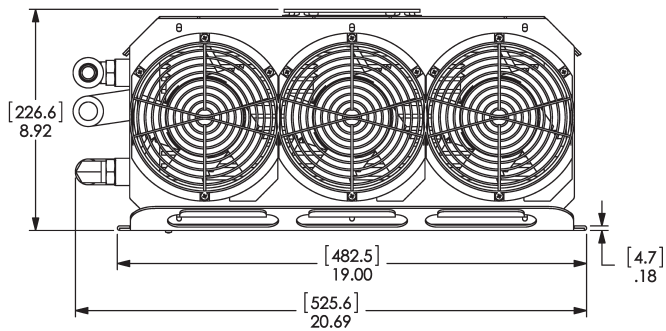
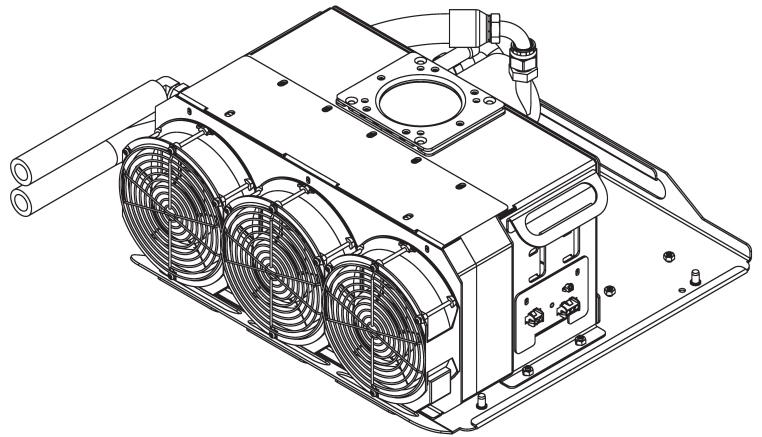
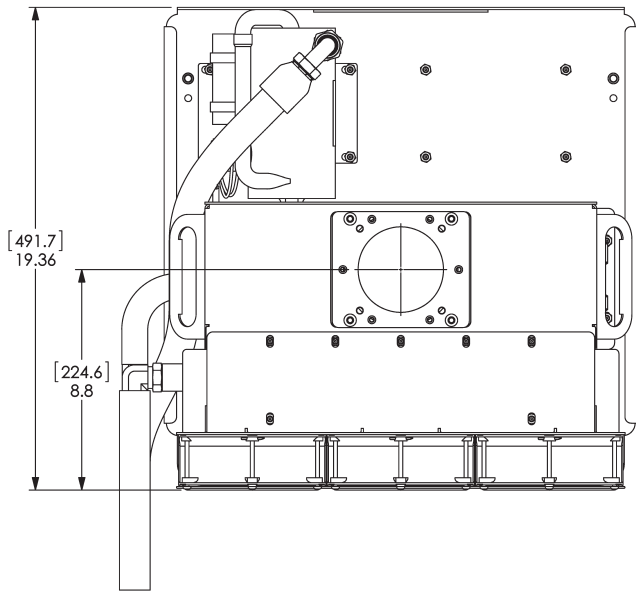
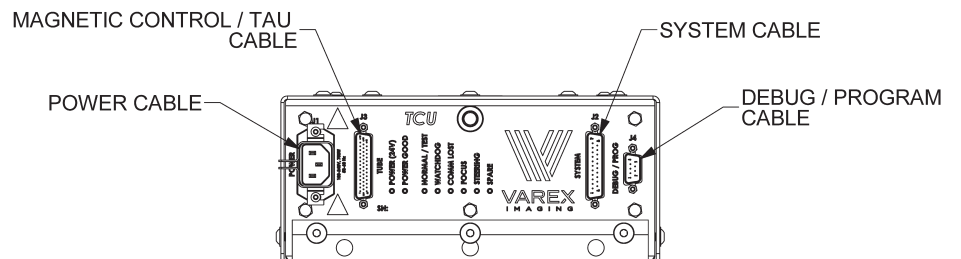
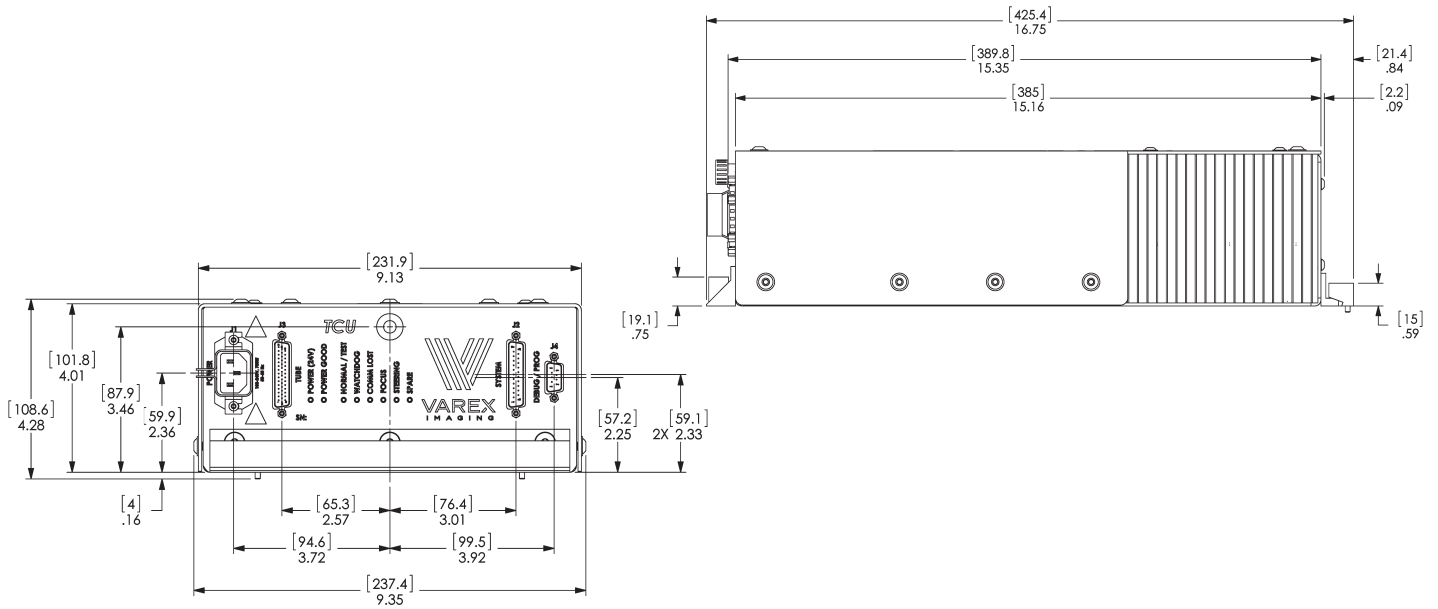
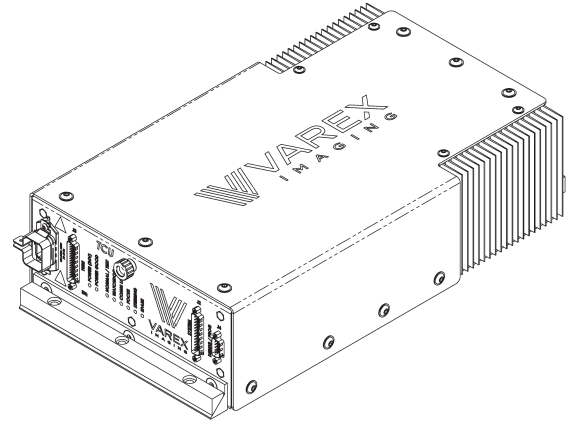
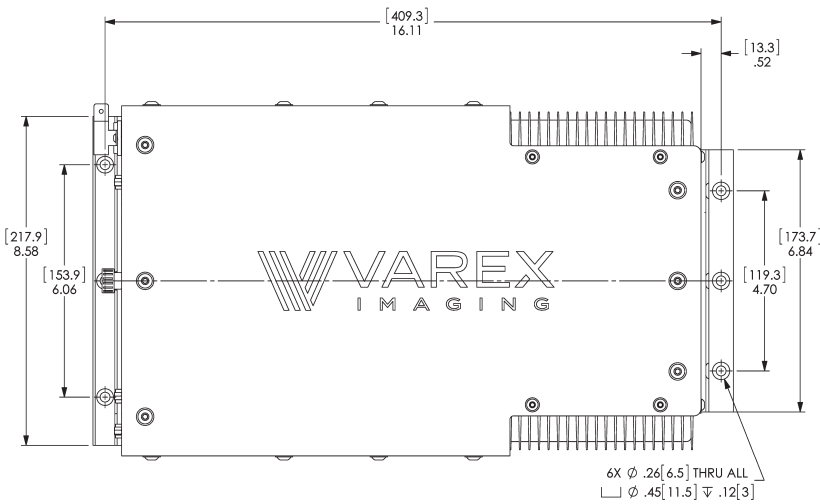
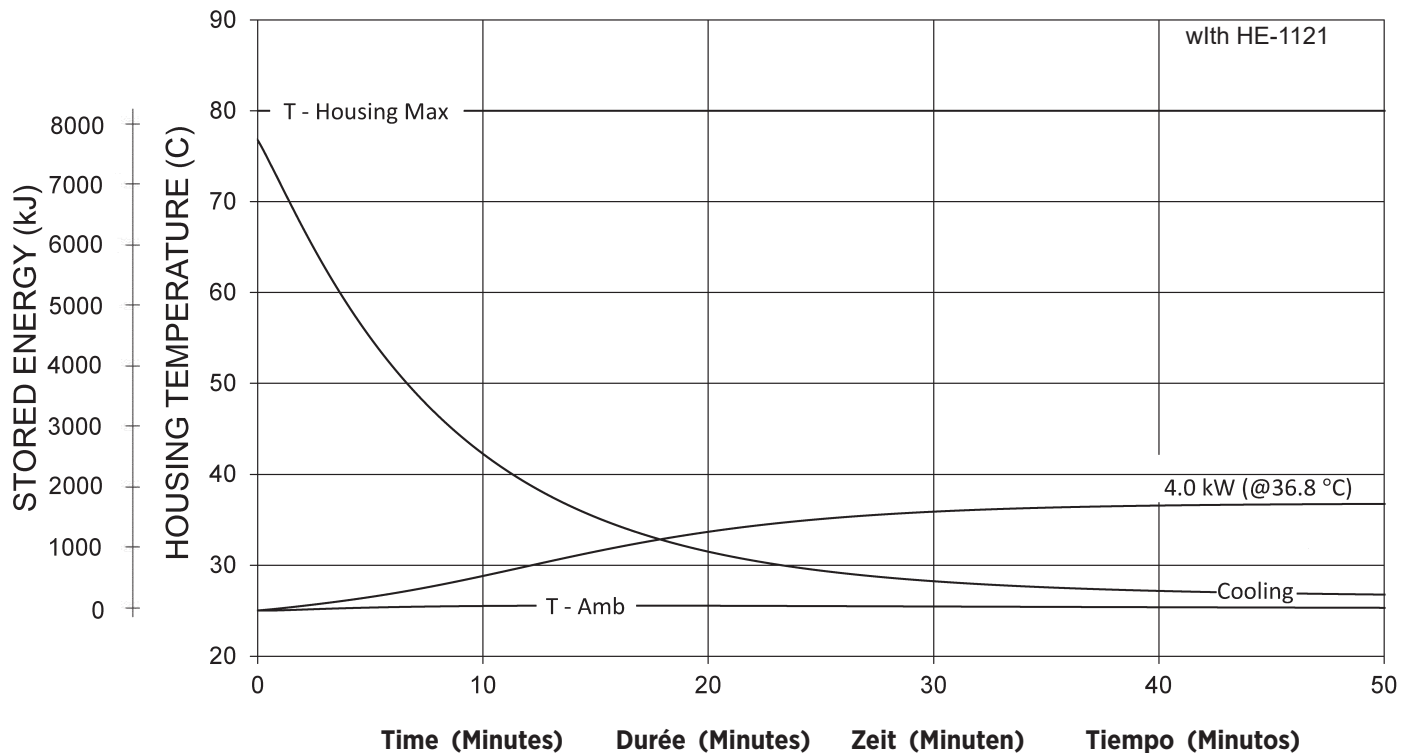
TUBE HOUSING ASSEMBLY HEATING AND COOLING

Note:

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.
- Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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
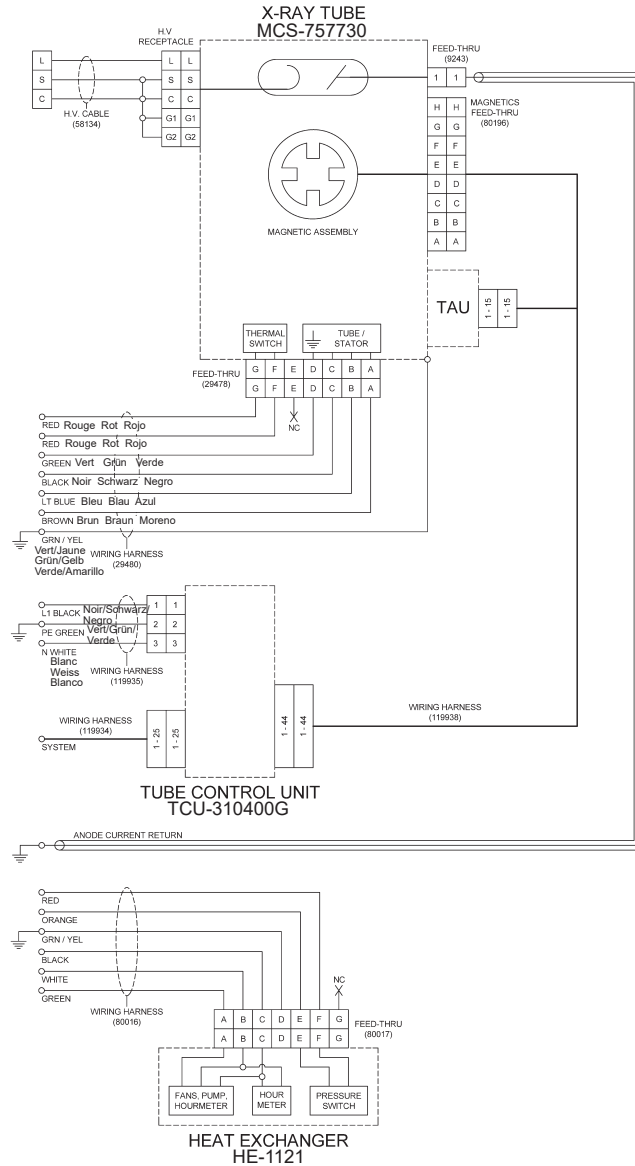
Stator Type: "3 Ø"		
Stator Coil Resistance: 2.2 Ohms ± 15%		
Starter Voltage:	Start	Run
95 Hz	360-440 VAC	110 VAC
130 Hz	360-440 VAC	170 VAC
Time to Full Speed:		
0 to 125 Hz		10 Sec.
95 to 125 Hz		10 Sec.
X-Ray Tube Assembly:		
MCS-757730/B-903H		IEC 60601-2-28

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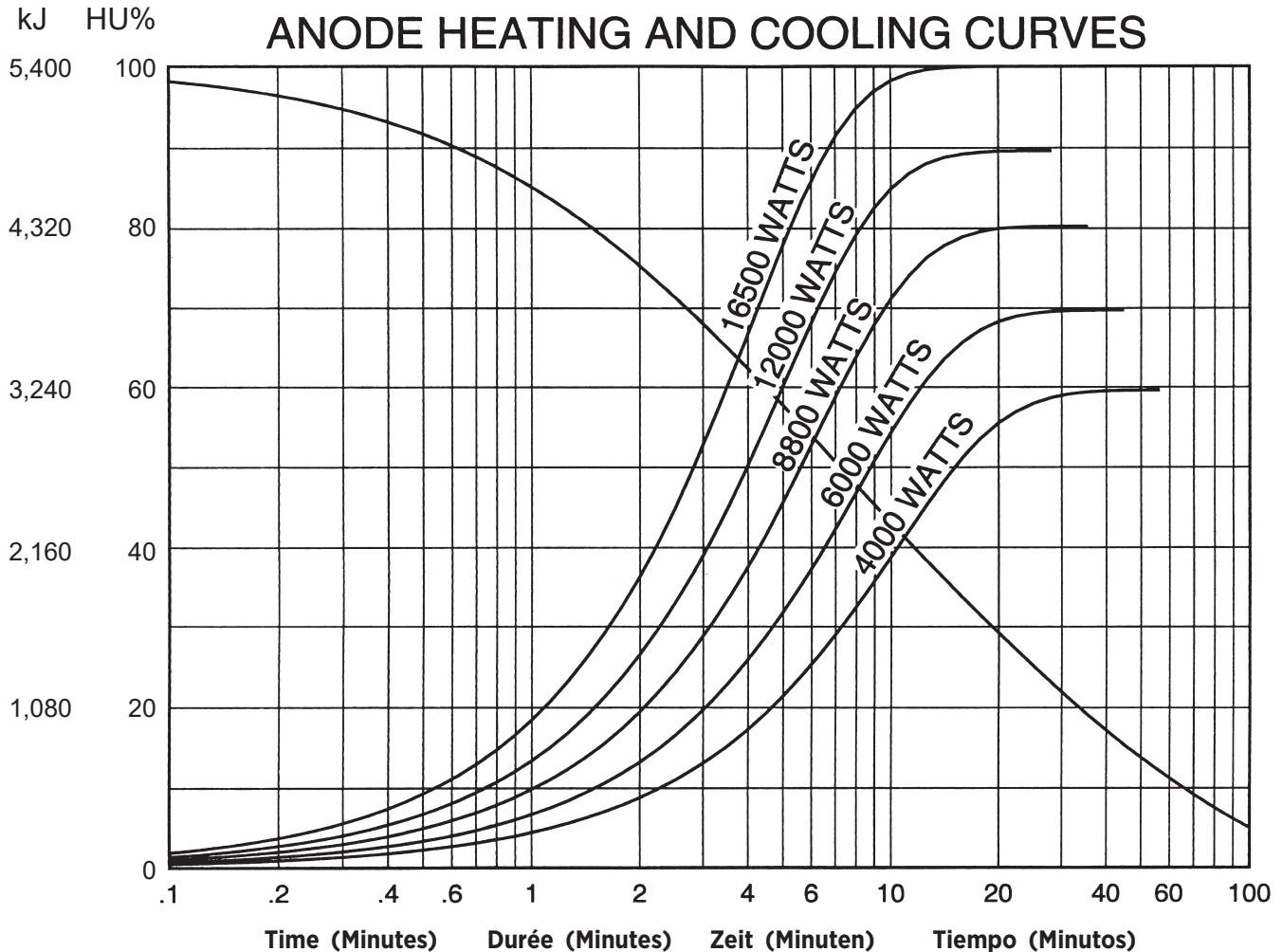Note:
1. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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