Rotating Anode X-Ray Tube
 Tubes Radiogénés à Anode Tournante
 Drehanoden - Röntgenröhre
 Tubos de Rayos-X con Ánodo Giratorio

Note: Document originally drafted in the English language.

Product Description

The MCS-757730 is a 8.27" (210 mm) 140 kV, 5.4 MJ (7.5 MHU) maximum anode heat content, rotating anode insert. This insert is specifically designed for CT Scanners, with capability of X-axis and/or Z-axis steering and adjustable focal spot sizing. This design needs to be coupled with Varex Imaging's Tube Control Unit (TCU) to control focal spot steering and focal spot size. The insert features a 7° tungsten-rhenium facing on molybdenum with a graphite backed target and is available with the following nominal focal spot values:

Small - 0.6 x 0.9
 Large - 1.3 x 1.5
 IEC 60336

Loading Factor for slit focal:

Small - 120 kV, 200 mA
 Large - 120 kV, 200 mA

Maximum Anode Cooling Rate:

16,500 W (23,100 HU/sec)

Maximum continuous anode heat dissipation:

12,000 W (16,800 HU/sec)

Nominal CT Anode Input Power:

Small - 45 kW IEC 60613:2010
 Large - 80 kW IEC 60613:2010

Nominal CT Scan Power Index:

Small - 45 kW IEC 60613:2010
 Large - 78 kW IEC 60613:2010

Reference Axis:

Perpendicular to port face.

This insert is intended for use in Varex Imaging B-903H housing.

The MCS-757730 utilizes dielectric oil in the housing & HE. This tube needs to be coupled with Varex Imaging's TCU-310400G (Generic).

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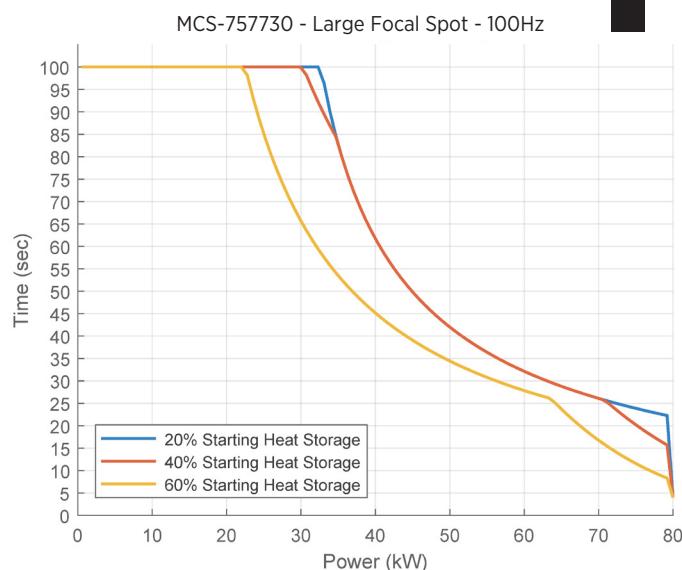
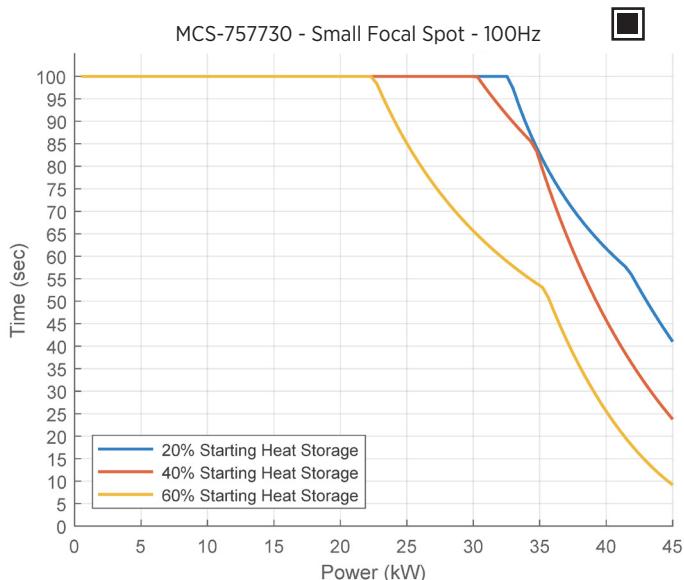
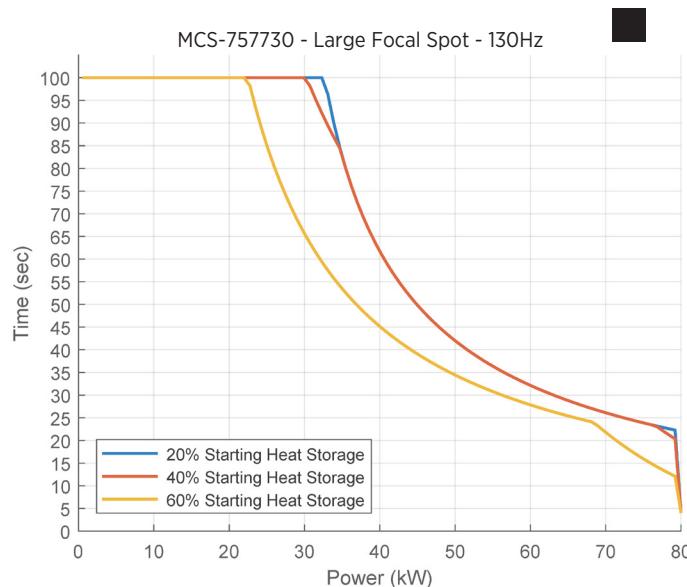
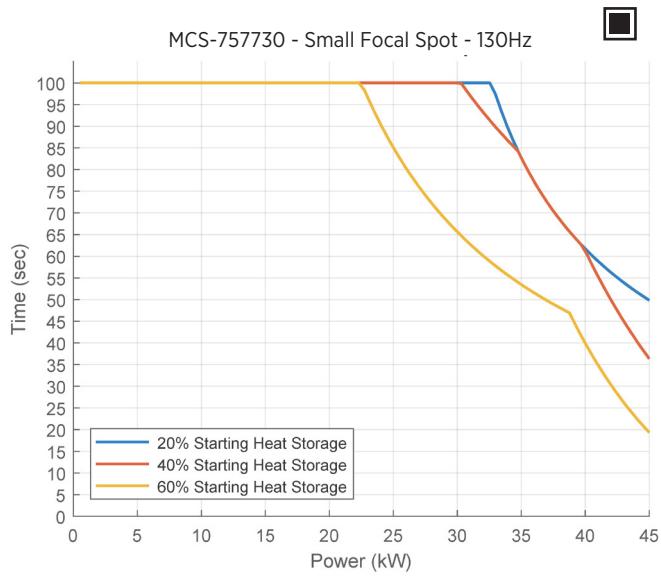
3 Ø 100 Hz

3 Ø 130 Hz

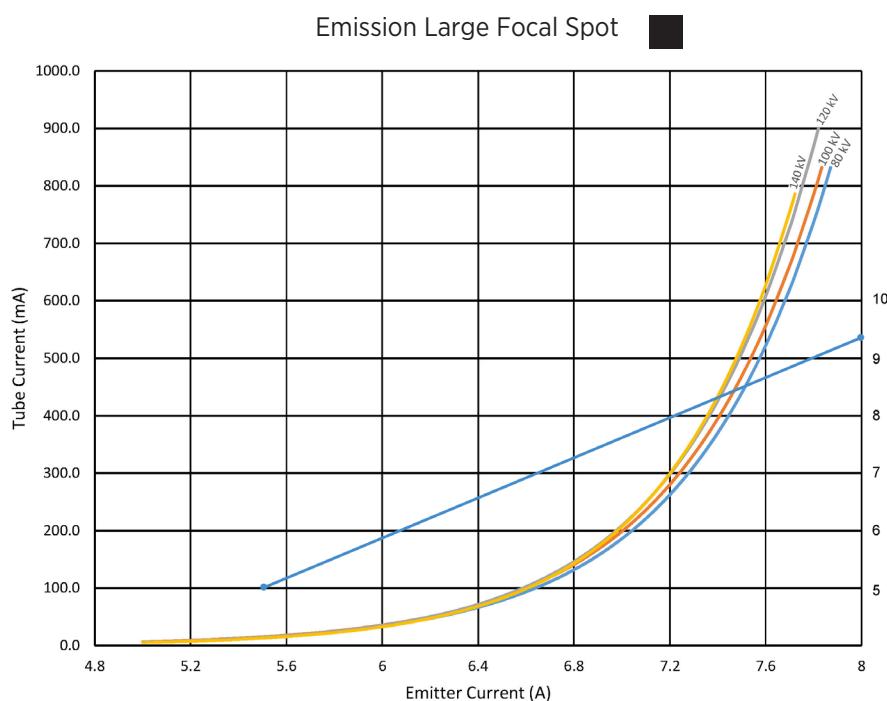
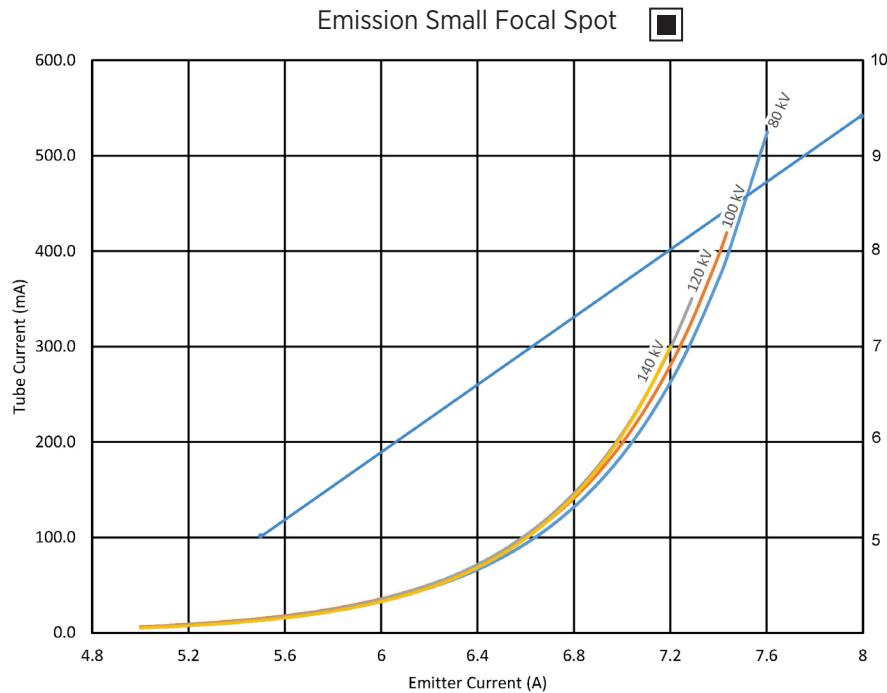
Note:
 Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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La Gaine B-903H

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Encaje de B-903H


Maximum Peak Voltage	140 kV
Anode to Ground	0 kV
Cathode to Ground	140 kV
Maximum X-ray Tube Assembly Heat Content	8.3 MJ (11.7 MHU)
Nominal Continuous Input Power (Includes stator heat)	4.0 kW (5.6 kHU/sec) IEC 60613:2010
Maximum Housing Temperature	78°C
Maximum Heat Exchanger Dissipation (at 86 kPa and inlet temperature difference of 31°C)	5.0 kW
Focal Point Position (Central Ray) Within 1mm (X, Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent filtration	1.0 mm Al IEC 60522/1999
Loading Factors for Leakage Radiation	140 kV, 29 mA
High Voltage Cable	Special
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to + 75°C
Humidity (non-condensing)	10% - 90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight - Housing with High Voltage Cable	64.7 kg (143 lbs)
Heat Exchanger Weight	22.8 kg (50 lbs)
TCU Weight	6.8 kg (15.0 lbs)
IEC Classification	Class 1
Safety Devices	
Housing	
Thermal Switch - Normally Closed Contacts	Opens at 85°C
Pressure Relief Valve	Opens at 42 PSI
Heat Exchanger	
Flow Switch - Normally Open Contacts, closes with adequate fluid flow	
Filament Frequency Limits	50 HZ - 45 kHz
Power Supply	DC

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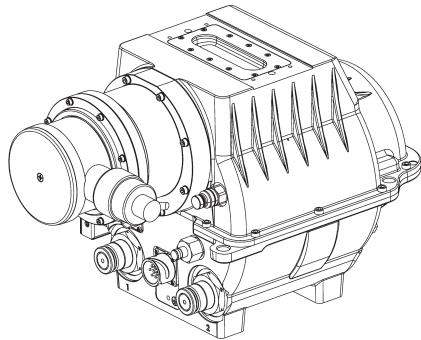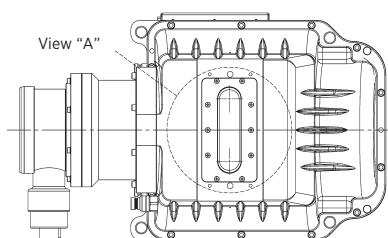
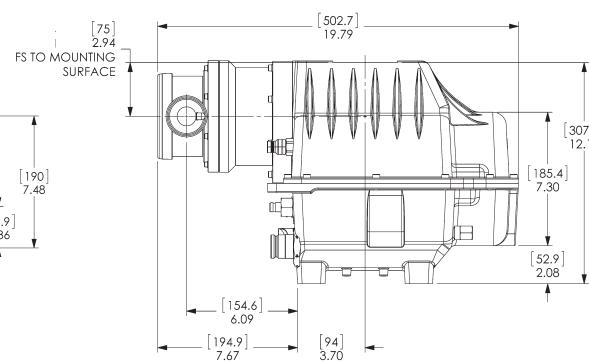
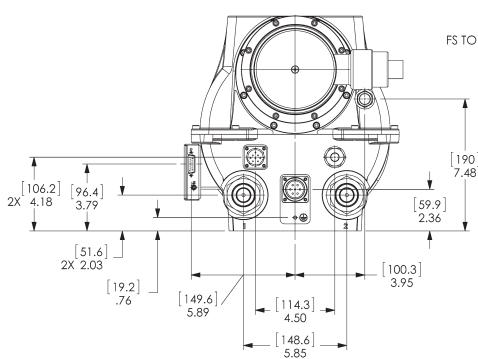
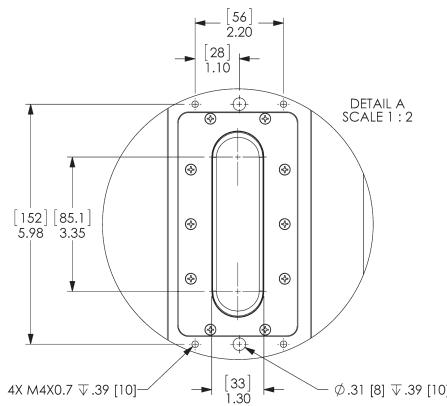
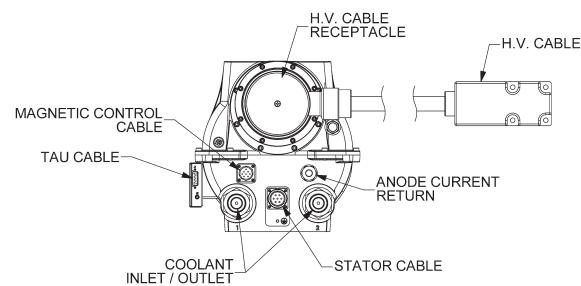
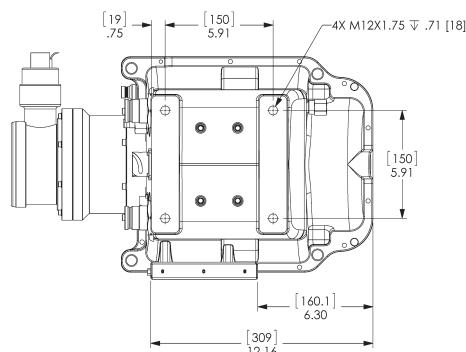
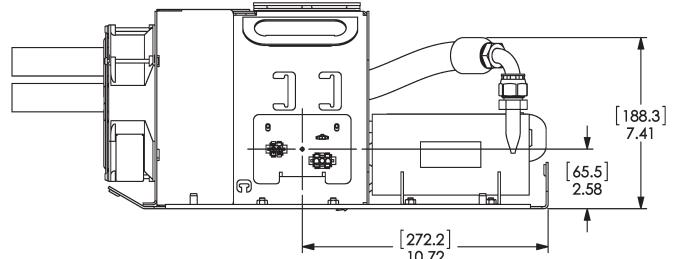
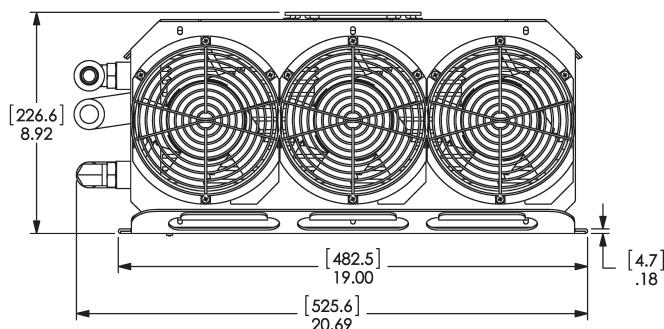
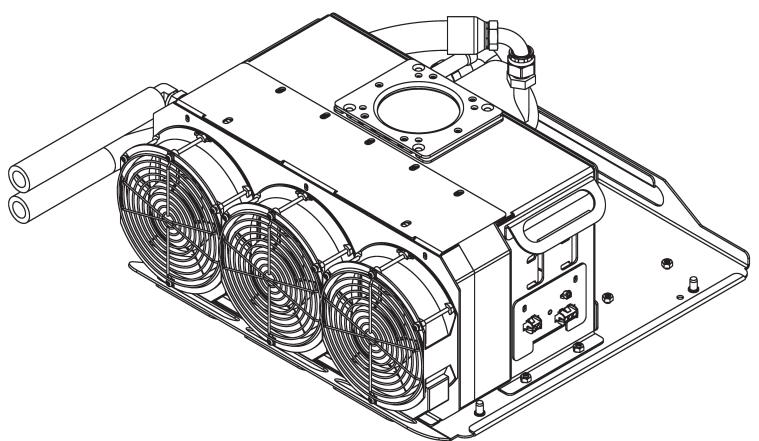
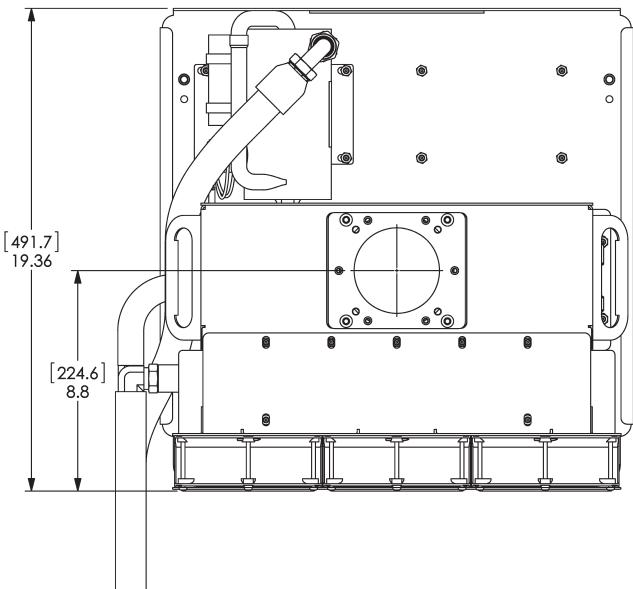
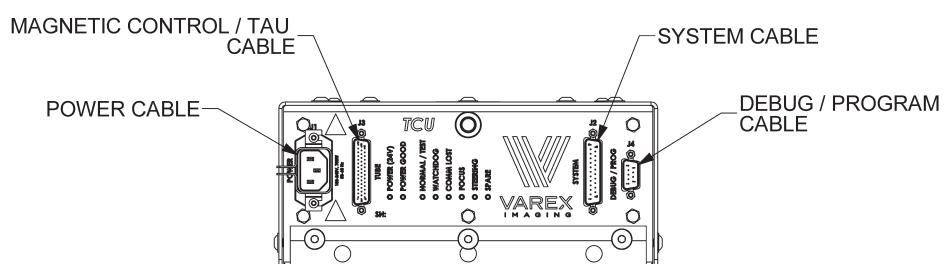
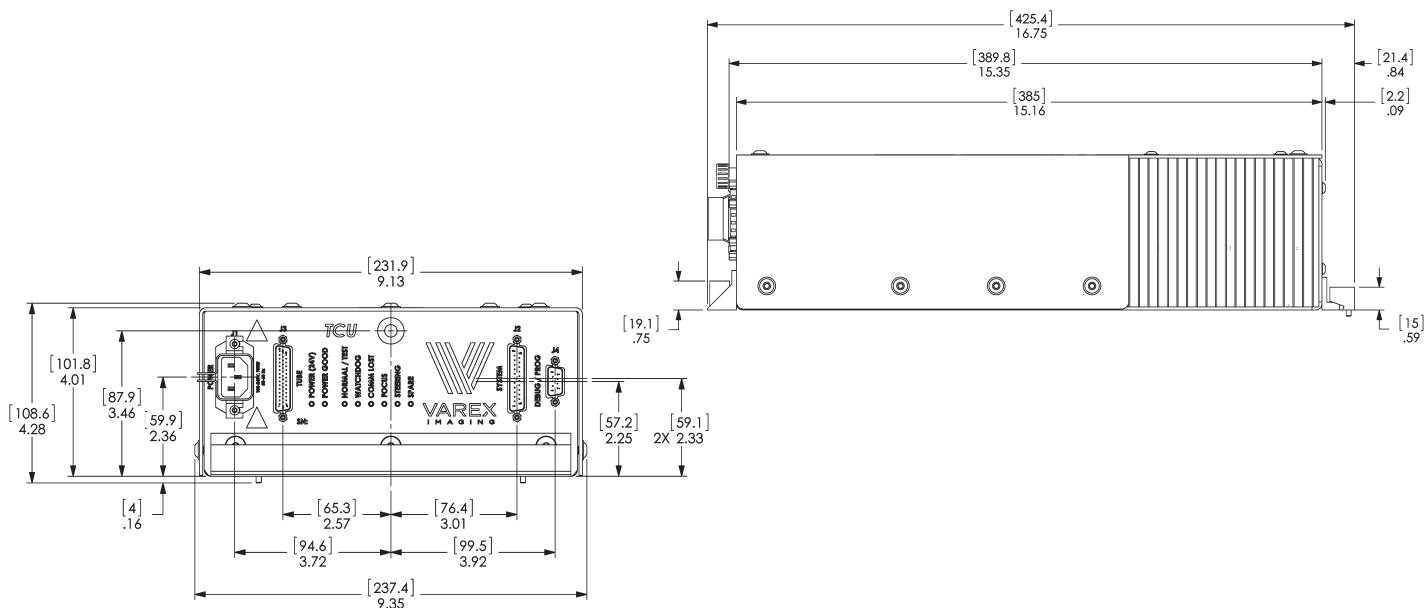
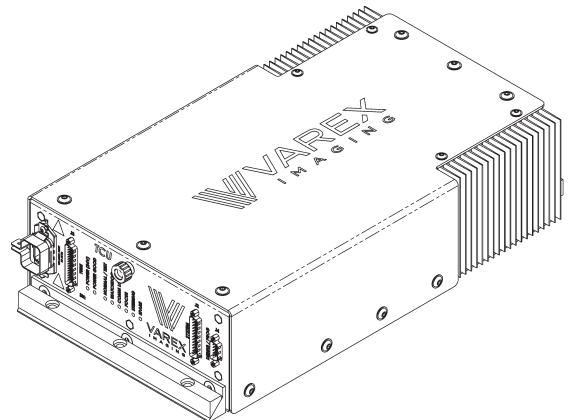
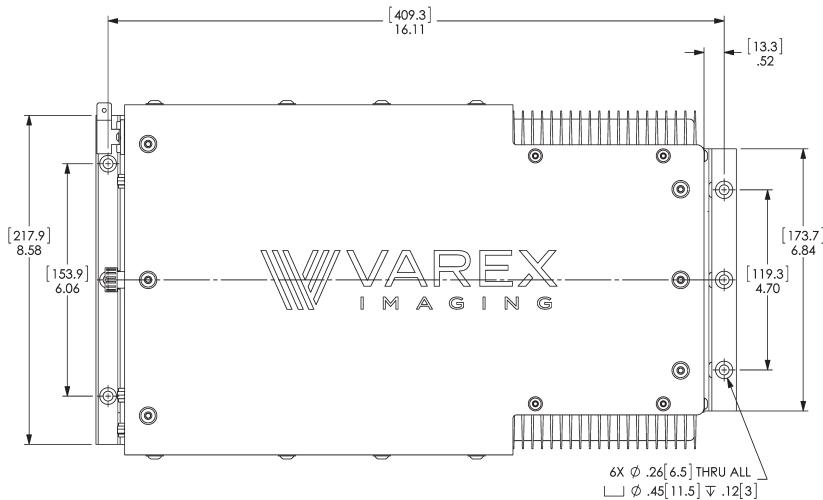
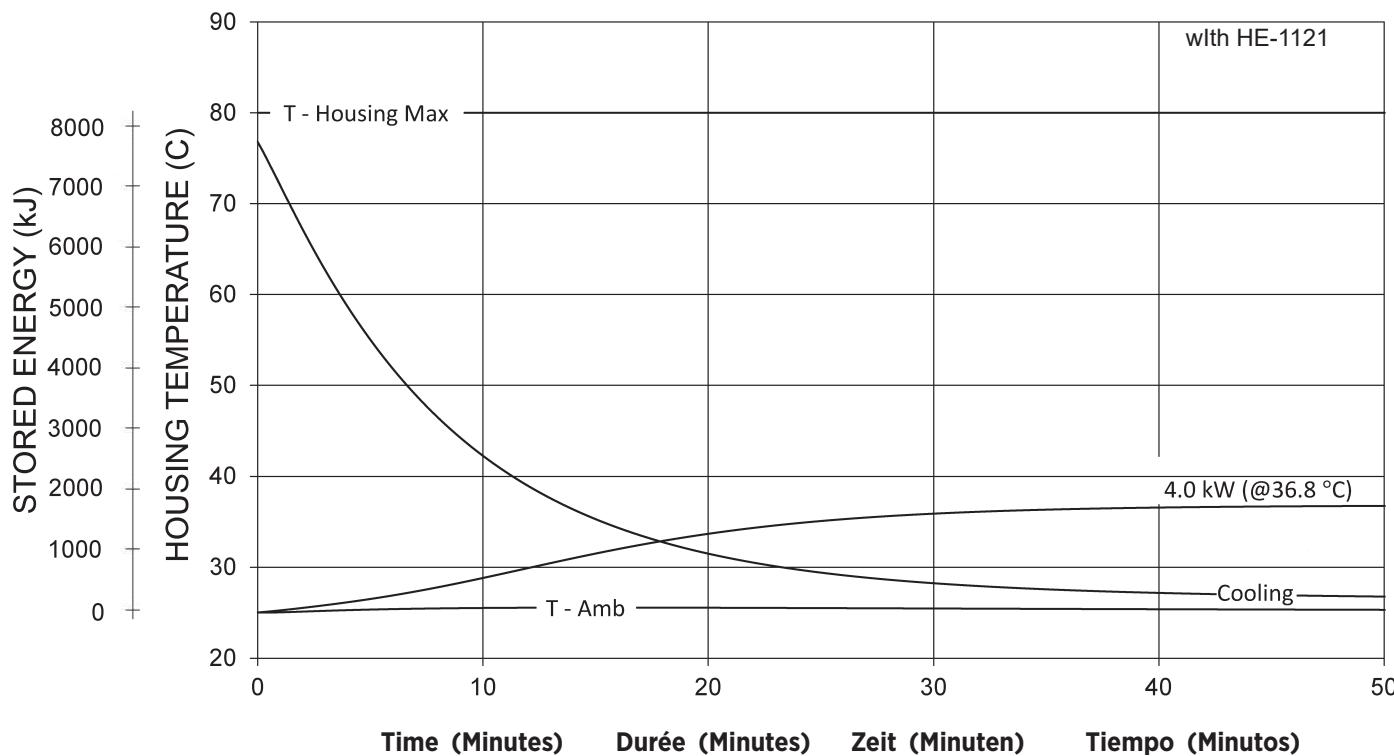
TUBE HOUSING ASSEMBLY HEATING AND COOLING

Note:
1. Heat inputs into housing include tube power, filament power, and stator power.
2. Heating curves based on no restrictions of natural convection around tube housing assembly.
3. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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