

Rotating Anode X-Ray Tube
 Tubes Radiogénés à Anode Tournante
 Röntgenröhre mit rotierender Anode
 Tubos de Rayos-X con Ánodo Giratorio

Large - Black
 Grand - Noir
 Gross - Schwarz
 Largo - Negro


Small - White
 Petit - Blanc
 Klein - Weiss
 Pequeño - Blanco

Stand - By
 Attente
 Bereit Stehen
 En Espera

Frame or Chassis
 Masse
 Chassis
 Soporte o Chasis

X-Ray Tube
 Tube Radiogène
 Röntgenröhre
 Tubo de Rayos X

Radiation Filter or Filtration
 Filtre de rayonnement
 Filtering
 Filtración de Radiación

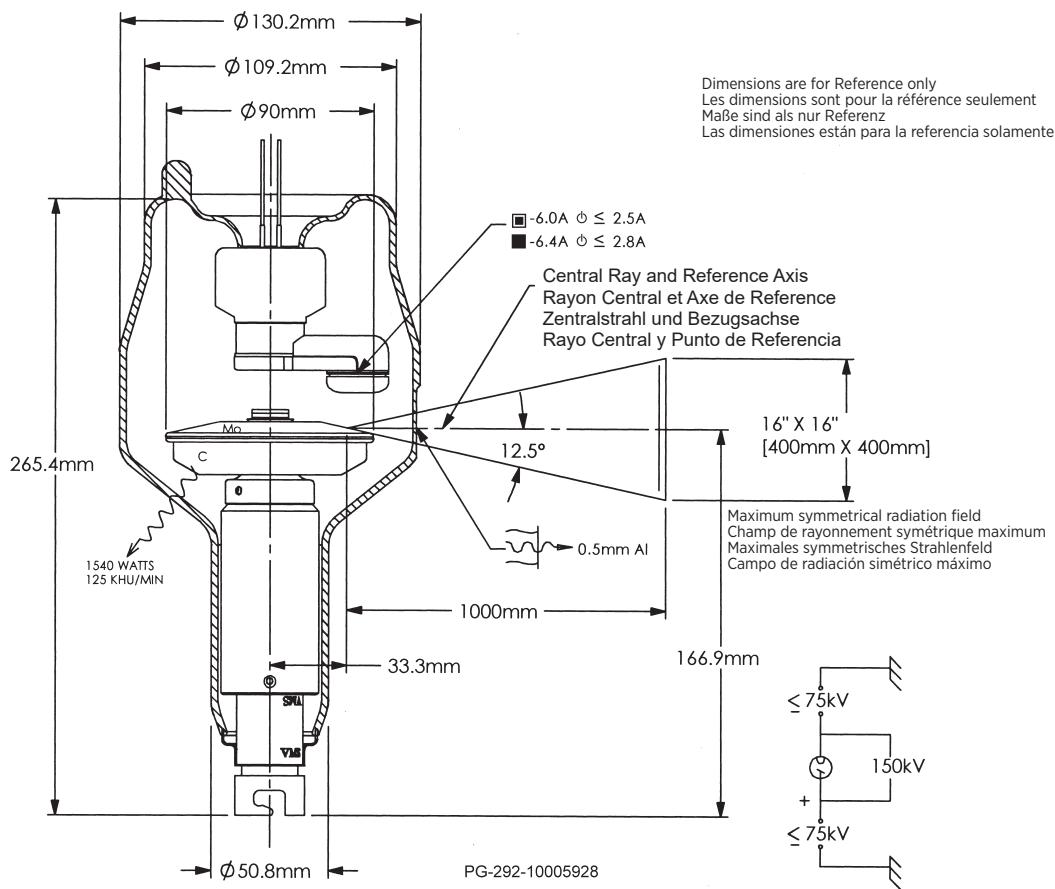

Note: Document originally drafted in the English language.

Product Description

The PG-292 is a 3.5" (90 mm) 150 kV, 370 kJ (500 kHU) maximum anode heat content, rotating anode insert. This insert is specifically designed for general radiography, cineradiography, digital and film screen angiography procedures. The insert features a 12.5° rhodium-tungsten facing on molybdenum with a graphite backed target and is available with the following nominal focal spots:

0.6 - 1.2
 IEC 60336

Nominal Anode Input Power

Small - 36 kW IEC 60613
 Large - 85 kW IEC 60613

For the equivalent anode input power of 175 Watts

Description du Produit

Le tube PG-292, à anode tournante de 90 mm, (3,5 pouces), 150 kV, avec une capacité calorifique maximale de 370 kJ (500 kUC) spécialement conçue pour les procédures radiographie générale, cinéradiographie et angiographie numériques et sur film. L'tube est pourvu d'une anode avec pente de 12.5° en rhénium - tungstène sur une base de molybdène et avec un doublage ge graphite. Il est disponible avec les foyers suivantes:

0.6 - 1.2
 CEI 60336

Puissance anodique nominale de l'anode

Petit foyer - 36 kW CEI 60613
 Grand foyer - 85 kW CEI 60613

Pour la puissance anodique d'équilibre thermique de 175 Watts

Produktbeschreibung

Die PG-292 ist eine 3.5" (90 mm) Doppelfokus Drehanoden-Röntgenröhre, mit einer Anoden Wärmespeicherkapazität von 370 kJ (500 kHU) und einer max. Spannungsfestigkeit von 150 kV. Die Röhre ist spezielle für den Radiographie-, Röntgenkinematographie-, digitale und Filmangiographieverfahren entwickelt. Der rückseitig graphitbeschichtete Rhodium-Wolfram- und Molybdän Anodensteller besitzt einen Winkel von 12.5°. Folgende Brennfleckkombination ist lieferbar:

0.6 - 1.2
 IEC 60336

Nominale Anodenbezugsleistung

Klein - 36 kW IEC 60613
 Gross - 85 kW IEC 60613

Gilt bei einer Äquivalent - Anodenleistung von 175 Watt

Descripción del Producto

El PG-292 es un tubo de ánodo giratorio de 90 mm (3.5"), 150 kV, 370 kJ (500 kHU) diseñado específicamente para procedimientos generales de radiografía, cineradiografía, digital, y angiografía con película de pantalla. El blanco emisor es una combinación de renio, tungsteno y molibdeno con grafito en la parte posterior con un rayo central de 12.5 grados. Disponible con las siguientes combinaciones de marcas focales:

0.6 - 1.2
 IEC 60336

Potencia nominal de entrada del anodo

Foco fine - 36 kW IEC 60613
 Foco grueso - 85 kW IEC 60613



Para una potencia equivalente del anodo de 175 Watts





3 Ø Constant Potential ---

Single Load Ratings IEC 60613
Abaques de Charge pour Pose Unique CEI 60613
Brennfleck - Belastungskurven IEC 60613
Diagramas de Exposición Radiográfica IEC 60613

50 Hz

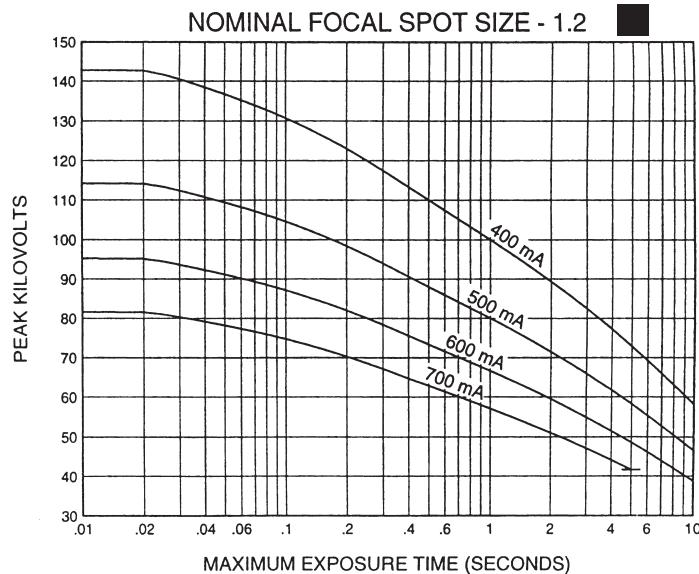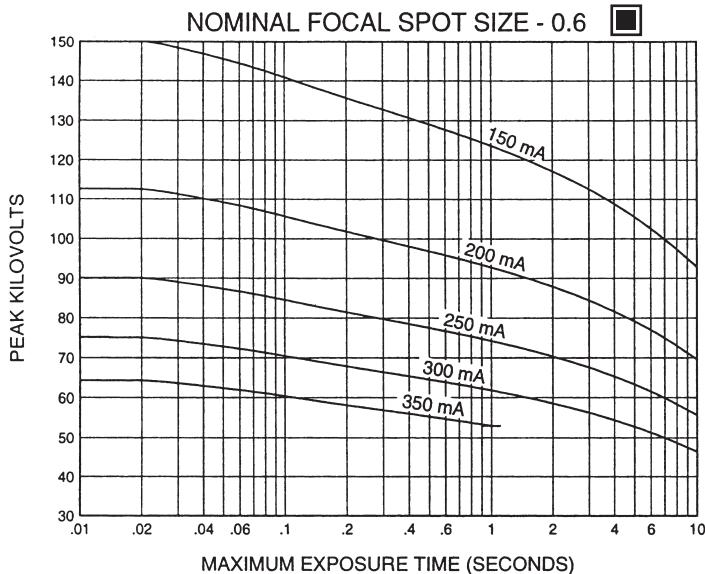

60 Hz

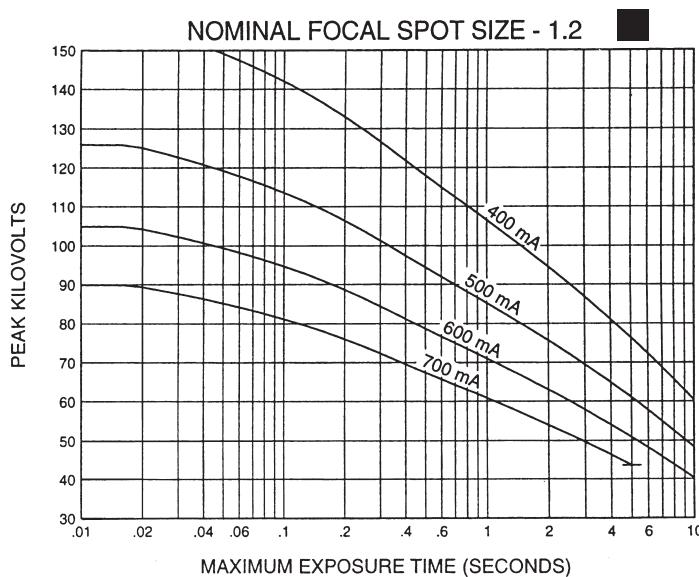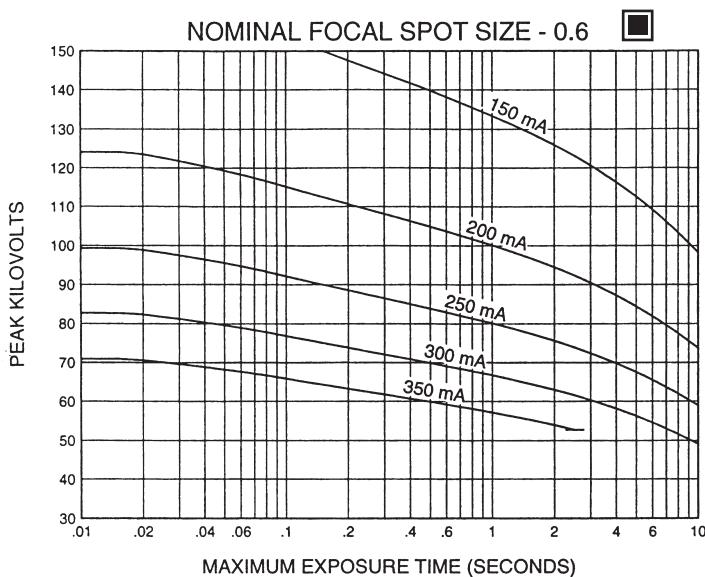

Nominal anode input power for the anode heat content 40%. IEC 60613

Puissance calorifique nominale de l'anode: 40%, CEI 60613

Thermische Anodenbezugsleistung bei einer Wärmespeicherung von 40%. IEC 60613



Aproximadamente el poder de penetración para obtener un almacenaje de calor del anodo de 40%. IEC 60613





3 Ø Constant Potential ---

Single Load Ratings IEC 60613
Abaques de Charge pour Pose Unique CEI 60613
Brennfleck - Belastungskurven IEC 60613
Diagramas de Exposición Radiográfica IEC 60613

150 Hz

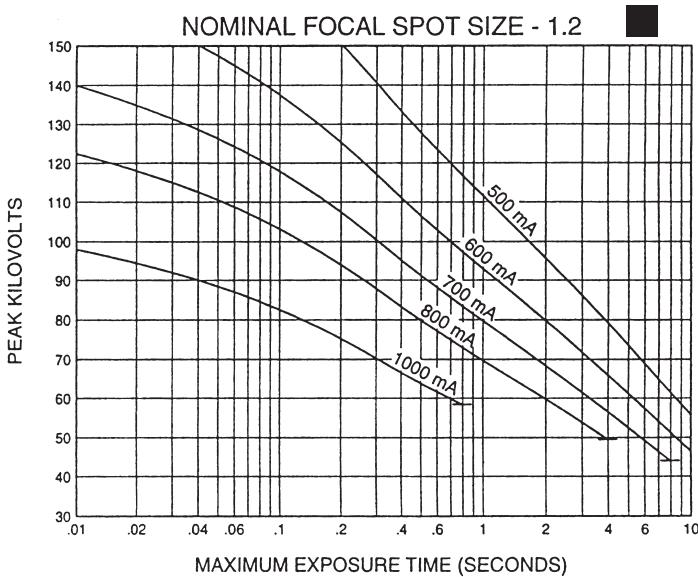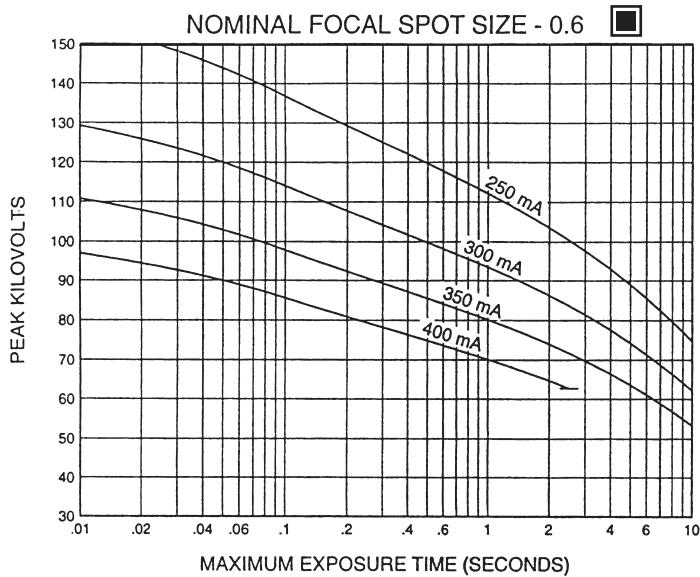

180 Hz

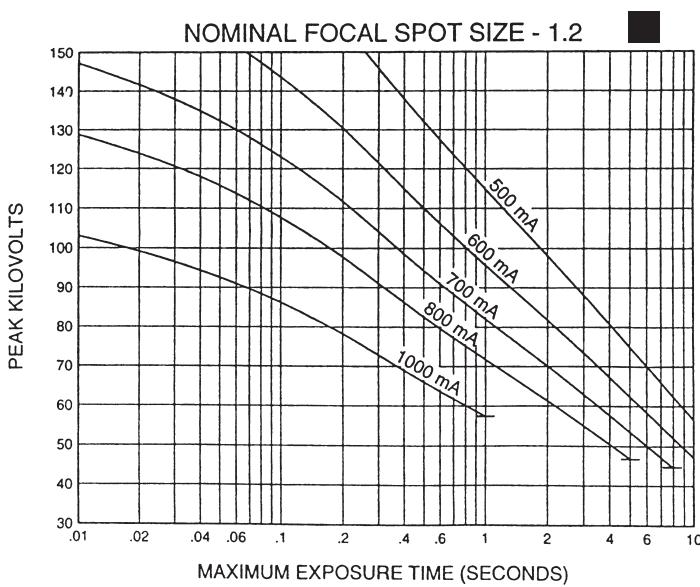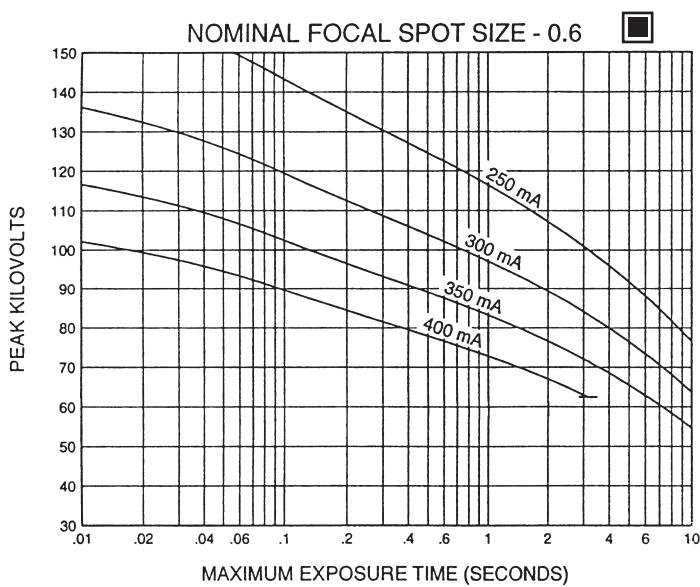

Nominal anode input power for the anode heat content 40%. IEC 60613

Puissance calorifique nominale de l'anode: 40%, CEI 60613

Thermische Anodenbezugsleistung bei einer Wärmespeicherung von 40%. IEC 60613

Aproximadamente el poder de penetración para obtener un almacenaje de calor del anodo de 40%. IEC 60613

CINERADIOGRAPHIC RATINGS

HOW TO USE CINERADIOGRAPHIC CHARTS

General: With the Cineradiographic rating chart we can determine the maximum allowable kW of the Cine pulse, or with a given kW determine maximum time in seconds the Cine run can progress.

The Most common way of using the charts is to determine maximum time of any expected Cine run and maximum duty factor. With a known duty factor and Cine run time kW can easily be determined.

Definition of Terms

Time in seconds: Total time of one Cine run, usually 5 to 12 seconds.

Duty Factor in Percent (DF%): Actual time during one second the x-ray tube is producing x-rays. If we select a 4 msec pulse width and 60 exposures per second the x-ray tube will be producing x-rays for a total of 240 msec each second or 24% of the time. The higher the DF number, the more load placed on the x-ray tube.

Peak Pulse Power: Peak energy in watts of any one Cine Pulse. Can be any combination of kV and mA allowed by Radiographic and Filament Emission curves.

Example: 80 kV at 400 mA equals

$$\frac{80,000 \text{ V} \times 400 \text{ mA}}{1000} = 32,000 \text{ W or } 32 \text{ kW}$$

USING THE CINE RATING CHARTS:

PG-292 150/180 Hz 3 Phase 1.2 Focal Spot

Example: Determine maximum kW allowed with the following known factors:

Maximum Pulse Width 5 msec
Exposures per Second 50
Maximum Cine Run Time10 seconds

Calculate Duty Factor: (DF%)

$$\text{DF\%} = \frac{\text{Pulse Width (mSec)} \times \text{Frames per Second}}{10}$$

$$\text{DF\%} = \frac{5 \text{ msec} \times 50 \text{ exp/sec}}{10} = \frac{250}{10} = 25\%$$

Refer to Rating Chart

PG-292 150/180 Hz 3 Phase 1.2 Focal Spot:

At bottom of chart find 10 second line. Move vertically to intersection with 25% DF curve. Make a horizontal reference to left side of rating chart and note kW rating of 45 kW.

kW = kV x mA. The kW of the exposure can be any combination of mA and kV allowed by the Radiographic and Filament Emission Charts.



The Cine rating charts are usable to 100% anode heat storage. Exceeding 100% anode heat storage will cause anode track erosion with a high risk of tube destruction.





3 Ø Constant Potential ---

Cineradiographic Exposure Charts IEC 60613
Abaques de Cinéradiographie CEI 60613
Belastungskurven für den Kinobetrieb IEC 60613
Diagramas de Exposición Cineradiográfica IEC 60613

50/60 Hz

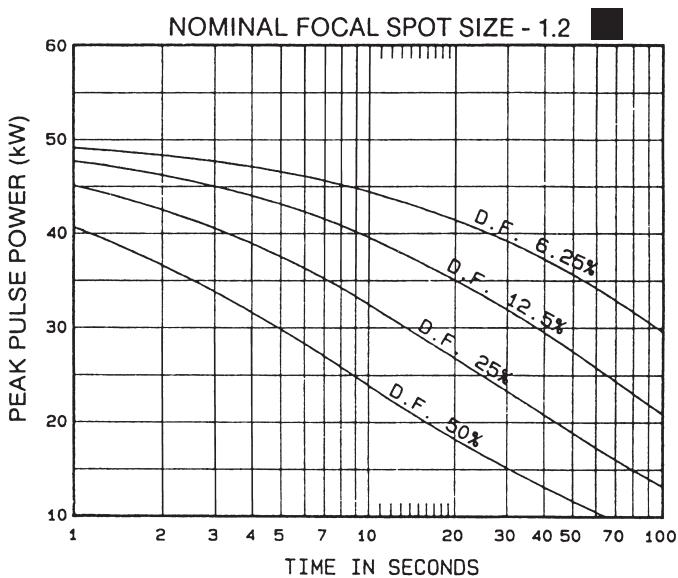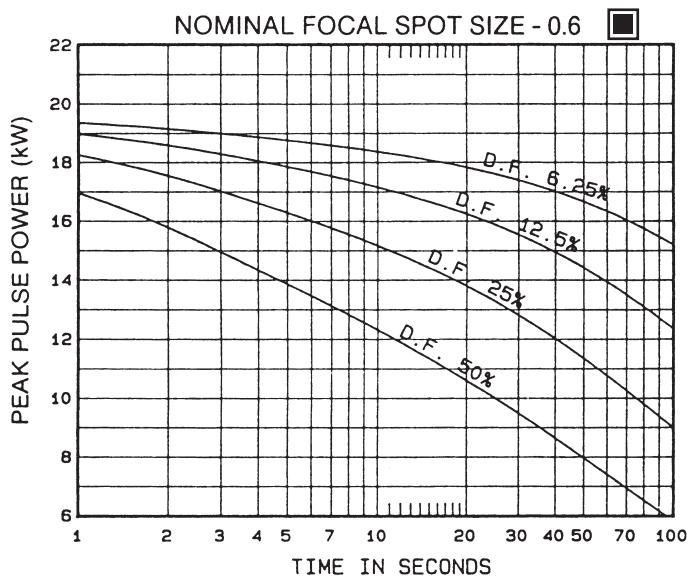

150/180 Hz

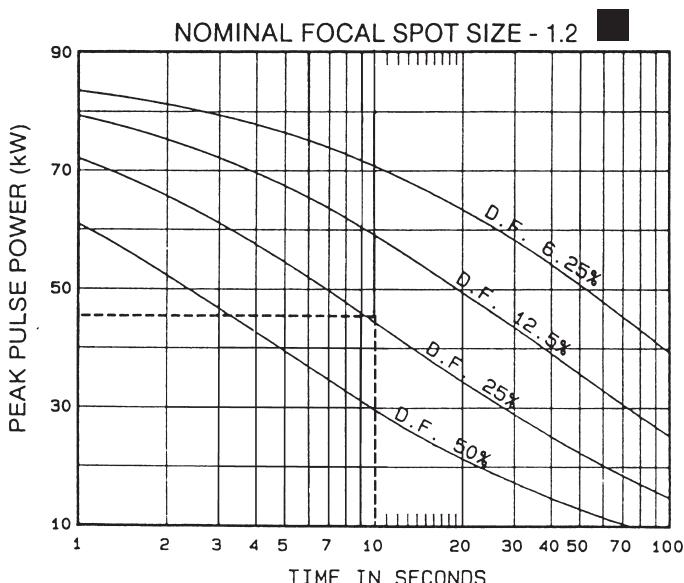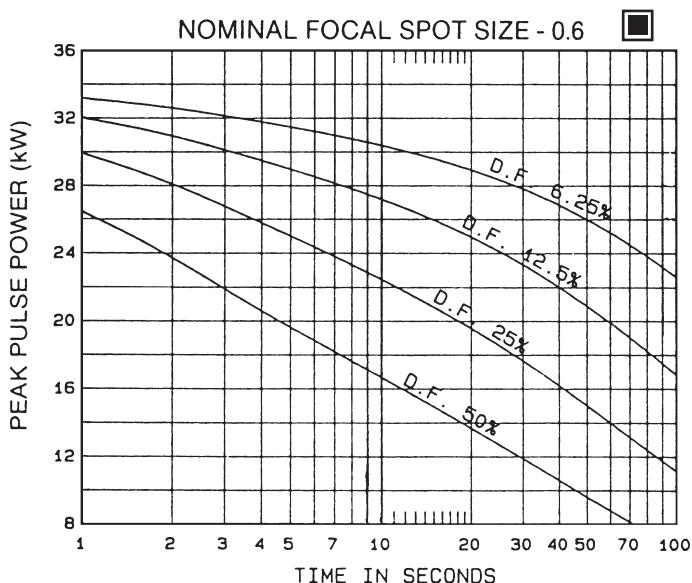

Nominal anode input power for the anode heat content 70%. IEC 60613

Puissance calorifique nominale de l'anode: 70%, CEI 60613

Thermische Anodenbezugsleistung bei einer Wärmespeicherung von 70%. IEC 60613

Aproximadamente el poder de penetración para obtener un almacenaje de calor del anodo de 70%. IEC 60613

ANGIOGRAPHIC RATINGS

HOW TO USE ANGIOGRAPHIC CHARTS

General: Serial Radiography puts a severe demand on the x-ray tube due to the large number of exposures made in rapid succession. Intervals between exposures are fixed and so short that it is not possible for the anode track to cool to any extent during the exposure series. Therefore, the temperature of the anode track increases from exposure to exposure. The kW values used in the angiographic charts have been determined to prevent damage to the anode. The angiographic rating charts are usable to 100% anode heat storage. Exceeding 100% anode heat storage will cause anode track erosion with high risk of tube destruction.

Definition of Terms

Number of Exposures in Series: The number of exposures made in succession or the number of exposures made during one contrast injection.

Exposure Rate: The number of exposures made per second. For a series of exposures where the exposure rate changes, it must be assumed that all exposures will be made at the maximum rate. For example, if during a series 10 exposures will occur at one per second and 30 exposures at 4 per second, use the kW ratings in the 40 exposure column at 4 per second rate.

Exposure Time: Time in seconds of each exposure.

USING THE CHARTS:

Determine the number of exposures in Series: With cut film angiography the number of exposures are known, however in Digital Angiography the number of exposures commonly are not known. When determining the number of exposures, assume worst case or past history.

Note: Most angiographic x-ray tubes fail from underestimating the number of exposures made in a series.

Determine kW of each exposure in Series: Referring to chart –find block under “Number of Exposures in Series” that is greater than or equal to expected number of exposures in Series. On left side directly opposite this block under “Exposure Rate per Second” column, select maximum rate per second that will be used for the exposure series. At the intersection of exposure rate and exposure time in seconds, find maximum kW allowed for each exposure.

kW = pkV x mA: The kW of the exposure can be any combination of mA and pkV allowed by the Radiographic and Filament Emission charts.

For Example: 80 pkV and 500 mA = 40 kW

Example: From chart PG-292 150/180 HZ 3 Phase
1.2 Focal Spot, determine kW allowed with following known factors.

Maximum number of exposures40
Exposure time .050 second (50 milliseconds)
Maximum Exposure per second4

From chart find 40 exposure block. On left side directly opposite this block under “Exposure Rate per Second” column, select 4 exposures per second. Find .050 seconds at top of chart. At intersection of exposure rate line and exposure time, find 47.5 kW.

0.6 Focal Spot 3Ø 50/60 Hz
 0.6 Dimension Focale 3Ø 50/60 Hz
 0.6 Brennfleck 3Ø 50/60 Hz
 0.6 De Marcas Focales 3Ø 50/60 Hz

Serial Load Ratings IEC 60613
 Abaques de charges successives CEI 60613
 Serienbetrieb-Belastungskurven IEC 60613
 Ratio de carga en serie IEC 60613

EXPOSURE RATE PER SECOND	TUBE LOAD (kW) AS A FUNCTION OF THE EXPOSURE TIME (SEC.) OF THE INDIVIDUAL RADIOPHOTOGRAPHS OF THE SERIES														NUMBER OF EXPOSURES IN SERIES	
	0.010	0.020	0.030	0.040	0.050	0.060	0.080	0.100	0.120	0.140	0.160	0.180	0.200	0.225	0.250	
1	19.1	19.1	18.8	18.5	18.3	18.1	17.7	17.4	17.1	16.8	16.6	16.3	16.1	15.9	15.7	
2	19.1	19.0	18.7	18.4	18.2	17.9	17.5	17.1	16.8	16.5	16.3	16.0	15.8	15.6	15.3	
3	19.1	19.0	18.6	18.3	18.0	17.8	17.4	17.0	16.6	16.3	16.0	15.8	15.6	15.3		
4	19.1	18.9	18.5	18.2	17.9	17.7	17.2	16.8	16.4	16.1	—	—	—	—	—	10
8	19.0	18.7	18.3	17.9	17.5	17.2	—	—	—	—	—	—	—	—	—	
15	18.8	18.4	17.8	17.4	—	—	—	—	—	—	—	—	—	—	—	
30	18.5	17.9	—	—	—	—	—	—	—	—	—	—	—	—	—	
1	19.1	18.9	18.6	18.2	18.0	17.7	17.2	16.8	16.4	16.1	15.8	15.5	15.2	14.9	14.6	
2	19.0	18.9	18.5	18.1	17.8	17.5	17.0	16.6	16.2	15.8	15.5	15.2	14.9	14.5	14.2	
3	19.0	18.8	18.4	18.0	17.7	17.4	16.8	16.3	15.9	15.5	15.2	14.8	—	—	—	
4	19.0	18.7	18.3	17.9	17.6	17.2	16.6	16.1	15.7	15.3	—	—	—	—	—	20
8	18.9	18.5	18.0	17.5	17.1	16.7	—	—	—	—	—	—	—	—	—	
15	18.6	18.1	17.4	16.8	—	—	—	—	—	—	—	—	—	—	—	
30	18.3	17.5	—	—	—	—	—	—	—	—	—	—	—	—	—	
1	19.0	18.7	18.2	17.8	17.4	17.1	16.4	15.9	15.4	14.9	14.5	14.1	13.8	13.4	13.0	
2	18.9	18.6	18.1	17.7	17.3	16.9	16.2	15.6	15.1	14.6	14.2	13.8	13.4	13.0	12.6	
3	18.9	18.5	18.0	17.5	17.1	16.7	16.0	15.4	14.8	14.3	13.9	13.4	—	—	—	
4	18.8	18.5	17.9	17.4	16.9	16.5	15.8	15.1	14.6	14.0	—	—	—	—	—	40
8	18.7	18.2	17.5	16.9	16.4	15.9	—	—	—	—	—	—	—	—	—	
15	18.4	17.7	16.9	16.2	—	—	—	—	—	—	—	—	—	—	—	
30	18.0	16.9	—	—	—	—	—	—	—	—	—	—	—	—	—	
1	18.8	18.4	17.9	17.4	16.9	16.5	15.7	15.1	14.5	13.9	13.4	13.0	12.6	12.1	11.7	
2	18.8	18.4	17.8	17.2	16.7	16.3	15.5	14.8	14.2	13.6	13.1	12.7	12.2	11.8	11.3	
3	18.7	18.3	17.7	17.1	16.6	16.1	15.3	14.6	13.9	13.3	12.8	12.4	—	—	—	
4	18.7	18.2	17.5	17.0	16.4	15.9	15.1	14.3	13.7	13.1	—	—	—	—	—	60
8	18.5	17.9	17.1	16.5	15.8	15.3	—	—	—	—	—	—	—	—	—	
15	18.3	17.4	16.5	15.7	—	—	—	—	—	—	—	—	—	—	—	
30	17.8	16.6	—	—	—	—	—	—	—	—	—	—	—	—	—	
1	18.7	18.2	17.6	17.0	16.4	15.9	15.1	14.3	13.6	13.1	12.5	12.0	11.6	10.3	9.2	
2	18.7	18.1	17.4	16.8	16.3	15.8	14.9	14.1	13.4	12.8	12.2	11.7	11.3	10.3	9.2	
3	18.6	18.0	17.3	16.7	16.1	15.6	14.6	13.8	13.1	12.5	12.0	11.5	—	—	—	
4	18.6	18.0	17.2	16.5	15.9	15.4	14.4	13.6	12.9	12.3	—	—	—	—	—	80
8	18.4	17.7	16.8	16.0	15.4	14.8	—	—	—	—	—	—	—	—	—	
15	18.1	17.1	16.1	15.3	—	—	—	—	—	—	—	—	—	—	—	
30	17.6	16.3	—	—	—	—	—	—	—	—	—	—	—	—	—	
1	18.6	18.0	17.3	16.6	16.0	15.4	14.5	13.7	12.9	12.3	11.6	10.3	9.2	8.2	7.4	
2	18.5	17.9	17.1	16.4	15.8	15.3	14.3	13.4	12.7	12.0	11.4	10.3	9.2	8.2	7.4	
3	18.5	17.8	17.0	16.3	15.7	15.1	14.1	13.2	12.4	11.8	11.2	10.3	—	—	—	
4	18.5	17.7	16.9	16.2	15.5	14.9	13.9	13.0	12.2	11.6	—	—	—	—	—	100
8	18.3	17.4	16.5	15.7	14.9	14.3	—	—	—	—	—	—	—	—	—	
15	18.0	16.9	15.8	14.9	—	—	—	—	—	—	—	—	—	—	—	
30	17.5	16.0	—	—	—	—	—	—	—	—	—	—	—	—	—	
1	18.3	17.5	16.5	15.7	15.0	14.3	13.2	12.2	10.3	8.8	7.7	6.9	6.2	5.5	4.9	
2	18.3	17.4	16.4	15.6	14.8	14.2	13.0	12.0	10.3	8.8	7.7	6.9	6.2	5.5	4.9	
3	18.2	17.3	16.3	15.4	14.7	14.0	12.8	11.8	10.3	8.8	7.7	6.9	—	—	—	
4	18.2	17.2	16.2	15.3	14.5	13.8	12.6	11.7	10.3	8.8	—	—	—	—	—	150
8	18.0	16.9	15.8	14.8	14.0	13.2	—	—	—	—	—	—	—	—	—	
15	17.7	16.4	15.1	14.1	—	—	—	—	—	—	—	—	—	—	—	
30	17.1	15.4	—	—	—	—	—	—	—	—	—	—	—	—	—	

Note:

- (kW) of Exposure Equals mA x kV. For Example: 70 kV x 300 mA = 21 kW.
- Exposures less than .010 seconds will have a kW rating same as .010 seconds.

Remarque:

- (kW) en exposition égale kV x mA. Par exemple: 70 kV x 300 mA = 21 kW.
- Les expositions inférieures à 0.010 sec. ent les mêmes valeurs en kW que celles de 0.010 sec.

Anmerkungen:

- (kW) der Belichtung ist gleich mA x kV. Zum Beispiel: 70 kV x 300 mA = 21 kW.
- Belichtungen von weniger als 0.010 Sekunden haben die gleichen kW Werte wie die von 0.010 Sekunden.

Nota:

- (kW) De exposición se calcula multiplicando mA x kV-por ejemplo: 70 kV x 300 mA = 21 kW.
- Para exposición de menos de .010 segundos, el resultado en (kW) sería lo mismo que el de .010 segundos.

Nominal anode input power for the anode heat content 70%. IEC 60613

Puissance calorifique nominale de l'anode: 70%, CEI 60613

Thermische Anodenbezugsleistung bei einer Wärmespeicherung von 70%. IEC 60613

Aproximadamente el poder de penetración para obtener un almacenaje de calor del anodo de 70%. IEC 60613

1.2 Focal Spot $3\varnothing$ 50/60 Hz
 1.2 Dimension Focale $3\varnothing$ 50/60 Hz
 1.2 Brennfleck $3\varnothing$ 50/60 Hz
 1.2 De Marcas Focales $3\varnothing$ 50/60 Hz

Serial Load Ratings IEC 60613
 Abaques de charges successives CEI 60613
 Serienbetrieb-Belastungskurven IEC 60613
 Ratio de carga en serie IEC 60613

EXPOSURE RATE PER SECOND	TUBE LOAD (kW) AS A FUNCTION OF THE EXPOSURE TIME (SEC.) OF THE INDIVIDUAL RADIOPHOTOGRAPHS OF THE SERIES															NUMBER OF EXPOSURES IN SERIES
	0.010	0.020	0.030	0.040	0.050	0.060	0.080	0.100	0.120	0.140	0.160	0.180	0.200	0.225	0.250	
1	48.3	47.7	46.5	45.4	44.4	43.5	42.0	40.6	39.4	38.3	37.3	36.3	35.4	34.4	33.5	
2	48.1	47.4	46.0	44.8	43.7	42.7	41.0	39.5	38.2	37.0	35.9	34.8	33.9	32.8	31.8	
3	47.9	47.0	45.6	44.2	43.1	42.0	40.1	38.5	37.1	35.8	34.6	33.6				
4	47.8	46.8	45.2	43.7	42.5	41.3	39.3	37.6	36.1	34.8						10
8	47.3	45.9	43.9	42.2	40.7	39.3										
15	46.7	44.7	42.3	40.3												
30	45.8	43.2														
1	47.8	46.9	45.3	43.9	42.6	41.5	39.5	37.8	36.2	34.8	33.6	32.4	31.3	30.1	29.0	
2	47.6	46.5	44.8	43.2	41.8	40.6	38.4	36.6	34.9	33.4	32.1	30.9	29.8	28.5	27.4	
3	47.4	46.1	44.2	42.6	41.1	39.8	37.5	35.5	33.8	32.2	30.8	29.6				20
4	47.2	45.7	43.8	42.0	40.4	39.0	36.6	34.5	32.7	31.1						
8	46.6	44.6	42.2	40.1	38.2	36.6										
15	45.7	43.0	40.1	37.6												
30	44.3	40.6														
1	47.0	45.4	43.2	41.3	39.7	38.2	35.6	33.3	31.4	29.7	28.2	25.7	23.1	20.6	18.5	
2	46.8	44.9	42.6	40.6	38.8	37.3	34.5	32.2	30.2	28.5	27.0	25.6	23.1	20.6	18.5	
3	46.5	44.5	42.1	39.9	38.1	36.4	33.6	31.2	29.2	27.4	25.9	24.5				40
4	46.3	44.1	41.5	39.3	37.3	35.6	32.7	30.2	28.2	26.4						
8	45.6	42.7	39.8	37.2	35.0	33.1										
15	44.5	40.9	37.4	34.5												
30	42.7	37.9														
1	46.3	44.0	41.4	39.1	37.1	35.4	32.4	29.9	25.7	22.0	19.3	17.1	15.4	13.7	12.3	
2	46.0	43.5	40.8	38.4	36.3	34.5	31.4	28.9	25.7	22.0	19.3	17.1	15.4	13.7	12.3	
3	45.8	43.1	40.2	37.7	35.6	33.7	30.6	28.0	25.7	22.0	19.3	17.1				60
4	45.5	42.7	39.7	37.1	34.9	33.0	29.7	27.2	25.0	22.0						
8	44.7	41.3	37.9	35.0	32.6	30.6										
15	43.5	39.3	35.4	32.3												
30	41.5	36.1														
1	45.6	42.7	39.7	37.1	34.9	33.0	28.9	23.1	19.3	16.5	14.5	12.8	11.6	10.3	9.2	
2	45.3	42.3	39.1	36.4	34.2	32.2	28.9	23.1	19.3	16.5	14.5	12.8	11.6	10.3	9.2	
3	45.0	41.8	38.5	35.8	33.5	31.4	28.1	23.1	19.3	16.5	14.5	12.8				80
4	44.8	41.4	38.0	35.2	32.8	30.8	27.4	23.1	19.3	16.5						
8	43.9	40.0	36.2	33.2	30.7	28.5										
15	42.7	37.9	33.8	30.5												
30	40.6	34.7														
1	44.9	41.5	38.2	35.4	33.0	30.8	23.1	18.5	15.4	13.2	11.6	10.3	9.2	8.2	7.4	
2	44.6	41.1	37.6	34.7	32.3	30.2	23.1	18.5	15.4	13.2	11.6	10.3	9.2	8.2	7.4	
3	44.3	40.6	37.0	34.1	31.6	29.5	23.1	18.5	15.4	13.2	11.6	10.3				100
4	44.1	40.2	36.5	33.5	31.0	28.9	23.1	18.5	15.4	13.2						
8	43.2	38.8	34.8	31.6	29.0	26.8										
15	41.9	36.8	32.4	29.0												
30	39.8	33.6														
1	43.2	38.8	34.8	30.8	24.7	20.6	15.4	12.3	10.3	8.8	7.7	6.9	6.2	5.5	4.9	
2	43.0	38.4	34.3	30.8	24.7	20.6	15.4	12.3	10.3	8.8	7.7	6.9	6.2	5.5	4.9	
3	42.7	37.9	33.8	30.5	24.7	20.6	15.4	12.3	10.3	8.8	7.7	6.9				150
4	42.5	37.6	33.3	30.0	24.7	20.6	15.4	12.3	10.3	8.8						
8	41.6	36.2	31.8	28.3	24.7	20.6										
15	40.3	34.3	29.6	26.0												
30	38.0	31.1														

Note:

- (kW) of Exposure Equals mA x kV. For Example: 70 kV x 300 mA = 21 kW.
- Exposures less than .010 seconds will have a kW rating same as .010 seconds.

Remarque:

- (kW) en exposition égale kV x mA. Par exemple: 70 kV x 300 mA = 21 kW.
- Les expositions inférieures à 0.010 sec. ont les mêmes valeurs en kW que celles de 0.010 sec.

Anmerkungen:

- (kW) der Belichtung ist gleich mA x kV. Zum Beispiel: 70 kV x 300 mA = 21 kW.
- Belichtungen von weniger als 0.010 Sekunden haben die gleichen kW Werte wie die von 0.010 Sekunden.

Nota:

- (kW) De exposición se calcula multiplicando mA x kV-por ejemplo: 70 kV x 300 mA = 21 kW.
- Para exposición de menos de .010 segundos, el resultado en (kW) sería lo mismo que el de .010 segundos.

Nominal anode input power for the anode heat content 70%. IEC 60613

Puissance calorifique nominale de l'anode: 70%, CEI 60613

Thermische Anodenbezugsleistung bei einer Wärmespeicherung von 70%. IEC 60613

Aproximadamente el poder de penetración para obtener un almacenaje de calor del anodo de 70%. IEC 60613

0.6 Focal Spot $3\varnothing$ 150/180 Hz
0.6 Dimension Focale $3\varnothing$ 150/180 Hz
0.6 Brennfleck $3\varnothing$ 150/180 Hz
0.6 De Marcas Focales $3\varnothing$ 150/180 Hz

Serial Load Ratings IEC 60613
Abaques de charges successives CEI 60613
Serienbetrieb-Belastungskurven IEC 60613
Ratio de carga en serie IEC 60613

EXPOSURE RATE PER SECOND	TUBE LOAD (kW) AS A FUNCTION OF THE EXPOSURE TIME (SEC.) OF THE INDIVIDUAL RADIOPHOTOGRAPHS OF THE SERIES															NUMBER OF EXPOSURES IN SERIES
	.010	.020	.030	.040	.050	.060	.080	.100	.120	.140	.160	.180	.200	.225	.250	
1	33.1	31.5	30.2	29.1	28.2	27.3	25.7	24.4	23.3	22.2	21.3	20.4	19.7	18.8	18.0	20
2	32.8	30.9	29.4	28.1	27.0	26.0	24.2	22.7	21.4	20.3	19.2	18.3	17.5	16.6	15.8	
3	32.5	30.4	28.8	27.4	26.1	25.0	23.1	21.5	20.1	18.9	17.9	16.9	0.0	0.0	0.0	
4	32.3	30.1	28.3	26.7	25.4	24.2	22.2	20.6	19.2	17.9	0.0	0.0	0.0	0.0	0.0	
8	31.7	29.1	27.0	25.2	23.7	22.4	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	31.1	28.0	25.6	23.7	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	30.2	26.5	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	32.7	30.7	29.1	27.7	26.6	25.5	23.7	22.1	20.8	19.6	18.6	17.6	16.8	15.9	15.1	
2	32.2	29.9	28.1	26.5	25.2	24.0	22.0	20.3	18.8	17.6	16.6	15.6	14.8	13.9	13.1	
3	31.9	29.3	27.3	25.6	24.1	22.8	20.7	18.9	17.5	16.2	15.1	14.2	0.0	0.0	0.0	
4	31.6	28.8	26.6	24.8	23.3	21.9	19.7	17.9	16.4	15.2	0.0	0.0	0.0	0.0	0.0	40
8	30.6	27.3	24.7	22.6	20.9	19.5	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	29.7	25.8	22.9	20.6	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	28.4	23.9	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	32.3	30.0	28.1	26.6	25.3	24.1	22.0	20.4	18.9	17.7	16.7	15.7	14.9	14.0	12.7	
2	31.8	29.1	27.1	25.3	23.8	22.5	20.4	18.6	17.1	15.9	14.8	13.9	13.1	12.2	11.4	
3	31.4	28.5	26.2	24.3	22.7	21.4	19.1	17.3	15.8	14.6	13.5	12.6	0.0	0.0	0.0	
4	31.0	27.9	25.5	23.5	21.8	20.4	18.1	16.3	14.8	13.5	0.0	0.0	0.0	0.0	0.0	
8	29.9	26.1	23.3	21.1	19.3	17.8	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	28.6	24.3	21.1	18.8	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	27.1	22.1	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	31.9	29.3	27.3	25.6	24.1	22.8	20.7	18.9	17.5	16.2	14.8	13.2	11.9	10.6	9.5	60
2	31.4	28.5	26.2	24.3	22.7	21.4	19.1	17.3	15.8	14.5	13.5	12.6	11.8	10.6	9.5	
3	30.9	27.8	25.3	23.3	21.6	20.2	17.9	16.0	14.5	13.3	12.3	11.4	0.0	0.0	0.0	
4	30.6	27.1	24.5	22.5	20.7	19.2	16.9	15.0	13.6	12.4	0.0	0.0	0.0	0.0	0.0	
8	29.3	25.3	22.3	20.0	18.2	16.6	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	27.9	23.2	19.9	17.5	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	26.1	20.8	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	31.5	28.7	26.5	24.7	23.1	21.8	19.5	17.7	15.8	13.6	11.9	10.6	9.5	8.4	7.6	
2	31.0	27.8	25.4	23.4	21.8	20.3	18.0	16.2	14.7	13.5	11.9	10.6	9.5	8.4	7.6	
3	30.5	27.1	24.5	22.4	20.7	19.2	16.8	15.0	13.5	12.3	11.3	10.5	0.0	0.0	0.0	
4	30.1	26.5	23.7	21.6	19.8	18.3	15.9	14.1	12.6	11.5	0.0	0.0	0.0	0.0	0.0	80
8	28.8	24.6	21.5	19.1	17.2	15.7	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	27.3	22.4	19.0	16.6	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	25.2	19.7	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	30.7	27.3	24.8	22.7	21.0	19.5	15.8	12.7	10.6	9.0	7.9	7.0	6.3	5.6	5.1	
2	30.1	26.5	23.7	21.5	19.7	18.2	15.8	12.7	10.6	9.0	7.9	7.0	6.3	5.6	5.1	
3	29.6	25.7	22.8	20.6	18.7	17.2	14.8	12.7	10.6	9.0	7.9	7.0	0.0	0.0	0.0	
4	29.2	25.1	22.1	19.8	17.9	16.4	14.0	12.3	10.6	9.0	0.0	0.0	0.0	0.0	0.0	
8	27.8	23.1	19.8	17.4	15.5	14.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	26.1	20.8	17.3	14.9	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	23.7	17.9	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	28.4	24.0	20.8	18.8	12.7	10.6	7.9	6.3	5.3	4.5	4.0	3.5	3.2	2.8	2.5	100
2	27.9	23.2	19.9	15.8	12.7	10.6	7.9	6.3	5.3	4.5	4.0	3.5	3.2	2.8	2.5	
3	27.4	22.5	19.2	15.8	12.7	10.6	7.9	6.3	5.3	4.5	4.0	3.5	0.0	0.0	0.0	
4	27.0	21.9	18.6	15.8	12.7	10.6	7.9	6.3	5.3	4.5	0.0	0.0	0.0	0.0	0.0	
8	25.5	20.1	16.6	14.2	12.4	10.6	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	23.6	17.9	14.4	12.1	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	21.0	15.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	

Note:

- (kW) of Exposure Equals mA x kV. For Example: 70 kV x 300 mA = 21 kW.
- Exposures less than .010 seconds will have a kW rating same as .010 seconds.

Remarque:

- (kW) en exposition égale kV x mA. Par exemple: 70 kV x 300 mA = 21 kW.
- Les expositions inférieures à 0.010 sec. ent les mêmes valeurs en kW que celles de 0.010 sec.

Anmerkungen:

- (kW) der Belichtung ist gleich mA x kV. Zum Beispiel: 70 kV x 300 mA = 21 kW.
- Belichtungen von weniger als .010 Sekunden haben die gleichen kW Werte wie die von .010 Sekunden.

Nota:

- (kW) De exposición se calcula multiplicando mA x kV-por ejemplo: 70 kV x 300 mA = 21 kW.
- Para exposición de menos de .010 segundos, el resultado en (kW) sería lo mismo que el de .010 segundos.

Nominal anode input power for the anode heat content 70%. IEC 60613

Puissance calorifique nominale de l'anode: 70%, CEI 60613

Thermische Anodenbezugsleistung bei einer Wärmespeicherung von 70%. IEC 60613

Aproximadamente el poder de penetración para obtener un almacenaje de calor del anodo de 70%. IEC 60613

1.2 Focal Spot $3\varnothing$ 150/180 Hz
1.2 Dimension Focale $3\varnothing$ 150/180 Hz
1.2 Brennfleck $3\varnothing$ 150/180 Hz
1.2 De Marcas Focales $3\varnothing$ 150/180 Hz

Serial Load Ratings IEC 60613
Abaques de charges successives CEI 60613
Serienbetrieb-Belastungskurven IEC 60613
Ratio de carga en serie IEC 60613

EXPOSURE RATE PER SECOND	TUBE LOAD (kW) AS A FUNCTION OF THE EXPOSURE TIME (SEC.) OF THE INDIVIDUAL RADIOPHOTOGRAPHS OF THE SERIES														NUMBER OF EXPOSURES IN SERIES	
	.010	.020	.030	.040	.050	.060	.080	.100	.120	.140	.160	.180	.200	.225	.250	
1	82.1	75.5	70.4	66.3	62.7	59.5	54.2	49.9	46.3	43.2	40.5	38.2	36.1	33.9	31.9	20
2	81.1	73.8	68.2	63.6	59.7	56.4	50.8	46.3	42.6	39.5	36.8	34.5	32.5	30.2	28.3	
3	80.4	72.6	66.7	61.9	57.9	54.4	48.7	44.1	40.4	37.3	34.6	32.3	0.0	0.0	0.0	
4	79.8	71.7	65.6	60.6	56.4	52.9	47.0	42.4	38.7	35.6	0.0	0.0	0.0	0.0	0.0	
8	78.1	69.0	62.2	56.8	52.3	48.6	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	76.0	65.8	58.4	52.6	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	73.1	61.5	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	79.4	70.9	64.6	59.5	55.2	51.6	45.7	41.1	37.4	33.9	29.7	26.4	23.8	21.1	19.0	40
2	77.8	68.4	61.5	56.1	51.6	47.8	41.8	37.2	33.5	30.5	28.0	26.0	23.8	21.1	19.0	
3	76.6	66.7	59.4	53.8	49.2	45.3	39.3	34.7	31.1	28.2	25.9	23.8	0.0	0.0	0.0	
4	75.8	65.4	58.0	52.2	47.5	43.6	37.6	33.1	29.5	26.7	0.0	0.0	0.0	0.0	0.0	
8	73.5	62.0	54.0	48.0	43.2	39.3	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	70.7	58.2	49.8	43.6	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	66.9	53.2	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	77.0	67.2	60.1	54.5	49.9	46.1	39.6	31.7	26.4	22.6	19.8	17.6	15.8	14.1	12.7	60
2	75.2	64.5	56.9	51.0	46.3	42.4	36.4	31.7	26.4	22.6	19.8	17.6	15.8	14.1	12.7	
3	73.8	62.5	54.5	48.5	43.8	39.9	33.9	29.6	26.2	22.6	19.8	17.6	0.0	0.0	0.0	
4	72.7	61.0	52.8	46.7	41.9	38.0	32.2	27.9	24.6	22.1	0.0	0.0	0.0	0.0	0.0	
8	69.9	57.1	48.6	42.3	37.6	33.8	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	66.8	53.1	44.3	38.1	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	62.6	47.9	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	74.9	64.0	56.3	50.4	45.7	39.6	29.7	23.8	19.8	17.0	14.8	13.2	11.9	10.6	9.5	80
2	72.9	61.2	53.1	47.0	42.2	38.4	29.7	23.8	19.8	17.0	14.8	13.2	11.9	10.6	9.5	
3	71.4	59.2	50.8	44.6	39.8	36.0	29.7	23.8	19.8	17.0	14.8	13.2	0.0	0.0	0.0	
4	70.2	57.5	49.0	42.7	38.0	34.2	28.5	23.8	19.8	17.0	0.0	0.0	0.0	0.0	0.0	
8	66.9	53.2	44.4	38.2	33.5	29.9	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	63.6	49.2	40.3	34.2	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	59.1	44.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	72.9	61.2	53.1	47.0	38.0	31.7	23.8	19.0	15.8	13.6	11.9	10.6	9.5	8.4	7.6	100
2	70.9	58.4	50.0	43.8	38.0	31.7	23.8	19.0	15.8	13.6	11.9	10.6	9.5	8.4	7.6	
3	69.3	56.3	47.7	41.5	36.7	31.7	23.8	19.0	15.8	13.6	11.9	10.6	0.0	0.0	0.0	
4	68.0	54.6	45.9	39.6	34.9	31.3	23.8	19.0	15.8	13.6	0.0	0.0	0.0	0.0	0.0	
8	64.4	50.1	41.2	35.1	30.6	27.1	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	60.9	46.0	37.1	31.2	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	56.3	40.9	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	68.5	55.3	42.2	31.7	25.3	21.1	15.8	12.7	10.6	9.0	7.9	7.0	6.3	5.6	5.1	150
2	66.5	52.7	42.2	31.7	25.3	21.1	15.8	12.7	10.6	9.0	7.9	7.0	6.3	5.6	5.1	
3	64.8	50.6	41.8	31.7	25.3	21.1	15.8	12.7	10.6	9.0	7.9	7.0	0.0	0.0	0.0	
4	63.5	49.0	40.1	31.7	25.3	21.1	15.8	12.7	10.6	9.0	0.0	0.0	0.0	0.0	0.0	
8	59.5	44.4	35.6	29.7	25.3	21.1	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	55.5	40.1	31.5	26.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	50.7	35.3	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
1	58.3	31.7	21.1	15.8	12.7	10.6	7.9	6.3	5.3	4.5	4.0	3.5	3.2	2.8	2.5	300
2	56.5	31.7	21.1	15.8	12.7	10.6	7.9	6.3	5.3	4.5	4.0	3.5	3.2	2.8	2.5	
3	55.0	31.7	21.1	15.8	12.7	10.6	7.9	6.3	5.3	4.5	4.0	3.5	0.0	0.0	0.0	
4	53.7	31.7	21.1	15.8	12.7	10.6	7.9	6.3	5.3	4.5	0.0	0.0	0.0	0.0	0.0	
8	49.8	31.7	21.1	15.8	12.7	10.6	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
15	45.5	30.4	21.1	15.8	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	
30	40.5	26.1	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	0.0	

Note:

- (kW) of Exposure Equals mA x kV. For Example: 70 kV x 300 mA = 21 kW.
- Exposures less than .010 seconds will have a kW rating same as .010 seconds.

Remarque:

- (kW) en exposition égale kV x mA. Par exemple: 70 kV x 300 mA = 21 kW.
- Les expositions inférieures à 0.010 sec. ent les mêmes valeurs en kW que celles de 0.010 sec.

Anmerkungen:

- (kW) der Belichtung ist gleich mA x kV Zum Beispiel: 70 kV x 300 mA = 21 kW.
- Belichtungen von weniger als 0.010 Sekunden haben die gleichen kW Werte wie die von 0.010 Sekunden.

Nota:

- (kW) De exposición se calcula multiplicando mA x kV-por ejemplo: 70 kV x 300 mA = 21 kW.
- Para exposición de menos de .010 segundos, el resultado en (kW) sería lo mismo que el de .010 segundos.

Nominal anode input power for the anode heat content 70%. IEC 60613

Puissance calorifique nominale de l'anode: 70%, CEI 60613



Thermische Anodenbezugsleistung bei einer Wärmespeicherung von 70%. IEC 60613

Aproximadamente el poder de penetración para obtener un almacenaje de calor del anodo de 70%. IEC 60613



3 Ø Full Wave

Filament Emission Charts IEC 60613
Abaques d' Émissions des Filaments CEI 60613
Glühfadenemissionsdiagramm IEC 60613
Curvas de Emisión de los Filamentos IEC 60613

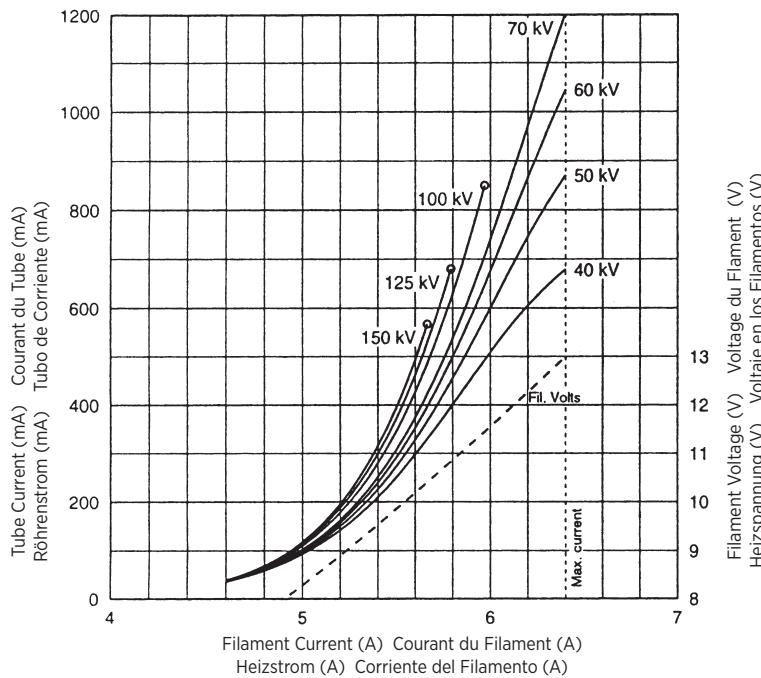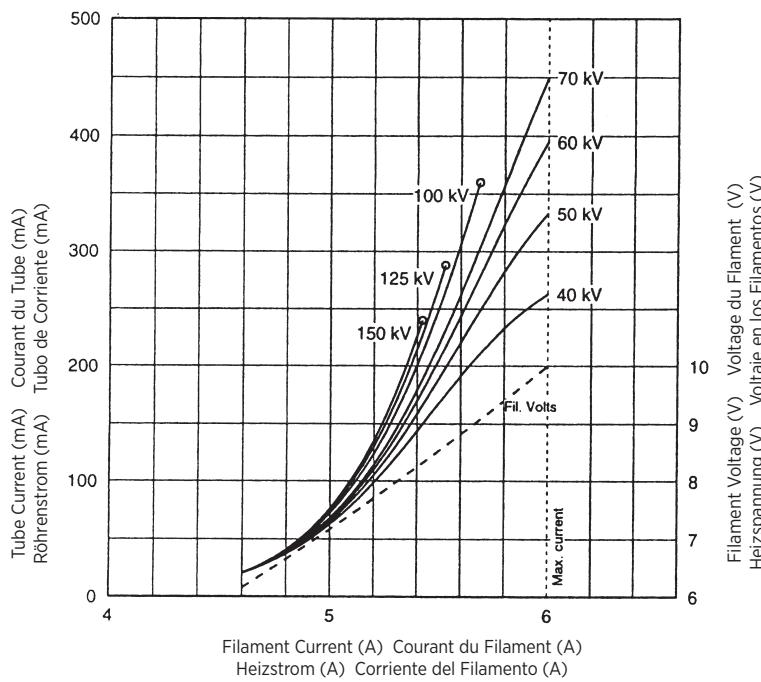

Note:
When using these emission curves for trial exposures, refer to the power rating curves shown for maximum kV, tube emission, filament current, exposure time, and target speed.

Remarque:
Lors de l'utilisation de ces abaques pour des expositions d'essai, référez-vous aux courbes maximales de kV, d'émission du filament, de temps d'exposition et de vitesse de rotation.

Anmerkung:
Wenn Sie diese Emissionskurven für Testaufnahmen verwenden, beziehen Sie sich hierbei auf die entsprechenden Nennleistungskurven für max. kV-Werte, Röhrenemission, Heizstrom, und Anodendrehzahl.

Nota:
Si utiliza estas curvas de emisión para exposiciones de prueba, refiérase a las curvas de gradación de potencia para el máximo de kV, tubo de emisión, corriente en los filamentos, tiempo de exposición, y a las curvas de velocidad del objetivo.



Anode Heating & Cooling Chart
Abaques d' Échauffement et de Refroidissement de L'Anode
Anoden Aufheiz - und Abkühl Kurven
Curvas de Calentamiento y Enfriamiento del Anodo

