Note: Document originally drafted in the English language.

Product Description

The B-110 housing is designed for Varex Imaging mammography inserts.

IEC Classification Class 1
 Weight, Approximate:
 Housing 10.2 kg (22.5 lbs)
 Stator Cord 4.5 m (14 Ft)
 Mounting Port
 X-Ray Tube Assembly (Complies to) IEC 60601-2-28

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Product Description

Nominal X-ray Tube Voltage
 Maximum Cathode to Ground 0 kV
 Maximum Anode to Ground 52 kV

Grid Control Voltages: (If Applicable)
 Typical Bias Voltage for Focals -5 to -100 Vdc

Maximum X-Ray Tube Heat Content 370 kJ (500 kHU)

Nominal Continuous Input Power 100 Watts (135 HU/sec)
 IEC 60613:2010

Maximum Housing Temperature 78°C

X-Ray Tube Assembly
 Permanent Filtration 0.63 mm Be IEC 60522/1999

Leakage Technique Factors 55 kV, 4.0 mA

Cable Receptacles Non Federal Standard
 55 kV - Rating

Thermal Switch Rating - 10 A @ 240Vac
 Normally closed Open @ 79.4°C ±3.9°C (175°F ±7°F)

Ambient Air Temperature Limits for Operation 5°C to 40°C

Temperature Limits for Storage and Transportation -10°C to 75°C
 Humidity 10% to 90%
 Atmospheric Pressure Range 70 kPa to 106 kPa

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Note 1 Dimensions listed are for the standard port boss
 Metric M6 X 1.0 available upon request

Note 2 **PORT BOSS DIMENSION**
 Dimensionelles de la Fenêtre Patron
 Abmessung Halterungscher
 Dimensiones de la Portales Patrón

For Determining Focal Spot to Port Distance
 Distance du Focales Suivantes de la Fenêtre
 Abstand Brennfleck zu Strahlenaustrittsfenster
 Punto Focal a la Distance de la Abertura Dimensiones de la Portales

M-101, M-101G, M-107, M-131, M-141, M-141G, M-143, RAD-85S

STANDARD/METRIC PORT BOSS
 DIM. F = 2.97 INCHES [75.4mm]
 DIM I = 1.85 INCHES [47mm]

EXTENDED PORT BOSS
 DIM. F = 3.20 INCHES [81.3mm]
 DIM. I = 2.08 INCHES [52.8mm]

M-145, M-147, M-147SP

STANDARD/METRIC PORT BOSS
 DIM. F = 2.97 INCHES [75.4mm]
 DIM. I = 1.74 INCHES [44.2mm]

EXTENDED PORT BOSS
 DIM. F = 3.20 INCHES [81.3mm]
 DIM. I = 1.97 INCHES [50.0mm]

M-105SP, M-109, M-149

STANDARD/METRIC PORT BOSS
 DIM. F = 2.97 INCHES [75.4mm]
 DIM. I = 1.66 INCHES [42.2mm]





EXTENDED PORT BOSS
 DIM. F = 3.20 INCHES [81.3mm]
 DIM. I = 1.89 INCHES [48mm]

BIANGULAR INSERTS
M-113 Series, M-171


STANDARD/METRIC PORT BOSS
 DIM. F = 2.97 INCHES [75.4mm]
 DIM. I = Refer to Drawing

EXTENDED PORT BOSS
 DIM. F = 3.20 INCHES [81.3mm]
 DIM. I = Refer to Drawing

Note: Dimensions in [mm] Inches
Remarque: Dimensions en [mm] Pouces
Hinweis: Abmessungen in [mm] Zoll
Nota: Dimensiones en [mm] Pulgadas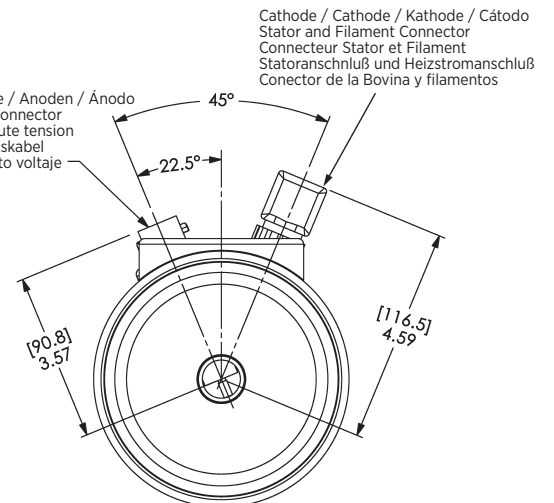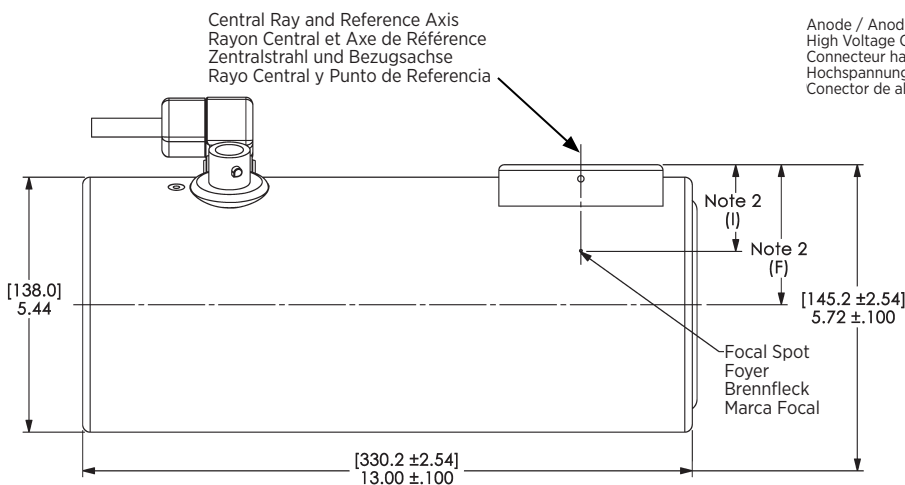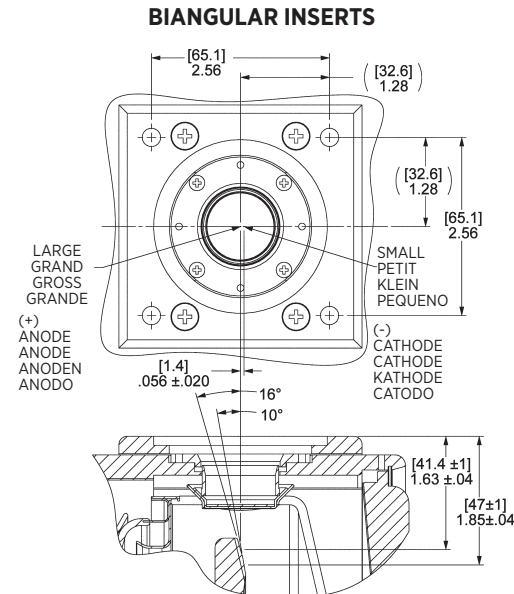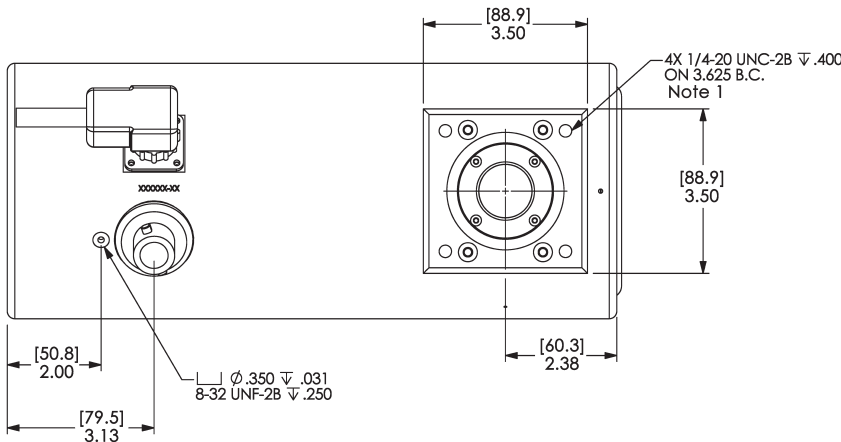

Stator Ratings and Characteristics
 Spécificités et Caractéristiques du Stator
 Statornennleistungen und Merkmale
 Características y Clarificación de la Bovina

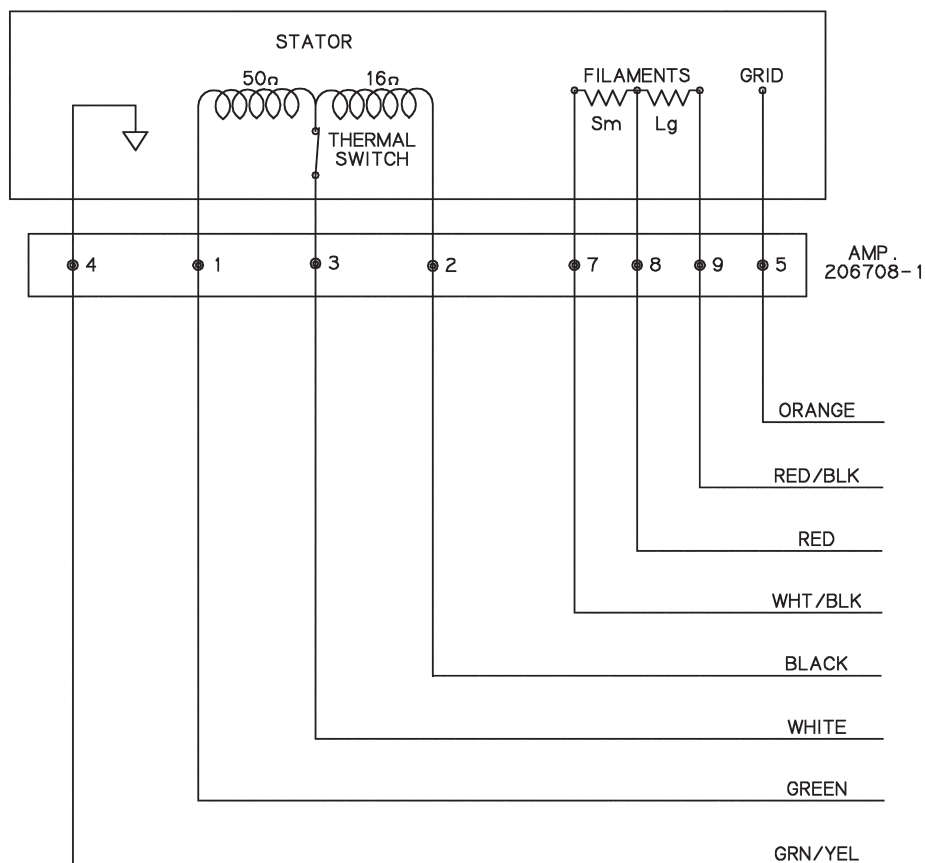| | |
|-------------------------------------------------------------|----------------------------|
| 1 Green | Phase Shift |
| Vert | Changement de Phase |
| Grün | Hilfsphase |
| Verde | Cambio de Fase del Estator |
| 2 Black | Phase |
| Noir | Phase |
| Schwarz | Phase |
| Negro | Fase |
| 3 White | Common |
| Blanc | Neutre |
| Weiss | Neutral |
| Blanco | Común |
| 4 Green/Yellow | Housing Ground |
| Vert/Jaune | Masse de la Gaine |
| Grün/Gelb | Masse des Gehäuses |
| Verde/Amerillo | Encaje a Tierra |
| 5 Orange | Grid |
| Orange | Grille |
| Orange | Gitter |
| Anaranjado | Rejillas |
| 6 Not used / Hors'd Usage / Nicht in Gebrauch/ No utilizado | |
| 7 White/Black | Small Filament |
| Blanc/Noir | Filament Petit |
| Weiss/Schwarz | Glüfaden Klein |
| Blanco/Negro | Filamento Pequeño |
| 8 Red | Common Filament |
| Rouge | Filament Neutre |
| Rot | Glüfaden Neutral |
| Rojo | Filamento Común |
| 9 Red/Black | Large Filament |
| Rouge/Noir | Filament Grand |
| Rot/Schwarz | Glüfaden Gross |
| Rojo/Negro | Filamento Largo |

| "STD" Stators | Stator "STD" | "STD" Stator | "STD" Bovina | |
|---------------|--------------|---------------|--------------|-------|
| Black/White | Noir/Blanc | Schwarz/Weiss | Negro/Blanco | 16 Ω |
| Black/Green | Noir/Vert | Schwarz/Grün | Negro/Verde | 66 Ω |
| Green/White | Vert/Blanc | Grün/Weiss | Verde/Blanco | 50 Ω |
| 180 Hz Cap | 180 Hz Cap | 180 Hz Cap | 180 Hz Cap | 6 μF |
| 60 Hz Cap | 60 Hz Cap | 60 Hz Cap | 60 Hz Cap | 30 μF |

| Stator Drive Frequency Fréquence d'entraînement du stator Statorantrieb Frequenz Frecuencia de la impulsión del estator | RPM |
|----------------------------------------------------------------------------------------------------------------------------------|---------------|
| 50 Hz | 2800 - 3000 |
| 60 Hz | 3400 - 3600 |
| 150 Hz | 8500 - 9000 |
| 180 Hz | 9500 - 10,800 |

Stator Power:
 Time to full speed of the anode is a function of the power rating of the "starter" and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible.

| Starter: | Start | Run |
|------------|---------|--------|
| 50/60 Hz | 115 VAC | 45 VAC |
| 150/180 Hz | 230 VAC | 90 VAC |

NOTE:
 For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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Manufactured by Varex Imaging Corporation
Fabrique par Varex Imaging Corporation
Hergestellt von Varex Imaging Corporation
Fabricado por Varex Imaging Corporation