Note: Document originally drafted in the English language.

Product Description

The B-130H housing is designed for Varex Imaging rotating Anode inserts having 102 mm (4 inch) diameter targets.

- IEC Classification Class 1
- Weight, Approximate:
- Housing 20.4 kg (45 lbs)
- With Trunnion Rings 22.9 kg (50.5 lbs)
- With Trunnion Rings and Fan Assembly
..... 24.1 kg (53.2 lbs)
- Stator Cord 6.1 M (20 Feet)
- Mounting Trunnion or Port
(using metric M6 screws)

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Product Description	
Nominal X-ray Tube Voltage	150 kV
Maximum Cathode to Ground	75 kV
Maximum Anode to Ground	75 kV
Grid to Cathode (If applicable)	-4 kV
Grid Control Voltages	
Typical Bias Voltage for Cutoff at 150 kV	-3600 Vdc
Grid Voltage for Exposure	0 Vdc
Maximum X-ray Tube Assembly Heat Content	1480 kJ (2,000 kHU)
Nominal Continuous Input Power (max. housing temperature 78°C)	
without air circulator	222 Watts (300 HU/sec) IEC 60613:2010
with air circulator	445 Watts (600 HU/sec) IEC 60613:2010
X-ray tube assembly cooling is provided by ambient air flow.	
X-Ray Tube Assembly	
Permanent Filtration	0.7mm Al/75 kV IEC 60522/1999
High Voltage Cable Receptacles	Per IEC 60526
Loading Factors for Leakage Radiation	
without Heat Exchanger	150 kV, 4 mA
with Heat Exchanger	150 kV, 4 mA for 300/400 KHU Anodes 150 kV, 6.7 mA for 600 KHU Anodes both limited by the anode
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Thermal Switch	Normally Closed
Open	7 A @ 120Vac or 30 Vdc max.
Open	85°C ±3.9°C (185°F ±7°F)
X-Ray Tube Assembly (Complies to)	IEC 60601-2-28

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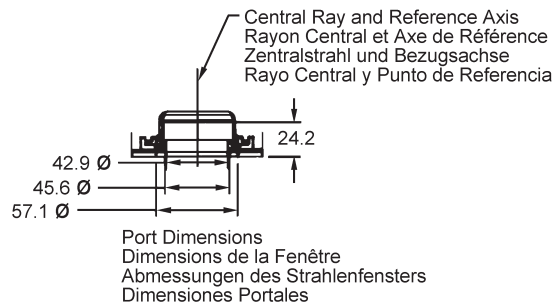
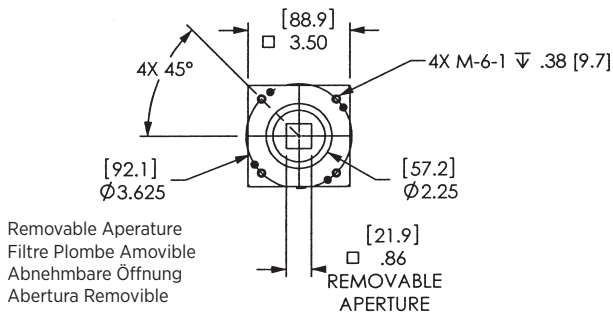
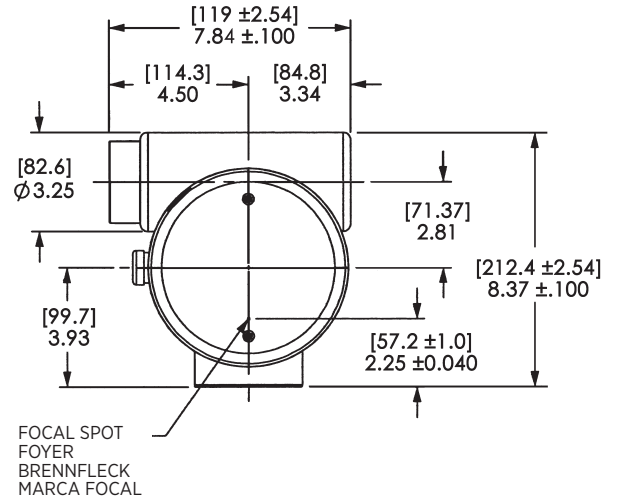
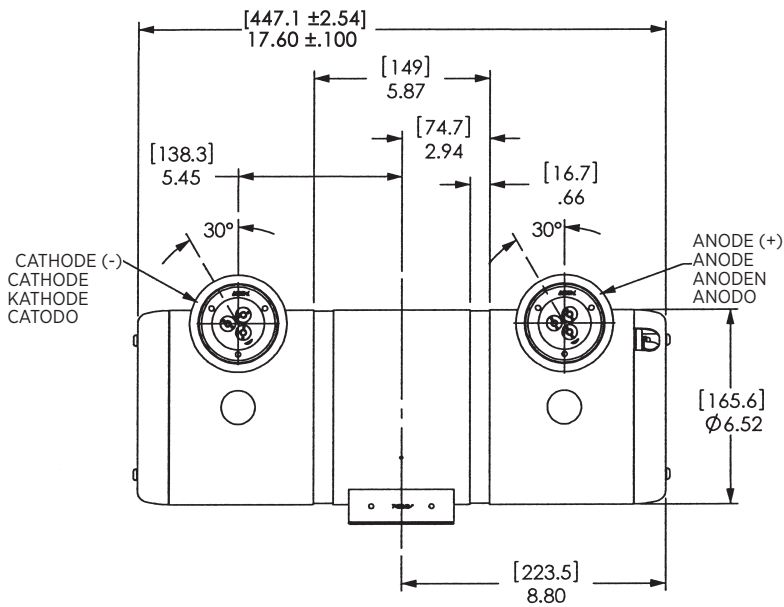
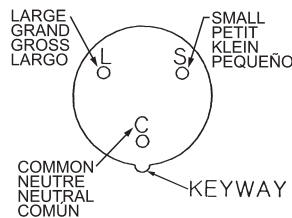
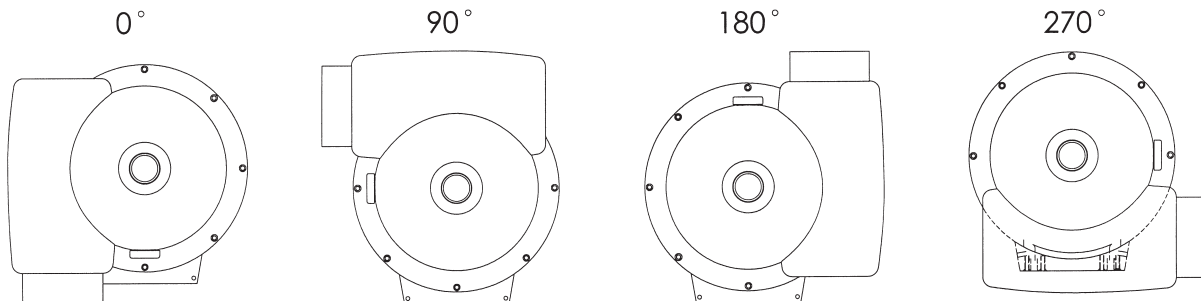
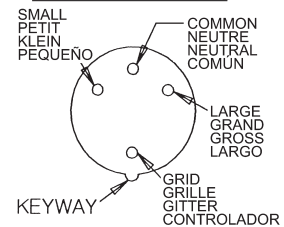
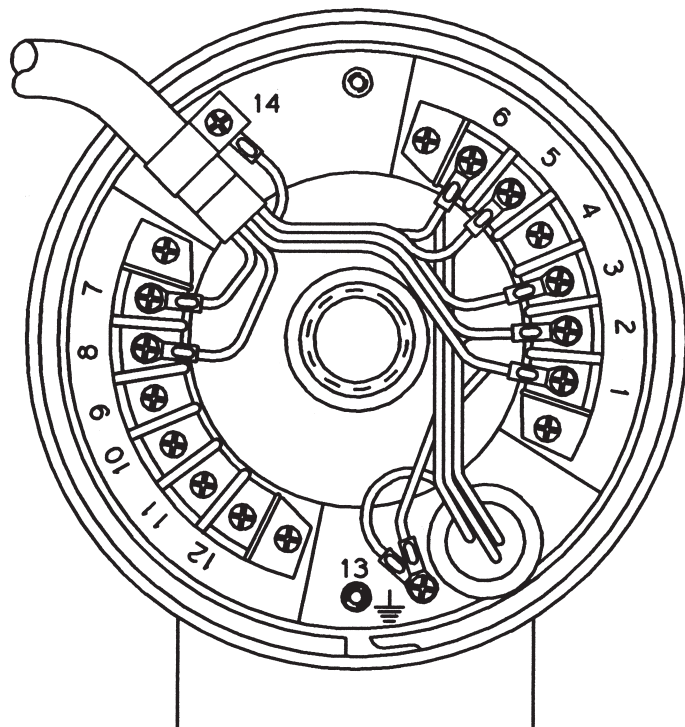
Stator Power:

Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 102 mm (4 inch) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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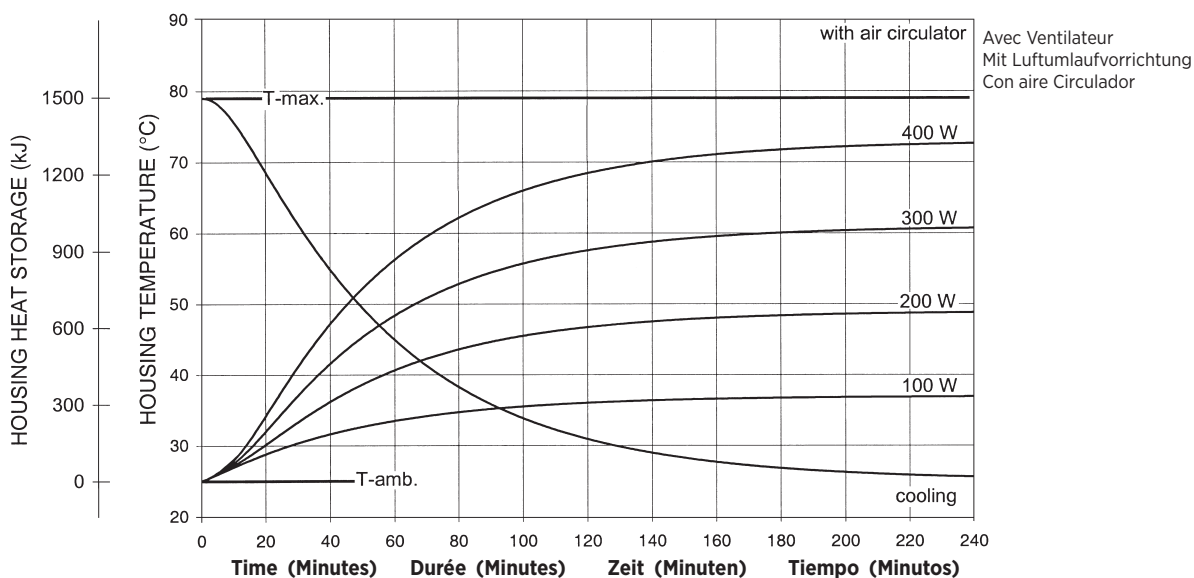
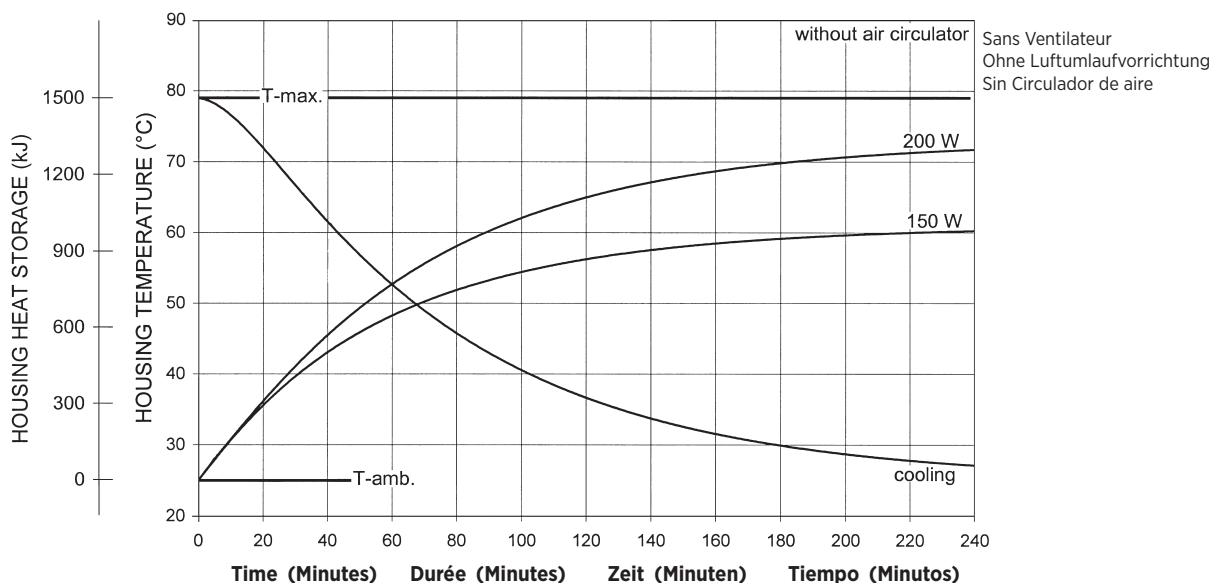
Caution:
 Housings configured with heat exchangers having long hoses (> 5 ft or 1.5 m) are not configured for operation separate from the associated heat exchanger. In such cases, the tube unit must not be operated unless properly coupled to the heat exchanger. Reference the specific heat exchanger's product data sheet for the associated cooling curves.

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Note:
 Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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Salt Lake City, UT 1-801-972-5000

For a complete listing of our global offices,
visit www.vareximaging.com

Manufactured by Varex Imaging Corporation
Fabrique par Varex Imaging Corporation
Hergestellt von Varex Imaging Corporation
Fabricado por Varex Imaging Corporation

Specifications subject to change without notice.
Spécifications susceptibles d'être modifiées sans préavis.
Technische Daten ohne Gewähr.
Especificaciones sujetas a cambio sin previo aviso.