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Product Description

Nominal X-ray Tube Voltage	125 kV
Maximum Cathode to Ground	63 kV
Maximum Anode to Ground	63 kV
Maximum X-Ray Tube Assembly Heat Content	1,750 kJ (2,450 kHU)
Nominal Continuous Input Power	1000 Watts - IEC 60613:2010 2 Liters/minute, 20°C to 40°C fluid
Maximum Housing Temperature	78°C
X-Ray Tube Assembly	
Permanent Filtration	1.0 mm Al/75 kV IEC 60522/1999
Loading Factors for Leakage Radiation	125 kV, 8.0 mA
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	.70 kPa to 106 kPa
Thermal Switch	Rating - 7A @ 120Vac or 30 Vdc max. Normally Closed
	Opens @ 85°C ±3.9°C (185°F ±7°F)

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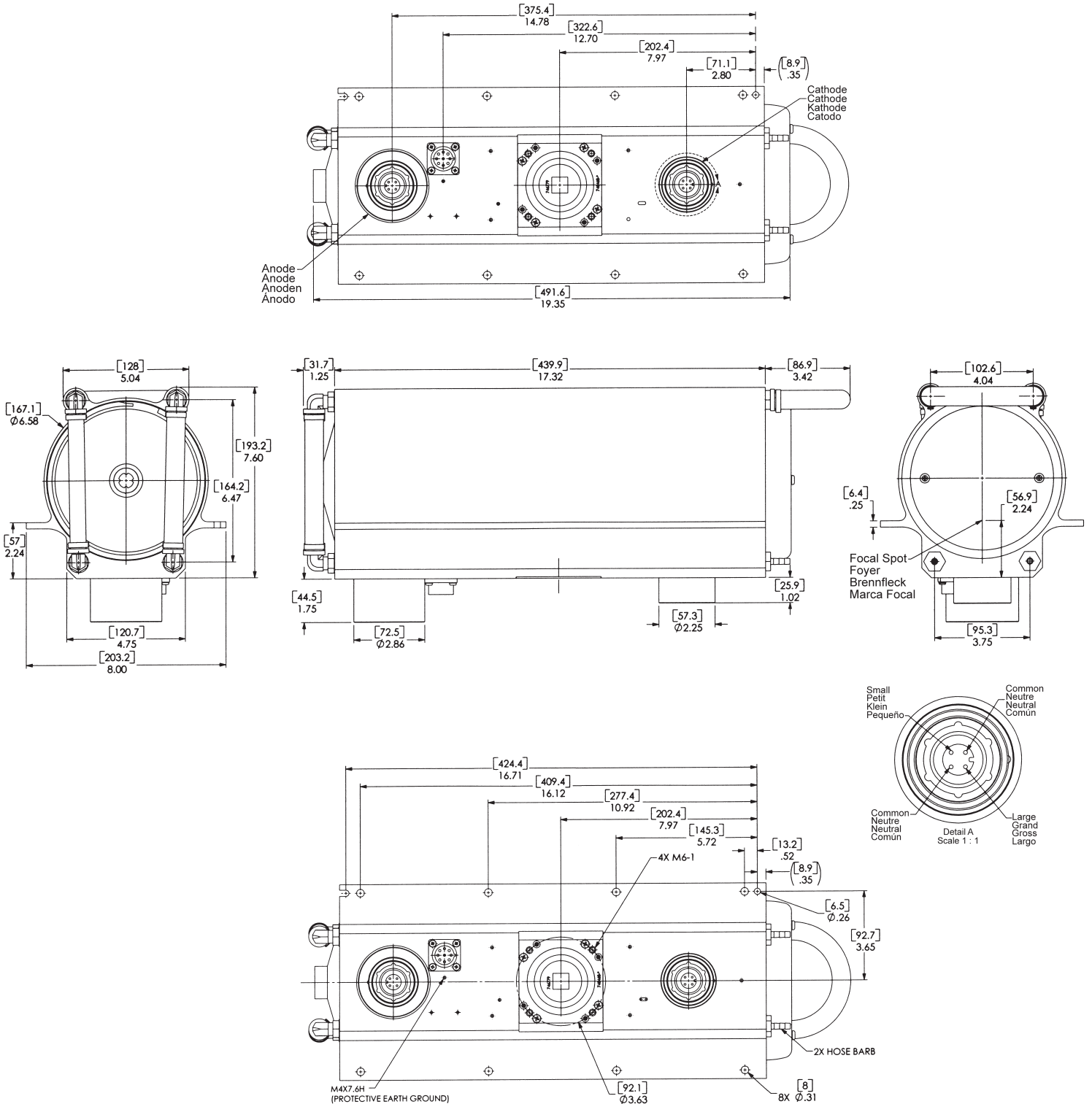
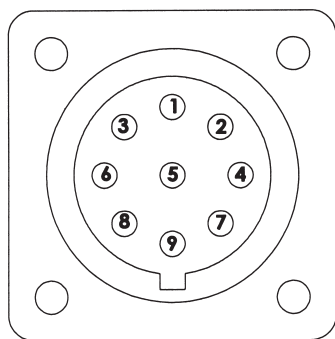
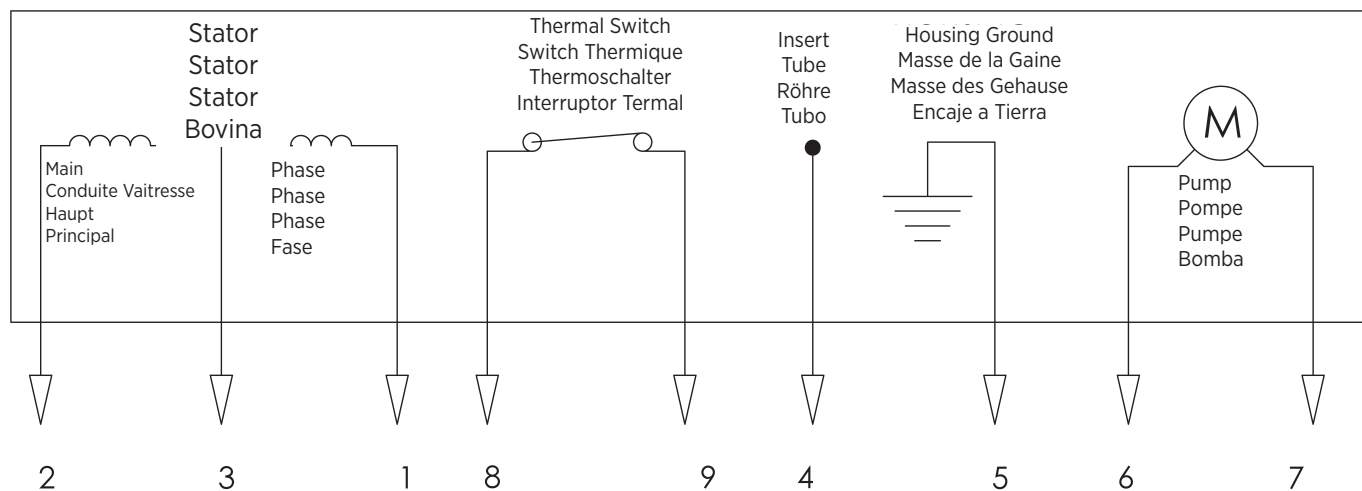
Stator Power:
 Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. The stator is rated for regular speed and high speed starters. Time to full speed of 102 mm (4 inch) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 3600 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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
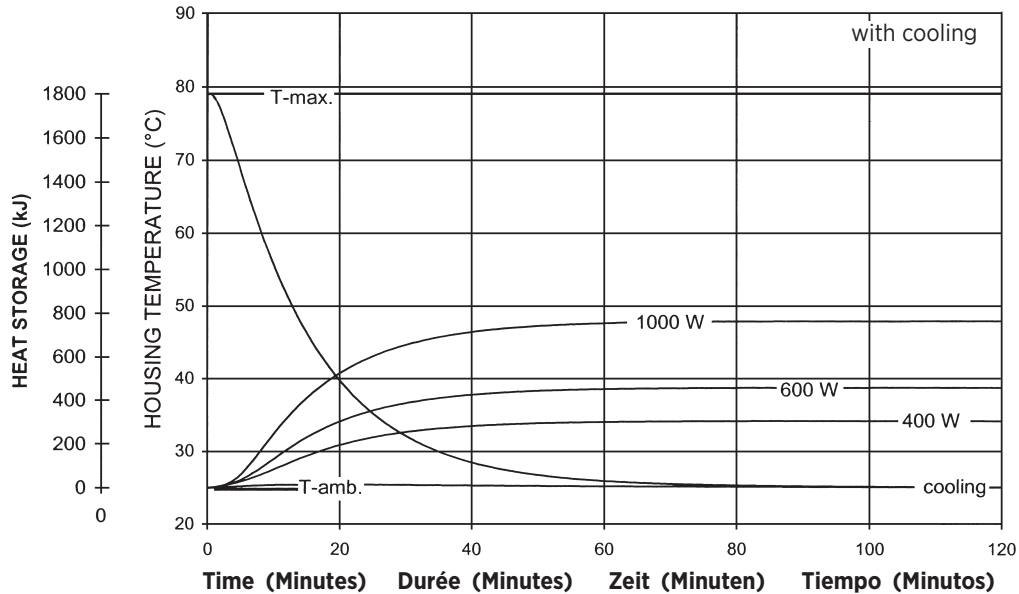
X-Ray Tube Assembly Heating and Cooling Curves

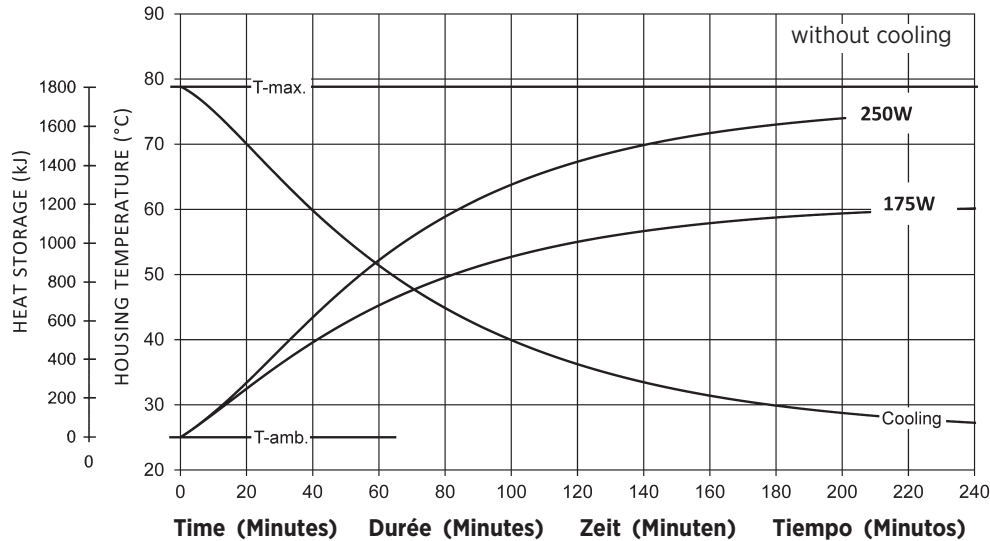X-Ray Tube Assembly Heating and Cooling Curves

Note:
 Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.
 Cooling performance measured with 25% propylene glycol and water at 2 liters per minute. Use an inhibitor to prevent corrosive of aluminum.

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Salt Lake City, UT 1-801-972-5000

*For a complete listing of our global offices,
visit www.vareximaging.com*

Manufactured by Varex Imaging Corporation
Fabrique par Varex Imaging Corporation
Hergestellt von Varex Imaging Corporation
Fabricado por Varex Imaging Corporation

Specifications subject to change without notice.
Spécifications susceptibles d'être modifiées sans préavis.
Technische Daten ohne Gewähr.
Especificaciones sujetas a cambio sin previo aviso.