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**Product Description**

Nominal X-ray Tube Voltage ..... 125 kV  
 Maximum Cathode to Ground ..... 63 kV  
 Maximum Anode to Ground ..... 63 kV

Maximum X-Ray Tube Assembly Heat Content ..... 1,750 kJ (2,450 kHU)

Nominal Continuous Input Power ..... 1000 Watts - IEC 60613:2010  
 2 Liters/minute, 20°C to 40°C fluid

Maximum Housing Temperature ..... 78°C

X-Ray Tube Assembly  
 Permanent Filtration ..... 0.7 mm Al/75 kV IEC 60522/1999

Loading Factors for Leakage Radiation ..... 125 kV, 8.0 mA

Temperature Limits for Storage and Transport ..... -20°C to +75°C  
 Humidity ..... +10% to +90%  
 Atmospheric Pressure Range ..... 70 kPa to 106 kPa

Thermal Switch ..... Rating - 7A @ 120Vac or 30 Vdc max.  
 Normally Closed ..... Opens @ 85°C ±3.9°C (185°F ±7°F)

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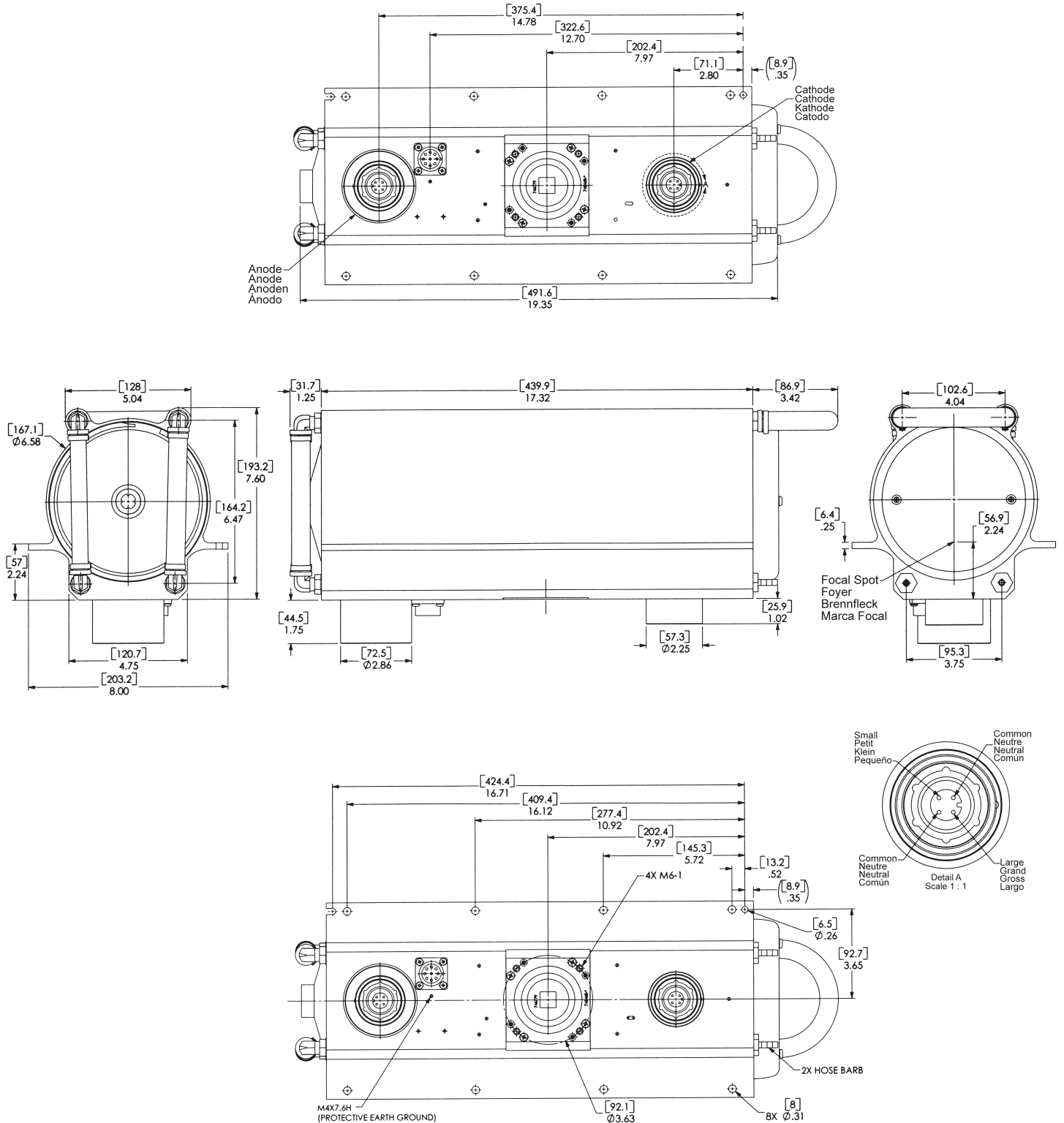
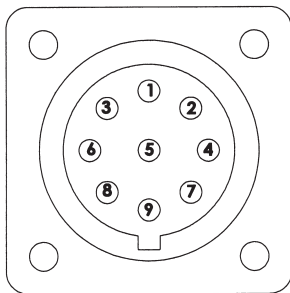
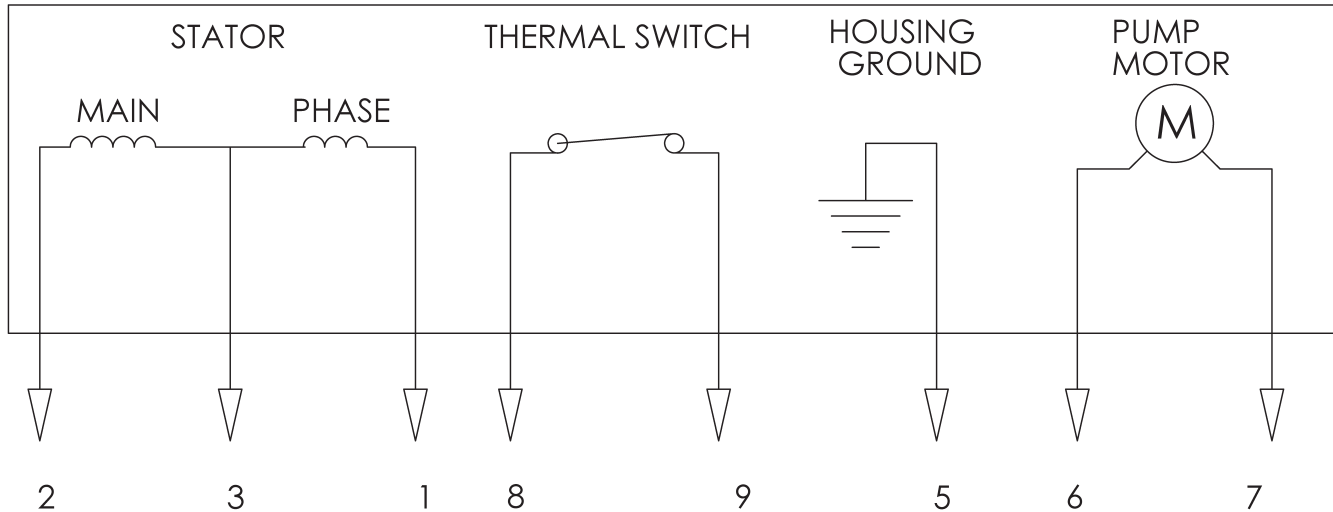
**Stator Power:**

Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. The stator is rated for regular speed and high speed starters. Time to full speed of 102 mm (4 inch) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 3600 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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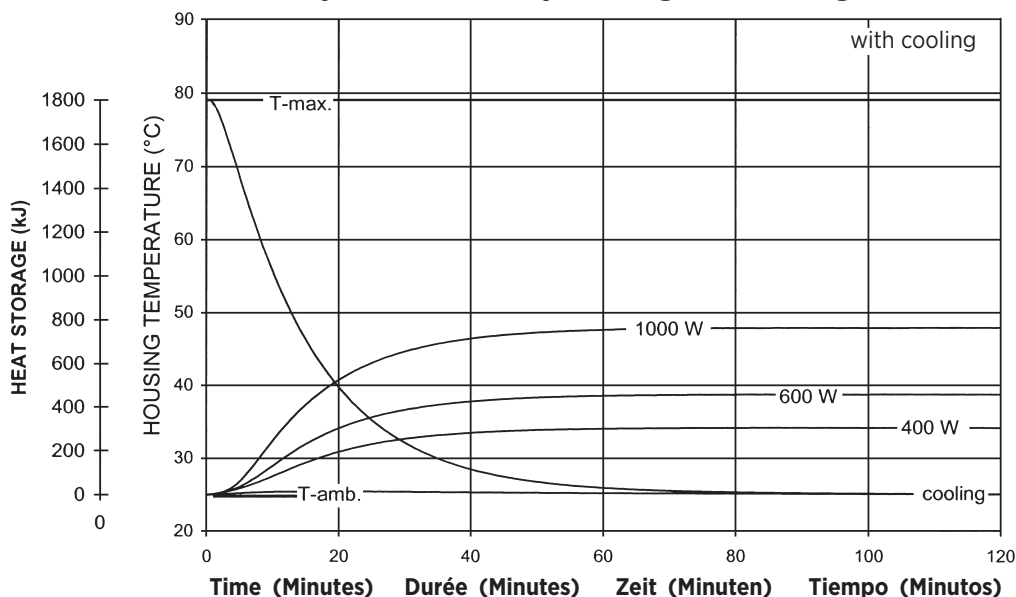
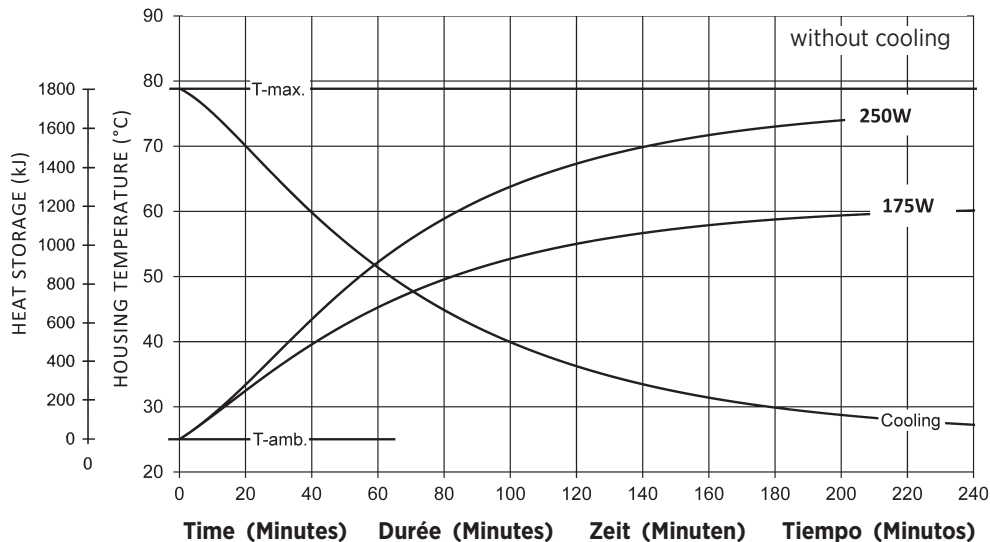
**X-Ray Tube Assembly Heating and Cooling Curves**

**X-Ray Tube Assembly Heating and Cooling Curves**

**Note:**

Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

Cooling performance measured with 25% propylene glycol and water at 2 liters per minute. Use an inhibitor to prevent corrosive of aluminum.

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Salt Lake City, UT 1-801-972-5000

*For a complete listing of our global offices,  
visit [www.vareximaging.com](http://www.vareximaging.com)*

Manufactured by Varex Imaging Corporation  
Fabrique par Varex Imaging Corporation  
Hergestellt von Varex Imaging Corporation  
Fabricado por Varex Imaging Corporation

Specifications subject to change without notice.  
Spécifications susceptibles d'être modifiées sans préavis.  
Technische Daten ohne Gewähr.  
Especificaciones sujetas a cambio sin previo aviso.