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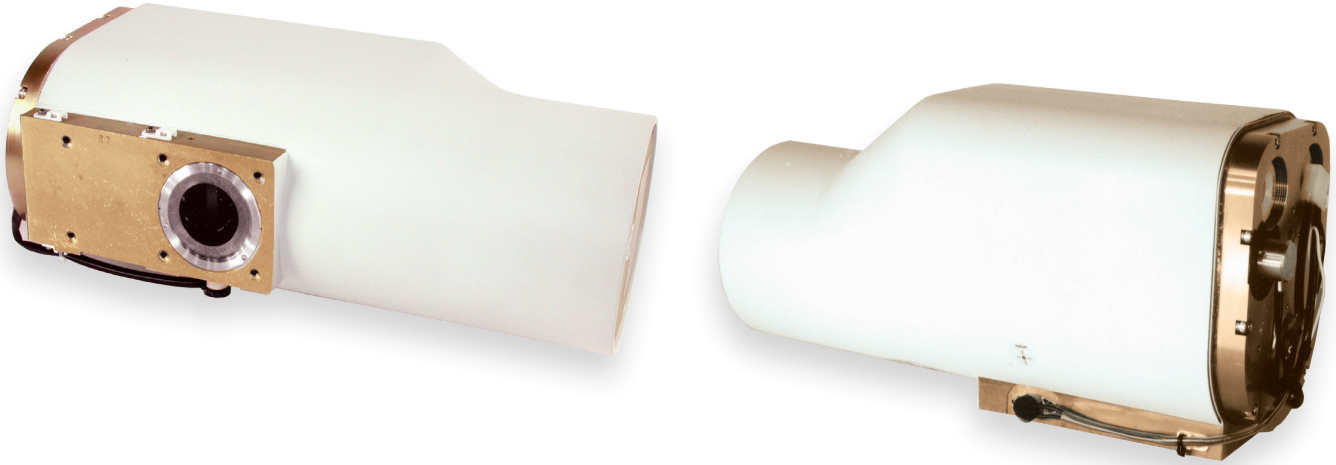
Note: Document originally drafted in the English language.

Product Description

The ORIII is a uniquely designed x-ray tube housing developed for use in surgical C-Arm applications. This design utilizes a high performance rotating anode tube, with inherently greater target loading capabilities than that of stationary anode tube designs. This housing provides ease of use at a nominal patient table height.

Housing: Shockproof, Aluminum. Lead lined; filled under vacuum with high dielectric insulating oil and hermetically sealed. The housing contains a stator for driving the rotor.

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Tube Housing Specifications

| | |
|----------------------|---------|
| Maximum Peak Voltage | |
| Anode to Cathode | 125 kV |
| Anode to Ground | 62.5 kV |
| Cathode to Ground | 62.5 kV |

Nominal Continuous Input Power 185 W (15 kHU**/Min.)
IEC 60613:2010

Maximum Housing Temperature 78°C

X-ray tube assembly cooling provided by ambient air flow.

Thermal Characteristics

Housing heat storage capacity 1,185 kJ (1.6 MHU**)

Weight 19.5 Kg (43 lbs.) without cables or accessories

X-Ray Tube Assembly

Permanent Filtration 0.7mm Al minimum at 75 kV - IEC 60522/1999

Loading Factors for Leakage Radiation 125 kV, 1.5 mA

X-Ray Coverage:

8.5° target - 12 in (30 cm) at 40 in (100 cm) Distance Source-Image

10° target - 14 in (35 cm) at 40 in (100 cm) Distance Source-Image

Thermal Switch

The switch must be used in conjunction with the control manufacturer's interlock circuitry. The thermal switch is normally closed. The switch will open when the temperature reaches 85°C ±2.8°C.

CAUTION

The thermal switch does not protect the insert against overheating.

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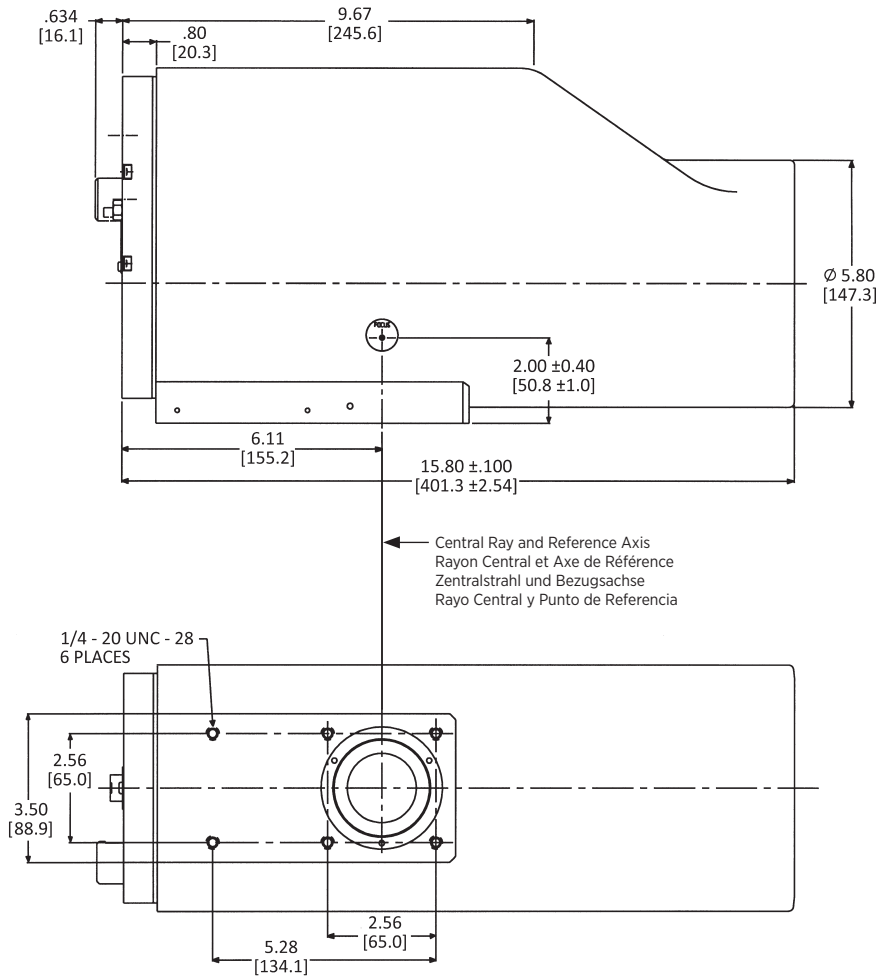
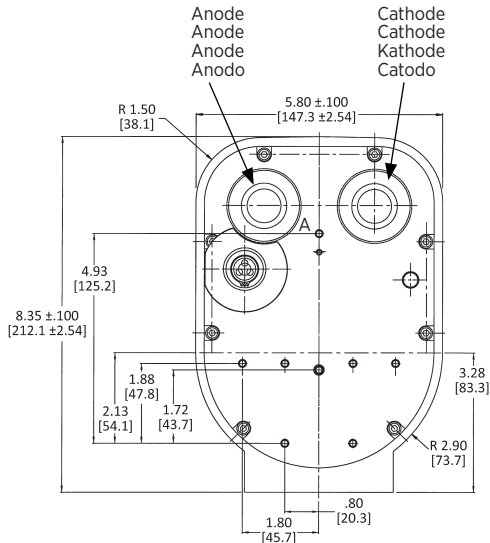
INCHES
 [mm]

Cable Receptacles:

Alden type, P702-1 (cathode) and P702 (anode), high voltage receptacles. The receptacles mate with Alden type (P7010S) high voltage connector and cable.

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Rayproofing: Stray radiation complies with DHHS standards under the "Radiation Control for Health and Safety Act 1968" (U.S.A.)

Etanchéité Radioactive:

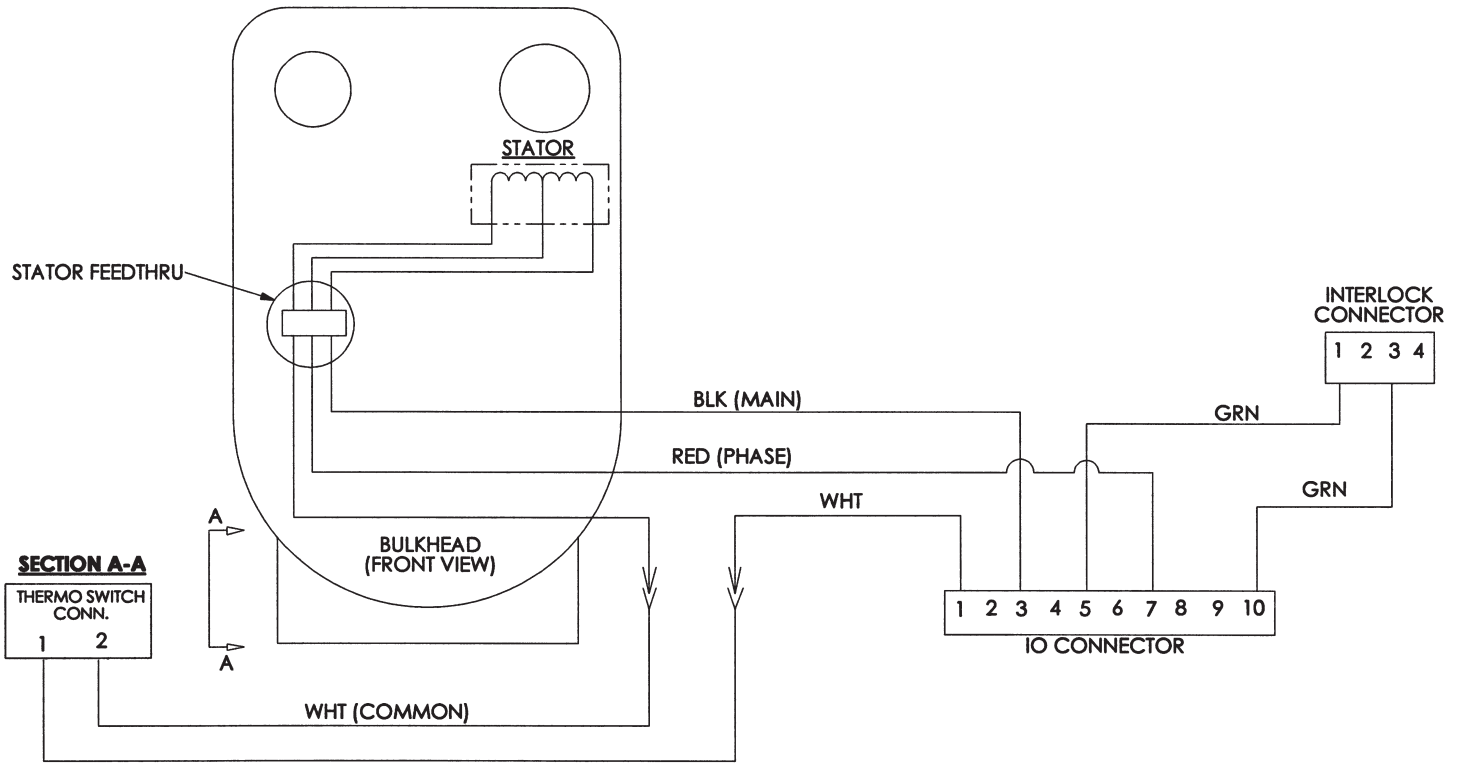
Les pertes en radiation sont conformes aux normes DHHS du "Radiation Control for Health and Safety Act 1968" (U.S.A.).

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Der Strahlenverlust entspricht den Vorschriften des DHHS "Radiation Control for Health and Safety Act 1968" (U.S.A.).

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Radiación de escape se ajusta a las normas de DHHS del "Radiation Control for Health and Safety Act 1968" (E.E. U.U.)

| Stator Drive Frequency Fréquence d'entraînement du stator Statorantrieb Frequenz Frecuencia de la impulsión del estator | RPM |
|--|---------------|
| 50 Hz | 2800 - 3000 |
| 60 Hz | 3400 - 3600 |
| 150 Hz | 8500 - 9000 |
| 180 Hz | 9500 - 10,800 |

| "R" Stators | Stator "R" | "R" Stator | "R" Bovina | Nominal | Acceptable |
|---------------|---------------|-----------------|----------------|---------|------------|
| Black - White | Noir - Blanc | Schwarz - Weiss | Negro - Blanco | 20 Ω | 18 - 22 |
| White - Red | Blanc - Rouge | Weiss - Rot | Blanco - Rojo | 50 Ω | 45 - 55 |
| Black - Red | Noir - Rouge | Schwarz - Rot | Negro - Rojo | 70 Ω | 63 - 77 |

Stator: The ORIII housing is equipped with a standard Varex Imaging stator requiring a 30 microfarad capacitor, providing 90° phase shift for 50/60 Hz., 2,850/3,450 rpm operation.

Stator Motor Ratings:

| Frequency (Hz.) | Volts | Common Leg Current |
|-----------------|-------|--------------------|
| 50-60 | 60* | Approx. = 2.0 Amps |
| 50-60 | 120 | Approx. = 4.0 Amps |
| 50-60 | 240 | Approx. = 8.0 Amps |

*Running Voltage

Stator Cord: A six wire shield-grounded stator cord should be connected from the stator terminal to the motor control. All listed voltage and amperage measurements should be taken at the housing end.

If the stator cord is not provided, the cord used must be UL listed or a UL recognized component for all products used in the USA. Outside the USA, the cord used must comply with all applicable regulatory and statutory requirements for electrical and safety. The cord must have the following characteristics: six conductor, 18AWG (0.9mm²) Cu, rated 600V, 90°C minimum, shielded cable with PVC jacket 0.032 inch (0.081mm) thick minimum. Lead terminations are provided and must be used with wire that matches the above description.

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| Frequenz (Hz) | Volts | Klemmen |
|---------------|-------|--------------------|
| 50-60 | 60* | Approx. = 2.0 Amps |
| 50-60 | 120 | Approx. = 4.0 Amps |
| 50-60 | 240 | Approx. = 8.0 Amps |

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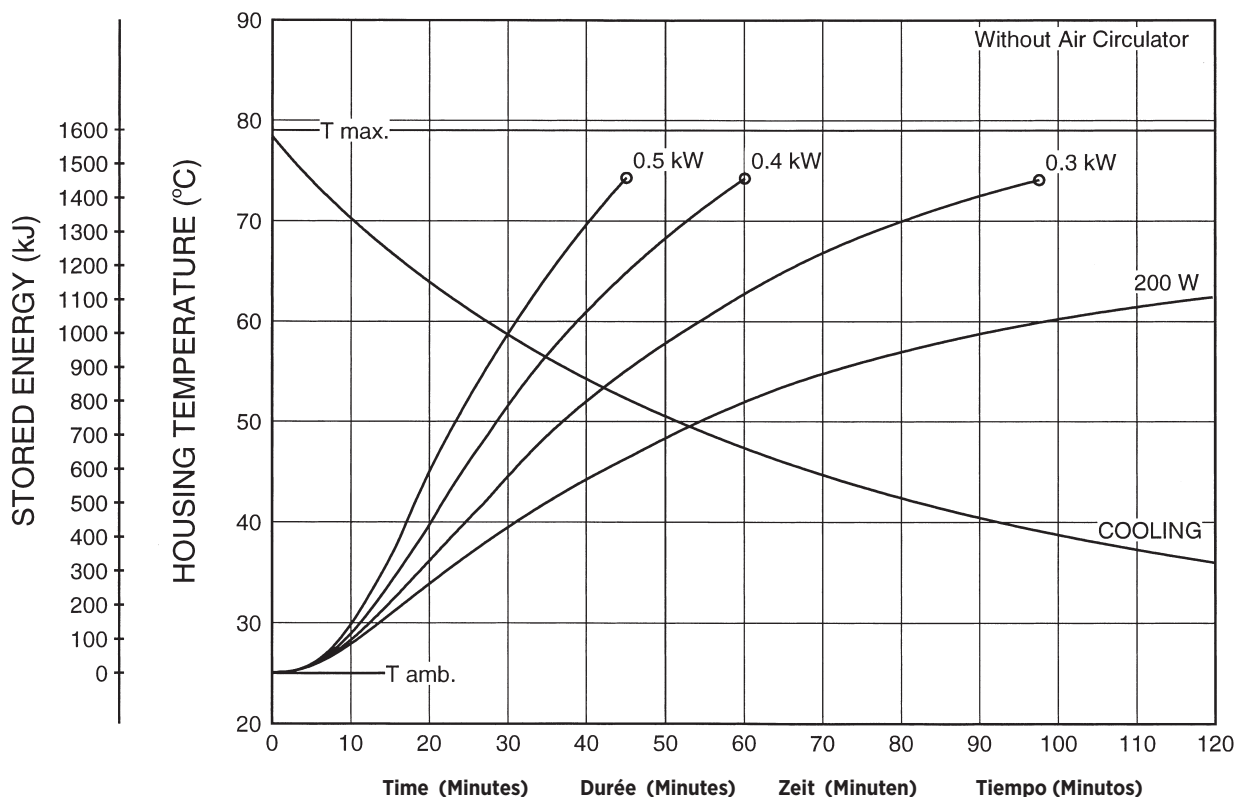
X-ray Tube Assembly Heating and Cooling Curve

Operating Temperature: Minimum ambient temperature for storage and transportation is -20°C (0°F). Operating range of the ORIII housing is 16° - 72°C (60° - 167°F).

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