Note: Document originally drafted in the English language.

Product Description

The ROT351™ housing is used for rotating anode inserts having 90mm (3.5 inch) diameter targets.

IEC Classification. Class 1

Weight, Approximate:

Housing & Tube. 24 kg (55 lbs)

Mounting Trunnion

™All trademarks property of the respective OEM's

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Product Description	
Maximum Potential Difference	150 kV
Cathode to Ground	85 kV
Anode to Ground	85 kV
Maximum X-Ray Tube Assembly Heat Content	1,250 kJ (1,700 kHU)
Maximum Housing Temperature	85°C
Maximum Continuous Heat Dissipation with cooling water	500 W (700 HU/sec)
X-ray tube assembly Permanent Filtration	0.7mm Al IEC 60522
Loading Factors for Leakage Radiation	150 kV, 4.0 mA
Temperature Limits for Storage and Transport	-25°C to + 70°C
Humidity	10% to 90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Thermal Switch	normally Closed 5A @ 125 Vac (Activates on Bellow extension)
X-Ray Tube Assembly (Complies to)	IEC 60601-2-28

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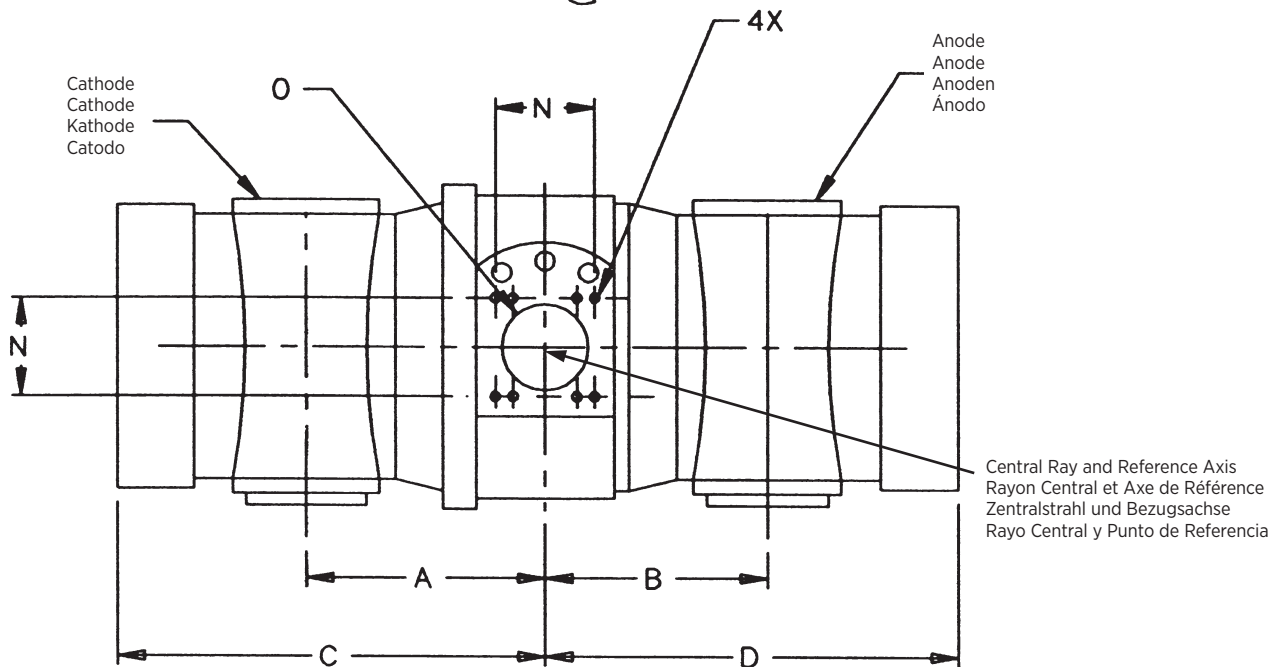
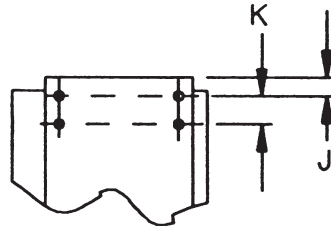
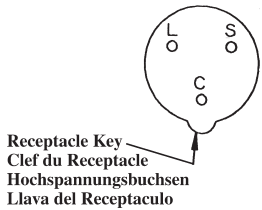
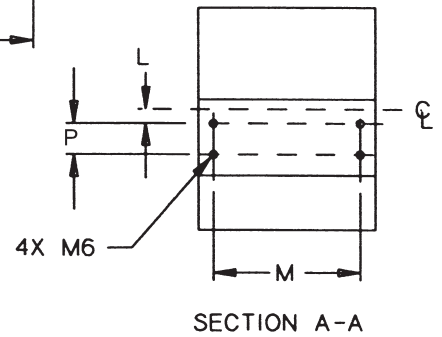
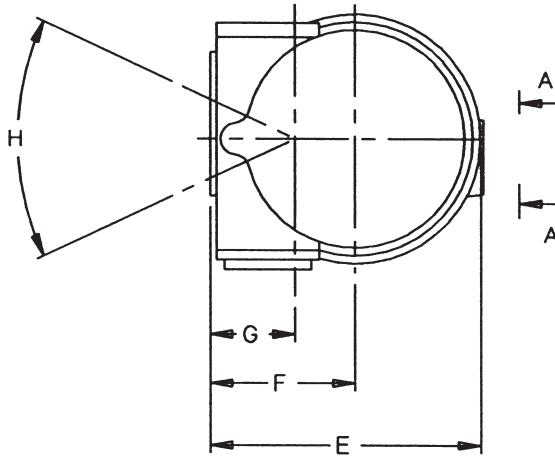
Stator Power:

Time to full speed of the anode is a function of the power rating of the “starter” and the weight / diameter of the anode. The ROT351 stator is rated for regular speed and high speed starters. Time to full speed (2800 RPM) of 3.5 inch (90mm) anode series tubes is approximately 1 second.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

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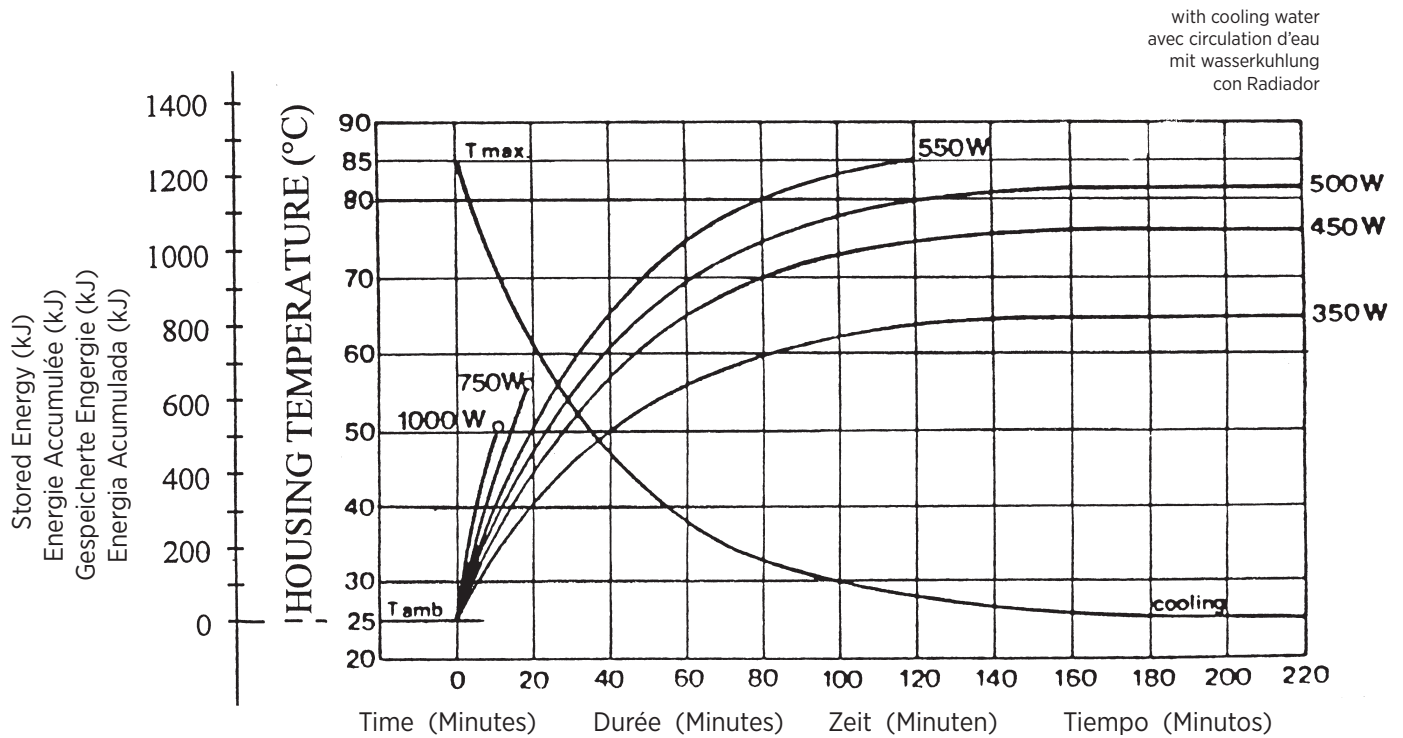
Note:

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.

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