Note: Document originally drafted in the English language.
注释：文件最初用英语起草。

Product Description

The GS-20721KD is a 5.25" (133 mm) 150 kV, 1,429 kJ (2.0 MHU) maximum anode heat content, rotating anode insert. This insert is specifically designed for Kangda CT Scanners. The insert features a 7° tungsten-rhenium facing on molybdenum with a graphite backed target and is available with the following nominal focal spot:

0.8
IEC 60336

Loading Factor for slit focal:

Large - 120 kV, 200 mA

Maximum Anode Cooling Rate:

4,000 W (5,600 HU/sec)

Continuous Anode Input Power:

3,000 W (4,200 HU/sec)

Nominal CT Anode Input Power:

Large - 29 kW IEC 60613:2010

Nominal CT Scan Power Index:

Large - 29 kW IEC 60613:2010

Reference Axis:

Perpendicular to port face.

This insert is intended for use in Varex Imaging B-220H housing.

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Note:
 1. Limits are based on maximum track rating except for the following codes:
 a - Limited by available heat storage.
 b - Limited by window heating.
 c - Limited by filament emission.
 2. H.S. = Heat Storage
 kV = Tube Voltage

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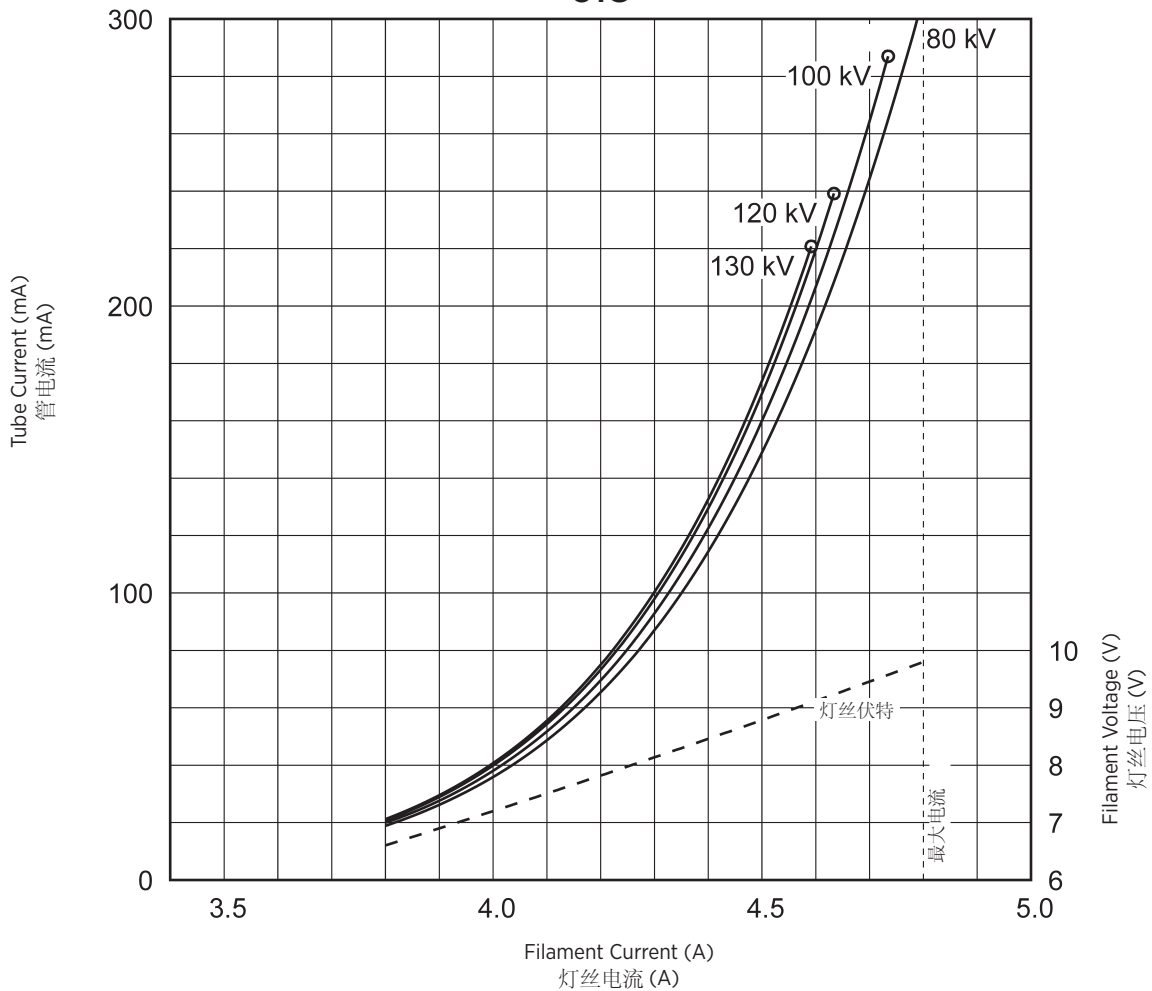
Note:
 Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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3 Ø ≡

THREE PHASE EMISSION ($\pm .15A$)
GS-20721KD 三相发射 ($\pm .15 A$)
0.8


<u>Product Description</u>	
Maximum Peak Voltage	150 kV
Anode to Ground	75 kV
Cathode to Ground	75 kV
Maximum X-ray Tube Assembly Heat Content ...	3.1 MJ (4.3 MHU)
Maximum Housing Temperature	78°C
Nominal Continuous Input Power (Includes stator heat)	3.2 kW (4,480 HU/sec) IEC 60613:2010
Maximum Heat Exchanger Dissipation	5.0 kW (7,050 HU/sec)
Focal Point Position (Central Ray) Within 1mm (X, Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent filtration	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation	150 kV, 20 mA
High Voltage Cable Receptacles	Per IEC 60526
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight - Housing	31.5 kg (69.4 lbs)
Heat Exchanger	16.3 kg (35.9 lbs)
IEC Classification	Class 1
Safety Devices: Thermal Switch	
Normally closed contact	Opening at 85°C ±3.9°C
Pressure Switch	
Normally Open contact	closes before 5.0 PSID Rising opens at 3.0 PSID ±0.2 PSID Falling
Filament Frequency Limits	50 Hz - 25 kHz
Power Supply	DC

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






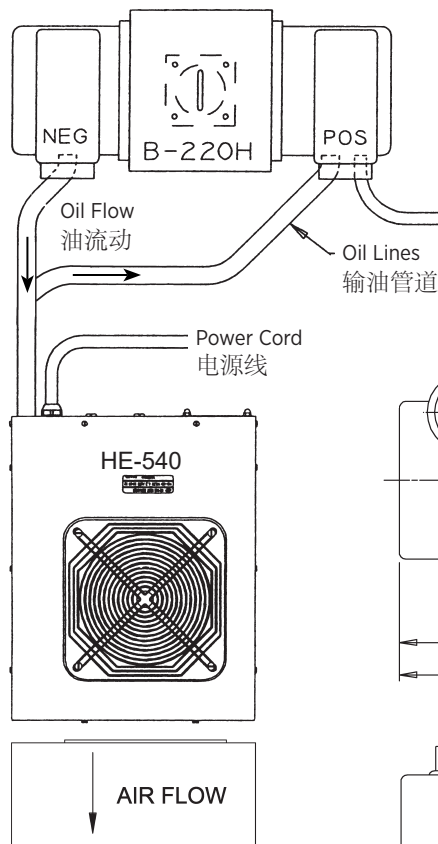
Stator Cord
定子电缆

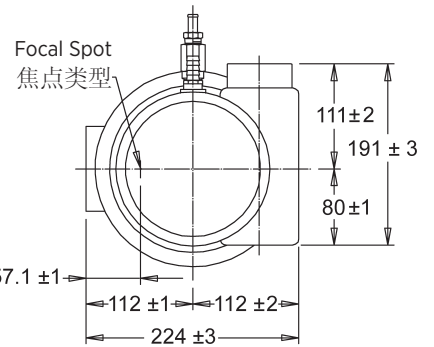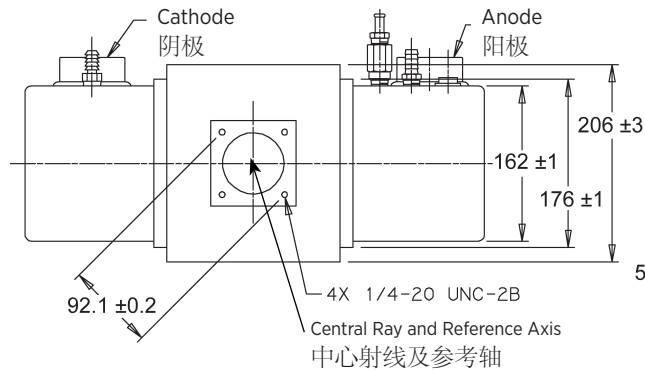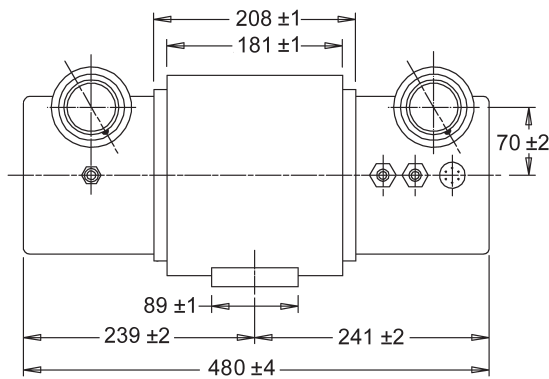HE-540

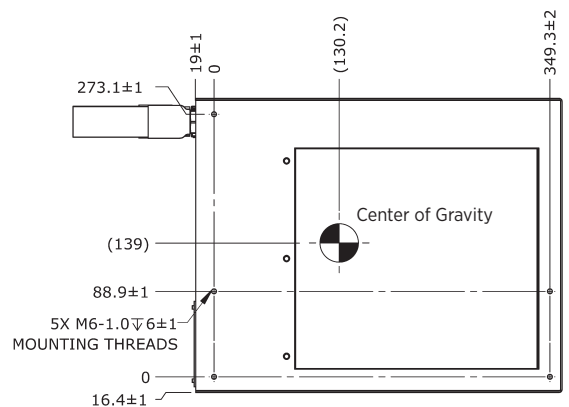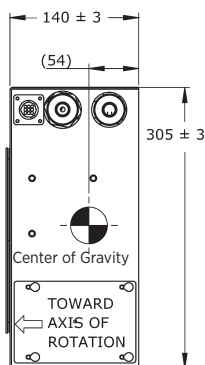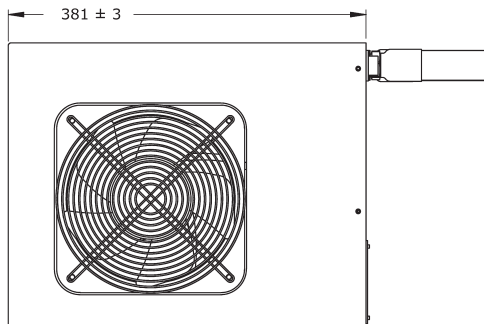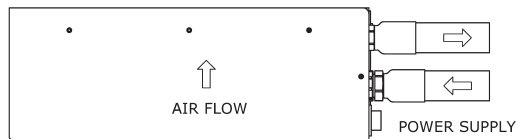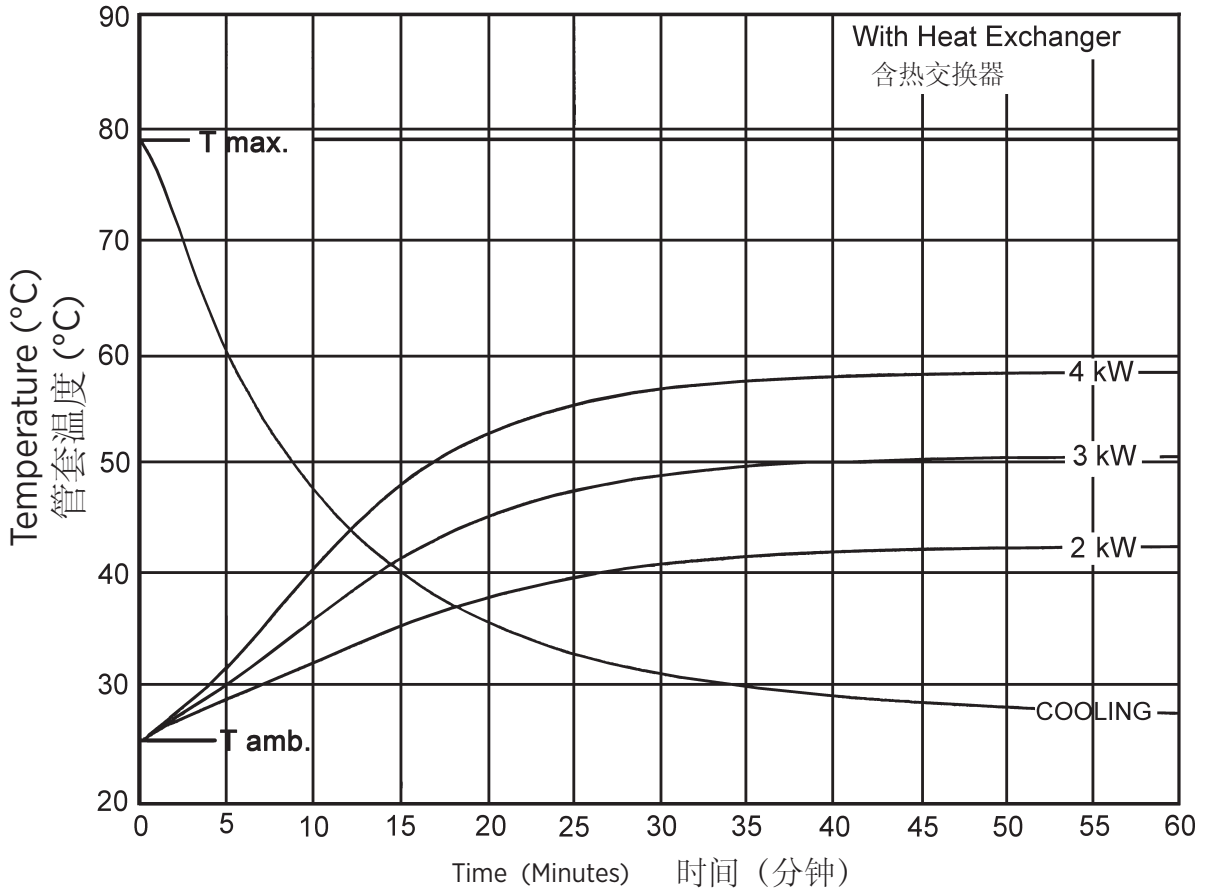X-Ray Tube Housing Assembly Heating and Cooling
X射线管组件加热/冷却曲线

Note:

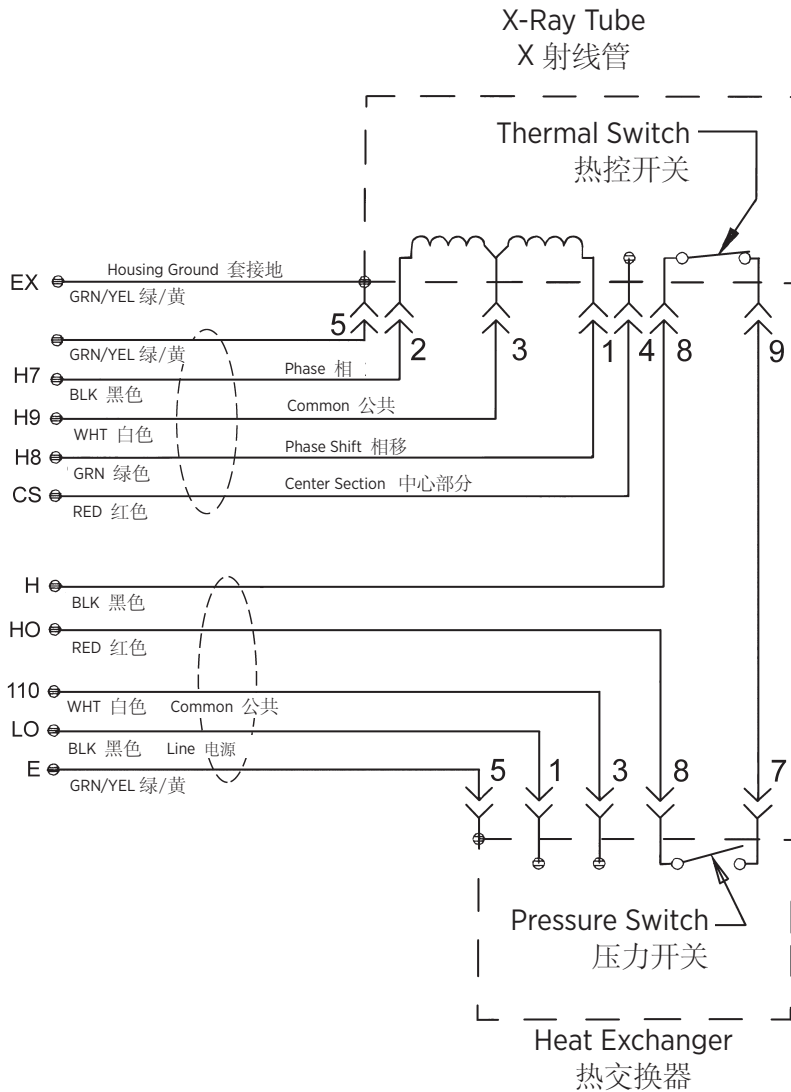
1. Heat inputs into housing include tube power, filament power, and stator power.
2. Heating curves based on no restrictions of natural convection around tube housing assembly.
3. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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Stator - Wiring Diagram 定子 — 连线图

Stator Ratings and Characteristics 定子额定值及特征

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
Stator Type: "R"	
Stator Coil Resistance:	
Black to White	14 Ohms ±15%
Green to White	46 Ohms ±15%
Starter Voltage:	
50/60 Hz	Start 222 VAC Run 72 VAC
Time to Full Speed:	
50/60 Hz	0 - 2700 RPM 8.5 Sec.
X-Ray Tube Assembly:	
GS-20721KD/B-220H	IEC 60601-2-28

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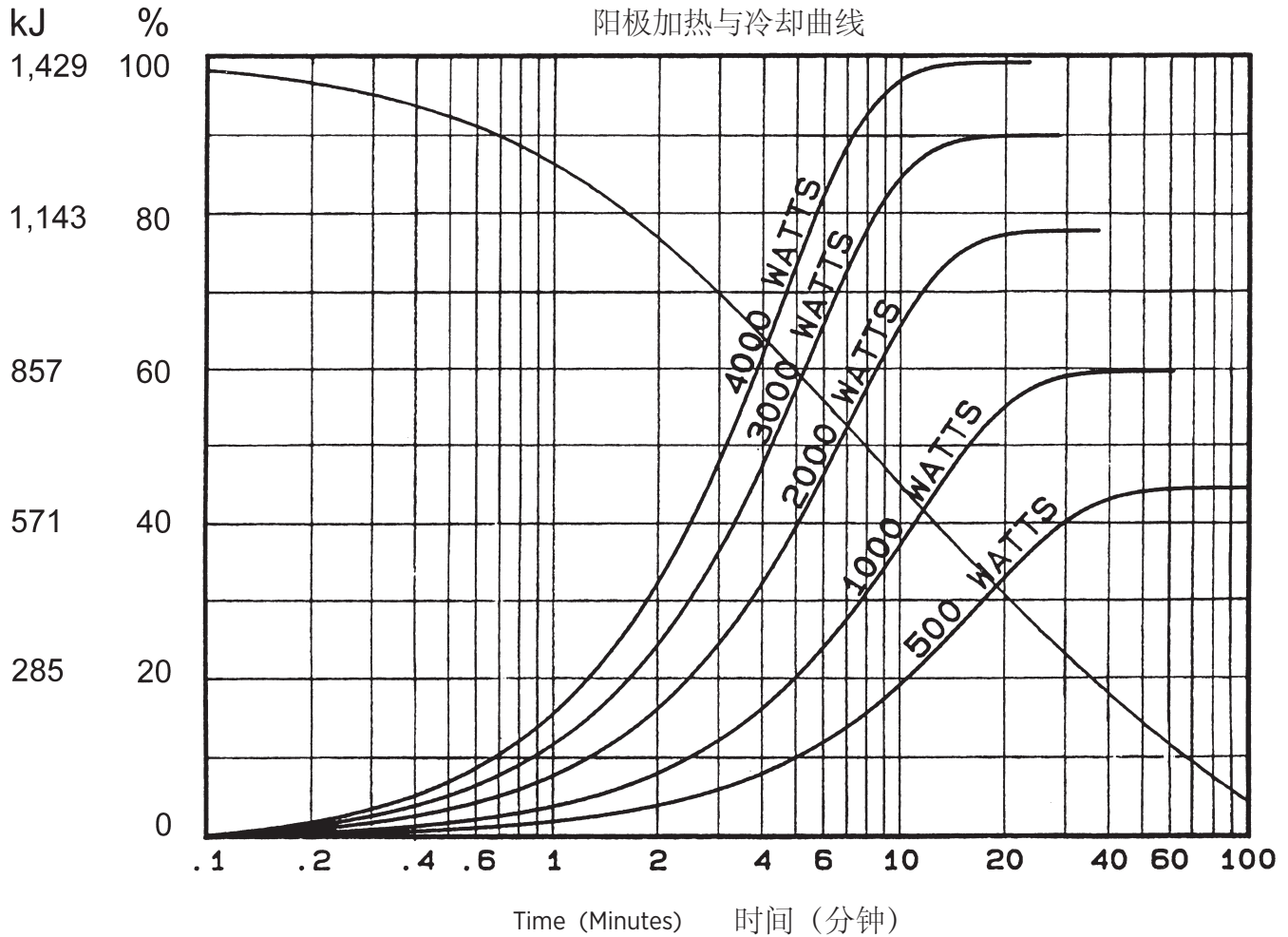Heating and Cooling Curves
阳极加热与冷却曲线

Note:
Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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