Note: Document originally drafted in the English language.

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Product Description

The GS-5171 is a 7.5" (190.5 mm) 140 kV, 3.8 MJ (5.3 MHU) maximum anode heat content, rotating anode insert. This insert is specifically designed for Anke CT Scanners. The insert features a 7° tungsten-rhenium facing on molybdenum with a graphite backed target and is available with the following nominal focal spot:

0.5 x 1.0
1.0 x 1.0
IEC 60336

Loading Factor for slit focal:
Small - 140 kV, 150 mA
Large - 140 kV, 210 mA

Maximum Anode Cooling Rate:
9,600 W (13,440 HU/sec)

Continuous Anode Input Power:
6,300 W (8,820 HU/sec)

Nominal CT Anode Input Power:
Small - 36 kW IEC 60613:2010
Large - 50.4 kW IEC 60613:2010

Nominal CT Scan Power Index:
Small - 36 kW IEC 60613:2010
Large - 49 kW IEC 60613:2010

Reference Axis:
Perpendicular to port face.

This insert is intended for use in Varex Imaging B-320H housing.

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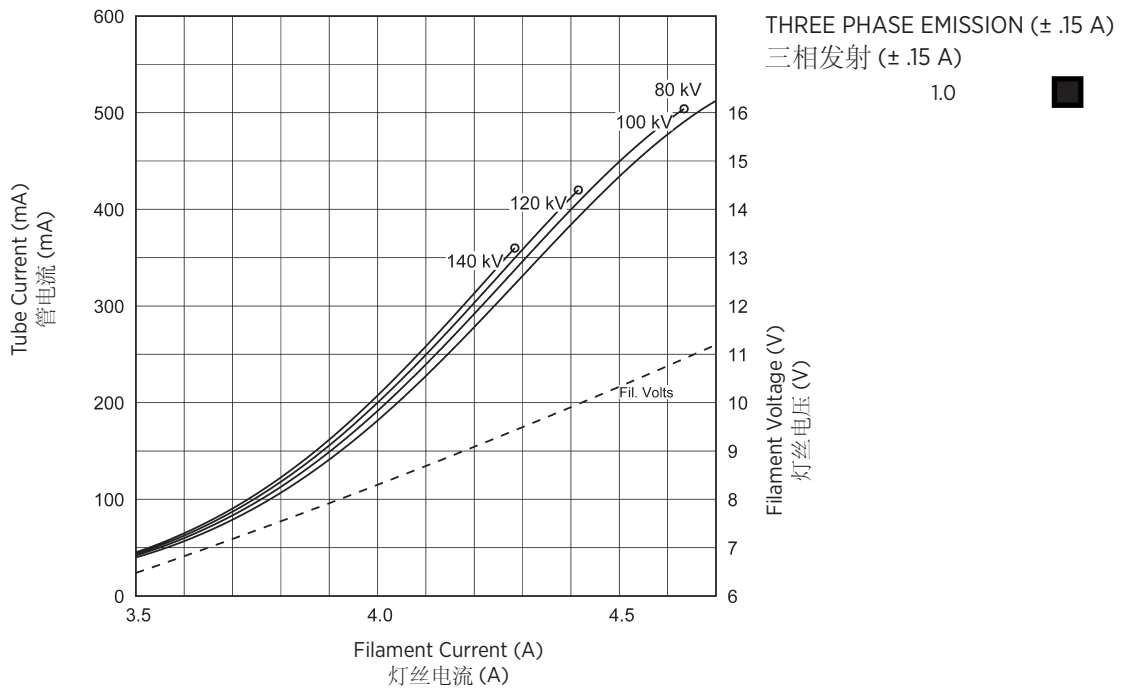
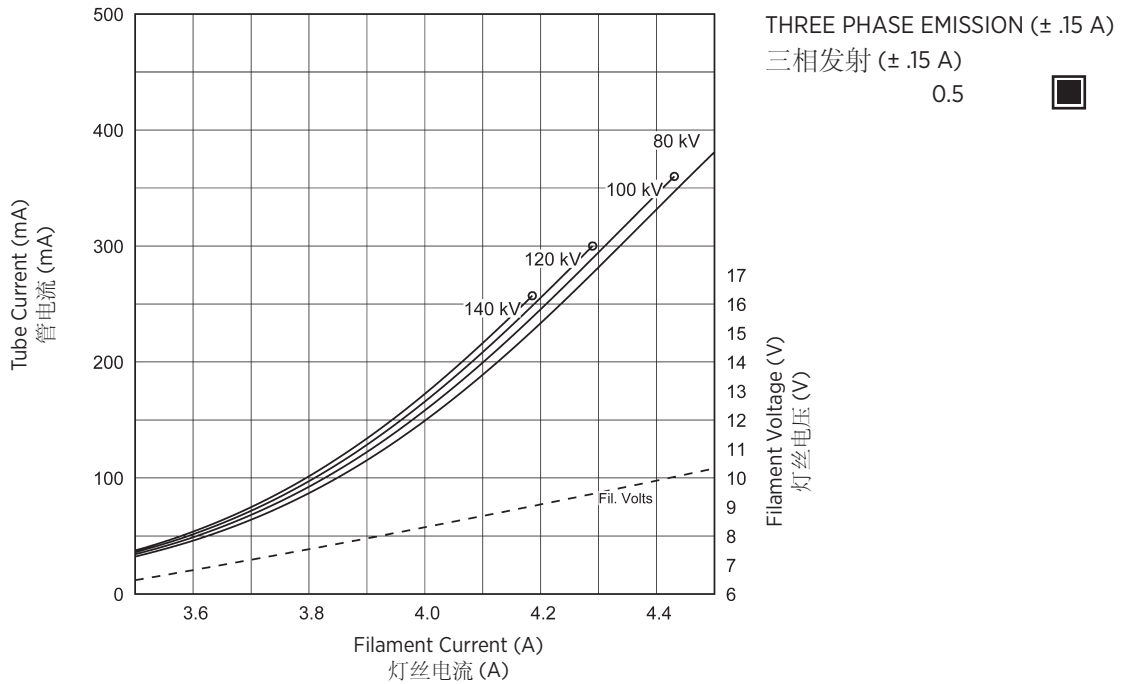
Note:
 1. Limits are based on maximum track rating except for the following codes:
 a - Limited by available heat storage.
 b - Limited by window heating.
 c - Limited by filament emission.
 2. H.S. = Heat Storage
 kV = Tube Voltage

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Note:
 Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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3 Ø 






<u>Product Description</u>	
Maximum Peak Voltage	140 kV
Anode to Ground	70 kV
Cathode to Ground	70 kV
Maximum X-ray Tube Assembly Heat Content ...	3.8 MJ (5.3 MHU)
Nominal Continuous Input Power (Includes stator heat)	6.4 kW (9.0 kHU/sec)
	IEC 60613:2010
Maximum Housing Temperature	80°C
Maximum Heat Exchanger Dissipation	6.4 kW (9.0 kHU/sec)
Focal Point Position (Central Ray) Within 1mm (X, Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent Filtration	0.5 mm Al @ 75 kV IEC 60522/1999
Additional Filtration	1.5 mm Al @ 75 kV IEC 60522/1999
Minimum Total Filtration ...	2.0 mm Al @ 75 kV IEC 60522/1999
Loading Factors for Leakage Radiation	140 kV, 43 mA
High Voltage Cable Receptacles	Per IEC 60526
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight - Housing & Heat Exchanger	216 lbs. ±2.0 lbs.
IEC Classification	Class 1
Safety Devices - Thermal Switches	
External - Normally Closed Contact	Opening at 80°C ±3.9°C
Internal - Normally Closed Contact	Opening at 110°C ±4.4°C
Pressure Switch	
Normally Open contact	closes at 3.0 PSID ±1.0 PSID
Filament Frequency Limits	50 HZ - 40 KHZ
Power Supply	DC

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
Dimensions are in millimeters
尺寸单位为毫米

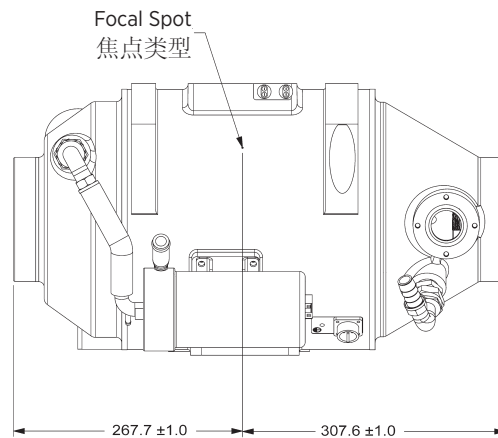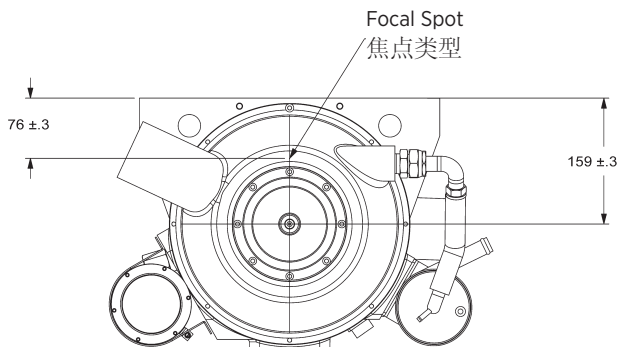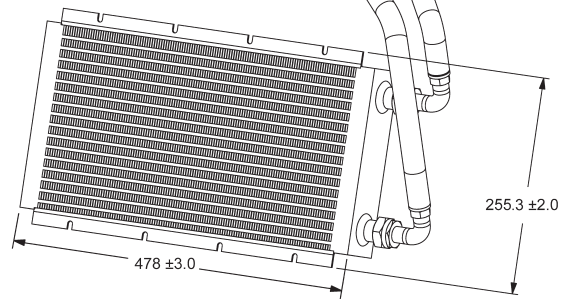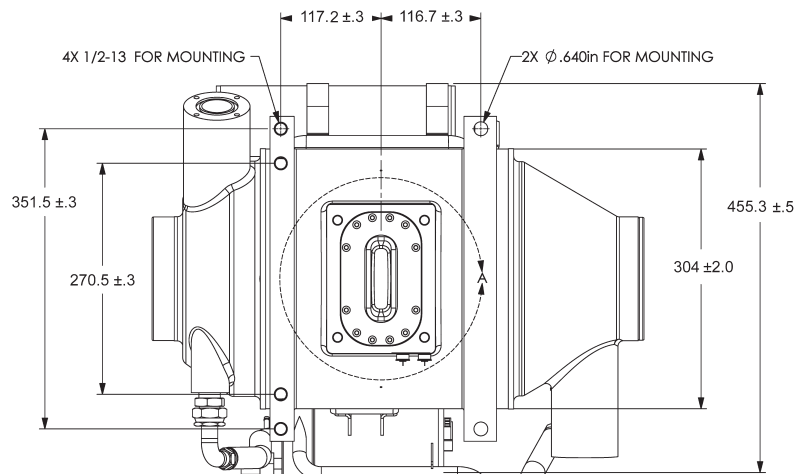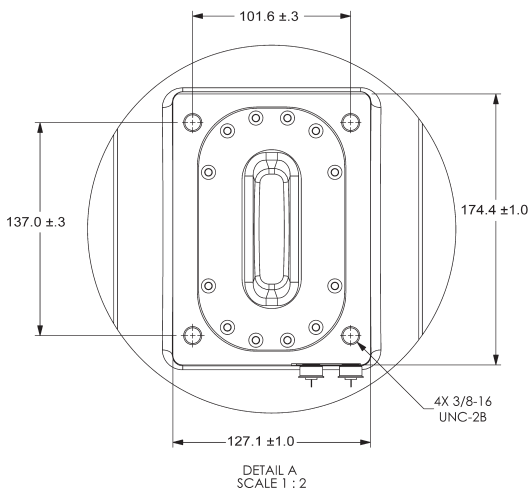Dimensions are for reference only
维度是供仅供参考

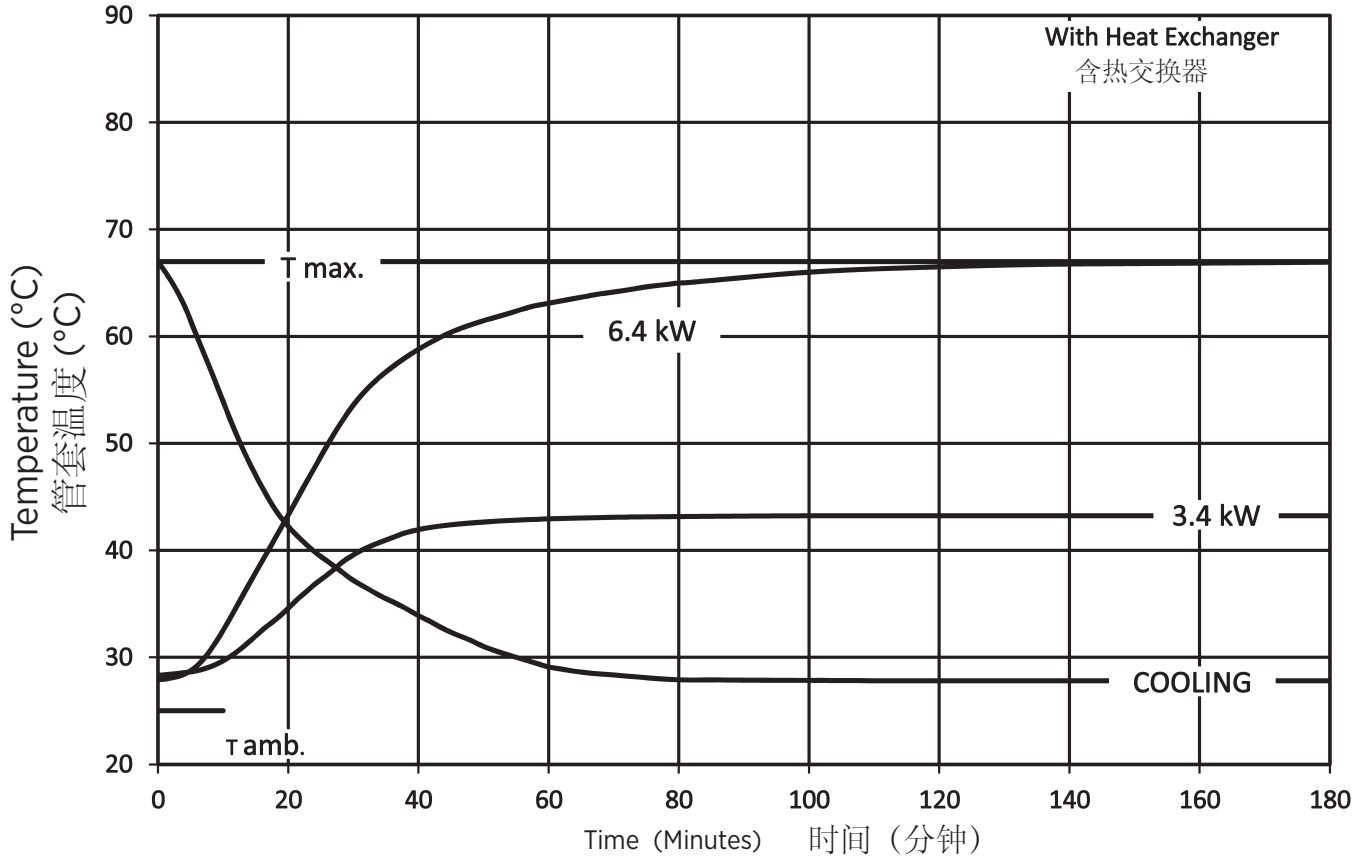
X-Ray Tube Housing Assembly Heating and Cooling
X射线管组件加热/冷却曲线

Note:
1. Heat inputs into housing include tube power, filament power, and stator power.
2. Heating curves based on no restrictions of natural convection around tube housing assembly.
3. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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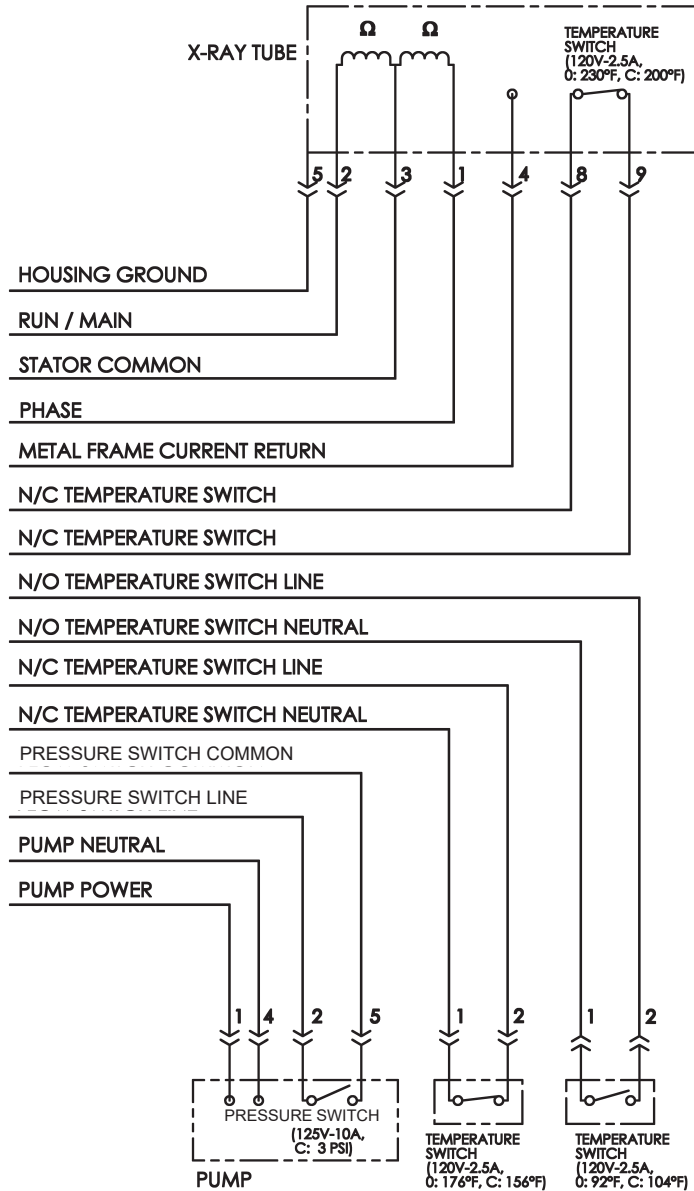
Stator Type:		
Stator Coil Resistance:		
Black to White	8.3 Ω	±15%
Green to White	8.3 Ω	±15%
Starter Voltage:	Start	Run
105 Hz	480 VAC	160 VAC
Starter Current:	Start	Run
105 Hz	9 Amps	3 Amps
Time to Full Speed:		
105 Hz	0 - 6300 RPM	20 Sec.
X-Ray Tube Assembly:	GS-5171/B-320H	IEC 60601-2-28

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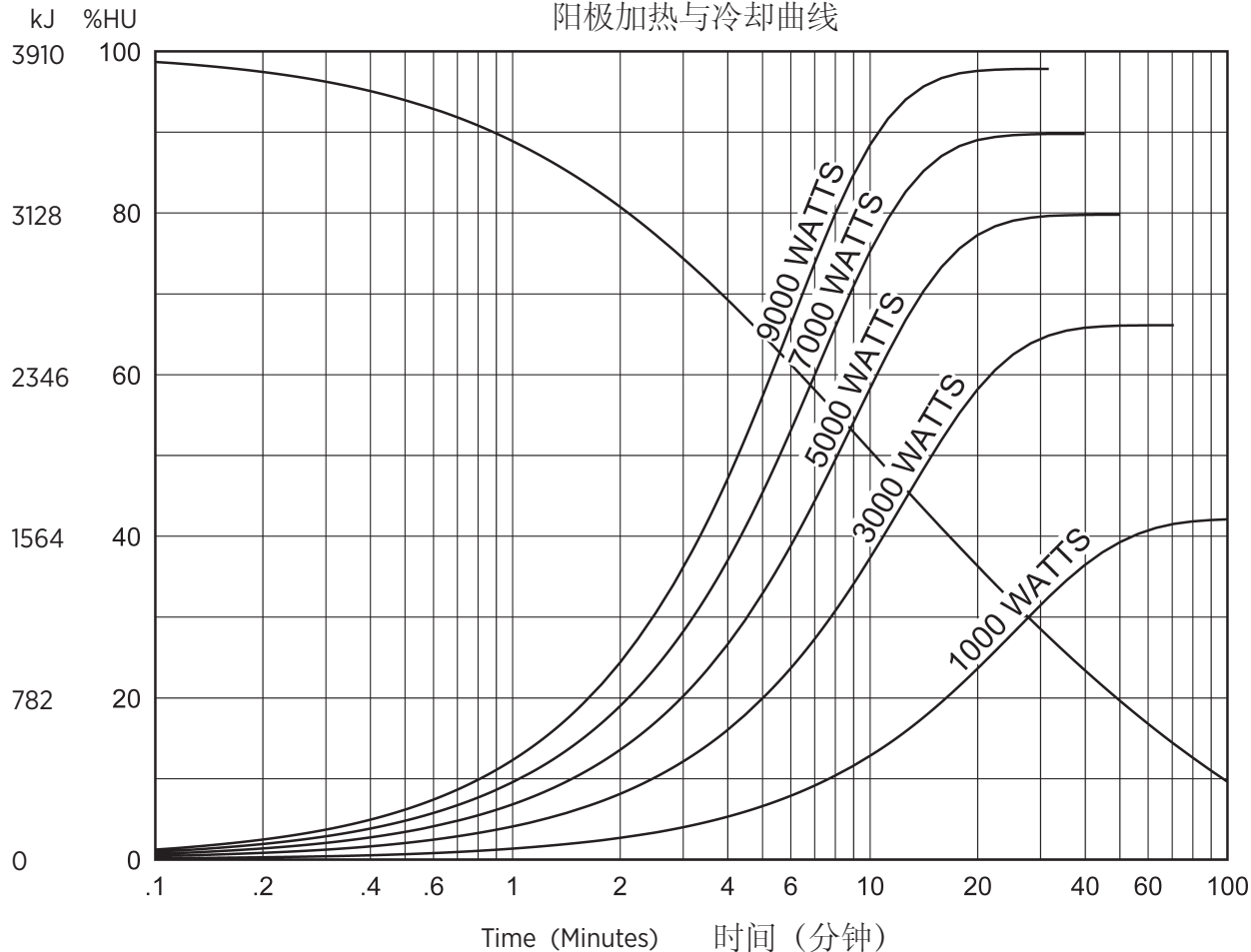Anode Heating and Cooling Curves
阳极加热与冷却曲线

Note:
Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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Manufactured by Varex Imaging Corporation
由万睿视影像有限公司生产