Rotating Anode X-Ray Tube Tubes  
Radiogènes à Anode Tournante  
Drehanoden - Röntgenröhre  
Tubos de Rayos-X con Ánodo Giratorio

Note: Document originally drafted in the English language.

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**Note:**  
 1. Limits are based on maximum track rating except for the following codes:  
 a - Limited by available heat storage.  
 b - Limited by window heating.  
 c - Limited by filament emission.  
 2. H.S. = Heat Storage  
 kV = Tube Voltage

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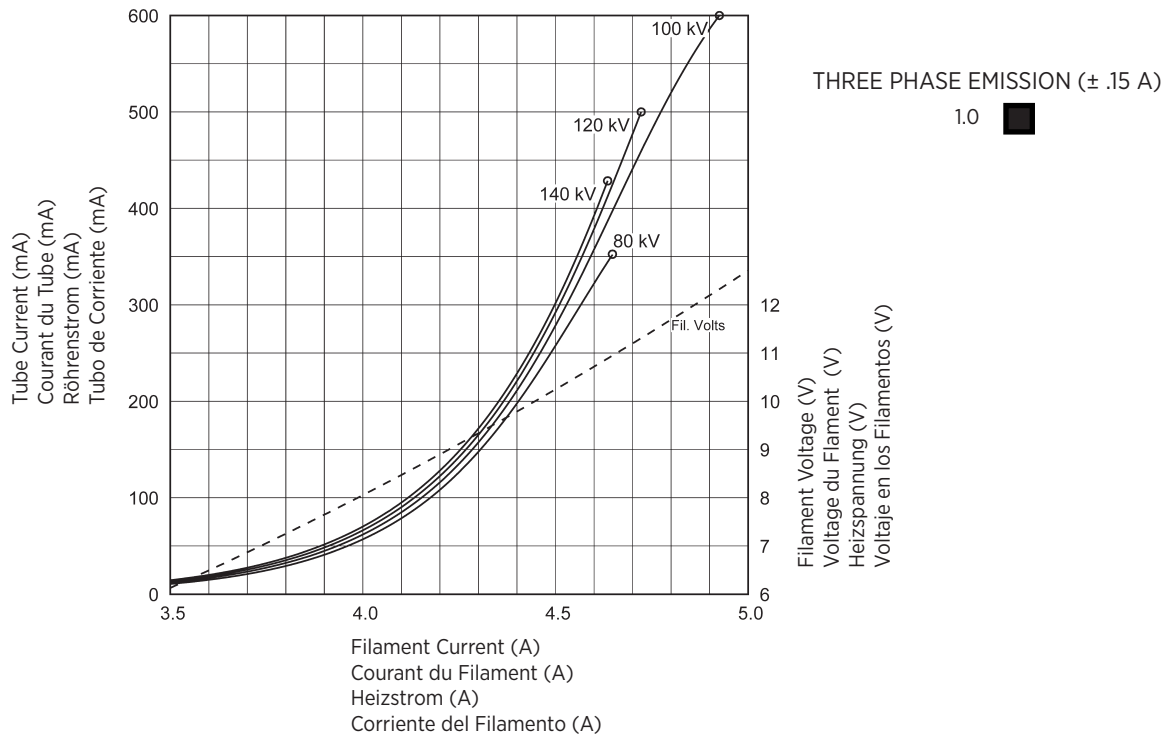
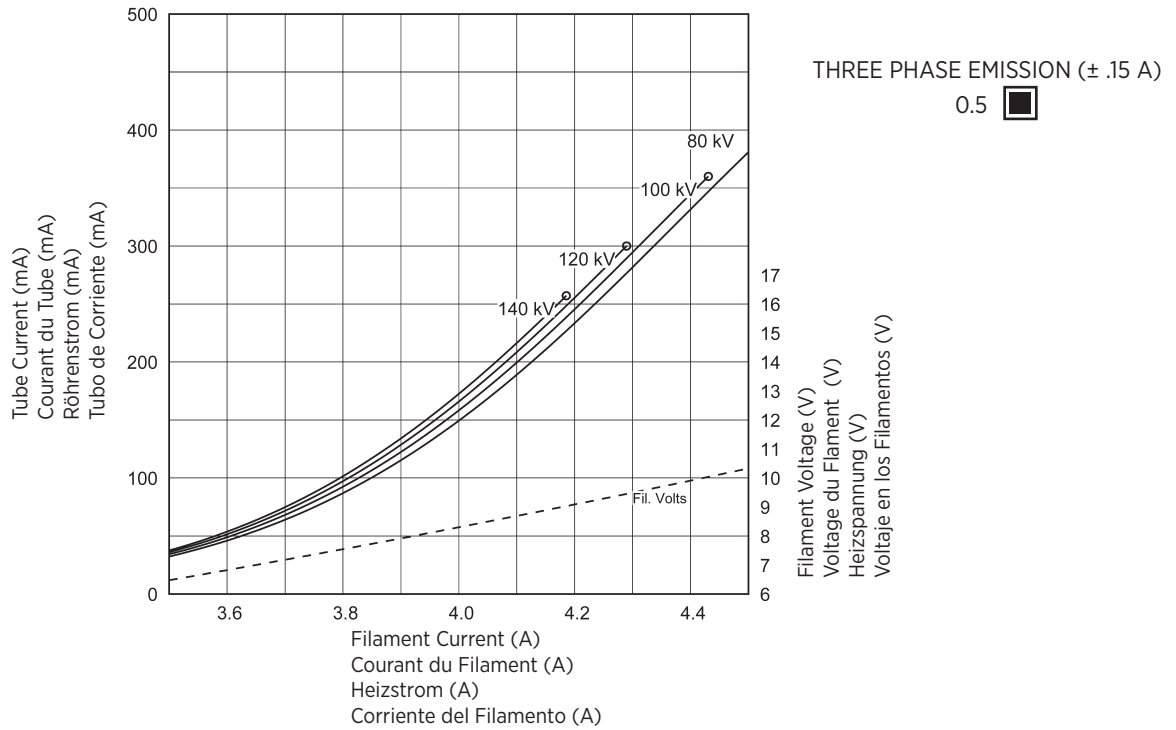
**Note:**  
 Rating charts reflect maximim tube performance. Tube operation is ultimately limited by system software.

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Maximum Peak Voltage	140 kV
Anode to Ground	70 kV
Cathode to Ground	70 kV
Maximum X-ray Tube Assembly Heat Content	3.8 MJ (5.3 MHU)
Nominal Continuous Input Power (Includes stator heat)	6.4 kW (9.0 kHU/sec) IEC 60613:2010
Maximum Housing Temperature	80°C
Maximum Heat Exchanger Dissipation	6.4 kW (9.0 kHU/sec)
Focal Point Position (Central Ray) Within 1mm (X, Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent Filtration	0.5 mm Al @ 75 kV IEC 60522/1999
Additional Filtration	1.5 mm Al @ 75 kV IEC 60522/1999
Minimum Total Filtration	2.0 mm Al @ 75 kV IEC 60522/1999
Loading Factors for Leakage Radiation	140 kV, 43 mA
High Voltage Cable Receptacles	Per IEC 60526
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight - Housing & Heat Exchanger	214.5 lbs. ±3.5 lbs.
IEC Classification	Class 1
Safety Devices - Thermal Switches	
External - Normally Closed Contact	Opening at 80°C ±3.9°C
Internal - Normally Closed Contact	Opening at 110°C ±4.4°C
Pressure Switch	
Normally Open contact	Contact closes at 3.0 PSID ±1.0 PSID
Filament Frequency Limits	50 HZ - 40 kHz
Power Supply	DC

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
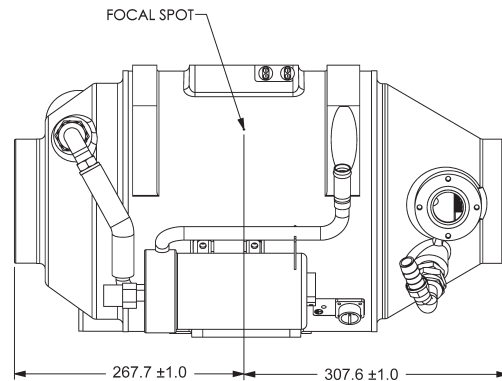
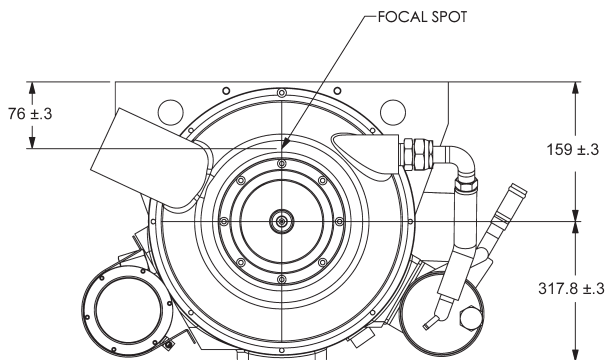
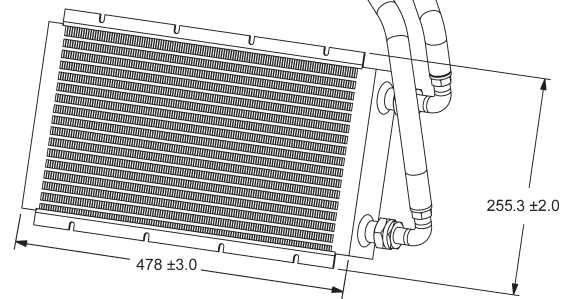
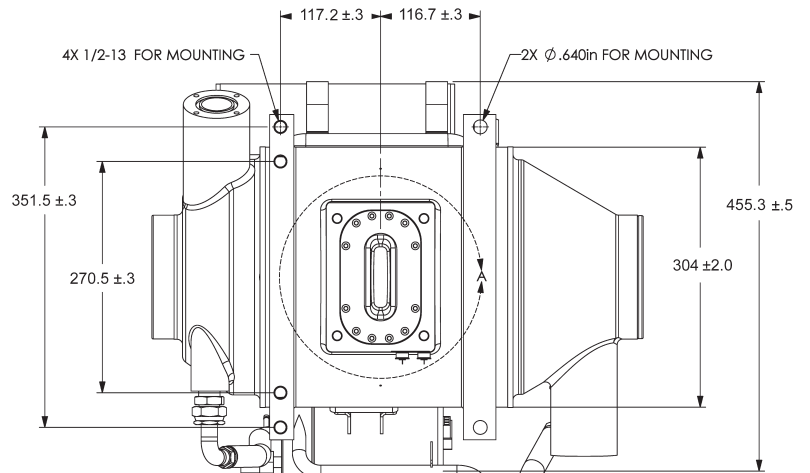
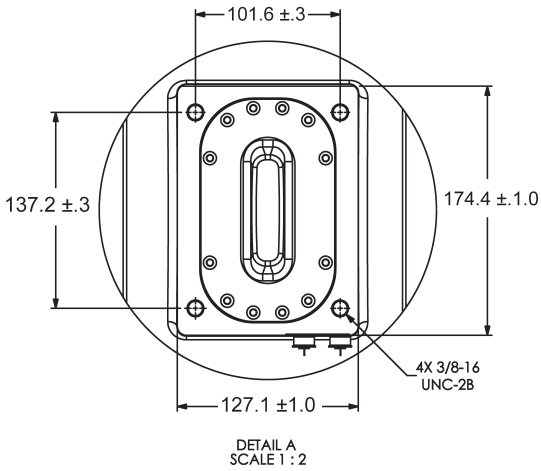
## TUBE HOUSING ASSEMBLY HEATING AND COOLING

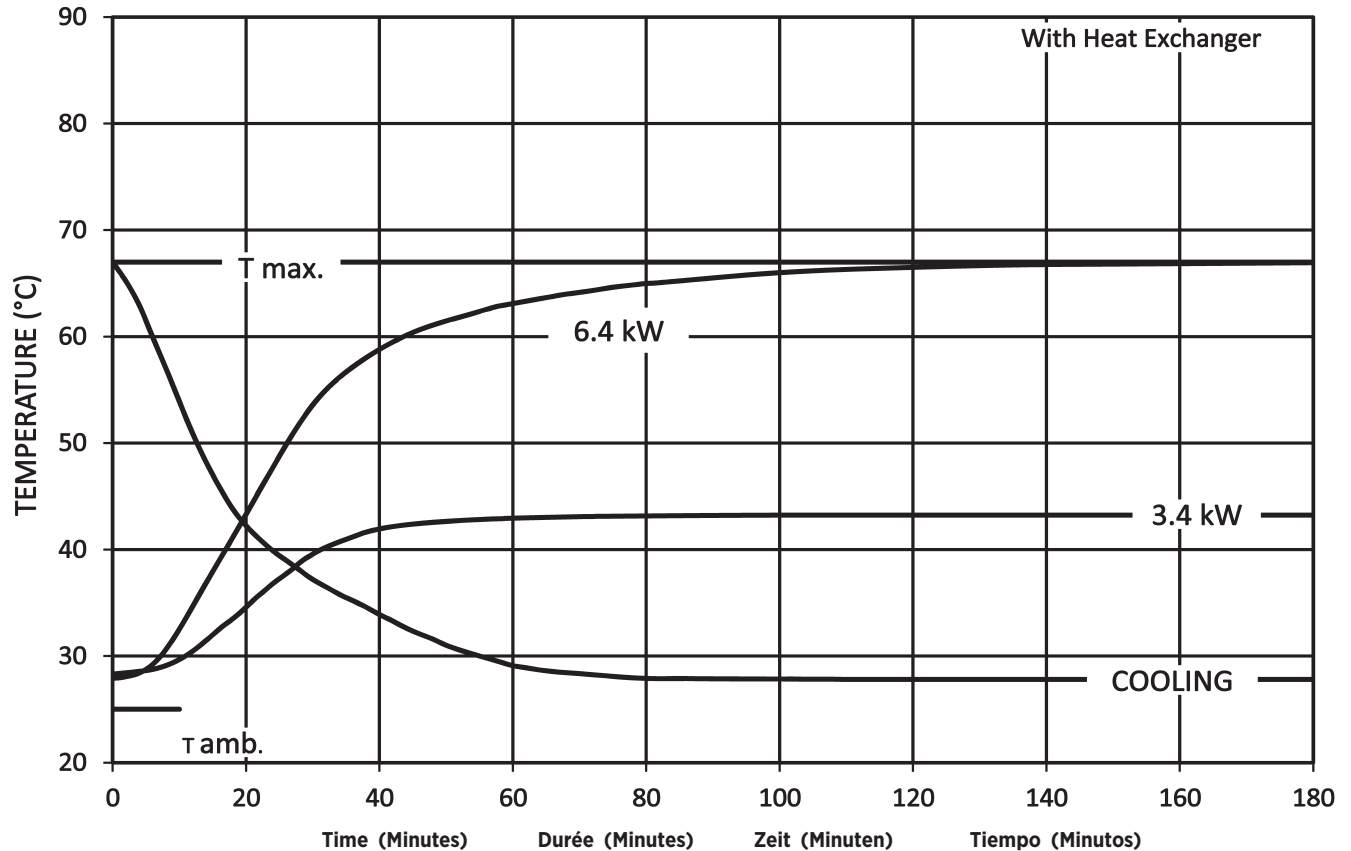**Note:**

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.
- Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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
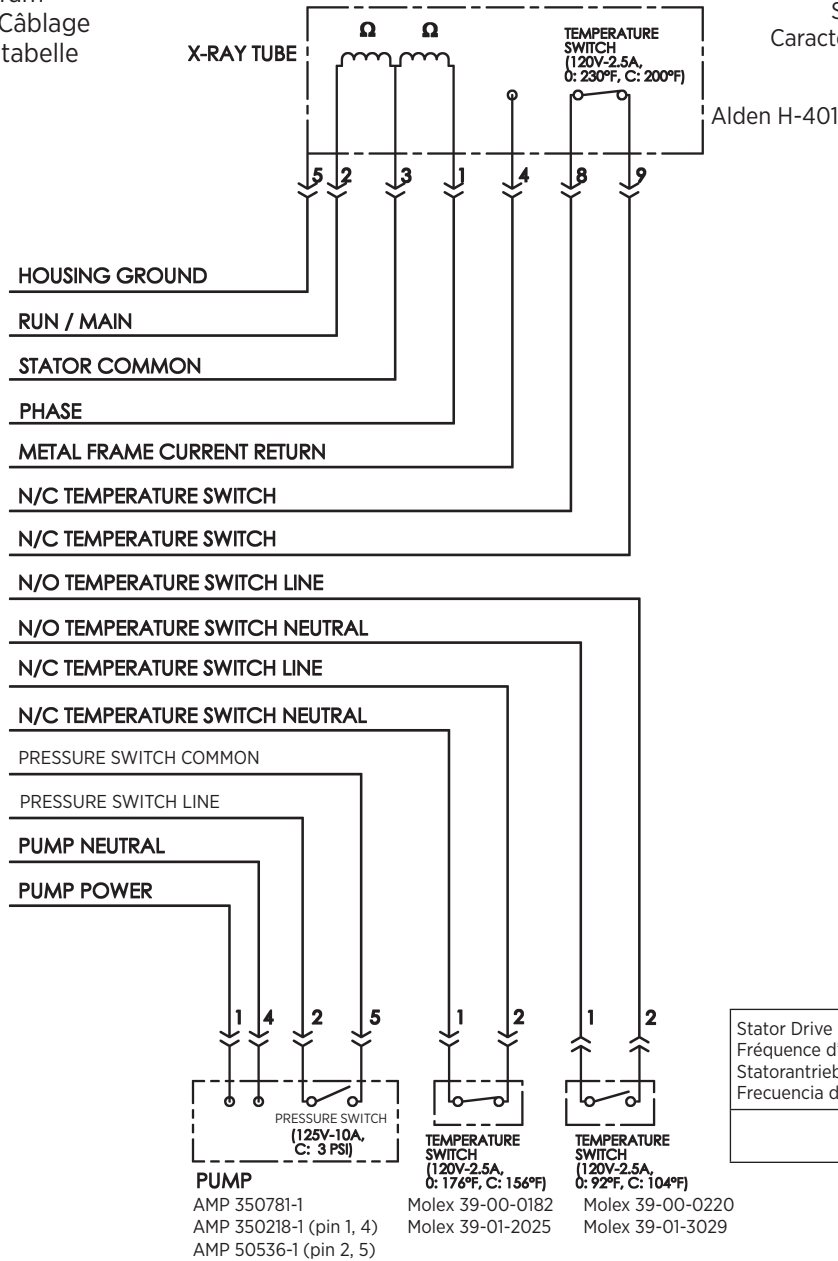
Stator Type:	
Stator Coil Resistance:	
Black to White	8.3 Ω ±15%
Green to White	8.3 Ω ±15%
Starter Voltage:	
105 Hz	Start 480 VAC    Run 160 VAC
Starter Current:	
105 Hz	Start 9 Amps    Run 3 Amps
Time to Full Speed:	
105 Hz	0 - 6300 RPM    20 Sec.
X-Ray Tube Assembly:	
GS-5177KD/B-320H	IEC 60601-2-28

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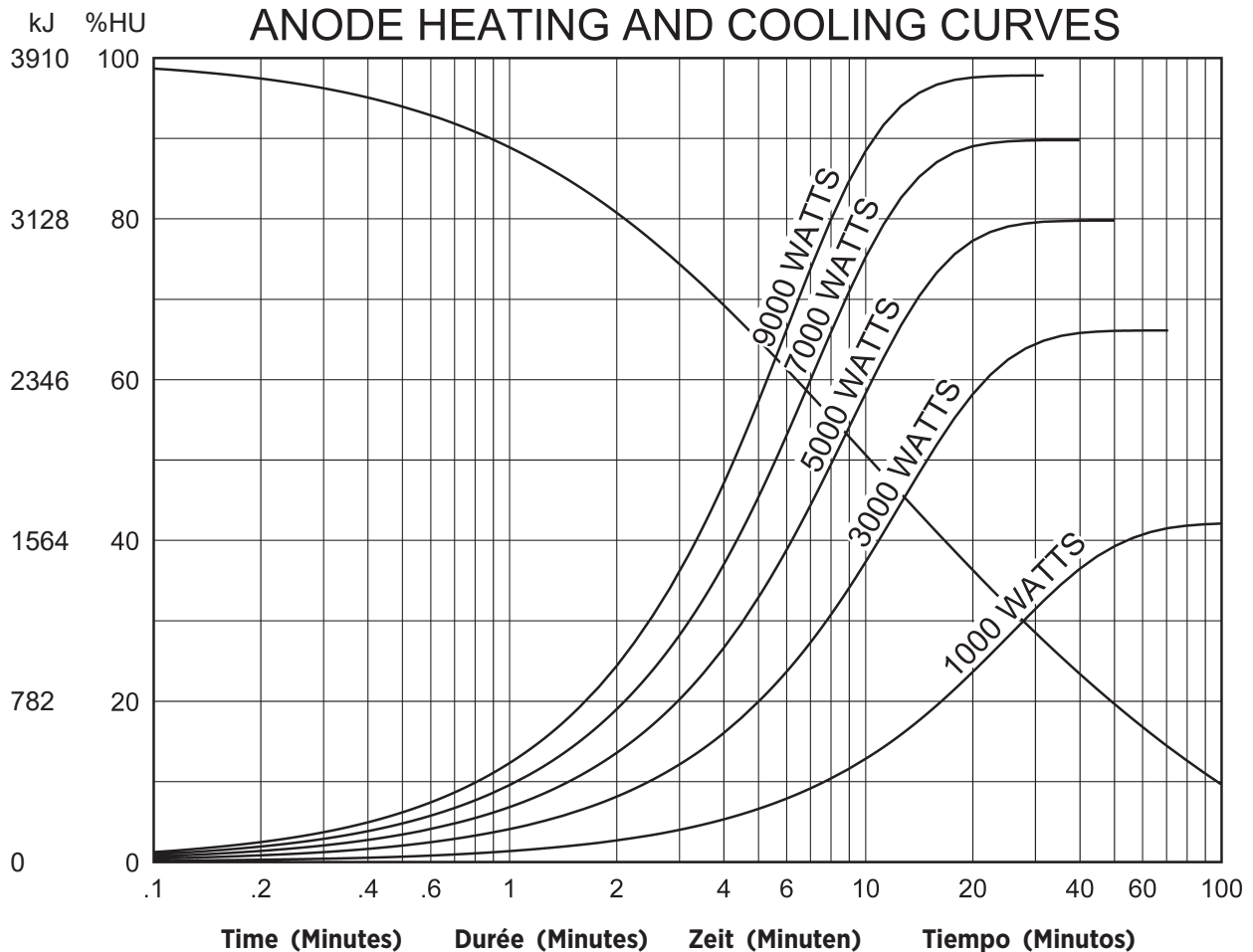**Note:**  
 1. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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