Note: Document originally drafted in the English language.

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Product Description

The GS-5179 is a 7.5" (190.5 mm) 140 kV, 3.8 MJ (5.3 MHU) maximum anode heat content, rotating anode insert. This insert is specifically designed for Neusoft CT Scanners. The insert features a 7° tungsten-rhenium facing on molybdenum with a graphite backed target and is available with the following nominal focal spot:

0.5 x 1.0
1.0 x 1.0
IEC 60336

Loading Factor for slit focal:
Small - 140 kV, 150 mA
Large - 140 kV, 210 mA

Maximum Anode Cooling Rate:
9,600 W (13,440 HU/sec)

Continuous Anode Input Power:
6,300 W (8,820 HU/sec)

Nominal CT Anode Input Power:
Small - 36 kW IEC 60613:2010
Large - 50.4 kW IEC 60613:2010

Nominal CT Scan Power Index:
Small - 36 kW IEC 60613:2010
Large - 49 kW IEC 60613:2010

Reference Axis:
Perpendicular to port face.

This insert is intended for use in Varex Imaging B-320H housing.

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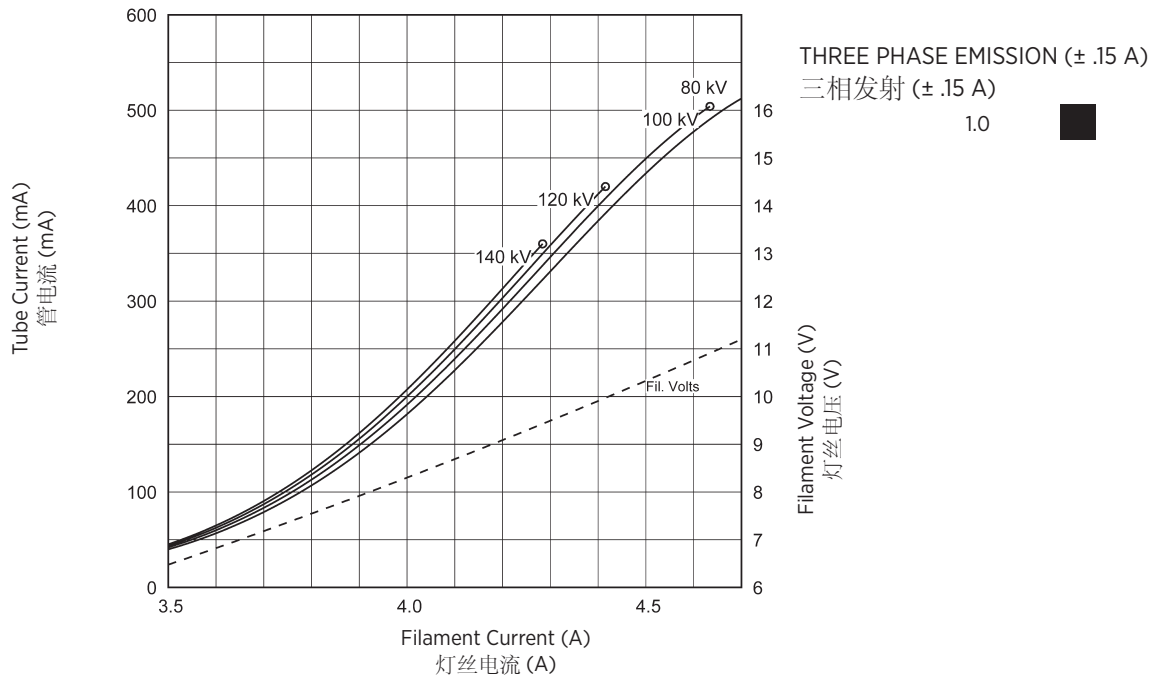
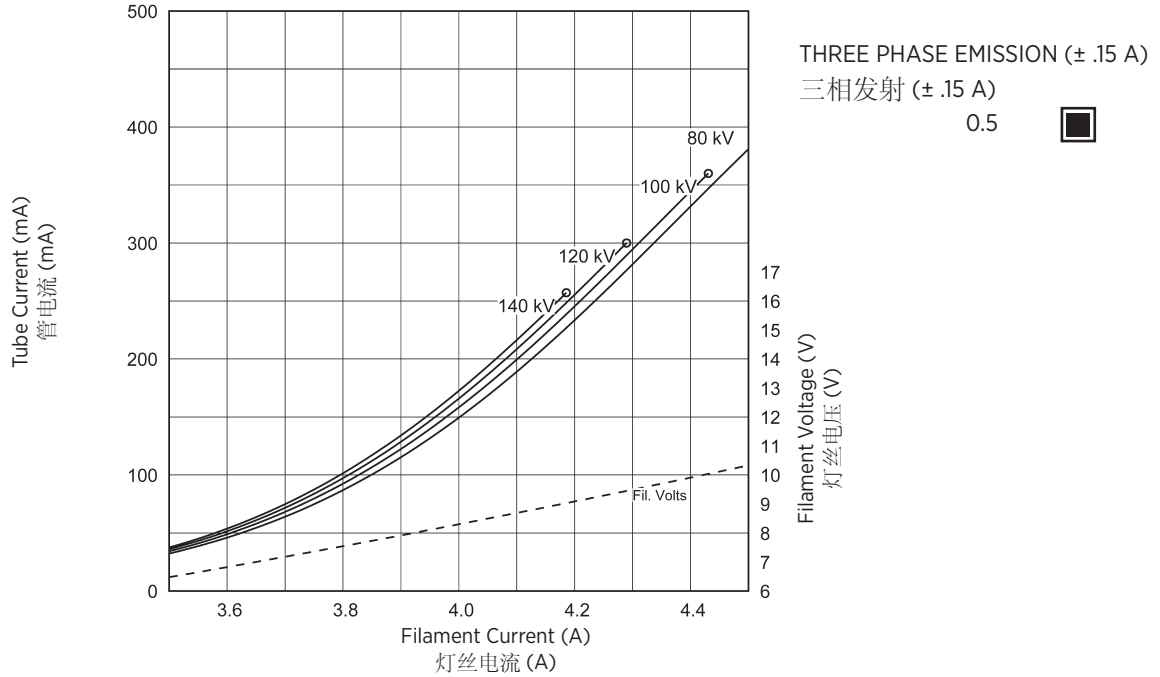
Note:
1. Limits are based on maximum track rating except for the following codes:
a - Limited by available heat storage.
b - Limited by window heating.
c - Limited by filament emission.
2. H.S. = Heat Storage
kV = Tube Voltage

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Note:
Rating charts reflect maximim tube performance. Tube operation is ultimately limited by system software.

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

Dimensions are for reference only
维度是供仅供参考

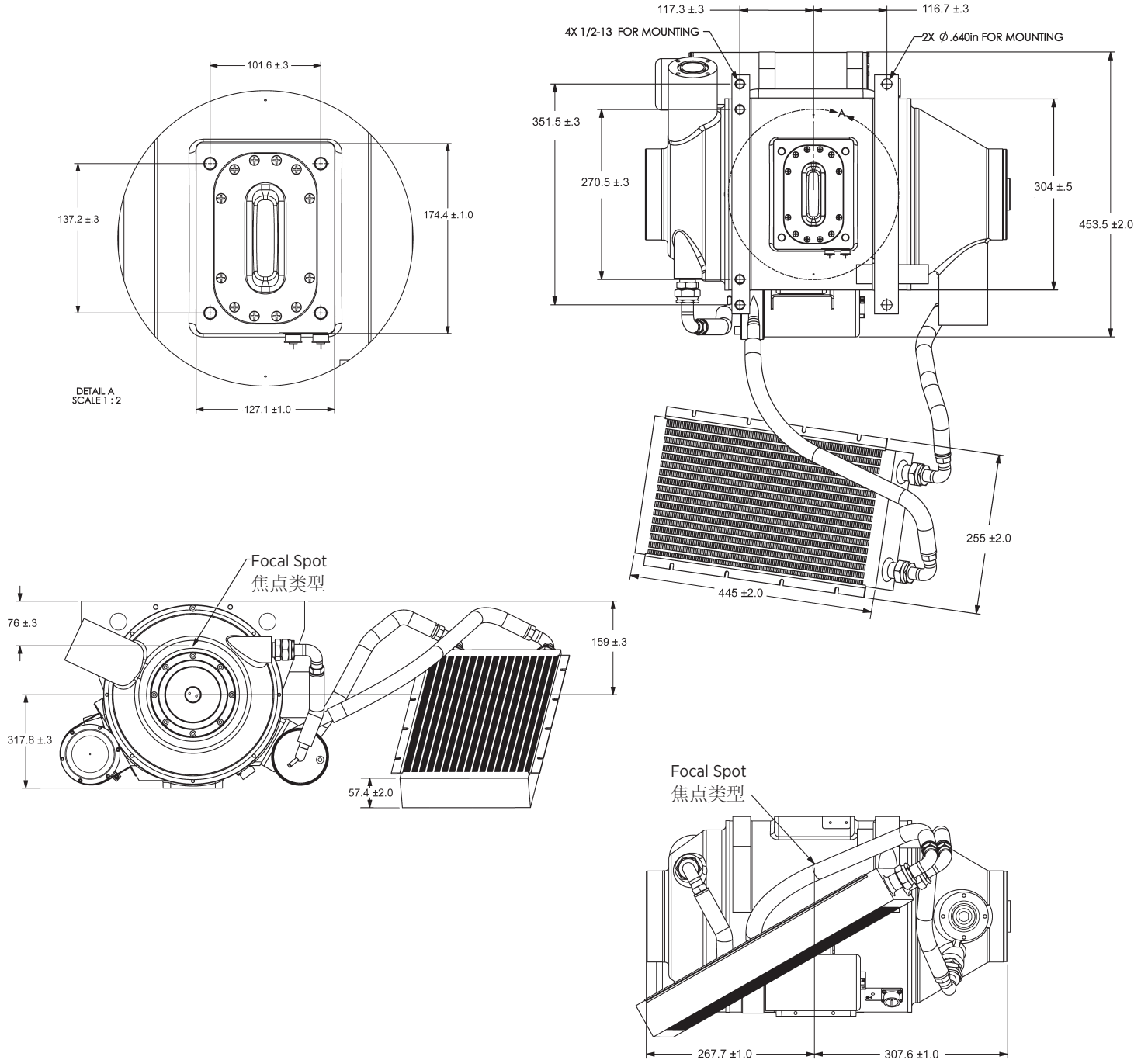Dimensions are in millimeters
尺寸单位为毫米

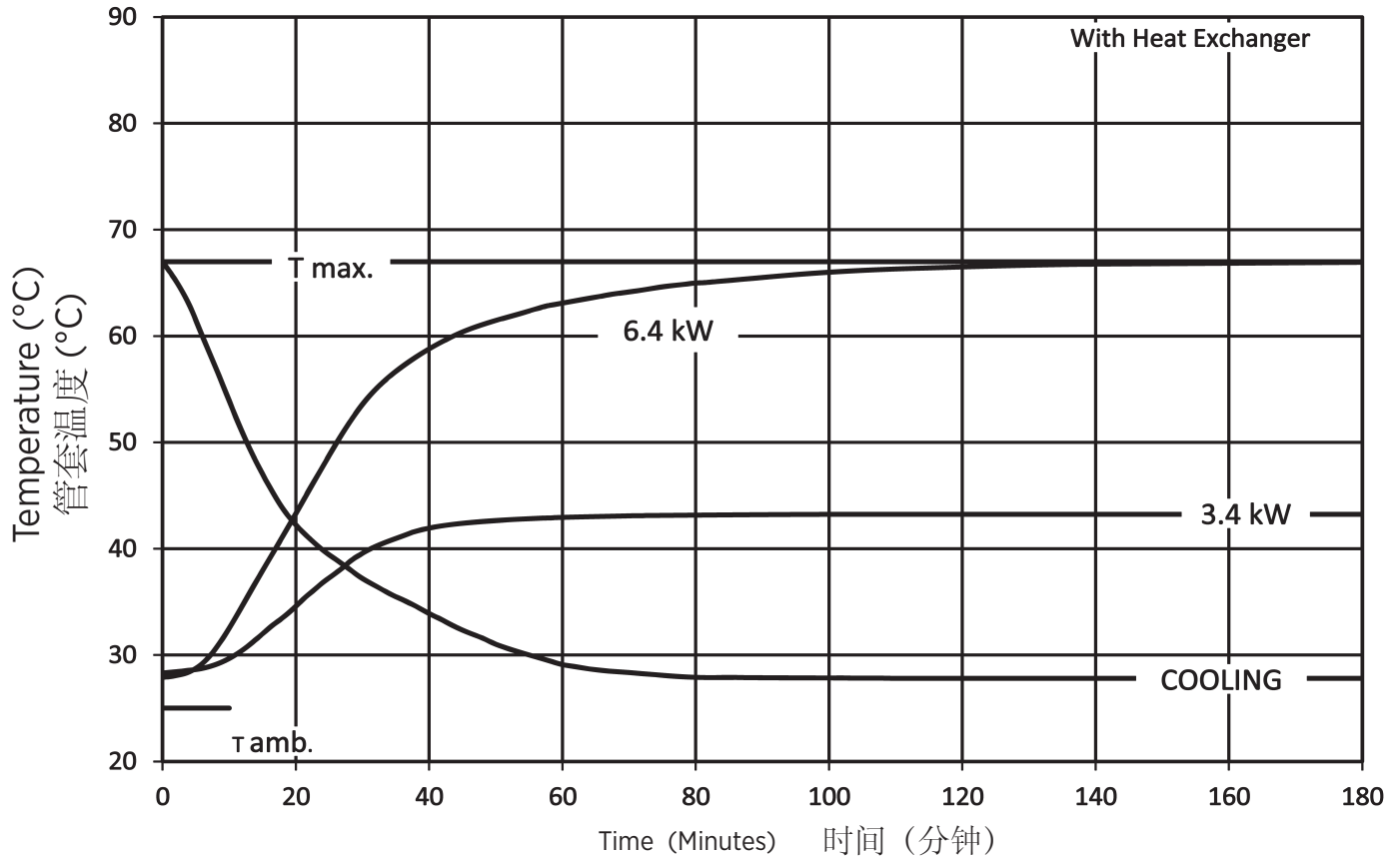
X-Ray Tube Housing Assembly Heating and Cooling
X射线管组件加热/冷却曲线

Note:
1. Heat inputs into housing include tube power, filament power, and stator power.
2. Heating curves based on no restrictions of natural convection around tube housing assembly.
3. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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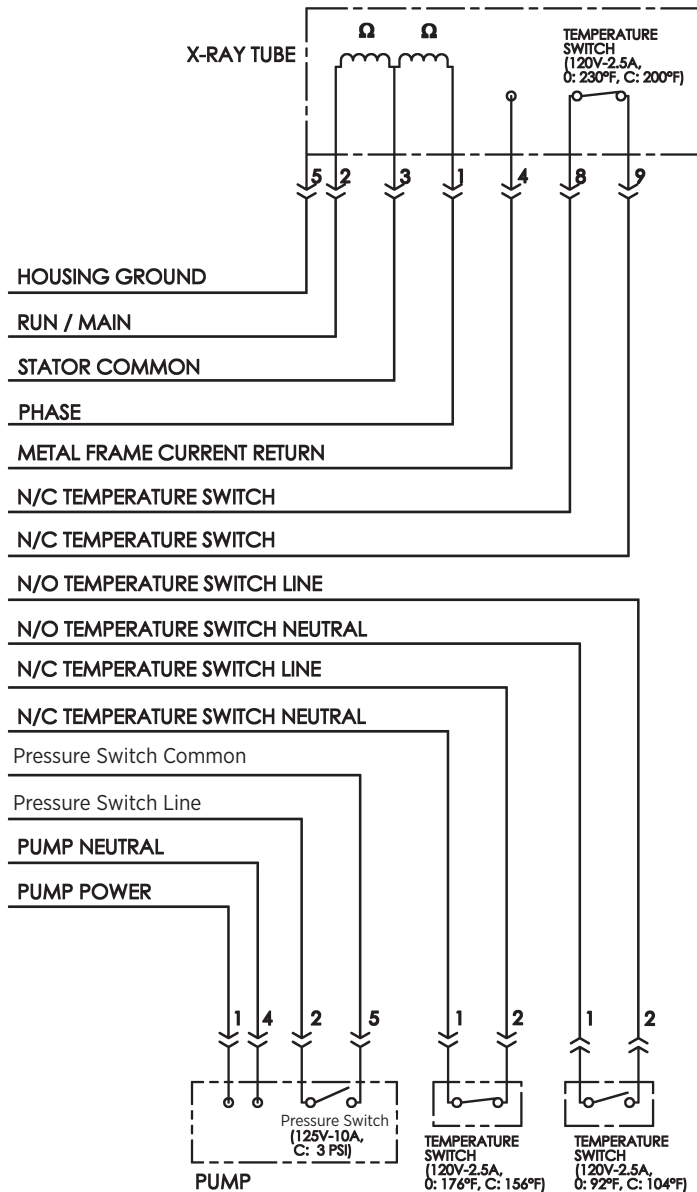
| | | |
|-------------------------|----------------|-------------|
| Stator Type: | | |
| Stator Coil Resistance: | | |
| Black to White | 8.3 Ω ±15% | |
| Green to White | 8.3 Ω ±15% | |
| Starter Voltage: | | |
| 105 Hz | Start 480 VAC | Run 160 VAC |
| Starter Current: | | |
| 105 Hz | Start 9 Amps | Run 3 Amps |
| Time to Full Speed: | | |
| 105 Hz | 0 - 6300 RPM | 20 Sec. |
| X-Ray Tube Assembly: | | |
| GS-5179/B-320H | IEC 60601-2-28 | |

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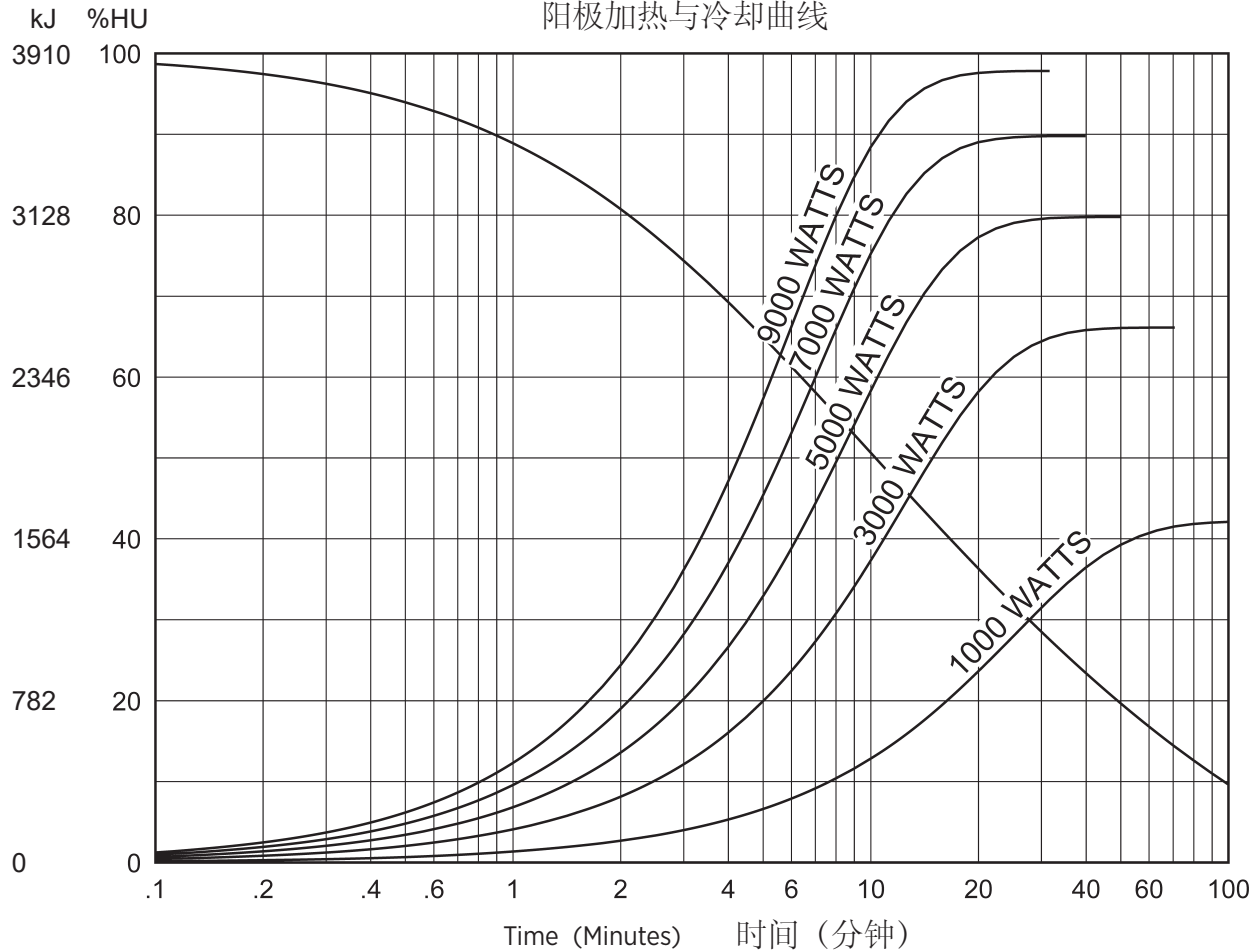Anode Heating and Cooling Curves
阳极加热与冷却曲线

Note:
Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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Manufactured by Varex Imaging Corporation
由万睿视影像有限公司生产