Note: Document originally drafted in the English language.

**Product Description**

The B-110 housing is designed for Varex Imaging mammography inserts.

IEC Classification ..... Class 1  
 Weight, Approximate:  
 Housing ..... 10.2 kg (22.5 lbs)  
 Stator Cord ..... 4.5 m (14 Ft)  
 Mounting ..... Port  
 X-Ray Tube Assembly (Complies to) ..... IEC 60601-2-28

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**Product Description**

Nominal X-ray Tube Voltage  
 Maximum Cathode to Ground ..... 0 kV  
 Maximum Anode to Ground ..... 52 kV

Grid Control Voltages: (If Applicable)  
 Typical Bias Voltage for Focals ..... -5 to -100 Vdc

Maximum X-Ray Tube Heat Content ..... 370 kJ (500 kHU)

Nominal Continuous Input Power ..... 100 Watts (135 HU/sec)  
 IEC 60613:2010

Maximum Housing Temperature ..... 78°C

X-Ray Tube Assembly  
 Permanent Filtration ..... 0.63 mm Be IEC 60522/1999

Leakage Technique Factors ..... 55 kV, 4.0 mA

Cable Receptacles ..... Non Federal Standard  
 55 kV - Rating

Thermal Switch ..... Rating - 10 A @ 240Vac  
 Normally closed ..... Open @ 79.4°C ±3.9°C (175°F ±7°F)

Ambient Air Temperature Limits for Operation ..... 5°C to 40°C

Temperature Limits for Storage and Transportation ..... -10°C to 75°C  
 Humidity ..... 10% to 90%  
 Atmospheric Pressure Range ..... 70 kPa to 106 kPa

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**Note 1** Dimensions listed are for the standard port boss  
 Metric M6 X 1.0 available upon request

**Note 2** **PORT BOSS DIMENSION**  
 Dimensionelles de la Fenêtre Patron  
 Abmessung Halterungscher  
 Dimensiones de la Portales Patrón

**For Determining Focal Spot to Port Distance**  
 Distance du Focales Suivantes de la Fenêtre  
 Abstand Brennfleck zu Strahlenaustrittsfenster  
 Punto Focal a la Distancia de la Abertura Dimensiones de la Portales

**M-101, M-101G, M-107, M-131, M-141, M-141G, M-143, RAD-85S**

**STANDARD/METRIC PORT BOSS**  
 DIM. F = 2.97 INCHES [75.4mm]  
 DIM I = 1.85 INCHES [47mm]

**EXTENDED PORT BOSS**  
 DIM. F = 3.20 INCHES [81.3mm]  
 DIM. I = 2.08 INCHES [52.8mm]

**M-145, M-147, M-147SP**

**STANDARD/METRIC PORT BOSS**  
 DIM. F = 2.97 INCHES [75.4mm]  
 DIM. I = 1.74 INCHES [44.2mm]

**EXTENDED PORT BOSS**  
 DIM. F = 3.20 INCHES [81.3mm]  
 DIM. I = 1.97 INCHES [50.0mm]

**M-105SP, M-109, M-149**

**STANDARD/METRIC PORT BOSS**  
 DIM. F = 2.97 INCHES [75.4mm]  
 DIM. I = 1.66 INCHES [42.2mm]





**EXTENDED PORT BOSS**  
 DIM. F = 3.20 INCHES [81.3mm]  
 DIM. I = 1.89 INCHES [48mm]

**BIANGULAR INSERTS**  
**M-103T, M-113 Series, M-171**


**STANDARD/METRIC PORT BOSS**  
 DIM. F = 2.97 INCHES [75.4mm]  
 DIM. I = Refer to Drawing

**EXTENDED PORT BOSS**  
 DIM. F = 3.20 INCHES [81.3mm]  
 DIM. I = Refer to Drawing

**Note:** Dimensions in [mm] Inches  
**Remarque:** Dimensions en [mm] Pouces  
**Hinweis:** Abmessungen in [mm] Zoll  
**Nota:** Dimensiones en [mm] Pulgadas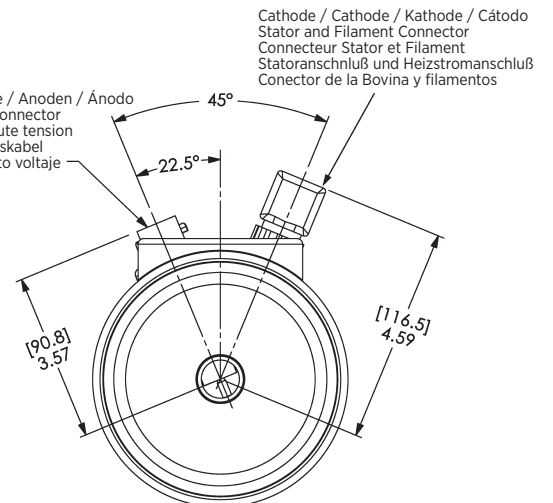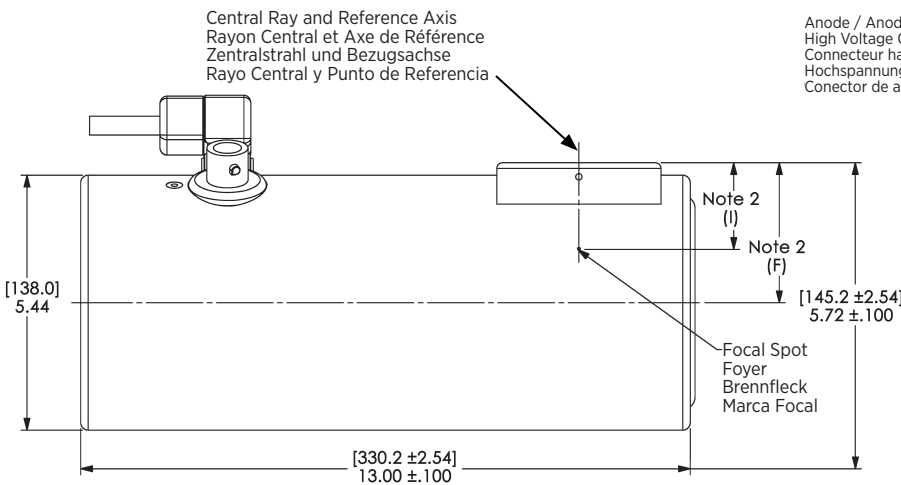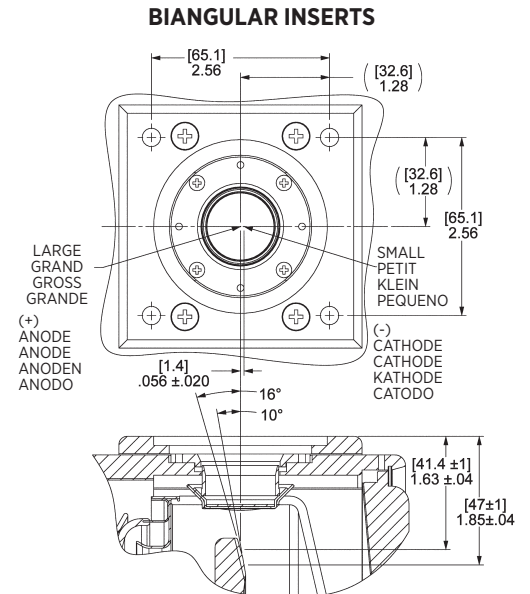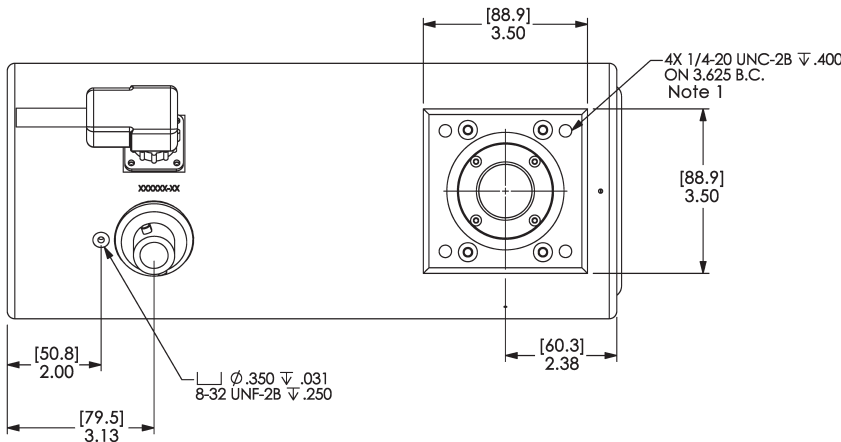

Stator Ratings and Characteristics  
 Spécificités et Caractéristiques du Stator  
 Statornennleistungen und Merkmale  
 Características y Clarificación de la Bovina

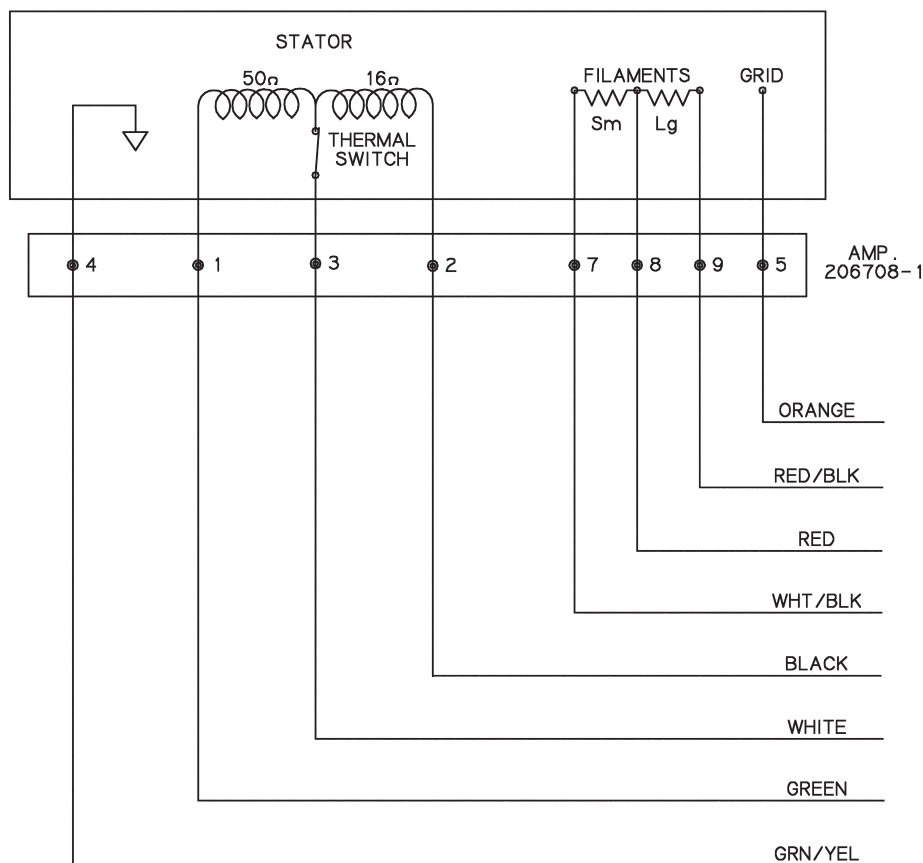1 Green	Phase Shift
Vert	Changement de Phase
Grün	Hilfsphase
Verde	Cambio de Fase del Estator
2 Black	Phase
Noir	Phase
Schwarz	Phase
Negro	Fase
3 White	Common
Blanc	Neutre
Weiss	Neutral
Blanco	Común
4 Green/Yellow	Housing Ground
Vert/Jaune	Masse de la Gaine
Grün/Gelb	Masse des Gehäuses
Verde/Amerillo	Encaje a Tierra
5 Orange	Grid
Orange	Grille
Orange	Gitter
Anaranjado	Rejillas
6 Not used / Hors'd Usage / Nicht in Gebrauch/ No utilizado	
7 White/Black	Small Filament
Blanc/Noir	Filament Petit
Weiss/Schwarz	Glüfaden Klein
Blanco/Negro	Filamento Pequeño
8 Red	Common Filament
Rouge	Filament Neutre
Rot	Glüfaden Neutral
Rojo	Filamento Común
9 Red/Black	Large Filament
Rouge/Noir	Filament Grand
Rot/Schwarz	Glüfaden Gross
Rojo/Negro	Filamento Largo

"STD" Stators	Stator "STD"	"STD" Stator	"STD" Bovina	
Black/White	Noir/Blanc	Schwarz/Weiss	Negro/Blanco	16 Ω
Black/Green	Noir/Vert	Schwarz/Grün	Negro/Verde	66 Ω
Green/White	Vert/Blanc	Grün/Weiss	Verde/Blanco	50 Ω
180 Hz Cap	180 Hz Cap	180 Hz Cap	180 Hz Cap	6 µF
60 Hz Cap	60 Hz Cap	60 Hz Cap	60 Hz Cap	30 µF

Stator Drive Frequency Fréquence d'entraînement du stator Statorantrieb Frequenz Frecuencia de la impulsión del estator	RPM
50 Hz	2800 - 3000
60 Hz	3400 - 3600
150 Hz	8500 - 9000
180 Hz	9500 - 10,800

**Stator Power:**  
 Time to full speed of the anode is a function of the power rating of the "starter" and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible.

Starter:	Start	Run
50/60 Hz	115 VAC	45 VAC
150/180 Hz	230 VAC	90 VAC

**NOTE:**  
 For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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Manufactured by Varex Imaging Corporation
Fabrique par Varex Imaging Corporation
Hergestellt von Varex Imaging Corporation
Fabricado por Varex Imaging Corporation