Note: Document originally drafted in the English language.

Product Description

The B-115 housing is designed for Varex Imaging mammography inserts.

IEC Classification ..... Class 1  
 Weight, Approximate:  
 Housing ..... 10.2 kg (22.5 lbs)  
 Stator Cord ..... 4.5 m (14 Ft)  
 Mounting ..... Port  
 X-Ray Tube Assembly (Complies to) ..... IEC 60601-2-28

Caution: Tubes ran in the vertical position must have the two thermal switch configuration.

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Product Description	
Nominal X-ray Tube Voltage	
Maximum Cathode to Ground	0 kV
Maximum Anode to Ground	39 - 49 kV
Grid Control Voltages: (If Applicable)	
Typical Bias Voltage for Focals	-5 to -100 Vdc
Maximum X-Ray Tube Heat Content	370 kJ (500 kHU)
Nominal Continuous Input Power	
Horizontal	100 Watts (135 HU/sec) IEC 60613:2010
Vertical	90 Watts (121.5 HU/sec) IEC 60613:2010
Maximum Housing Temperature	78°C
X-Ray Tube Assembly	
Permanent Filtration	0.63 mm Be IEC 60522/1999 (0.0mm Al/49 kV)
Leakage Technique Factors	49 kV, 2.0 mA
Cable Receptacles	Non Federal Standard 55 kV - Rating
Thermal Switch	normally closed
Open	79.4°C ±3.9°C (175°F ±7°F) Rating - 10A @ 240VAC
Second Thermal Switch for Vertical Applications	85°C ±3°C (185°F ±5.4°F) Rating - 10A @ 240VAC
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transportation	-10°C to 75°C
Humidity	10% to 90%
Atmospheric Pressure Range	70 kPa to 106 kPa

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**Note 1** Dimensions listed are for the standard port boss  
 Metric M6 X 1.0 available upon request

**Note 2** **PORT BOSS DIMENSION**  
 Dimensionelles de la Fenêtre Patron  
 Abmessung Halterungscher  
 Dimensiones de la Portales Patrón

**For Determining Focal Spot to Port Distance**  
 Distance du Focales Suivantes de la Fenêtre  
 Abstand Brennfleck zu Strahlenaustrittsfenster  
 Punto Focal a la Distancia de la Abertura Dimensiones de la Portales

**M-101, M-101G, M-107, M-131, M-141, M-141G, M-143, RAD-85S**

**STANDARD/METRIC PORT BOSS**  
 DIM. F = 2.97 INCHES [75.4mm]  
 DIM I = 1.85 INCHES [47mm]

**EXTENDED PORT BOSS**  
 DIM. F = 3.20 INCHES [81.3mm]  
 DIM. I = 2.08 INCHES [52.8mm]

**M-145, M-147, M-147SP**

**STANDARD/METRIC PORT BOSS**  
 DIM. F = 2.97 INCHES [75.4mm]  
 DIM. I = 1.74 INCHES [44.2mm]

**EXTENDED PORT BOSS**  
 DIM. F = 3.20 INCHES [81.3mm]  
 DIM. I = 1.97 INCHES [50.0mm]

**M-105SP, M-109, M-149**

**STANDARD/METRIC PORT BOSS**  
 DIM. F = 2.97 INCHES [75.4mm]  
 DIM. I = 1.66 INCHES [42.2mm]

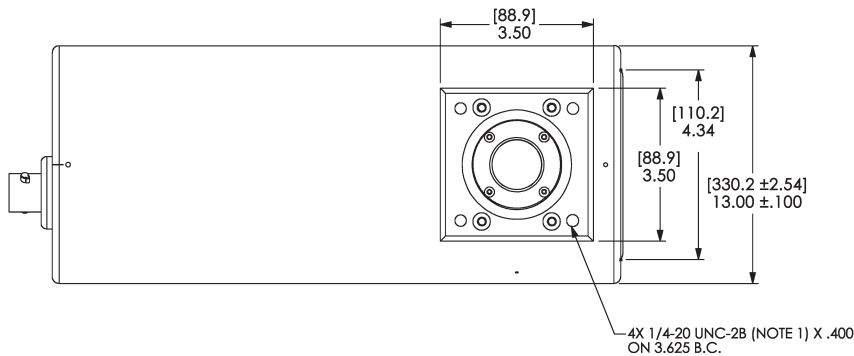
**EXTENDED PORT BOSS**  
 DIM. F = 3.20 INCHES [81.3mm]  
 DIM. I = 1.89 INCHES [48mm]

**BIANGULAR INSERTS**  
**M-103T, M-113 Series, M-171**


**STANDARD/METRIC PORT BOSS**  
 DIM. F = 2.97 INCHES [75.4mm]  
 DIM. I = Refer to Drawing

**EXTENDED PORT BOSS**  
 DIM. F = 3.20 INCHES [81.3mm]  
 DIM. I = Refer to Drawing


Note: Dimensions in [mm] Inches  
 Remarque: Dimensions en [mm] Pouces  
 Hinweis: Abmessungen in [mm] Zoll  
 Nota: Dimensiones en [mm] Pulgadas

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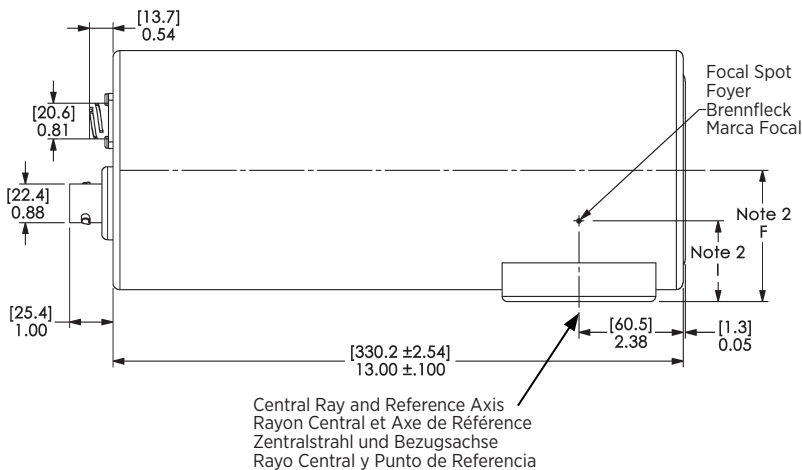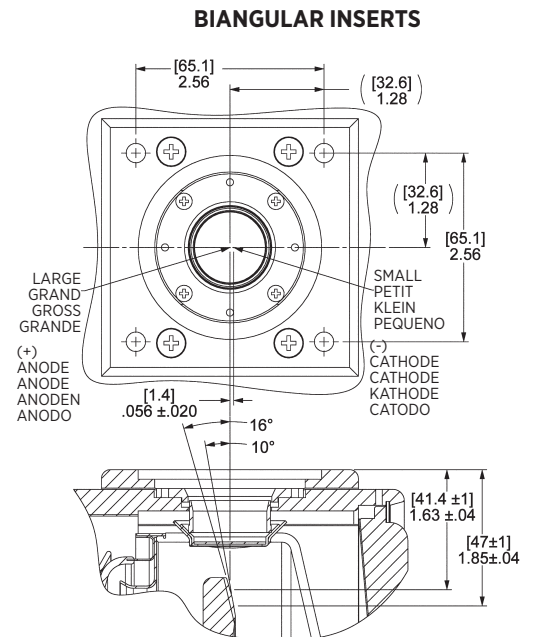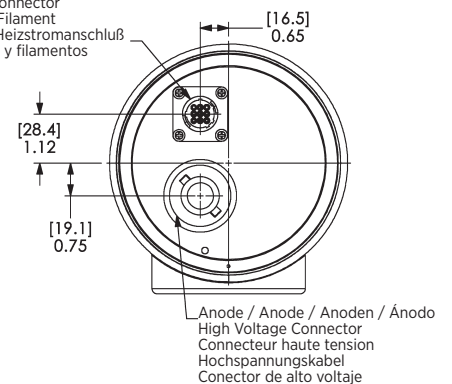
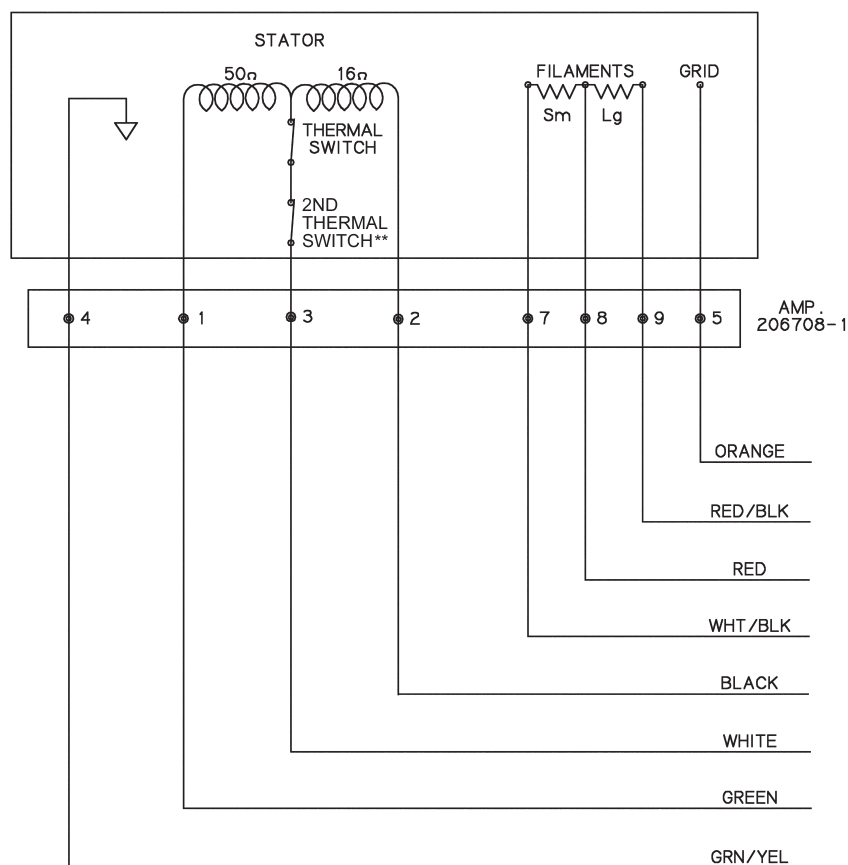
**Stator Power:**  
 Time to full speed of the anode is a function of the power rating of the “starter” and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.  
 Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.  
 No more than two high speed starts per minute are permissible.  
**Starter:**  

	<b>Start</b>	<b>Run</b>
50/60 Hz	115 VAC	45 VAC
150/180 Hz	230 VAC	90 VAC

**NOTE:**  
 For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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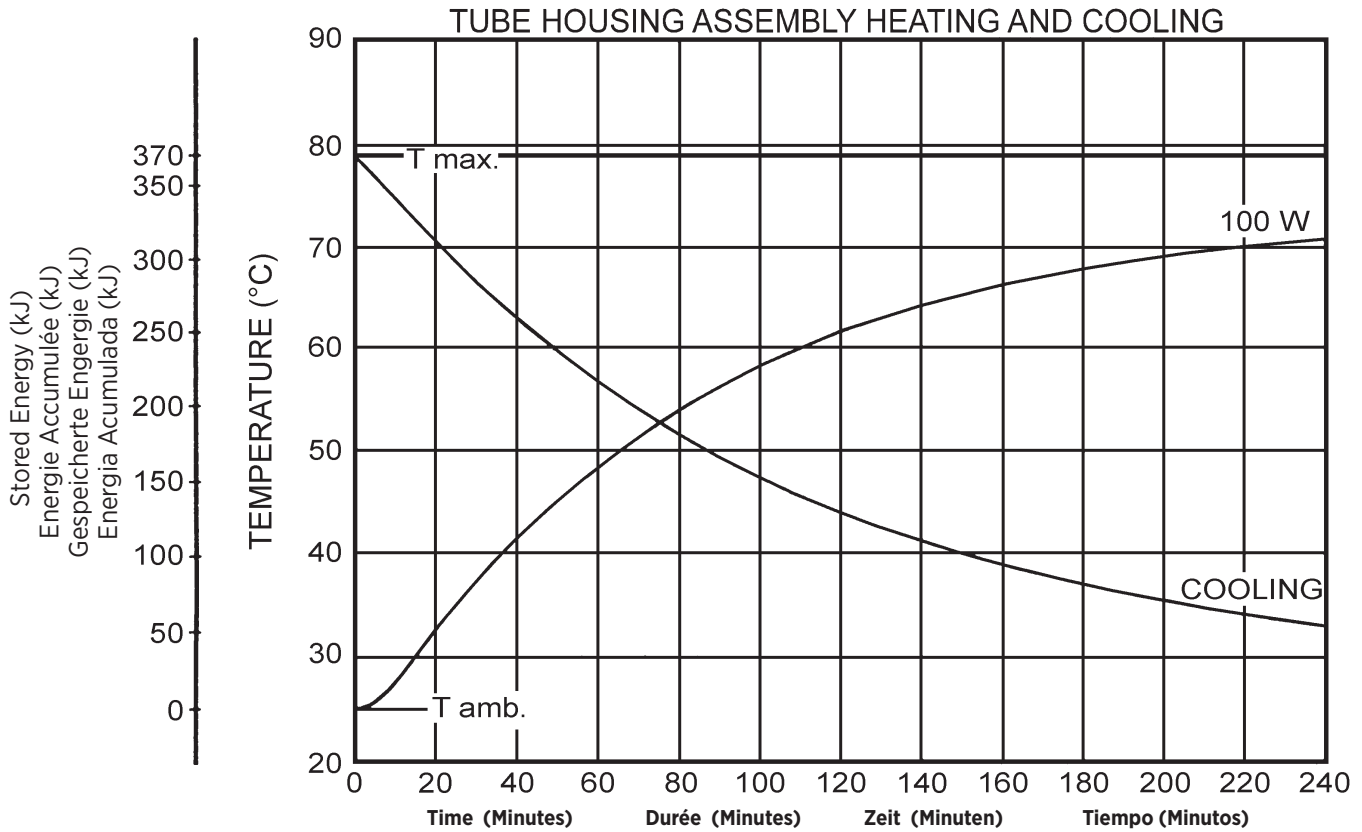
**Note:**  
 Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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